

SonoSite
FUJIFILM

X-PORTE

User Guide

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution | Federal (United States) law restricts this device to sale by or on the order of a physician.

SonoMB, SonoSite, Steep Needle Profiling, X-Porte, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

DICOM is a registered trademark of the National Electrical Manufacturers Association.

All other trademarks are the property of their respective owners

Patents: US 8,216,146; US 8,213,467; US 8,147,408; US 8,137,278; US 8,088,071; US 8,066,642; US 8,052,606; US 7,819,807; US 7,804,970; US 7,740,586; US 7,686,766; US 7,604,596; US 7,591,786; US 7,588,541; US 7,534,211; US 7,449,640; US 7,169,108; US 6,962,566; US 6,648,826; US 6,575,908; US 6,569,101; US 6,471,651; US 6,416,475; US 6,383,139; US 6,364,839; US 6,203,498; US 6,135,961; US 5,893,363; US 5,817,024; US 5,782,769; US 5,722,412; AU: 730822; AU: 727381; CA: 2,372,152; CA: 2,371,711; CN 98108973.9; CN: 98106133.8; CN: 97113678.5; DE: 69831698.3; DE: 69830539.6; DE: 69730563.5; DE: 602004027882.3; DE: 602004023816.3; DE: 60034670.6; DE: 60029777.2; EP: 1589878; EP: 1552792; EP: 1180971; EP: 0875203; EP: 0815793; EP 1180970; EP 0881492; ES: 2229318; ES: 159878; ES: 1552792; ES: 0881492; FR: 158978; FR: 1552792; FR: 1180970; FR: 0881492; FR: 0875203; FR: 0815793; GB: 158978; GB: 1552792; GB: 1180971; GB: 1180970; GB: 0881492; GB: 0875203; GB: 0815793; IT: 1589878; IT: 1552792; IT: 0881492; IT: 0815793; JP: 4696150; KR: 532359; KR: 528102; NO: 326814; NO: 326202

Copyright © 2014 FUJIFILM SonoSite, Inc. All Rights reserved.

Part Number: P14645-02

Publication Date: May 2014

CE
0086

STENTEN CONTENTS

CHAPTER 1

Introduction	1
About the X-Porte User Guide	1
Welcome to the X-Porte User Guide	2
Document conventions	2
Getting help	3

CHAPTER 2

Getting started	5
About the system	5
Accessories and peripherals	6
Preparing the system	7
Turning on the system	7
Adjusting the height and angle	8
USB devices	9
USB memory sticks	9
General interaction	10
Clinical monitor	10
Touch panel	11
Onscreen keyboard	12
Preparing transducers	13
Connecting transducers	13
Selecting a transducer and exam type	16
Gel	20
Sheaths	21
Ports	21
Battery charge indicator	21
Transporting the system	22
Visual Guide videos	23
Intended uses	25
Contraindications	26

CHAPTER 3

Setting up the system	27
-----------------------------	----

CONTENTS

Administration settings	28
About security settings	29
Managing the Administrator account	29
Protecting patient information	30
Adding and managing user accounts	31
Logging in	34
Audio settings	34
Calculations settings	35
Cardiac calculations settings	35
Obstetrics calculations settings	36
Specifying gestational age and growth analysis	36
Creating custom obstetrics measurements	36
Viewing obstetric calculation tables	38
Setting up custom obstetric tables	38
Exporting and importing custom obstetric tables and measurements	39
Deleting custom obstetric tables	40
CDA Report settings	41
Connectivity settings	43
Importing and exporting connectivity settings	43
Date and Time settings	45
DICOM	46
Configuring the system for DICOM transfer	47
Connecting to the network	47
DICOM configuration pages	50
Configuration setup fields	52
Associating devices with locations	59
Display Information settings	61
Labels	62
Logs	63
Network Status settings	65
Power and Battery settings	66
Presets settings	66
General preferences	67
Brightness	67
Labels	68
Exam types	69
Customizing exam types	70
User profile settings	71
Importing and exporting	73
Routing selections	74
Associating routing selections with exams	75

CONTENTS

Specifying educational DICOM archivers	76
System Information settings	77
USB settings	77
Limitations of JPEG format	79
eFilm Lite image-viewer	79

CHAPTER 4

Imaging	81
Understanding imaging modes	81
2D	82
M-line	83
M Mode	83
Color	85
D-line	86
Doppler	86
Dual	88
Scanning	88
2D	89
Color	90
Dual	91
Doppler	93
M Mode	96
Imaging controls	99
Controls in 2D	101
Controls in frozen 2D	108
Controls in Color	110
Controls in frozen Color	115
Controls in M Mode	116
Controls in frozen M Mode	120
Controls in Doppler	120
Controls in D-line	121
Controls in scrolling Doppler	127
Adjusting depth and gain	135
Depth	135
Gain	136
Freezing, viewing frames, and zooming	136
Freezing the image	136
Viewing the cine buffer	138
Zooming in on the image	138
Visualizing needles	140

CONTENTS

Needle size and angle	143
Additional recommendations	143
Labeling images	143
Text	143
Arrows	145
Pictographs	146
Setting the home position	147
Labeling during review	147
Entering patient information	150
Editing patient information	150
Entering patient information manually	151
Entering patient information from the worklist	151
Updating and querying the worklist	153
Ending the exam	155
Patient form fields	156
Images and clips	158
Reviewing	158
Patient list	158
Reviewing images and video clips	161
Printing images	164
Archiving and exporting	165
Archiving exams	165
Exporting exams	168
Saving images and video clips	169
Saving images	170
Saving video clips	171
Managing internal storage space	172
DVR recording	173
ECG	174

CHAPTER 5

Measurements and calculations	177
Measuring	177
Calipers	177
Viewing and deleting measurement results	178
Basic measurements in 2D	178
Basic measurements in M Mode	180
Basic measurements in Doppler	180
Automatic trace results	182
Assigning measurements to calculations	183
About calculations	183

CONTENTS

Overview	184
Percent reduction calculations	186
Volume calculation	188
Volume flow calculation	188
Required measurements	189
Considerations	189
Calculating volume flow	189
Exam-based calculations	190
Arterial calculations	190
Cardiac calculations	192
Gynecological calculations	207
Measuring uterus	207
Measuring ovaries	208
Measuring follicles	209
Obstetrics calculations	209
Measuring gestational sacs	215
Small Parts calculations	215
Acute Care calculations	217
Worksheets and reports	220
Report preview	221
Arterial and cardiac reports	221
Obstetrics reports	222
Obstetrics graphs	222
Acute Care and MSK worksheets	223
Printing reports and worksheets	224
Displaying reports and worksheets after the exam has ended	225

CHAPTER 6

Measurement references	227
Measurement accuracy	227
Sources of measurement errors	229
Measurement publications and terminology	229
Cardiac references	230
Obstetrical references	238
Gestational age tables	240
Abdominal Circumference (AC)	240
Biparietal Diameter (BPD)	240
Cisterna Magna (CM)	240
Crown Rump Length (CRL)	240
Femur Length (FL)	241

CONTENTS

Fetal Trunk Cross-Sectional Area (FTA)	241
Gestational Sac (GS)	241
Head Circumference (HC)	241
Humerus (HL)	241
Occipito-Frontal Diameter (OFD)	242
Tibia	242
Transverse Trunk Diameter (TTD)	242
Growth analysis tables	242
Abdominal Circumference (AC)	242
Biparietal Diameter (BPD)	242
Estimated Fetal Weight (EFW)	242
Femur Length (FL)	243
Head Circumference (HC)	243
Head Circumference (HC)/Abdominal Circumference (AC)	243
Ratio calculations	243
FL/AC Ratio	243
FL/BPD Ratio	243
FL/HC Ratio	244
HC/AC Ratio	244
General references	244

CHAPTER 7

Troubleshooting and maintenance	249
Troubleshooting	249
Software licensing	251
Maintenance	252
Disinfecting	252
System backups	252
Patient data	252
System configuration settings	252
Servicing	253
Cleaning and disinfecting	255
Introduction	255
Ultrasound system	255
Cleaning and disinfecting the ultrasound system	256
Compatible disinfectants for the ultrasound system	256
Transducers	257
Cleaning and disinfecting transducers	258
Compatible disinfectants for standard transducers	259
Compatible disinfectants for specialty transducers	263
Accessories	264

CONTENTS

Cleaning and disinfecting accessories	264
Cleaning and disinfecting the stand or Triple Transducer Connect (TTC)	264
Cleaning the footswitch or ECG cable	265
Compatible disinfectants for accessories	265

CHAPTER 8

Safety	271
Ergonomic safety	271
Position the system	272
Position yourself	272
Take breaks, exercise, and vary activities	273
Electrical safety	273
Electrical safety classification	275
Isolating the X-Porte ultrasound system from power	275
Equipment safety	276
Clinical safety	277
Hazardous materials	278
Electromagnetic compatibility	278
Wireless transmission	279
Electrostatic discharge	279
Separation distance	281
Compatible accessories and peripherals	281
Manufacturer's declaration	284
Immunity testing requirements	288
Labeling Symbols	288
Specifications	293
Dimensions	293
Environmental limits	293
Operating	294
Shipping and storage	294
Electrical	294
Imaging modes	294
Image and video clip storage capacity	294
Standards	295
Electromechanical safety standards	295
EMC standards classification	295
DICOM standard	296
HIPAA standard	296

CONTENTS

CHAPTER 9

Acoustic output	297
ALARA principle	297
Applying the ALARA principle	298
Direct, indirect, and receiver controls	298
Acoustic artifacts	299
Guidelines for reducing MI and TI	299
Output display	300
MI and TI output display accuracy	301
Factors that contribute to display uncertainty	301
Related guidance documents	302
Transducer surface temperature rise	302
Acoustic output measurement	303
In Situ, derated, and water value intensities	303
Tissue models and equipment survey	304
Acoustic output tables	305
Acoustic measurement precision and uncertainty	345
Terminology in acoustic output tables	346
Glossary	349
General terms	349
Terminology in acoustic output tables	350

CHAPTER 1


Introduction

This section introduces the X-Porte ultrasound system and this user guide.

About the X-Porte User Guide

The X-Porte User Guide is intended for a user familiar with ultrasound. It does not provide training in sonography, ultrasound, or clinical practices. Before using the X-Porte ultrasound system, you must complete such training.

Welcome to the X-Porte User Guide

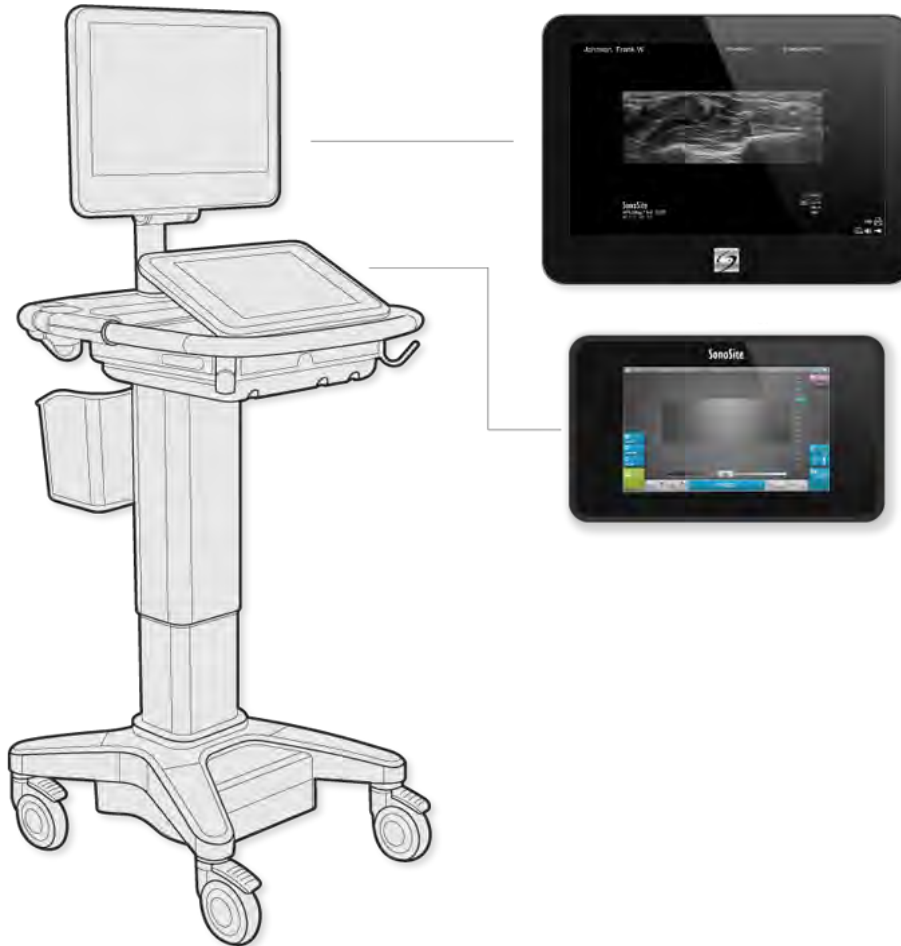

Document conventions

- ▶ A WARNING describes precautions necessary to prevent injury or loss of life.
- ▶ A Caution describes precautions necessary to protect the products.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" on page 288.

Getting help

In addition to the *X-Porte User Guide*, the following are available:

- ▶ Visual Guide videos. See "Visual Guide videos" on page 23.
- ▶ On-system Help: tap **MORE**, and then tap **Help**.
- ▶ *X-Porte Getting Started Guide*.
- ▶ Service manual.
- ▶ FUJIFILM SonoSite Technical Support

Phone (U.S. or Canada)	(877) 657-8118
Phone (Outside U.S. or Canada)	(425) 951-1330, or call your local representative
Fax	(425) 951-6700
E-mail	service@sonosite.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020
	English support: +44 14 6234 1151
	French support: +33 1 8288 0702
	German support: +49 69 8088 4030
	Italian support: +39 02 9475 3655
	Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5589

Printed in the U.S.

CHAPTER 2

Getting started

WARNING | Do not use the system if it exhibits erratic or inconsistent behavior. Such behavior indicates a hardware failure. Contact FUJIFILM SonoSite Technical Support.

About the system

X-Porte is a portable device that acquires and displays high-resolution, real-time ultrasound images. Features available depend on your system configuration, transducer, and exam type.

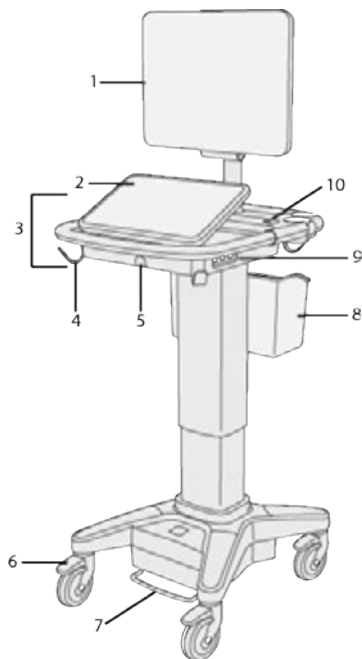


Figure 2-1 X-Porte front view

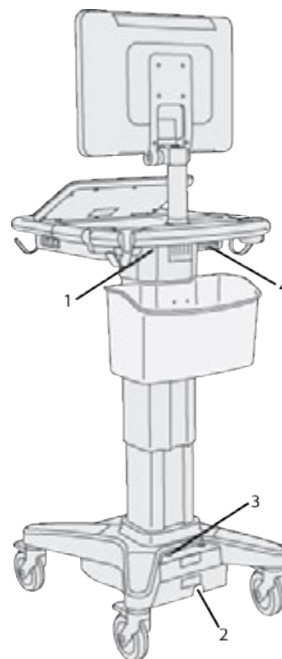


Figure 2-2 X-Porte rear view

1. Clinical monitor, 2. Touch panel, 3. Platform, 4. Hook (4), 5. Transducer connector, 6. Locking wheel (4), 7. Height-adjustment pedal, 8. Basket, 9. USB ports (3), 10. Power button

1. Ports on dock, 2. Power cord connector, 3. Battery charge indicator, 4. Ports on dock

A license key is required to activate the software. See "Software licensing" on page 251.

Basic operating steps

- 1 Connect a transducer. See "Connecting transducers" on page 13.
- 2 Turn on the system. See "Turning on the system" on the facing page.
- 3 Select the transducer and exam type (or use the default selections). See "Selecting a transducer and exam type" on page 16.
- 4 (Optional) Enter patient information. See "Entering patient information" on page 150.
- 5 Scan. See "Scanning" on page 88.

Accessories and peripherals

The system supports various accessories and peripherals. See "Compatible accessories and peripherals" on page 281.

Preparing the system

Turning on the system

- WARNING** Verify that the hospital supply voltage corresponds to the power supply voltage range.
- WARNING** Plug the system only into a grounded hospital-grade outlet.
- WARNING** Use only power cords provided by FUJIFILM SonoSite with the system.
- Caution** Do not use the system if an error message appears on the clinical monitor. Note the error code and turn off the system. Call FUJIFILM SonoSite or your local representative.
- Caution** When using AC power, position the system to allow easy access to disconnect it.

To turn on the system

The system can be powered by the internal battery or by AC power.

- 1 If you are operating the system using AC power, connect the AC power cord to the stand, and then connect the AC power cord to a hospital-grade outlet.
- 2 Press the power button.
The power button turns green when the system is ready for scanning.

If the system does not maintain expected battery charge, or if the battery icon on the clinical monitor does not display the battery charge status, disconnect and reconnect the system to AC power.

Connect the system to AC power to maintain battery charge, especially if the system will not be used for several days.

To connect the system to AC power (battery charge maintenance)

- 1 Turn off the system.
- 2 Disconnect the system from AC power.
- 3 Check the battery switches; ensure that all three switches are depressed to the  symbol, which is the ON position.
 - Note** The system will not charge and maintain the batteries if the battery switches are depressed to the  symbol, which is the OFF position.
- 4 Reconnect the system to AC power.
The battery charge indicator at the base of the stand blinks green, and the battery icon on the clinical monitor displays the battery charging state.

To turn off the system

- Note** If the system appears unresponsive, wait several minutes before restarting it. Restarting the system while it is performing data-intensive background activities, such as transferring patient files, can result in loss of patient data. To power down an unresponsive system, press and hold the power button until the system shuts down. This procedure may take 5 seconds or longer.
- Press the power button.

Adjusting the height and angle

WARNING Lock the wheels whenever the system is unattended or stationary.

WARNING To avoid possible injury from an unexpected clinical monitor collapse during system transport, collapse the clinical monitor before system transport (see "To collapse the clinical monitor" below).

To raise or lower the platform

- ▶ While pressing down the height-adjustment pedal, grasp both sides of the platform and push down or pull up to the desired height.

To lock a wheel

- ▶ Press down the lever on the wheel.

To unlock the wheel, press up on the bottom of the lever.

To adjust the clinical monitor angle

- ▶ Grasping the clinical monitor on both sides, tilt or rotate it.

To adjust the touch-panel angle

- ▶ Grasping the sides of the touch panel, pull it forward or push it backward to the desired angle.

To collapse the clinical monitor

Always collapse the clinical monitor before system transport.

1. Adjust the touch panel angle to the lowest position.
2. Grasping the clinical monitor on both sides, align it squarely above the touch panel.
3. Fold the clinical monitor downward over the touch panel.

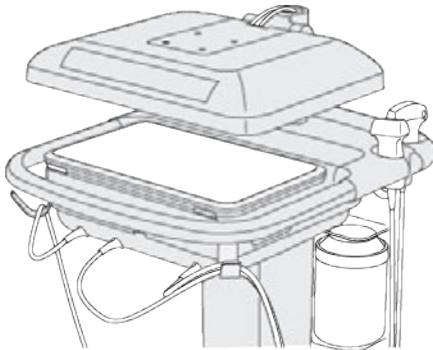


Figure 2-3 Clinical monitor collapsed for system transport

USB devices

You can use the USB ports on the system for connecting devices such as a USB printer or a USB memory stick. (For a list of supported devices, see "Compatible accessories and peripherals" on page 281.

One of the USB ports at the back of the system is for DVR recording only. See "Ports" on page 21 and "DVR recording" on page 173.

USB memory sticks

You can use a USB memory stick to export patient exams, import and export logs and setup configurations, and to import custom obstetric calculation tables.

Note The system does not support password-protected USB memory sticks. Make sure that the USB memory stick you use does not have password protection enabled.

Caution To avoid damaging the USB memory stick and losing patient data from it, observe the following:

- ▶ Do not remove the USB memory stick or turn off the ultrasound system while the system is exporting.
- ▶ Do not bump or otherwise apply pressure to the USB memory stick while it is in a USB port on the ultrasound system. The connector could break.

Caution If the USB icon  does not appear in the system status area on the clinical monitor, the USB memory stick may be defective or password-protected. Replace the USB memory stick.

To connect a USB memory stick for importing or exporting

- ▶ Insert the USB memory stick into a USB port (see "About the system" on page 5).

The USB memory stick is ready when the USB icon  appears onscreen.

To view information about the device, see "USB settings" on page 77.

To disconnect a USB memory stick

Disconnecting the USB memory stick while the system is exporting to it may cause the exported files to be corrupted or incomplete.

- 1 If exporting, wait at least five seconds after the USB animation icon  stops.
- 2 Remove the USB memory stick from the port.

General interaction

Clinical monitor

WARNING FUJIFILM SonoSite recommends against using a monitor other than the clinical monitor provided by FUJIFILM SonoSite. Using a different monitor may result in image distortion and degradation of image quality.

The clinical monitor displays the ultrasound image as well as details about the exam and system status.



Figure 2-4 Clinical monitor layout

Legend:

1. Patient header
2. Measurement and calculation area
3. Selected transducer, exam type, and MI and TI values
4. Imaging mode or modes, controls selected
5. System status area
6. Depth scale

- 7. Ultrasound image
- 8. Orientation marker

Touch panel

The touch panel is where you adjust settings; select the exam type, transducer, and imaging mode; enter patient information; and more. As you adjust the image settings or controls, the results appear on the clinical monitor. When an image is frozen, the touch panel displays an outline of the image.

You interact with the touch panel the same as with many other touchscreen devices:

- **Swipe:** Move your finger quickly across the panel. Faster than dragging.
- **Drag:** Move one or two fingers across the panel, usually to move an object from one location to another.
- **Tap:** Quickly touch the panel once; for example, to activate a control.
- **Double-tap:** Quickly touch the panel twice with one or more fingers.
- **Pinch or spread:** Slide two fingers together or apart on the panel.

Use these gestures to perform these actions:

Gesture	Action
Swipe	Steer D-line (linear transducers only)
	Steer color box (linear transducers only)
	Scroll through pages in forms, such as the patient form, worksheets, and thumbnails in Review
	Select previous or next images in full-screen Review
Drag	Adjust depth or gain
	Move color or zoom box
	Move calipers
	Move D-line or M-line
	Move Doppler baseline
	With two or more fingers, drag anywhere on the touch panel to move or resize the active object, such as the Color box or the Doppler gate.
	Move depth marker in biopsy guide
	Change the D-line angle selection
	Move labels, pictograms, and transducer marker
	Move through frames in the cine buffer
	Move controls to the Controls bar
	Pan a frozen zoomed 2D image (panning is disabled if measurements or labels exist on the frozen zoomed image)
	Unfreeze a frozen image by dragging the Slide to Unfreeze slider
Tap	Freeze
	Adjust depth
	Select calipers
	Select image in dual

Gesture	Action
	Select mode in split screen (2D, D-line, or Doppler trace)
Double-tap	With two or more fingers, double-tap to freeze or unfreeze
	Double-tap with one finger in the zoom box to zoom
	Double-tap with one finger on a live zoomed image to unzoom
Pinch or spread	Sample volume size
	Resize color or zoom box

Onscreen keyboard

You can enter text into text boxes (for example, on the patient form) using the onscreen keyboard.

To enter text using the onscreen keyboard

- 1 Tap a text box (for example, in the patient form).
The onscreen keyboard appears.
- 2 Tap keys as needed.



Figure 2-5 Onscreen keyboard

Legend:

1. Alpha keys: Tap and hold the alpha keys to reveal options for selecting accented versions of the characters
2. Shift key: Changes alphabet keys to uppercase and international characters
3. Jumps to previous text box
4. Advances to next text box
5. Spacebar

6. Displays keys for numbers, symbols, and special characters

Note To display international characters on the keyboard, tap , and then tap the Shift key.

7. Closes the keyboard
8. Starts a new line or advances to next text box
9. Deletes character on left side of pointer

Preparing transducers

WARNING Some transducer sheaths contain natural rubber latex and talc, which can cause allergic reactions in some individuals. FUJIFILM SonoSite recommends that you identify your latex and talc-sensitive patients and be prepared to treat allergic reactions promptly.

WARNING Some gels and sterilants can cause an allergic reaction on some individuals.

Caution To avoid damage to the transducer, use only gels recommended by FUJIFILM SonoSite. Using other gels can damage the transducer and void the warranty. If you have questions about gel compatibility, contact FUJIFILM SonoSite or your local representative.

Caution Clean transducers after each use. See "Cleaning and disinfecting " on page 255.


Connecting transducers

This section provides instructions to connect a transducer, with or without the Triple Transducer Connect (TTC), as well as instructions to remove a transducer.


To connect a transducer to the TTC

The TTC is a three-transducer module that lets you simultaneously connect up to three transducers to the ultrasound system. The TTC is standard with the system.


- 1 Pull the transducer latch handle up, and rotate it clockwise.
- 2 Insert the transducer connector into one of the three TTC connector ports on the bottom of the TTC.

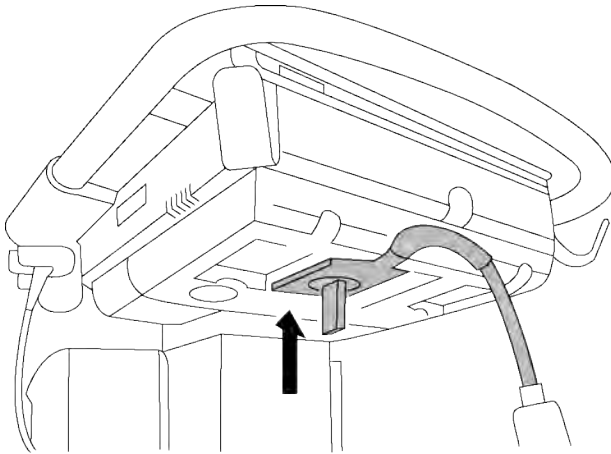

- 3 Make sure that the connector is firmly seated, and then turn the latch handle counterclockwise.

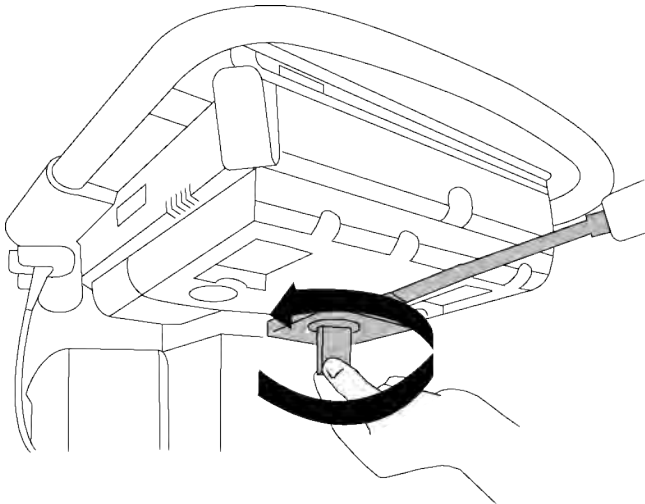


- 4 Press the latch handle up, securing the transducer connector to the TTC.

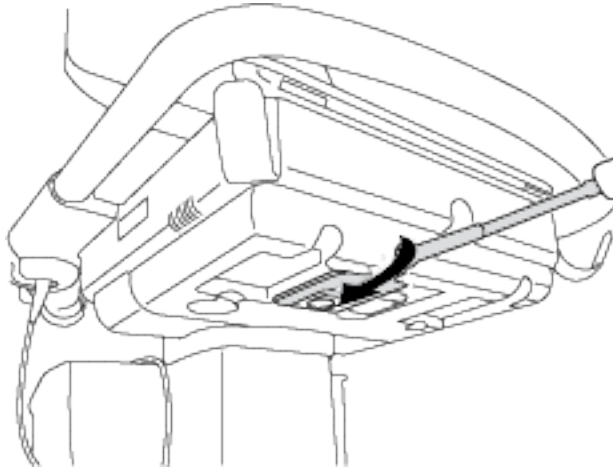


To connect a transducer to the system (without a TTC)

If the TTC is not installed on your system, you can connect one transducer directly to the system.


- 1 Pull the transducer latch up, and rotate it clockwise.
- 2 Insert the transducer connector into the transducer port underneath the stand platform.

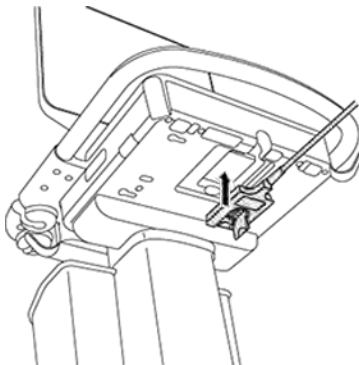

- 3 Make sure that the connector is firmly seated, and then turn the latch handle counterclockwise.

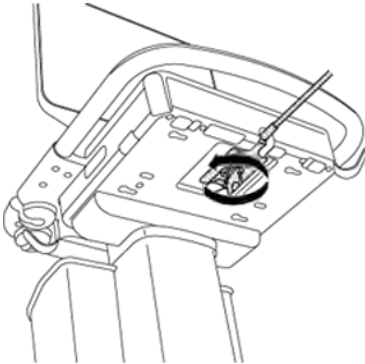

- 4 Press the latch up, securing the transducer connector to the system.

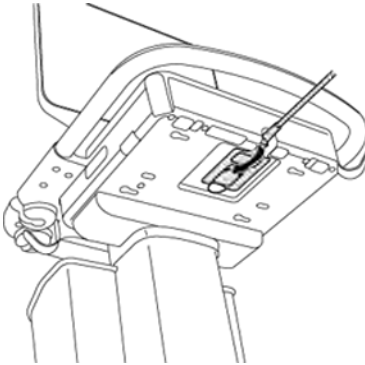

To remove a transducer

Caution To avoid equipment damage that could lead to image quality degradation, do not disconnect a transducer while it is in use. Either freeze the image or switch to another transducer before disconnecting.

- 1 Pull the transducer latch up, and rotate it clockwise.
- 2 Pull the transducer connector away from the system.

Selecting a transducer and exam type

WARNING To prevent misdiagnosis or harm to the patient, use the correct transducer for the application. The diagnostic capability differs for each transducer, exam type, and imaging mode. Transducers are developed to specific criteria depending on their physical application. These criteria include biocompatibility requirements. Understand the system's capabilities prior to use.

Before scanning, select a transducer and exam type.

To select a transducer and exam type

- 1 Do one of the following:
 - ▶ On the start select screen, tap **SELECT** if present. (To set up the start select screen, see "User profile settings" on page 71.)
 - ▶ Tap **TRANSDUCERS & EXAMS** at the top of the touch panel.



Cards for the available transducers appear (see "Connecting transducers" on page 13).

- 2 On the card for the appropriate transducer, do one of the following:
 - ▶ Double-tap the exam type.
 - ▶ Tap the exam type, and then tap **SCAN**, or tap **Cancel** to cancel.

Scrolling the list of exam types displays any hidden items.

You can also select the exam type on the patient form. See "Entering patient information" on page 150.

Table 2-1 *Imaging modes and exam types on transducers*

Transducer	Exam type	Imaging mode				
		2D M Mode	CPD	Color	PW Doppler*	CW Doppler
C60xp	Abdomen	✓	✓	✓	✓	—
	Gynecology	✓	✓	✓	✓	—
	Musculoskeletal	✓	✓	✓	✓	—
	Nerve	✓	✓	✓	✓	—
	Obstetrics	✓	✓	✓	✓	—

Table 2-1 Imaging modes and exam types on transducers (continued)

Transducer	Exam type	Imaging mode				
		2D M Mode	CPD	Color	PW Doppler*	CW Doppler
HFL38xp	Arterial	✓	✓	✓	✓	—
	Breast	✓	✓	✓	✓	—
	Musculoskeletal	✓	✓	✓	✓	—
	Nerve	✓	✓	✓	✓	—
	Small Parts	✓	✓	✓	✓	—
	Venous	✓	✓	✓	✓	—
HFL50xp	Breast	✓	✓	✓	✓	—
	Musculoskeletal	✓	✓	✓	✓	—
	Nerve	✓	✓	✓	✓	—
	Small Parts	✓	✓	✓	✓	—

Table 2-1 Imaging modes and exam types on transducers (continued)

Transducer	Exam type	Imaging mode				
		2D M Mode	CPD	Color	PW Doppler*	CW Doppler
HSL25xp	Arterial	✓	✓	✓	✓	—
	Breast	✓	✓	✓	✓	—
	Musculoskeletal	✓	✓	✓	✓	—
	Nerve	✓	✓	✓	✓	—
	Small Parts	✓	✓	✓	✓	—
	Venous	✓	✓	✓	✓	—
ICTxp	Gynecology	✓	✓	✓	✓	—
	Obstetrics	✓	✓	✓	✓	—
L25xp	Musculoskeletal	✓	✓	✓	✓	—
	Nerve	✓	✓	✓	✓	—
	Superficial	✓	✓	✓	✓	—
	Arterial	✓	✓	✓	✓	—
	Venous	✓	✓	✓	✓	—
	Ophthalmic	✓	✓	✓	✓	—
	Lung	✓	✓	✓	✓	—

Table 2-1 Imaging modes and exam types on transducers (continued)

Transducer	Exam type	Imaging mode				
		2D M Mode	CPD	Color	PW Doppler*	CW Doppler
L38xp	Nerve	✓	✓	✓	✓	—
	Small Parts	✓	✓	✓	✓	—
	Arterial	✓	✓	✓	✓	—
	Venous	✓	✓	✓	✓	—
	Lung	✓	✓	✓	✓	—
P21xp	Abdomen	✓	✓	✓	✓	—
	Cardiac	✓	—**	✓	✓	✓
	Lung	✓	✓	✓	✓	—
	Obstetrics	✓	✓	✓	✓	—

* For the cardiac exam type, PW TDI is also available.

**The system uses Variance Color Doppler instead of CPD for the cardiac exam type on the P21xp transducer.

Related topics

Intended uses25

Gel

Use acoustic coupling gel on the transducer during exams. Although most gels provide suitable acoustic coupling, some gels are incompatible with some transducer materials. FUJIFILM SonoSite recommends Aquasonic gel and provides a sample with the system.

For general use, apply a liberal amount of gel between the transducer and the body. For interventional use, apply a transducer sheath.

Sheaths

WARNING Use market-cleared, sterile transducer sheaths and sterile coupling gel for transrectal, transvaginal, or guided-needle procedures. Do not apply the transducer sheath and coupling gel until you are ready to perform the procedure. After use, remove and discard the single-use sheath, and clean and disinfect the transducer using a FUJIFILM SonoSite-recommended high-level disinfectant.

To apply a transducer sheath

- 1 Place gel inside the sheath. Make sure that the gel is at the end of the sheath.
- 2 Insert the transducer into the sheath.
- 3 Pull the sheath over the transducer and cable until the sheath is fully extended.
- 4 Secure the sheath using the bands supplied with the sheath.
- 5 Check for and eliminate air bubbles between the face of the transducer and the sheath.
Air bubbles between the face of the transducer and the sheath may affect the ultrasound image.
- 6 Inspect the sheath to ensure that there are no holes or tears.

Ports

The system provides ports for USB, digital video-in, audio-in, DVR, E-SATA (unsupported), and Ethernet.

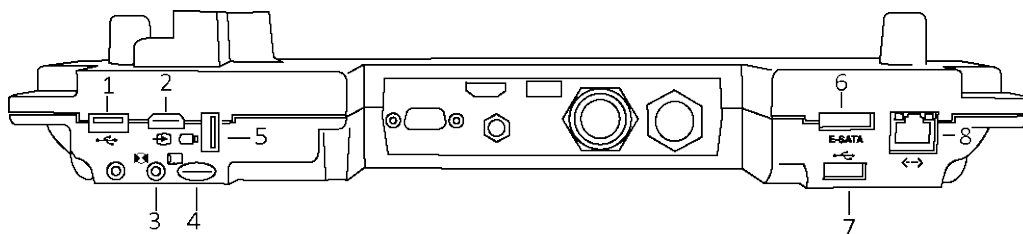


Figure 2-6 Ports on the dock

1. USB 2. Digital video-in 3. Audio in 4. MicroSD - DVR software upgrade 5. USB storage - DVR 6. E-SATA (unsupported) 7. USB 8. Ethernet

Battery charge indicator

WARNING Connect the system to AC power when the battery charge is low.
Periodically check to make sure that the battery charges fully. If the battery fails to charge fully,

contact FUJIFILM SonoSite Technical Support (see "FUJIFILM SonoSite Technical Support" on page 3).

Caution Use only FUJIFILM SonoSite batteries with the system.

The battery charge indicator on the base of the stand blinks while the battery is charging. The battery charging indicator is steady when the battery is fully charged. Icons in the connectivity status area on the clinical monitor show the power and battery status:



The battery is partially charged. The length of the blue bar indicates the level of battery charge. If Beep Alert is on (see "Audio settings" on page 34), the system beeps when the battery charge is below 14%.



The system is on AC power only, either because the batteries are switched off or because no batteries are installed.



The system is on AC power, and the battery is charged. The length of the blue bar indicates the level of the battery charge.

Transporting the system

If you need to transport the system, make sure to observe the following:

- ▶ Collapse the clinical monitor. See "To collapse the clinical monitor" on page 8.
- ▶ To wheel the system, push forward on the bar on the platform instead of pushing downward on the bar or pushing the clinical monitor.

Caution Make sure to turn the battery switches off before transporting the system for shipping: Open the battery compartment door located at the base of the stand and depress all three battery switches to the **•O** symbol, which is the OFF position.

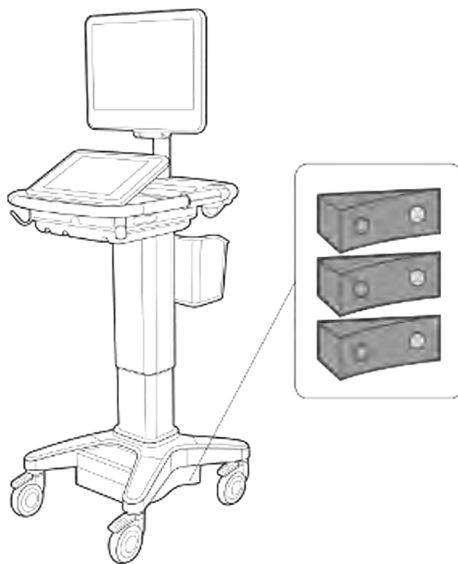


Figure 2-7 *Battery switches in off positions*

Visual Guide videos

The Visual Guide library contains animated 3D videos that describe scanning techniques, terminology, and more. Scanning is active while a video plays, so you can practice scanning techniques in conjunction with the visual guides.

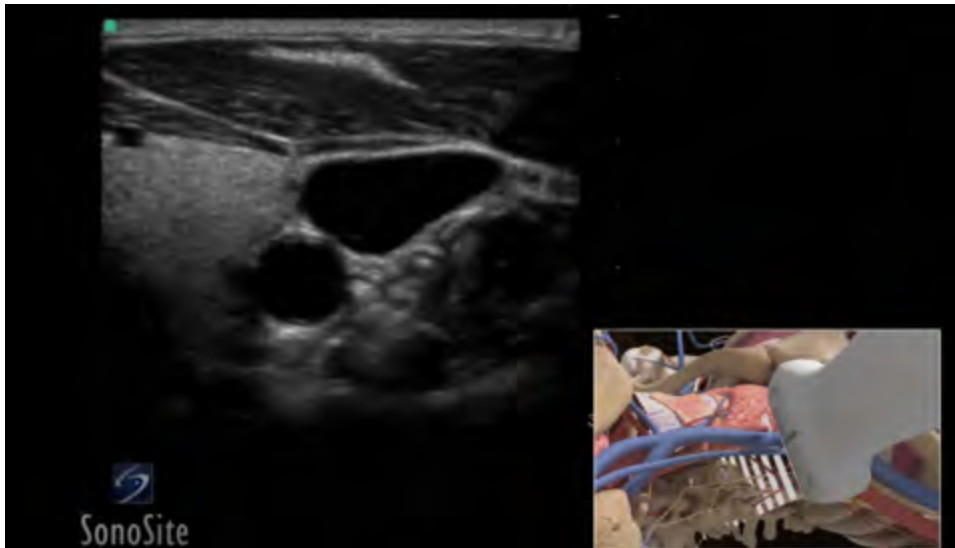


Figure 2-8 A Visual Guide video

Note Some Visual Guide videos may display incorrect representations of the X-Porte user interface due to using different versions of the software.

To play a Visual Guide video

- 1 Display the Visual Guide screen by doing one of the following:
 - Tap **LEARN** on the startup screen.
 - In the upper right corner of the touch panel, tap **LEARN**, or tap **MORE** and then tap **LEARN**, depending on your setup.

To specify the location of this command, see "Presets settings" on page 66.

- 2 At the top of the Visual Guide screen, tap a category (for example, **Basics**).
The current selection is highlighted.
- 3 Tap a video selection in one of the cards that appear.
- 4 Tap the left and right controls to show additional cards.



The video player starts. Tap to play your selection.

5 Do any of the following:

- ▶ Restart or pause the video using the controls  at the bottom of the player.

- ▶ Tap the volume control  to mute the audio. (Tap again to unmute.)
- ▶ Drag the volume slider to adjust the volume.
- ▶ Tap **Hide** to hide the player. (You can tap **LEARN** to show the player again.)
- ▶ Tap another video in the list to play it.
- ▶ Tap **LEARN** to return to the Visual Guide screen.

Intended uses

The system is used with a transducer attached and is powered either by battery or by AC electrical power. The clinician is positioned beside the patient and places the transducer onto the patient's body where needed to obtain the desired ultrasound image.

The system transmits ultrasound energy into the patient's body to obtain ultrasound images as listed below.

For the intended transducer and imaging modes for each exam type, see "Selecting a transducer and exam type" on page 16.

Abdominal imaging applications You can assess the liver, kidneys, pancreas, spleen, gallbladder, bile ducts, transplanted organs, abdominal vessels, and surrounding anatomical structures for the presence or absence of pathology transabdominally. You can evaluate and perform interventional abdominal procedures and evaluate blood flow in abdominal organs.

Arterial and venous imaging applications You can assess the carotid arteries, deep veins, and arteries in the arms and legs, superficial veins in the arms and legs, great vessels in the abdomen, and various small vessels feeding organs for the presence or absence of pathology.

Cardiac imaging applications You can assess the cardiac valves, the great vessels, heart size, cardiac function, hemodynamic assessment, and surrounding anatomical structures for the presence or absence of pathology.

You can use the ECG option to display the patient's heart rate. (ECG is available only on the cardiac exam type.)

WARNING To prevent misdiagnosis, do not use the ECG trace to diagnose cardiac rhythms. The FUJIFILM SonoSite ECG option is a non-diagnostic feature.

Gynecology and infertility imaging applications You can assess the uterus, ovaries, adnexa, and surrounding anatomical structures for the presence or absence of pathology transabdominally or transvaginally. You can assess blood flow in pelvic organs transabdominally and transvaginally.

Interventional department imaging applications You can use the system for ultrasound guidance in biopsy and drainage procedures, vascular line placement, peripheral nerve blocks, amniocentesis and other obstetrical procedures.

Obstetrical imaging applications You can assess the fetal anatomy, viability, estimated fetal weight, fetal heart rate, fetal position, gestational age, amniotic fluid, and surrounding anatomical structures for the presence or absence of pathology transabdominally or transvaginally.

You can evaluate the blood flow of the fetus, placenta, umbilical cord, and surrounding maternal structures.

Color Power Doppler (CPD) and Color Velocity Doppler (CVD) imaging tools are intended to evaluate the blood flow of the fetus, placenta, umbilical cord, and surrounding maternal structures in all cases, including high-risk pregnancies. High-risk pregnancy indications include, but are not limited to, multiple pregnancies, fetal hydrops, placental abnormalities, maternal hypertension, diabetes, and lupus. CPD and Color imaging tools are not intended as a sole means of diagnosis nor as a sole method of high-risk pregnancy screening.

WARNING CPD or Color images can be used as an adjunctive method, not as a screening tool, for the detection of structural anomalies of the fetal heart and as an adjunctive method, not as a screening tool, for the diagnosis of Intrauterine Growth Retardation (IUGR).

WARNING To prevent injury or misdiagnosis, do not use this system for Percutaneous Umbilical Blood Sampling (PUBS) or *in vitro* Fertilization (IVF). The system has not been validated to be proven effective for these two uses.

Small parts, breast, musculoskeletal, nerve, superficial, and lung imaging applications You can assess the breast, thyroid, testicle, lymph nodes, hernias, musculoskeletal structures, soft tissue structures, ophthalmic structures, and surrounding anatomical structures for the presence or absence of pathology. You can detect normal lung motion for the presence or absence of pathology. You can demonstrate blood flow in superficial anatomical structures.

You can use the system for ultrasound guidance in biopsy and drainage procedures, vascular line placement, and peripheral nerve blocks.

WARNING To avoid injury to the patient, use only an Ophthalmic exam type when performing imaging through the eye. The FDA has established lower acoustic energy limits for ophthalmic use. The system will not exceed these limits only if the Ophthalmic exam type is selected.

Contraindications

The X-Porte ultrasound system has no known contraindications.

CHAPTER 3

Setting up the system

System Settings is where you customize the system and set preferences.

Note Not all system settings are available if you log in to the system as a guest (see "Logging in" on page 34).


Note Reset functions are provided on some system settings pages. For example, you can reset the More Controls window for your user profile to the factory default. For more information about user profile settings, see "User profile settings" on page 71.

To display a settings page

- ▶ Tap **MORE**, and then tap **System Settings**.



Tapping a settings page in the list displays its settings on the right side. The active selection is highlighted.

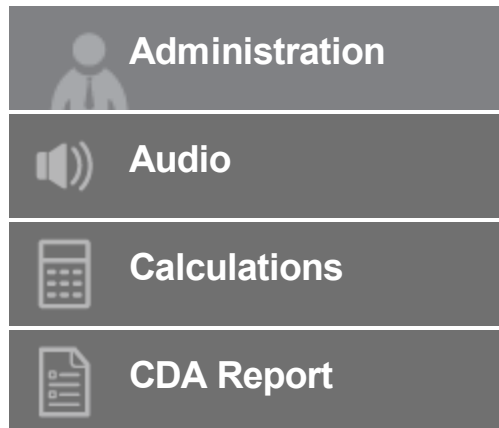

	Connectivity
	Date and Time
	Display Information
	Network Status
	Power and Battery
	Presets
	System Information
	USB
	Logs

Administration settings

On the Administration settings page, you can configure the system to require users to log in and enter passwords. Required login helps protect patient information. You can also add and delete users and change passwords.

To display the Administration Login settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Administration** in the list on the left.



About security settings

WARNING Healthcare providers who maintain or transmit health information are required by the Health Insurance Portability and Accountability Act (HIPAA) of 1996 and the European Union Data Protection Directive (95/46/EC) to implement appropriate procedures: to ensure the integrity and confidentiality of information and to protect against any reasonably anticipated threats or hazards to the security or integrity of the information or unauthorized uses or disclosures of the information.

The Administration settings page includes security settings that help you to meet the applicable security requirements listed in the HIPAA standard. Users are ultimately responsible for ensuring the security and protection of all electronic protected health information collected, stored, reviewed, and transmitted on the system.


Managing the Administrator account

To log in as Administrator

The default Administrator login name is **Administrator**. If you don't have the administrator password, contact FUJIFILM SonoSite. (See "Getting help" on page 3.)

Any user with administrative privileges can log in as an administrator.

- 1 On the Administration Login settings page, type the login name of a user with administrative privileges in the **Name** text box.
- 2 Type the password for the login name in the **Password** text box.
- 3 Tap **Login**.

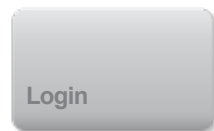


To log out as Administrator

Do one of the following:

- Tap any other settings page.
- Turn off or restart the system.

To change the administrator password

- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Under **User List**, tap **Administrator**.
- 3 Type the new password in the **Password** text box and in the **Confirm** text box.
To ensure security, choose a password that contains uppercase characters (A-Z), lowercase characters (a-z), and numbers (0-9). Passwords are case-sensitive.
- 4 Tap **Save**.

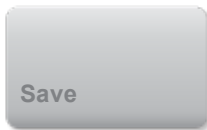

Protecting patient information

To require user login

You can set the system to require a user name and password at startup. If you require user login, you also need to set up user accounts. (See "Adding and managing user accounts" on the facing page.)

- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Select **On** from the **User Login** list.
- 3 Tap **Done**.




To hide patient information

You can choose not to display the patient name and ID number on the clinical monitor and to remove it from exported images and video clips.

- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Tap **Administrator** in the **User List**.


- 3 Select **Disable Patient Header**.
- 4 Tap **Save**.

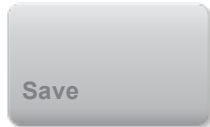

To disable export and recording of patient data

You can prevent users from exporting patient data to USB memory sticks and from recording video displayed on the clinical monitor to a USB memory stick inserted in the DVR (see "DVR recording" on page 173).

- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Tap **Administrator** in the **User List**.
- 3 Select **Disable USB Export**.
- 4 Tap **Save**.

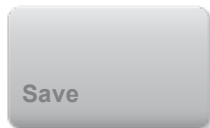


Adding and managing user accounts

You can add up to 20 user accounts (including the default Administrator account).

To add a new user

Note | Once you add a new user, you cannot edit the user's name.


- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Tap **New**.

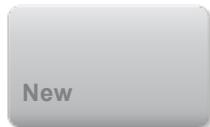

- 3 Under **User Information**, fill in the **Name**, **Password**, and **Confirm** text boxes.
To ensure security, choose a password that contains uppercase characters (A-Z), lowercase characters (a-z), and numbers (0-9). Passwords are case-sensitive.
- 4 (Optional) In the **User** box, type the user's name. This information appears by default in the **User** field in the patient form and on the clinical monitor.

5 (Optional) Select **Administration Access** to allow access to all administration privileges.

6 Tap **Save**.

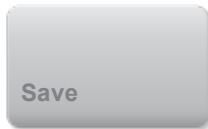

7 Tap **Done**.




To modify a user

1 On the Administration Login settings page, log in as a user with administrative privileges.

2 Under **User List**, select the user.

3 Make changes to the **Name**, **User**, or **Administration Access** entries.

4 Tap **Save**.

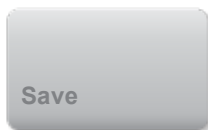

5 Tap **Done**.




To change a user password

1 On the Administration Login settings page, log in as a user with administrative privileges.

2 Under **User List**, select the user.

3 Type the new password in the **Password** and **Confirm** text boxes.

4 Tap **Save**.

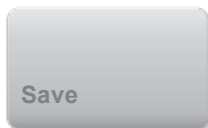


- 5 Tap **Done**.



To allow users to change their password

- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Under **User List**, tap **Administrator**.
- 3 Select **Password Changes**.
- 4 Tap **Save**.

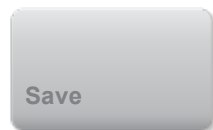

- 5 Tap **Done**.



To delete a user

- 1 On the Administration Login settings page, log in as a user with administrative privileges.
- 2 Select the user under **User List**.
- 3 Tap .
- 4 Tap **Yes**.
- 5 Tap **Done**.



Logging in

If user login is required, the user login screen appears when you turn on the system. (See "To require user login" on page 30.)

To log in as a user

- 1 Turn on the system.
- 2 In the login screen, type your user name and password, and then tap **OK**.

To log in as guest

Guests can scan, save images and clips, view worksheets, and access Help and Visual Guide videos. Guests cannot access patient information or system settings (except **Monitor Brightness** and **Touch Panel Brightness**).

When you log in as a guest, the system always displays the 2D imaging screen, no matter which startup mode is specified in User Profile settings (see "User profile settings" on page 71).

- 1 Turn on the system.
- 2 In the login screen, tap **Guest**.

To change your password

You can change your password only if the administrator has enabled password changes. See "To allow users to change their password" on the previous page.

- 1 Turn on the system.
- 2 In the login screen, tap **Change Password**.
- 3 Type your old and new passwords, confirm the new password, and then tap **Change**.

Audio settings

On the Audio settings page, you can specify sound settings and adjust the volume for beeps and clicks.

To display the Audio settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Audio** in the list on the left.



To specify sound settings

- ▶ On the Audio settings page, do either or both of the following:
 - ▶ Select **Button Clicks** for keys to click when tapped.
 - ▶ Select **Beep Alert** for the system to play sounds.

To adjust the volume for beeps and clicks

- ▶ On the Audio settings page, drag the **Beeps and Clicks Volume** slider. To mute the volume for



Related topics

Saving images and video clips	169
Setting up the system	27

Calculations settings

On the Calculations settings page, you can specify measurement names that appear in the Tissue Doppler Imaging (TDI) calculations list and in the reports for cardiac exams.

You can select authors for obstetrics calculation tables and import custom obstetrics calculation tables.

You can also set up obstetrics custom measurements that appear in the obstetrics calculations lists and in the report for obstetrics exams.

To display the Calculations settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Calculations** in the list on the left.



Cardiac calculations settings

To specify cardiac measurement names

- 1 On the Calculations settings page, tap **Cardiac**.



- 2 Under **TDI Walls**, select a name for each wall.
- 3 Tap **Done**.




Obstetrics calculations settings

Specifying gestational age and growth analysis

To specify gestational age and growth analysis

- 1 On the Calculations settings page, tap **OB**.
- 2 Select the desired authors (or select **None**) in the measurement lists under **Gestational Age** and **Growth Analysis**.
Selecting a gestational age author places the associated measurement in the calculations list.
- 3 To reset the authors for the **Gestational Age** and **Growth Analysis** measurement lists to the factory default, tap **Reset**.

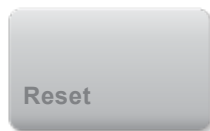

Creating custom obstetrics measurements

You can create up to five custom measurements that appear in the obstetrics calculations lists, in the list of custom measurements in Calculations settings, and in the obstetric report.

You can create custom measurements on the system or in the X-Porte OB Custom Table Editor, a Microsoft Excel file available at www.sonosite.com. You can associate table data with a custom measurement only if you create the measurement in the OB Custom Table Editor.


WARNING When you create, remove, or import a custom obstetric measurement, the system clears all saved measurements and calculations for the current patient.

To create custom obstetrics measurements on the system

- 1 End the current exam.
- 2 On the Calculations settings page, tap **OB**.



- 3 In the **OB Custom Measurements** area, tap  **Add New**.
- 4 In the **Name** box, type a unique name.
- 5 In the **Type** list, select the desired measurement type.
- 6 Tap **Save**.

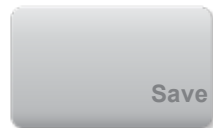

To create custom obstetrics measurements in the OB Custom Table Editor

Before you use the OB Custom Table Editor, make sure that your system meets the minimum requirements for the editor and specify the language for the editor (see "Setting up custom obstetric tables" on the next page).

Note | The language of the OB Custom Table Editor must match the language of the X-Porte system on which you import the measurements.


- 1 On the **Setup-First** tab of the OB Custom Table Editor, click the Age-User Table to which you want to add a measurement.
- 2 In the **Measurement Name** cell, type a unique name of up to six characters.
- 3 In the **Measurement Type** cell, select the desired measurement type.
- 4 Enter table data.

Note | If you create a custom measurement with a name that matches the name of a custom measurement already on the system, the system overwrites the custom measurement on the system with the new custom measurement when you import it.

Once you've created custom measurements in the custom table editor, you can export the editor to a USB memory stick and then import the custom measurements from the editor onto the system (see "Exporting and importing custom obstetric tables and measurements" on page 39).

To delete a custom obstetric measurement

- 1 End the current exam.
- 2 On the Calculations setup page, tap **OB**.

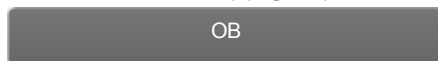

- 3 In the **OB Custom Measurements** area, tap  next to the measurement you want to delete.
- 4 Tap **Yes** to confirm the deletion.
The measurement and its associated table is removed from the system.


Viewing obstetric calculation tables

You can view obstetric calculation tables on the system, including tables you've imported using the OB Custom Table Editor.

You cannot modify or delete default (system-defined) obstetric calculation tables.

To view obstetric calculations tables

- 1 On the Calculations setup page, tap **Tables**.

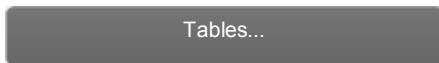

- 2 Select the desired table type and measurement/author.

Setting up custom obstetric tables

You can use the X-Porte OB Custom Table Editor, a Microsoft Excel file available at www.sonosite.com, to set up custom obstetric gestational age and growth analysis tables.

After you enter custom tables in the editor, you export the tables to a USB memory stick and then import the tables onto the X-Porte system. (See "Exporting and importing custom obstetric tables and measurements" on the facing page.)

Before you use the X-Porte OB Custom Table Editor, make sure that your computer meets the following minimum requirements:

Windows

- Windows Vista, 32-bit or 64-bit, or later
- Microsoft Excel 2007 or later, configured to enable all macros and to trust access to the VBA project object model (see Excel Help for instructions).

Mac OS

- Mac OS X 10.5 (Leopard) or later
- Microsoft Excel 2011 or later

To specify the language of the OB Custom Table Editor

- On the **Setup-First** tab in the editor, select a language from the **Please Specify Language** list.
The language selected must match the language of the X-Porte system on which you import the tables.

To set up custom obstetric tables in the OB Custom Table Editor

You can set up custom tables of up to 120 entries (age tables) or 210 entries (growth tables).

You must enable all macros in Excel when you use the OB Custom Table Editor in Windows.

When you are finished using the OB Custom Table Editor, you can revert the macro settings in Excel if desired.
For instructions, see Excel Help.

Entries in the table editor cannot be negative and cannot contain slashes, asterisks, brackets, commas, or semi-colons.

WARNING Prior to use, verify that custom table data entries are correct. The system does not confirm the accuracy of the custom table data that you enter.

- 1 Open the editor in Excel.
- 2 (Mac OS only) Click **Enable Macros**.
- 3 On the **Setup-First** tab, click the Growth or Age table that you want to edit.
- 4 Enter a unique name with a maximum of 24 characters in the **Author** field. You cannot use author names that occur in system-defined OB calculation tables.
- 5 Do one of the following:
 - (Growth tables) Enter values in the **Weeks**, **Min**, **Mid**, and **Max** fields.
Values in the **Weeks** column must increase.
 - (Age tables) Enter values in the **Value**, **Age (weeks)**, **Age (days)**, **Range (weeks)**, and **Range (days)** fields.
Values in the **Value** column and the totals of the values in the **Age (weeks)** and **Age (days)** columns must increase.
- 6 A graphical representation of the values you enter appears to the right of the cells in each tab.
- 7 Save the data.

Exporting and importing custom obstetric tables and measurements

To export custom obstetric tables and measurements from the OB Custom Table Editor

When you export custom tables from the editor, all content from all tabs, including custom measurements, is included in the exported CSV file.

- 1 Insert a USB memory stick into your computer.
- 2 Click **Export** on any tab in the editor.

- 3 In the **Please Select the Export File Folder** window, navigate to the USB memory stick and click **OK**. You cannot change the file name of the exported file.

Note Do not modify the exported CSV file. The system will not import data from the CSV file if the file has been modified.

To import custom obstetric tables and measurements


On import, the system updates the custom tables and custom measurements.

- WARNING** When you import custom obstetric tables and measurements, the system replaces existing tables with tables from the OB Custom Table Editor.
- WARNING** When you create, remove, or import a custom obstetric measurement, the system clears all saved measurements and calculations for the current patient.
- WARNING** Prior to use, verify that custom table data entries are correct. The system does not confirm the accuracy of the custom table data entered by the user.

- 1 End the current exam.
- 2 Insert the USB memory stick that contains the exported CSV file into a USB port on the side of the system.
- 3 On the Calculations settings page, tap **Tables**.



- 4 Tap **Import**.

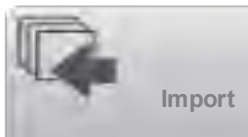

- 5 Select the USB memory stick that contains the CSV file, and then tap **Import**.



The system restarts.

Deleting custom obstetric tables

You cannot delete system-defined obstetric tables.

- 1 On the Calculations settings page, tap **Tables**.
- 2 Select **Gestational Age** or **Growth Analysis**, and then select the **Measurement** and **Author** for the table you want to delete.
- 3 Tap **Delete**.

CDA Report settings

When you select **Include CDA** on the DICOM **Archivers** configuration page, the ultrasound system archives the patient report in DICOM Encapsulated CDA (Clinical Document Architecture) format using the profile you specify on the CDA Report settings page.

Some CDA report settings appear in the patient report (see "Displaying reports and worksheets after the exam has ended" on page 225). For example, the **Custodian/Provider Organization Name** appears in the report's banner.

You can configure up to four CDA profiles on the system.

To display the CDA Report settings page

- 1 Tap **MORE**, and then tap **System Settings**.




- 2 Tap **CDA Report** in the list on the left.



To configure a CDA profile

Required fields are marked with asterisks.


- 1 On the CDA Report settings page, tap **Config**.

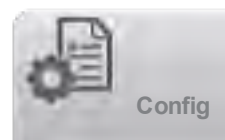

- 2 On the CDA Report page, tap **New**.



- 3 To change the profile name, type a new profile name in the **System/Author ID** box. Each profile name must be unique.
- 4 Complete the fields in the **Custodian/Provider Organization** and **Author/Authenticator Organization** sections.
- 5 To enter information about the authenticator, select **Include?** in the **Authenticator** section and complete the fields in the **Authenticator** section.
- 6 Tap **Save**.

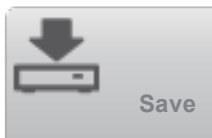


To specify a CDA profile


- On the CDA Report settings page, select a profile from the **Active Configuration** list.

To delete a CDA profile

- 1 On the CDA Report settings page, tap **Config**.

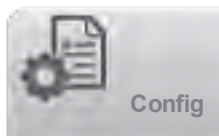

- 2 Select the CDA profile you want to delete in the **CDA List**, and tap **Delete**.

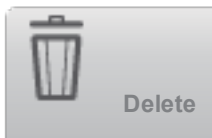

Related topics

DICOM	46
Setting up the system	27

Connectivity settings

On the Connectivity settings page, you specify the system location, and specify options for alerts when internal storage is full. You can also import and export connectivity settings.

You can also manage wireless certificates and specify settings for DICOM, which is an optional feature. See "DICOM" on page 46.

To display the Connectivity settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Connectivity** in the list on the left.



To specify the system location

The location you specify in Connectivity settings represents the active location of the system.

- On the Connectivity settings page, select a location from the **Location** list.

To receive storage alerts

- On the Connectivity settings page, select **Internal Storage Capacity Alert**.
The system displays a message if internal storage is near capacity when you end an exam.

Related topics

Entering patient information.....	150
Network Status settings.....	65
DICOM.....	46
Setting up the system.....	27

Importing and exporting connectivity settings

You can import and export all location and connectivity settings from and to another X-Porte system. These settings include DICOM configuration data for locations, wireless settings, archivers, printers, storage commit servers, worklist servers, and MPPS servers.


The system does not import IP addresses or AE titles when you import configuration data from another system.

Caution


To avoid losing data from or damaging the USB memory stick, do not remove the USB memory stick or turn off the ultrasound system while exporting. In addition, do not bump or apply pressure to the USB memory stick while it is connected to the system. The connector could break.

To import connectivity settings


- 1 Connect the USB memory stick that contains the settings (see "To connect a USB memory stick for importing or exporting" on page 9).
- 2 On the Connectivity settings page, tap **DICOM Setup**.

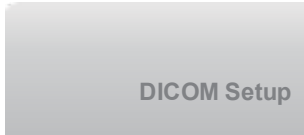

- 3 Tap **Config**.

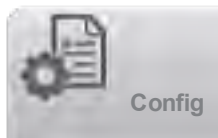

- 4 Tap **Import**.

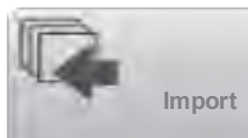


- 5 Select the USB memory stick, and then tap **Import**.




The system restarts. All configurations on the system are replaced with imported data.

To export connectivity settings


- 1 Connect a USB memory stick (see "To connect a USB memory stick for importing or exporting" on page 9).
- 2 On the Connectivity settings page, tap **DICOM Setup**.

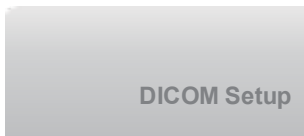

- 3 Tap **Config**.

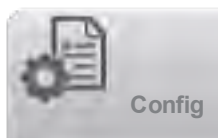

- 4 Tap **Export**.

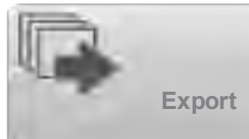

- 5 Select the USB memory stick, and then tap **Export**.



All configurations on the USB memory stick are replaced with exported data. Five seconds after the exporting completes, you can safely remove the USB memory stick and use it to import the data onto another X-Porte system.

Date and Time settings


WARNING To obtain accurate obstetrics calculations, an accurate date and time are critical. Verify that the date and time are accurate before each use of the system.

To display the **Date and Time settings page**

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Date and Time** in the list on the left.

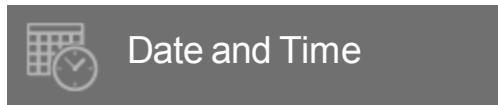

To set the date and time

- ▶ On the Date and Time settings page, do the following:
 - ▶ In the **Date** boxes, type the current year, month, and day.
 - ▶ In the **Time** boxes, type the current time in hours and minutes.
 - ▶ To obtain the system time from a time server, select **Use Time on Time Server** and enter the **Server Address**. These options are available only if the DICOM licensed feature is enabled on the system.
- Note

If you select **Use Time on Time Server**, you cannot edit the **Date** and **Time**. If you want to edit the **Date** and **Time**, deselect **Use Time on Time Server** first.
- ▶ To specify that the system automatically adjusts for daylight savings time changes, select **Daylight Savings Time**. This option is available only if you select **Use Time on Time Server**.
 - ▶ Select your time zone from the **Time Zone** list.

Related topics

Setting up the system 27

DICOM

Using data in the Digital Imaging and Communications in Medicine (DICOM) standard, the ultrasound system can do the following:

- ▶ Transfer patient exam data to an archiver, DICOM printer, or storage commitment server. See "Archiving and exporting" on page 165.
- ▶ Import patient data from a worklist. See "Entering patient information" on page 150.
- ▶ Export exam data as DICOMDIR files to a USB storage device. See "USB settings" on page 77.

DICOM is a licensed feature. The system conforms to the DICOM standard as specified in the *X-Porte DICOM Conformance Statement*, available at www.sonosite.com.

The *X-Porte DICOM Conformance Statement* provides information about the purpose, characteristics, configuration, and specifications of the network connections supported by the system.

The system connects over a local area network (LAN) to the following:

- ▶ Picture Archive and Communication Systems (PACS) archivers
- ▶ Worklist servers
- ▶ Modality Performed Procedure Step (MPPS) servers
- ▶ DICOM printers
- ▶ Storage commitment servers

The system can transfer to one or more devices and connect to different networks, depending on how you configure it. The DICOM log collects network errors and events, typically to support diagnostics (see "Logs" on page 63).

Related topics

Logs	63
Entering patient information	150
Worksheets and reports	220
CDA Report settings	41
Archiving and exporting	165
Network Status settings	65
Connectivity settings	43
Routing selections	74

Configuring the system for DICOM transfer

To configure the system for DICOM transfer, you do the following tasks (typically completed by a network administrator or PACS manager):

- 1 Back up default DICOM configuration settings to a USB memory stick kept in a secure location. You can revert to the default settings if necessary. See "Importing and exporting connectivity settings" on page 43.
- 2 Connect to the network. See "Connecting to the network" below.
- 3 Complete the configuration pages for locations and devices. See "DICOM configuration pages" on page 50.
- 4 Associate devices with locations. See "Associating devices with locations" on page 59.

Connecting to the network

Before configuring the system for DICOM transfer, you must connect to the network. You can connect by Ethernet or wirelessly.

If you connect by Ethernet, the X-Porte ultrasound system must be connected to a standard Ethernet 10/100/1000 network.

To connect to the network by Ethernet

Devices connected to the system's Ethernet port must comply with the IEC 60601-1 or IEC 60950 standards.

- ▶ Connect the Ethernet cable to the Ethernet port on the back of the dock (see "Ports" on page 21). With the system on, the LAN link light (green LED) next to the Ethernet connector indicates physical connection to the network.

To connect to the network wirelessly

Wireless connectivity is a licensed feature.


- 1 Tap **MORE**, and then tap **System Settings**.




- 2 Tap **Connectivity** in the list on the left.



- 3 On the Connectivity setup page, tap **DICOM Setup**.

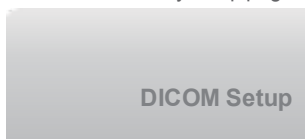


- 4 Tap **Config**.

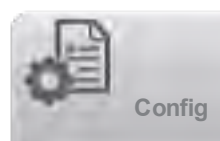


- 5 Tap **Location**.

- 6 Tap **New** and complete the fields on the **DICOM Location** configuration page. Required fields are marked with asterisks. See "DICOM configuration pages" on page 50.

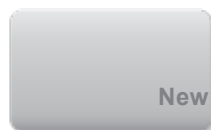


- 7 Select **Wireless** on the **Location** configuration page.

- 8 Tap **Wireless Configuration**.

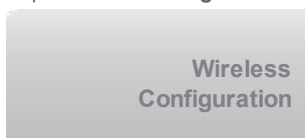

- 9 Complete the wireless connectivity fields. Required fields are marked with asterisks. See "DICOM configuration pages" on page 50.

- 10 Tap **Save**.

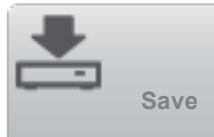

The wireless icon  indicates the status of the wireless connection (the number of white bars indicates the strength of the connection).

(Optional) To import or delete wireless certificates


If your security scheme requires it, you can import wireless certificates. The system supports the following file types: CER, PFX, and PVK.

Caution To avoid losing data from or damaging the USB memory stick, do not remove the USB memory stick or turn off the ultrasound system while exporting. In addition, do not bump or apply pressure to the USB memory stick while it is connected to the system. The connector could break.

- 1 Tap **MORE**, and then tap **System Settings**.




- 2 Tap **Connectivity** in the list on the left.

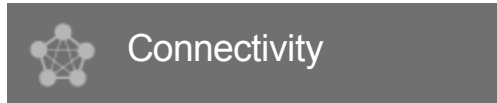

- 3 On the Connectivity setup page, tap **Wireless Certificates**.
A list of certificates on the system appears.


4 Do one of the following:

- **Import certificates:** Connect the USB memory stick that contains the certificates (see "To connect a USB memory stick for importing or exporting" on page 9). Verify that the certificates on the system, plus those on the USB memory stick, do not exceed 20. Delete certificates if necessary. Then tap **Import**.

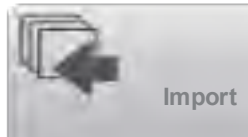

Note | Certificates on the USB memory stick replace certificates on the system.

- **Delete certificates:** Select the certificates to delete, and then tap **Delete**.

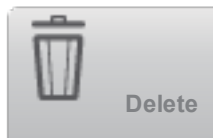

5 Tap **Done**.



6 Restart the system.

DICOM configuration pages

The DICOM configuration pages are as follows:

Location Configuration for network settings, including settings for a wireless network connection. Also specifies which devices you want to associate with that network. For example, configure a location called "Office," and then associate a printer and archiver with it. You can configure up to eight locations. (See "Associating devices with locations" on page 59.)

Archive Configuration for PACS archivers, devices for storing patient exams (images and clips). Only one archiver per location can receive in-progress image transfers. You can associate up to four archivers per location.

Printer Configuration for DICOM printers for printing images. You can associate up to two printers per location.

Storage Commit Configuration for storage commitment servers, devices that take responsibility for and provide receipt of content sent by the ultrasound system.

Worklist Configuration for worklist servers, devices that contain scheduled patient procedure data. You can import the data into the patient form (see "Entering patient information" on page 150).

MPPS Configuration for MPPS servers, devices that coordinate detailed information about the exam performed. You can associate one MPPS server per location.


To configure a new location or device

Note | Changing the configuration can require a system restart.

- 1 Tap **MORE**, and then tap **System Settings**.




- 2 Tap **Connectivity** in the list on the left.

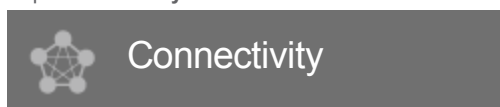


- 3 On the Connectivity setup page, tap **DICOM Setup**.

- 4 Tap **Config**.

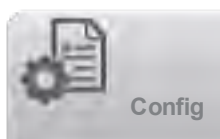


- 5 From the **DICOM** list, select a configuration page.

- 6 Tap **New**.

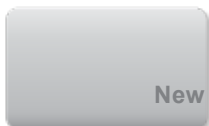


- 7 Complete the configuration setup fields, and tap **Save**.

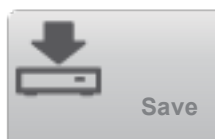

To delete a location or device

- 1 On the proper configuration page, select the name from the list of locations or devices.
- 2 Tap **Delete**.

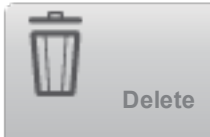

- 3 Tap **Yes** to confirm the deletion.
- 4 Tap **Done**.



Configuration setup fields

Location

Required fields are marked with asterisks.

Name

Unique network name for the ultrasound system. The default is SonoSite.

Alias

Name that identifies the network location of the ultrasound system.

AE Title

DICOM Application Entity Title.

Port

Device port number. IP port 104 is typically assigned for DICOM.

DHCP

If desired, select to enable Dynamic Host Configuration Protocol. The IP Address, Subnet Mask, and Default Gateway fields become unavailable.

IP Address

Unique identifier of the ultrasound system location. Cannot be between 127.0.0.0 and 127.0.0.8.

Subnet Mask

Identifies a network subdivision. The default value is 255.255.0.0.

Default Gateway

IP address where network connects to another network. Cannot be between 127.0.0.0 and 127.0.0.8.

Transfer Images

Specify when to transfer images: during or at end of exam.

JPEG Compression

Select High, Medium, or Low. A high compression has a smaller file size but less detail. For best image quality, select Low. See "Limitations of JPEG format" on page 79.

Wireless

Select if configuring a wireless location.

The following fields are available for wireless network connections:

Profile Name

Name of the profile set for this location. For each wireless location, you can have up to 10 profiles.

Network Name (SSID)

Network Name Service Set Identifier for the router.

Security Policy

Security type that authenticates the network:

- **Open** No security.
- **WEP** or **Shared WEP** The following fields appear:
 - **Encryption** Encryption key type (64 bit or 128 bit)
 - **Key index** WEP key index 1-4. Network location where a specific key is stored.
 - **Key** WEP key value used to encrypt data.

- **RADIUS** Remote Access Dial-Up User Service -802.1x Key Exchange. The following fields appear:
 - **Authentication Type** Select one of the following:
 - **EAP-TLS** Extensible Authentication Protocol-Transport Layer Security. The following fields appear:
 - **User Name** Name of designated user.
 - **Client Certificate** Select from the list of client certificates installed on the system.
 - **Private Key** One of a pair of keys (public and private) that is provided only to the requestor and never shared.
 - **Private Key Password** A unique combination of letters and symbols that allows user access.
 - **Certification Authority** Validates the authentication server or certificate authority. The list is populated by wireless certificates that you have imported. (See "(Optional) To import or delete wireless certificates" on page 49.) Select the desired certificate from the list.
 - **EAP-PEAPV0**
 - **User Name** Name of designated user.
 - **Password** A unique combination of letters and symbols that allows user access.
 - **Certification Authority** Validates the authentication server or certificate authority. The list is populated by wireless certificates that you have imported. (See "(Optional) To import or delete wireless certificates" on page 49.) Select the desired certificate from the list.

- **WPA or WPA2** Wi-Fi Protected Access. The following fields appear:
 - **Authentication** If you select **PERSONAL**, the **Passphrase** field appears; enter the WPA Shared Key entry used in configuring the router.
 - **Encryption** Encryption protocol for the router.
 - **Authentication Type** Select one of the following:
 - **EAP-TLS** Extensible Authentication Protocol-Transport Layer Security. The following fields appear:
 - **User Name** Name of designated user.
 - **Client Certificate** Select from the list of client certificates installed on the system.
 - **Private Key** One of a pair of keys (public and private) that is provided only to the requestor and never shared.
 - **Private Key Password** A unique combination of letters and symbols that allows user access.
 - **Certification Authority** Validates the authentication server or certificate authority. The list is populated by wireless certificates that you have imported. (See "(Optional) To import or delete wireless certificates" on page 49.) Select the desired certificate from the list.
 - **EAP-PEAPv0** Extensible Authentication Protocol-Protected Extensible Authentication Protocol. The following fields appear:
 - **User Name** Name of designated user
 - **Password** A unique combination of letters and symbols that allows user access.
 - **Certification Authority** Validates the authentication server or certificate authority. The list is populated by wireless certificates that you have imported. (See "(Optional) To import or delete wireless certificates" on page 49.) Select the desired certificate from the list.

Archiver

Required fields are marked with asterisks.

Name

Network host name for an archiver.

Alias

Personalized name for an archiver.

AE Title

Archiver DICOM Application Entity Title.

Images

Defines how images are sent to the archiver: RGB (uncompressed), Mono (uncompressed), or JPEG.

Image Type

List of archiver image types, based on capture mode.

IP Address

Unique identifier for the archiver.

Port

Device port number. IP port 104 is typically assigned for DICOM.

Ping

Tap to determine whether the IP address is accessible. The system displays OK or Failed.

Exclude Video Clips

If the check box is selected, only images (no video clips) are transferred. Available only if Image Type is set to Ultrasound.

Include CDA

Select this check box to send the CDA report to the archiver.

Include SR

Select this check box to send the Structured Report to the archiver.

Educational

Select this check box to associate the archiver with the **Educational** routing select. See "Routing selections" on page 74.

Printer

Required fields are marked with asterisks.

Name

Network host name for a printer.

Alias

Personalized name for a printer.

IP Address

Unique identifier for the printer.

Port

Device port number. IP port 104 is typically assigned for DICOM.

AE Title

Printer DICOM Application Entity Title.

Ping

Select to determine whether the IP address is accessible. The system displays OK or Failed.

Model

List of Agfa, Codonics, and Kodak printer models. If your model is not listed, choose a generic model at the end of the list.

Film Size

Film sizes supported by the printer.

Film Type

Film medium supported by the printer.

Destination

Location film is placed after it is printed.

Format

Number of columns and rows in the image printout.

Orientation

Film layout.

Settings

Defines how images are sent to the printer, either as Color (RGB) or Monochrome images.

Copies

Number of copies to print for each image.

Priority

Importance of the print job.

Max

Maximum density of the black value (in hundredths of optical density (OD)).

Min

Minimum density of the white value (in hundredths of optical density (OD)).

Border

Density of the areas surrounding and between film images (in hundredths of optical density (OD)) .

Empty

Empty image density (in hundredths of optical density (OD)).

Magnification

Type of interpolation used during printing.

Configure

Printer-specific configuration value. If using generic printer settings, no configuration strings are available.

Storage Commit

Required fields are marked with asterisks.

Name

Unique network name for the Storage Commit server. Default is SonoSite.

Alias

Name that identifies the network location of the Storage Commit server.

AE Title

DICOM Application Entity Title.

IP Address

Unique identifier of the storage commitment server.

Port

Device port number. IP port 104 is typically assigned for DICOM.

Ping

Tap to determine whether the IP address is accessible. The system displays OK or Failed.

Worklist

Required fields are marked with asterisks.

Name

Network host name for a worklist server.

Alias

Personalized name for a worklist server.

AE Title

Application Entity Title.

Worklist

Today; Yesterday, today, tomorrow; All.

This Device Only

Restricts the query to patient procedures that are scheduled for the system based on its AE Title.

Modality

US (Ultrasound) is the default procedure type.

IP Address

Unique identifier for the worklist server.

Port

Device port number. IP port 104 is typically assigned for DICOM.

Automatic Query

Turns automatic query on/off.

Occurs Every

In an automatic query, length of time between automatic updates.

Start Time

In an automatic query, start time for the automatic update (displayed in 24 hour time).

Ping

Tap to determine whether the IP address is accessible. The system displays OK or Failed.

MPPS

Required fields are marked with asterisks.

Name

Network host name for the MPPS server.

Alias

Name that identifies the network location of the MPPS system.

AE Title

DICOM Application Entity Title.

IP Address

Unique identifier of the MPPS server.

Port

Device port number. IP port 104 is typically assigned for DICOM.

Ping

Tap to determine whether the IP address is accessible. The system displays OK or Failed.

Associating devices with locations

For each location, select which devices (archivers and printers) you want to receive the data that you transfer, which archivers you want to designate as an MPPS or storage commitment server, and which worklist server you want to receive data from. Once these selections are complete, select the location you want to use.

Note | Changing the configuration requires a system restart.

To associate devices with a location

The devices must be configured before you can associate them. See "To configure a new location or device" on page 51.

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Connectivity** in the list on the left.



- 3 On the Connectivity setup page, tap **DICOM Setup**.
- 4 In the **Location** list, select the location of the system.

- 5 In the list of devices, select the check box next to one or more archivers, printers, or worklist servers. You can select a maximum of two printers, four archivers, and one worklist server for each location. Only one archiver can be selected to receive in-progress transfers. Selected devices have a check mark in the left-hand column.
- 6 If you want to use the MPPS service, associate the MPPS server with the archiver:
 - a. Select the check box of the MPPS server that you want. (MPPS servers appear near the end of the list.)
 - b. Select the check box of the archiver.
 - c. Select the check box in the archiver's MPPS column.
- 7 If you want to use the storage commitment service, associate the storage commitment server with the archiver:
 - a. Select the check box for the storage commitment server that you want. (Storage commitment servers appear at the end of the list.)
 - b. Select the check box for the archiver.
 - c. Select the check box in the archiver's SC column.
- 8 Complete any additional configuration tasks, and then tap **Done**.



To disassociate devices with a location

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Connectivity** in the list on the left.



- 3 On the Connectivity setup page, tap **DICOM Setup**.
- 4 In the **Location** list, select the location.
- 5 Do any of the following:
 - To disassociate a printer, archiver, or worklist server, select its check box.
 - To disassociate an MPPS server from an archiver, select the check box for the MPPS server.
 - To disassociate a storage commitment server from an archiver, select the check box for the storage commitment server.
 - To disassociate an archiver from all servers, deselect and then reselect its check box.

- 6 Tap **Done**.



To verify the connection status of devices

- 1 On the Connectivity setup page, tap **DICOM Setup**.
- 2 Select the device you want to verify, and tap **Verify** to confirm that the associated devices are connected. (If **Verify** is unavailable, check cable and wireless connections. Restart the system if you changed configuration. If the problem continues, see your system administrator.)



The connection status of the devices appears in the **Status** column:

- ▶ **Failed** DICOM cannot communicate with the device.
- ▶ **Success** DICOM can communicate with the device.
- ▶ **Unknown** The device does not support the C-ECHO (for example, Verify query command).

Display Information settings


On the Display Information settings page, you can specify which details appear on the clinical monitor during imaging.

To display the Display Information settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Display Information** in the list on the left.

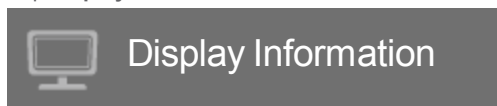

To specify details that appear on the clinical monitor

- ▶ On the Display Information settings page, select settings in the following sections:
Patient Header Information that appears in the patient header, including the **Patient Name**, **Patient ID**, **Department ID**, **User**, **Institution**, and **Date and Time**.
Mode Data Imaging information for **2D**, **Doppler**, **Color**, or **M Mode**.

Related topics

Setting up the system 27


Labels

You can customize predefined labels, create new labels, delete labels, and set the preference for clearing labels when unfreezing images.

To manage predefined labels

Each exam type supports up to 35 predefined labels.

- 1 On the Presets settings page, tap **Labels**.

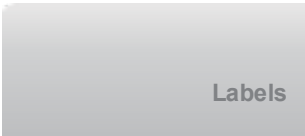


2 Tap an exam type and do any of the following:


- To customize a label, tap it and edit the text in the text box at the top of the form. Tap **Apply**.



- To create a new label, tap an empty label, and type the new label in the text box. Tap **Apply**.
- To delete a label, tap the **X** next to the label.
- To reset labels to the factory default, tap **Reset**.

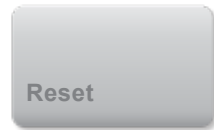

- To select a new exam type, tap **Exam Type**.

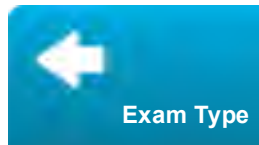

To specify label retention when unfreezing

You can specify that all labels be cleared from an image when you unfreeze the image.

- 1 On the Presets settings page, tap **Labels**.



- 2 Select **Clear Labels on Unfreeze**.

Logs

On the Logs settings page, you can view the following logs:

- **User** Collects information about user logins and user creation, as well as information about when the log was exported or cleared.

- ▶ **DICOM** Collects network errors and events, typically to support diagnostics (see "DICOM" on page 46).
- ▶ **Assert** Collects processor exceptions and software-generated assertions to support diagnostics. The assert log is visible only to users with administrative privileges.
- ▶ **System** Collects errors and events to support diagnostics. The system log is visible only to users with administrative privileges.

The logs are text files that you can open in a text-editing application (for example, Notepad or TextEdit). You can export the logs to a USB memory stick and read them on a PC.

Log contents are saved as entries are generated. The logs have limited space and overwrite existing content when full.

To display the Logs settings page

- 1 Tap **MORE**, and then tap **System Settings**.




- 2 Tap **Logs** in the list on the left.



To export a log

Caution To avoid losing data from or damaging the USB memory stick, do not remove the USB memory stick or turn off the ultrasound system while exporting. In addition, do not bump or apply pressure to the USB memory stick while it is connected to the system. The connector could break.

- 1 Connect a USB memory stick (see "To connect a USB memory stick for importing or exporting" on page 9).
- 2 On the Logs settings page, tap the log.
- 3 Tap **Export**.

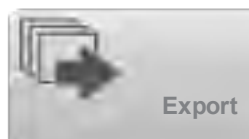

A list of USB devices appears.


- 4 Select the appropriate USB memory stick, and tap **Export**.



- 5 Tap **Yes** to confirm the export.
Five seconds after the exporting completes, you can safely remove the USB memory stick.

To clear a log

- 1 On the Logs settings page, tap the log.
- 2 Tap **Clear** to delete all text.

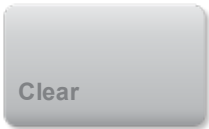

- 3 Tap **Yes** to confirm the deletion.

Related topics

DICOM.....	46
Setting up the system.....	27

Network Status settings

The Network Status settings page displays information on system IP address, location, subnet mask, default gateway, and the Ethernet MAC address.

To display the Network Status settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Network Status** in the list on the left.



Related topics

Connectivity settings.....	43
DICOM.....	46
Setting up the system.....	27

Power and Battery settings


On the Power and Battery settings page, you can specify the period of inactivity before the system goes into sleep mode or turns off.

To display the Power and Battery settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Power and Battery** in the list on the left.

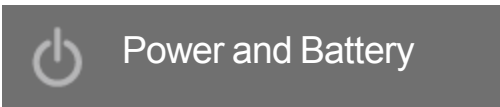

To specify power settings

- ▶ On the **Power and Battery** settings page, select from the following lists:
 - ▶ **Sleep Delay** Select **Off**, or **5** or **10** minutes to specify the period of inactivity before the system goes into sleep mode.
The system will automatically turn off if it is in sleep mode and the battery charge level drops below 14%.

Note | The system will not sleep if you are in Procedure mode (see "Controls in 2D" on page 101) or if the system is archiving or exporting exams.
 - ▶ **Power Delay** Select **Off**, or **15** or **30** minutes to specify the period of inactivity before the system automatically turns off.

Related topics

Setting up the system, 27

Presets settings

The Presets settings page has settings for general preferences, brightness, labels, exam types, and user profile settings. You can export and import these settings.

To display the Presets settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **Presets** in the list on the left.



General preferences

To set general preferences

- ▶ On the Presets settings page, select from the following lists:
 - Note** Changing **Doppler Scale** or **Units** clears all measurements and calculations for the current exam.
 - ▶ **Doppler Scale** Select **cm/s** or **kHz**.
 - ▶ **Thermal Index** Select **TIS**, **TIB**, or **TIC**. The setting is retained for the current session. When the exam is ended or another transducer or exam type is selected, the exam type-based default setting is used.
 - ▶ **Units** Units for patient height and weight in cardiac exams: **in/ft/lbs** or **cm/m/kg**.
 - ▶ **Auto Save Patient Form** When this option is on, the system saves the patient form as an image in the patient's exam.
 - ▶ **Generate Patient ID** When this option is on, the system automatically generates a unique patient ID upon starting an exam from the patient form, unless a patient ID is entered manually or from the worklist. This option can be used to accommodate certain workflows.
 - ▶ **Prefix** Defines a prefix string attached to the beginning of the automatically generated patient ID. This enables you to easily search for the patient ID on the image storage system at a later time. This option is available only when the Generate Patient ID option is on.

Brightness

You can adjust the brightness of the clinical monitor or the touch panel to compensate for the level of light in the room in which the system is located.

To adjust brightness

- ▶ Do either or both of the following on the Presets settings page:
 - ▶ Drag the **Monitor Brightness** slider. The range is **0-10**.
 - ▶ Drag the **Touch Panel Brightness** slider. The range is **2-10**.


Labels

You can customize predefined labels, create new labels, delete labels, and set the preference for clearing labels when unfreezing images.

To manage predefined labels

Each exam type supports up to 35 predefined labels.

- 1 On the Presets settings page, tap **Labels**.

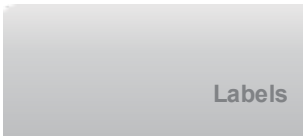


- 2 Tap an exam type and do any of the following:


- To customize a label, tap it and edit the text in the text box at the top of the form. Tap **Apply**.



- To create a new label, tap an empty label, and type the new label in the text box. Tap **Apply**.
- To delete a label, tap the **X** next to the label.
- To reset labels to the factory default, tap **Reset**.

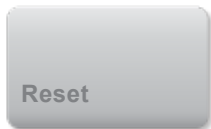

- To select a new exam type, tap **Exam Type**.

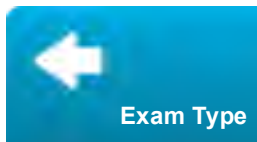

To specify label retention when unfreezing

You can specify that all labels be cleared from an image when you unfreeze the image.

- 1 On the Presets settings page, tap **Labels**.






- 2 Select **Clear Labels on Unfreeze**.

Exam types

The system displays both system-defined and user-defined exam types, organized by transducer. You cannot share user-defined exam types among transducers.

To manage exam types

- 1 On the Presets settings page, tap **Exam Types**.

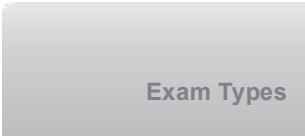

- 2 Tap the transducer that includes the exam type you want to manage.

Note If the transducer that includes the exam type you want to manage does not appear, make sure that the transducer is connected to the system.

- 3 Do any of the following:

- ▶ (User-defined exam types only) To rename the exam type, tap the exam type name and type a new name.
- ▶ (User-defined exam types only) To delete the exam type, tap  .
- ▶ To show or hide the exam type in the transducer & exam types list, tap the check box next to the exam type in the  column.
- ▶ To rearrange the exam type, tap it and then tap **Up** or **Down**.
- ▶ To return to the transducer and exam types list, tap **Transducer**.

- 4 Tap **Done**.



Customizing exam types

You can customize, or *define*, any exam type to use your favorite settings. When you select the new exam type (that is, the *user-defined exam type*) the system automatically uses the settings you specified.

The system displays user-defined exam types along with the original exam types in the transducer-selection lists.


You can customize up to five exam types for each exam type/ transducer combination. For example, you can customize five different exam types for Abdomen on the P21xp transducer and five different exam types for Abdomen on the C60xp transducer.

You can rename, delete, export, show, hide, reorder, and import user-defined exam types. See "Presets settings" on page 66.

To customize an exam type


- 1 Select a transducer and exam type (see "Selecting a transducer and exam type" on page 16).
- 2 Adjust imaging controls to your favorite settings.
You can set gain, AutoGain adjust, depth, dynamic range, image orientation, sweep speed, color settings, color Doppler type, Doppler invert, Doppler angle correct, wall filter level, scale, baseline, audio volume level, optimization, centerline display, simultaneous display of 2D and M (Simultaneous M Mode), and trace settings. For more information about Simultaneous M Mode, see "Controls in M Mode" on page 116.

- 3 Tap **Save Exam Type**.

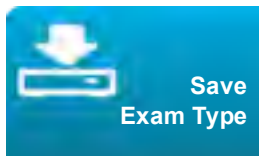


If this control is hidden, tapping **More Controls** displays it.

- 4 In the **Save User-Defined Exam** window, tap **New**.

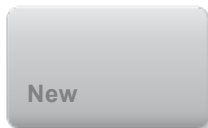


- 5 Type a name for the exam type in the Exam box, and then tap **Save**. The exam type must have a unique name.

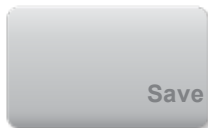


To modify a user-defined exam type

- 1 Adjust imaging controls to the new settings.
- 2 Tap **Save Exam Type**.

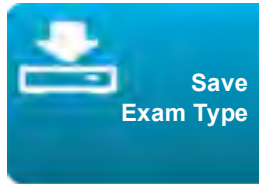

If this control is hidden, tapping **More Controls** displays it.

- 3 In the **Save User-Defined Exam** window, tap the exam type and tap **Update**.

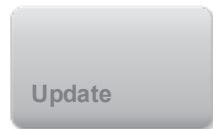


The current settings overwrite the previously saved settings.

User profile settings

To specify a startup mode

You can specify what mode the system will be in when you start it, end an exam, or log in.

- 1 On the Presets settings page, tap **User Profile**.

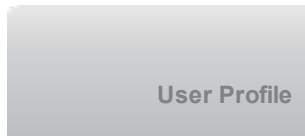


- 2 Select an option in the **Startup** section:
 - **Start Select Screen** Displays the home screen, which offers you the option of scanning, entering patient information, select a transducer and exam type, or viewing visual guides.
 - **Scanning** Displays the 2D imaging screen.
 - **Transducer/Exam Select** Displays the options for selecting transducers and exam types.
 - **Patient Info** Displays the patient form.

To automatically query the worklist from a bar code scanner

You can specify that the system query the worklist for patient information when you scan a patient ID bar code.

- 1 On the Presets settings page, tap **User Profile**.

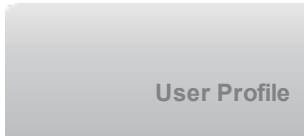


- 2 Select **Bar Code Auto Query**.

To program the footswitch

You can specify the function of the left and right footswitches: **None**, **Save Video Clip**, **Record**, **Freeze**, **Save Image**, or **Print**.

- 1 On the Presets settings page, tap **User Profile**.

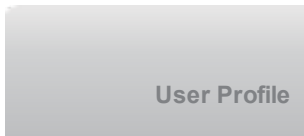


- 2 Select options from the **Footswitch (L)** and **Footswitch (R)** lists.

To configure custom controls

By default, the four controls that appear next to the **Freeze** control in live imaging are **Auto Gain**, **Save Image**, **Save Video Clip**, and **End Exam**.

You can specify the function of these controls.

- 1 On the Presets settings page, tap **User Profile**.

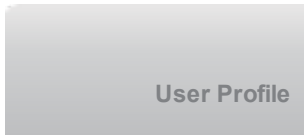

- 2 Select options from the **Program Control A**, **Program Control B**, **Program Control C**, and **Program Control D** lists.

The selected options for **Program Control A** and **Program Control B** appear to the left of the **Freeze** control on the touch panel. The selected options for **Program Control C** and **Program Control D** appear to the right of the **Freeze** control on the touch panel.

To reset controls

You can reset the More Controls window to the factory default.

If you have moved controls from the More Controls window to the Controls bar, resetting the controls removes them from the Controls bar. Similarly, resetting the controls moves controls you have moved from the Controls bar to the More Controls window.

- 1 On the Presets settings page, tap **User Profile**.




- 2 Tap **Reset More Controls**.



To show or hide the Learn button

You can specify that the system hide the Learn button on the touch panel. If you hide the Learn button, you can still access the visual guides by selecting **Learn** from the **More** menu.

- 1 On the Presets settings page, tap **User Profile**.

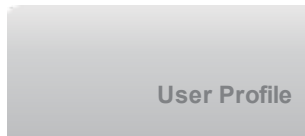

- 2 Deselect **Show Learn Button**.

Importing and exporting


When you import or export from the Presets settings page, you import and export all labels, exam types, user accounts, and system settings, except for connectivity settings.

Note You must have administrative privileges to import or export labels, exam types, user accounts, and system settings.

To import or export

When you import from the Presets settings page, all system preferences on the system are replaced with the imported preferences. When you export from the Presets settings page, all system preferences on the USB memory stick are replaced with the exported preferences.

- 1 Insert a USB memory stick.
- 2 On the Presets settings page, do either of the following:
 - To import labels, exam types, user accounts, and system settings, tap **Import**, and then tap **Yes**.

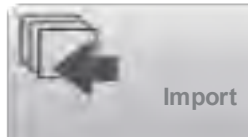


Select the desired USB memory stick, and then tap **Import**.



All labels, exam types, user accounts, and system settings are replaced with those from the USB memory stick.

- To export labels, exam types, user accounts, and system settings, tap **Export**, and then tap **Yes**.

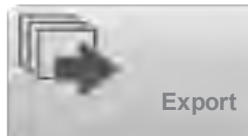

Select the desired USB memory stick, and then tap **Export**.



A copy of all labels, exam types, user accounts, and system settings saves to the USB memory stick. Passwords for user accounts are encrypted.

Routing selections

Routing selections let you control where patient exams are archived. You can associate one or more routing selection with an exam.

The system supports three routing selections:

- Diagnostics
- Educational/Accreditation
- Procedural

You can associate the Educational/Accreditation routing selection with a DICOM archive device (see "Specifying educational DICOM archivers" on the next page).

Related topics

Reviewing.....	158
DICOM.....	46
Entering patient information.....	150

Associating routing selections with exams

When you associate one or more routing selections with an exam, the system reflects the routing selections in the patient report and in the Acute Care worksheet (if the Acute Care Worksheets licensed option is installed). See "Worksheets and reports" on page 220.

You can associate an exam with a routing selection during an exam or after ending an exam.

To associate a routing selection during an exam

- 1 Tap **PATIENT**.



- 2 Select one or more routing selections in the **Routing Selection** section of the patient form.


To associate a routing selection after an exam

When you change the routing selection after an exam has ended, the system updates the patient report.


- 1 Tap **REVIEW**.



- 2 Tap **Patient List**.

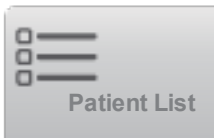

- 3 Select the check box next to the exam, and then tap **Route Exam**.

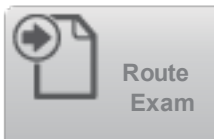

- 4 Select one or more options in the **Routing Selection** section.

Specifying educational DICOM archivers

When you associate the Educational/Accreditation routing selection with a DICOM archiver, the system archives exams associated with the Educational/Accreditation routing selection to that archiver.

Archivers that are not specified as educational are *procedural* (billing) archivers. The system archives exams associated with the Diagnostic and Procedural routing selections to these archivers.

If you do not specify an archiver as an educational archiver, exams associated with the Educational/Accreditation routing selection are also sent to the procedural (billing) archiver.

If an exam is associated with more than one routing selection, the system archives it to both the procedural (billing) archiver and to the educational archiver.

To specify an educational archiver

Note If you specify an archiver as an educational archiver, exams that aren't associated with a routing selection will not transfer to any archiver. (You can associate an exam with a routing selection once the exam has ended. See "To associate a routing selection after an exam" on the previous page.)


- 1 Make sure that the system is configured for DICOM transfer. See "DICOM" on page 46. Tap **MORE**, and then tap **System Settings**.




- 2 Tap **Connectivity**.



- 3 On the Connectivity system setup page, tap **DICOM Setup**.

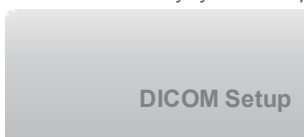

- 4 Tap **Config**.

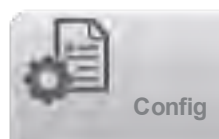

- 5 From the **DICOM** list, select the **Archive** configuration page for the archiver.

- 6 Select **Educational**.
- 7 Tap **Done**.



System Information settings

The System Information settings page displays system hardware and software versions, patents, and license information.


See also "Software licensing" on page 251.

To display the System Information settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **System Information** in the list on the left.

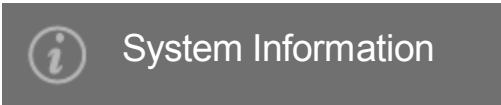

Related topics

Setting up the system,	27
------------------------------	----

USB settings

On the USB settings page, you can view information about connected USB devices, including space availability.

You can also specify a file format and options for images and video clips in patient exams that you export to a USB memory stick, and you can enable automatic export of ended exams.

To display the USB settings page

- 1 Tap **MORE**, and then tap **System Settings**.



- 2 Tap **USB** in the list on the left.



To specify export options

- 1 On the USB settings page, select an **Export Type**:
- ▶ **DICOM Export** creates DICOMDIR files readable by a DICOM reader. DICOM is an optional feature.
 - ▶ **Multimedia Export** organizes files in a standard folder structure. Video clips export in MJPEG video saved as AVI. To view them, FUJIFILM SonoSite recommends Windows Media Player or QuickTime 7.0 or later.
- 2 Select an **Image Format** for your export type. For JPEG image format, also select a JPEG compression. A high compression has a smaller file size but less detail. (See "Limitations of JPEG format" on the facing page.) The image format affects both still images and video clips.
For optimal DICOM image quality, select RGB image format and low compression.
- 3 (**Multimedia Export** only) Select a sort order from the **Sort By** list.
- 4 Select **Auto Export** to enable automatic export of ended exams. See also "To export patient exams automatically to a USB memory stick" on page 169.
- 5 (**DICOM Export** only) Select any of the following:
- ▶ **Include eFilm Lite** to include a copy of the eFilm Lite image-viewer with exams that you export to a USB memory stick. See "eFilm Lite image-viewer" on the facing page.
 - ▶ **Include CDA** to export the CDA report. See "CDA Report settings" on page 41.
 - ▶ **Include SR** to export the Structured Report.
- 6 Tap **Done**.



Related topics

USB devices	9
Archiving and exporting	165

Limitations of JPEG format

When transferring or exporting images in JPEG format, the system uses *lossy compression*. Lossy compression may create images that have less absolute detail than BMP format and that don't render identically to the original images.

In some circumstances, lossy-compressed images may be inappropriate for clinical use.

For more information on using lossy-compressed images, consult the industry literature.

eFilm Lite image-viewer

You can include a copy of the eFilm Lite image-viewer with exams that you export to a USB memory stick in DICOM format (see "To specify export options" on the previous page). eFilm Lite lets you view DICOM-format-
ted images on a computer running Windows.

eFilm Lite is included with the DICOM licensed feature.

To start eFilm Lite image-viewer after exporting exams

- 1 Insert the USB memory stick into your computer.
- 2 Display the USB memory stick's contents.
- 3 Double-click **eFilmLite.bat**.
eFilmLite.bat starts the executable file in the eFilmLite folder. The eFilmLite folder contains the eFilm Lite software and related files. See also the *eFilm Lite User's Guide*, a PDF file in the eFilmLite folder.

CHAPTER 4

Imaging


This section describes imaging with the X-Porte ultrasound system.

Understanding imaging modes

This section describes the imaging modes available on the ultrasound system. For a list of imaging modes available by transducer and exam type, see "Selecting a transducer and exam type" on page 16.

2D

2D is the system's default imaging mode. The system displays echoes in two-dimensional view by assigning a brightness level based on the echo signal amplitude. For scanning instructions, see "2D" on page 89.

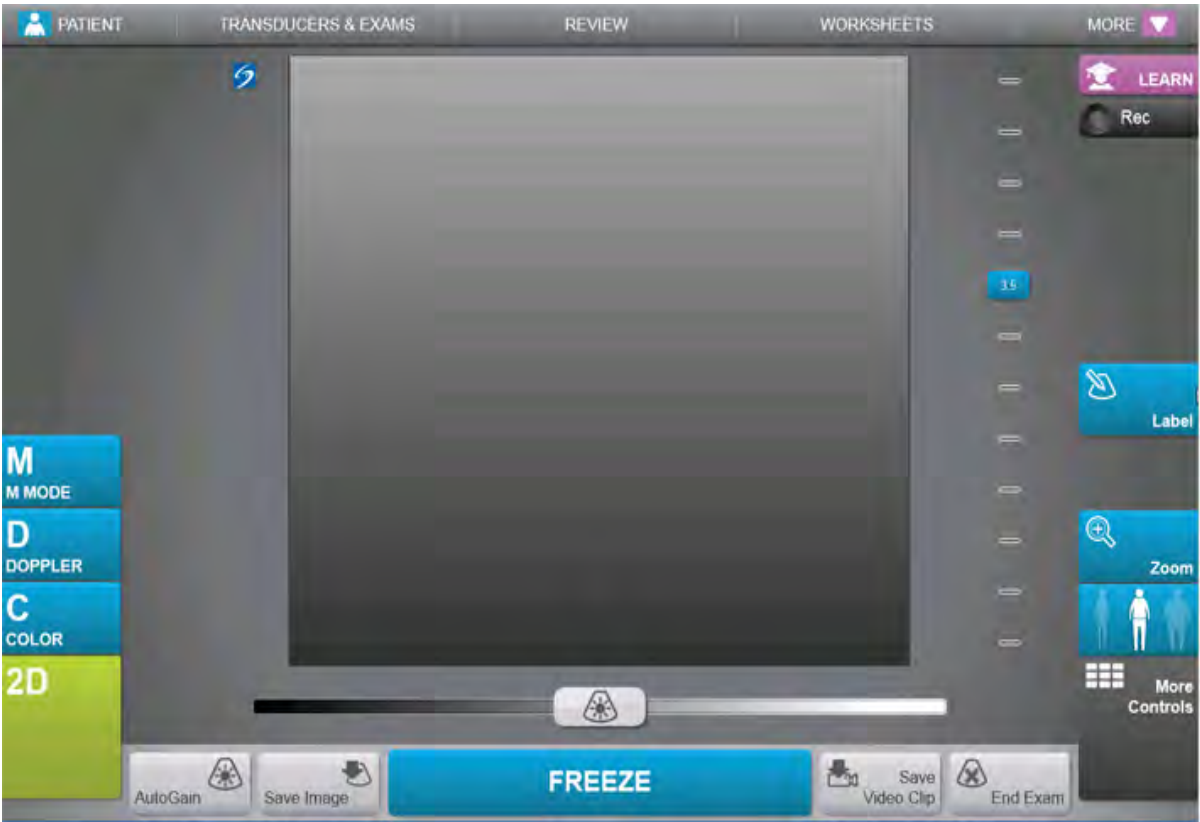


Related topics

Controls in 2D.....	101
Scanning.....	88
Doppler.....	86
Color.....	85
M Mode.....	83
Dual.....	88

M-line

When you tap **M Mode**, the M-line appears. You can drag the M-line to the area of interest before you display the M Mode trace. For scanning instructions, see "M Mode" on page 96.

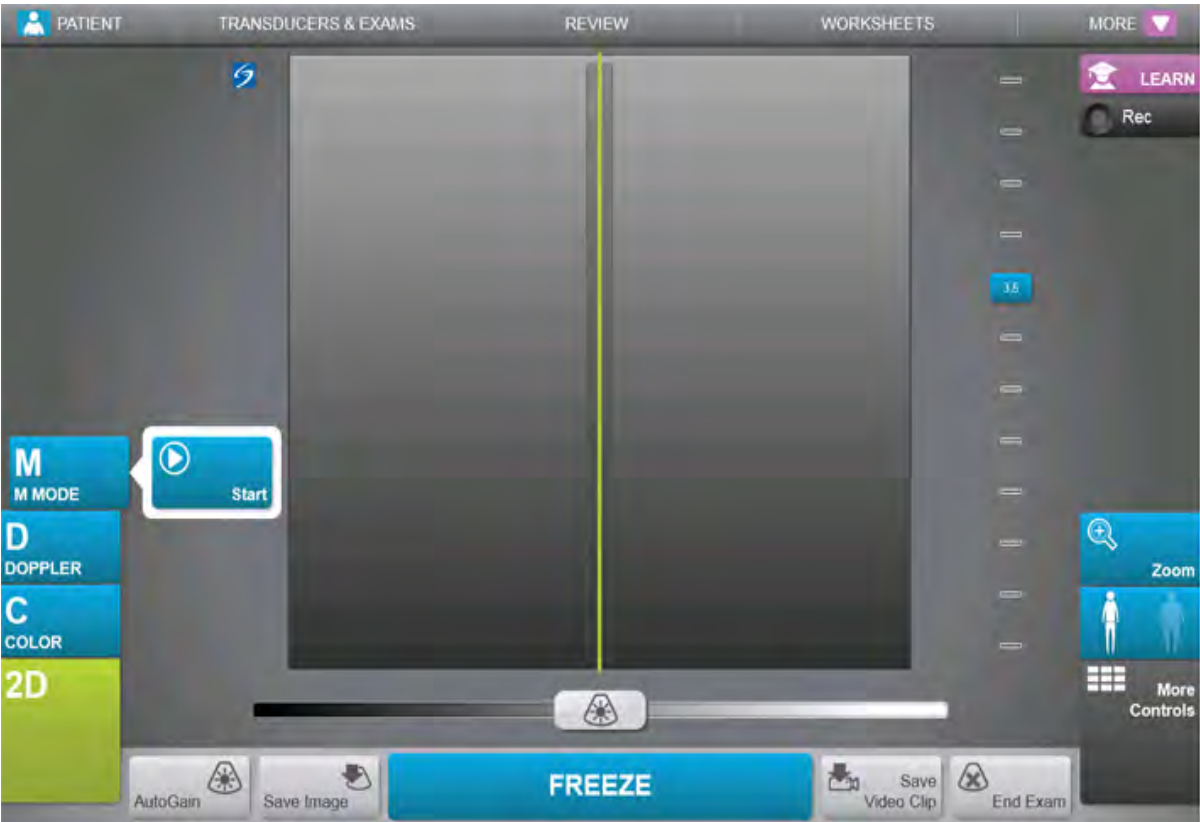

Related topics

M Mode.....	83
Scanning.....	88
Controls in M Mode.....	116

M Mode

Motion mode (M Mode) is an extension of 2D. It provides a trace of the 2D image displayed over time. A single beam of ultrasound is transmitted, and reflected signals are displayed as dots of varying intensities, creating

lines across the screen. During Simultaneous M Mode, 2D and M Mode are displayed in synchrony. For scanning instructions, see "M Mode" on page 96,




Related topics

Controls in M Mode.....	116
Scanning.....	88
Doppler.....	86
Color.....	85
2D.....	82
Dual.....	88
M-line.....	83

Color

Color imaging shows the presence, velocity, and direction of blood flow toward and away from the transducer. In Color mode, the area within the highlighted box displays blood flow in color. For scanning instructions, see "Color" on page 90.

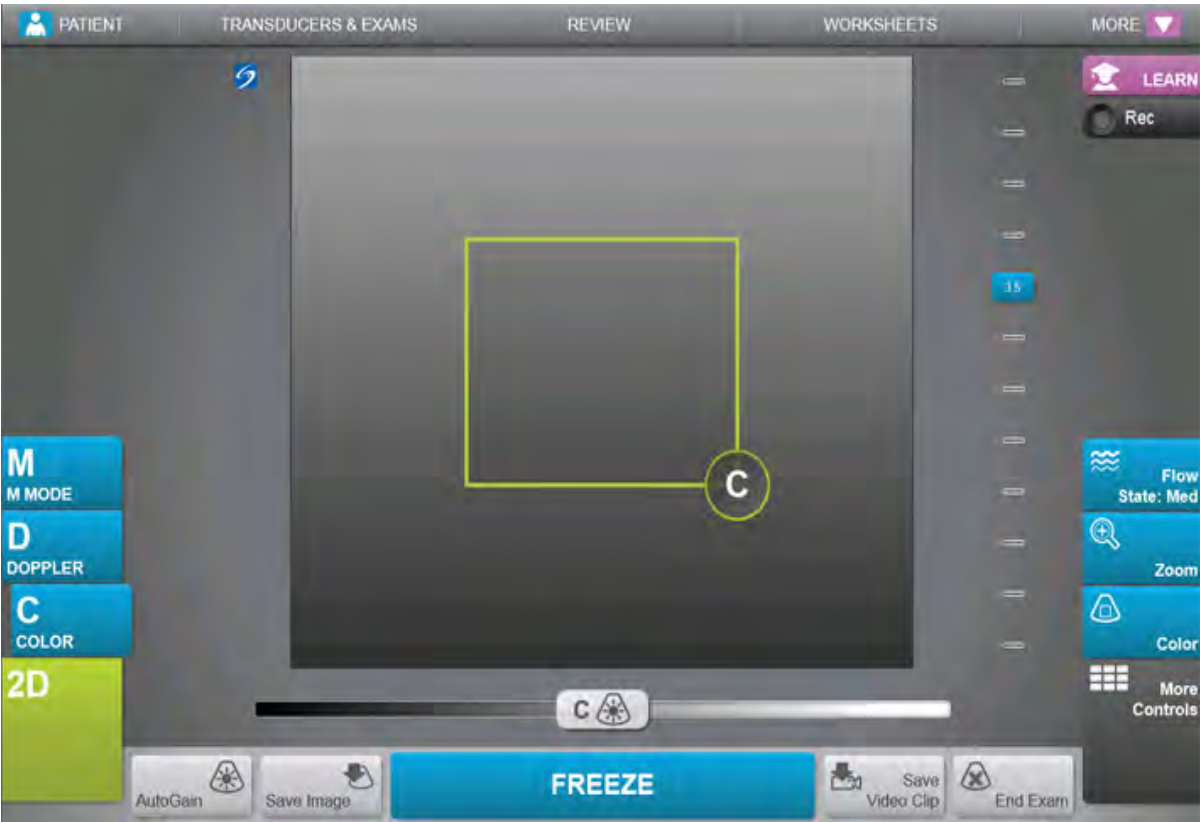


Related topics

Scanning	88
Doppler	86
M Mode	83
2D	82
Dual	88
Controls in Color	110

D-line

When you tap **Doppler**, the D-line appears. You can position the D-line, adjust the gate size, or correct the angle of the D-line before you display the Doppler trace. For scanning instructions, see "Doppler" on page 93.

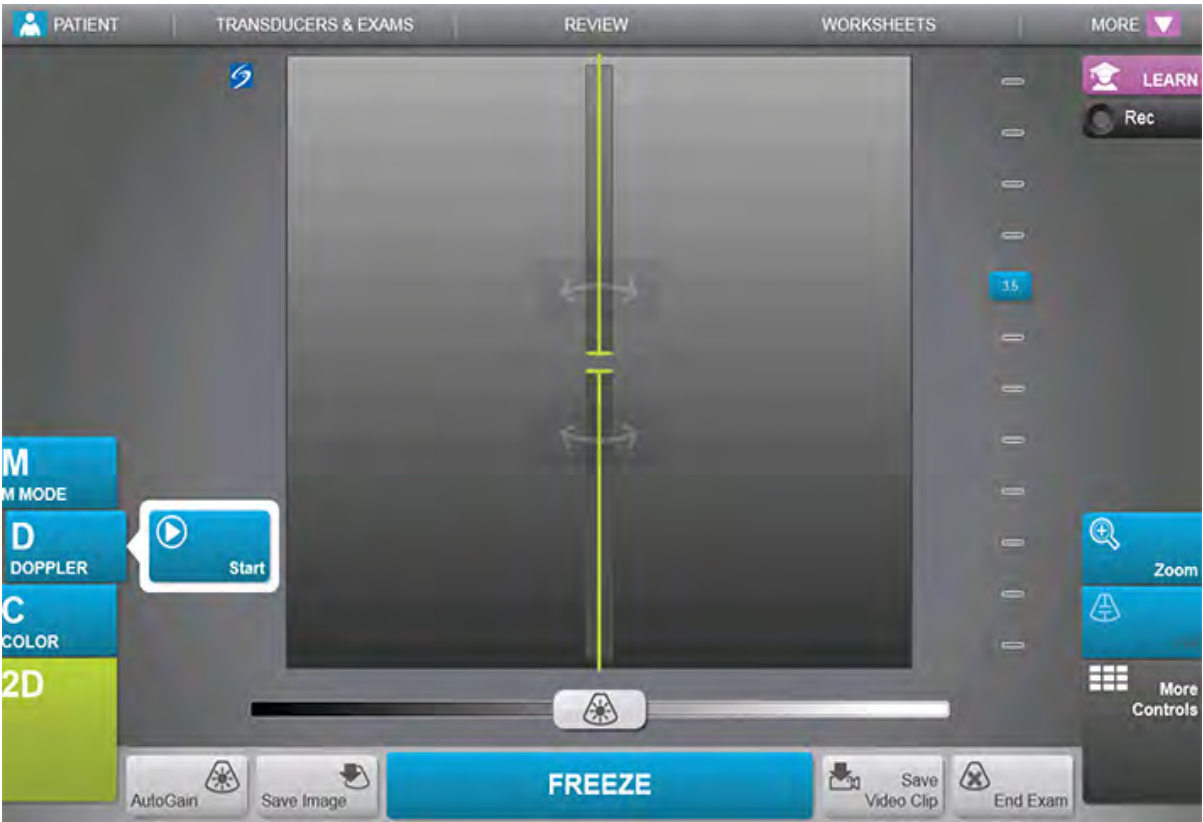


Related topics

Doppler.....	86
Controls in Doppler.....	120
Scanning.....	88

Doppler

Doppler imaging is a display of a spectrum of flow velocities over time. The amplitude of the signal is indicated as a shade of gray. Doppler imaging can be used to display any type of movement, including blood flow and tissue movement. Pulsed

wave (PW), pulsed wave Tissue Doppler Imaging (PW TDI), and continuous wave (CW) Doppler are available. CW Doppler is not range specific. For scanning instructions, see "Doppler" on page 93.

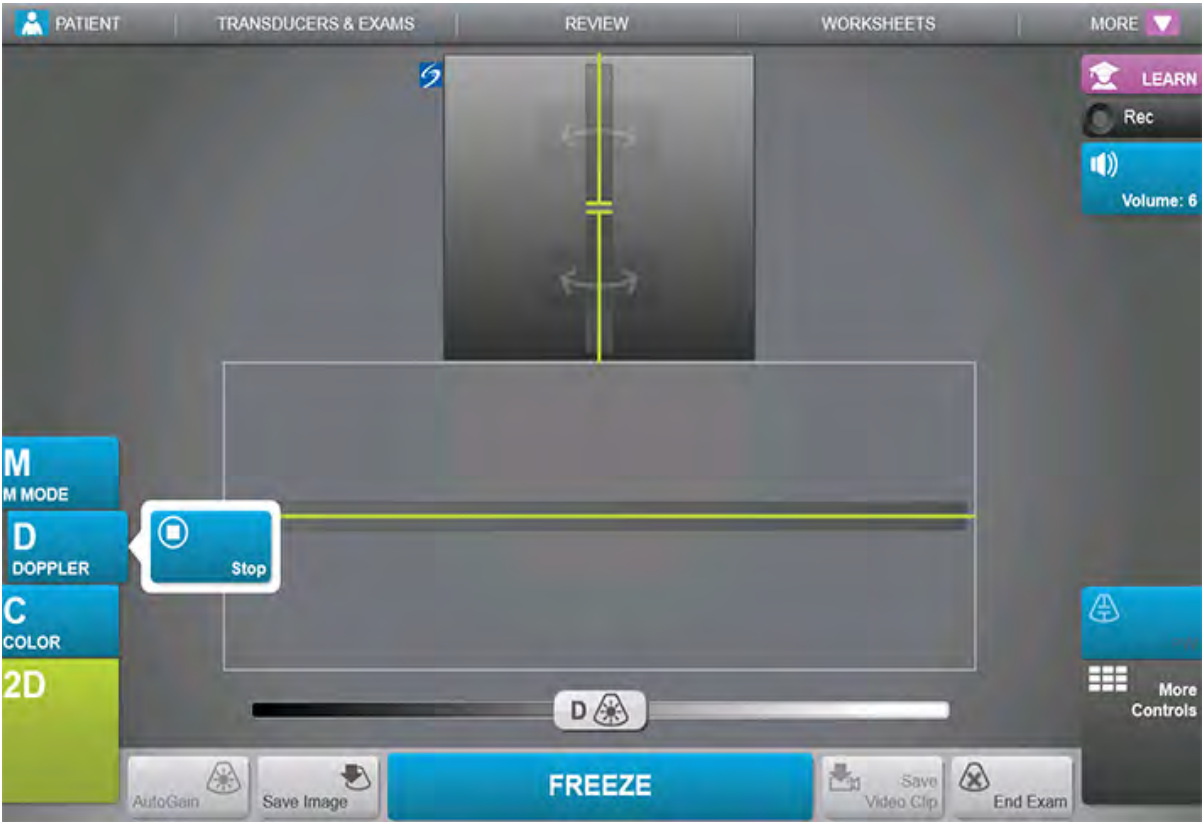


Related topics

Scanning.....	88
Color.....	85
M Mode.....	83
2D.....	82
Dual.....	88
Controls in Doppler.....	120
D-line.....	86

Dual

Dual displays side-by-side 2D images. The system displays independent image information for each image, for example, Depth, color maps, and orientation markers. You can switch between the two images and adjust some controls (for example, Depth and Gain) independently. You can view frames in the cine buffer for both images independently.

Dual is available in 2D and in Color. For scanning instructions, see "Dual" on page 91.

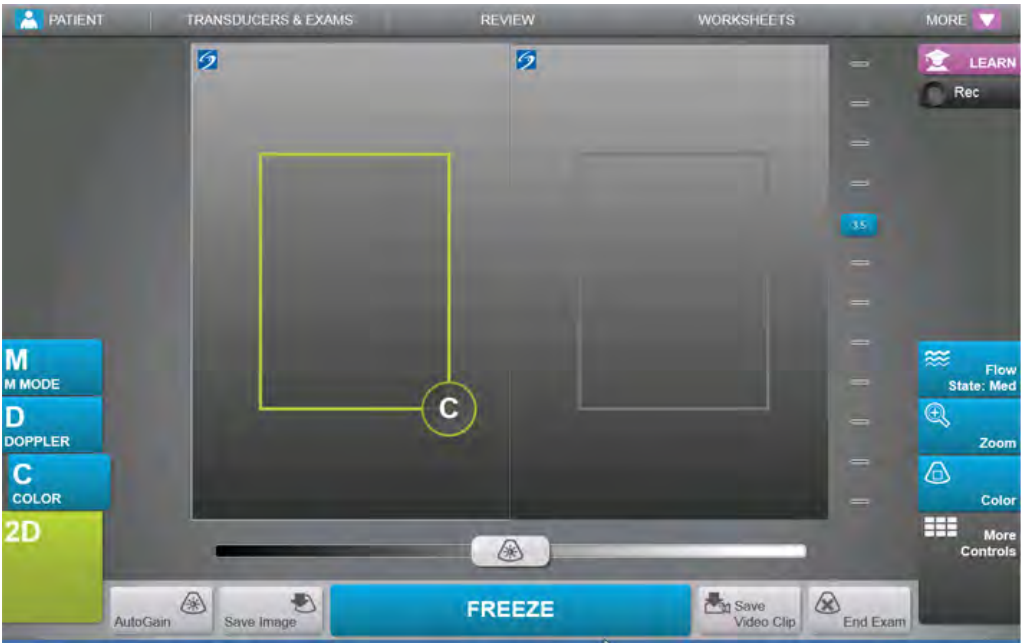

Related topics

Scanning.....	88
Doppler.....	86
Color.....	85
M Mode.....	83
2D.....	82

Scanning

X-Porte offers four different imaging modes: 2D (see "2D" on the facing page), Color, (see "Color" on page 90), Doppler (see "Doppler" on page 93), and M Mode (see "M Mode" on page 96). You can display 2D and M Mode simultaneously (see "Controls in M Mode" on page 116); you can also view 2D or color images in Dual (see "Dual" on page 91).

For information about working with controls, see "Imaging controls" on page 99.

Note When you change display formats (for example, from a single image to Dual, or by changing the Display Format), the images may overlap. If the M line or D line is located near the side of the image, it may be hidden. Toggle the trace to expose the M line or D line.

Related topics

Doppler.....	86
Color.....	85
M Mode.....	83
2D.....	82
Dual.....	88
Controls in 2D.....	101
Controls in Color.....	110
Controls in Doppler.....	120
Controls in M Mode.....	116
M-line.....	83
D-line.....	86

2D

1 Do one of the following:

▸ Tap **2D**.



▸ In Transducers & Exams, tap **SCAN**.


2 Adjust controls as needed. See "Controls in 2D" on page 101.

Color

- 1 Tap **Color**.



The color box appears.

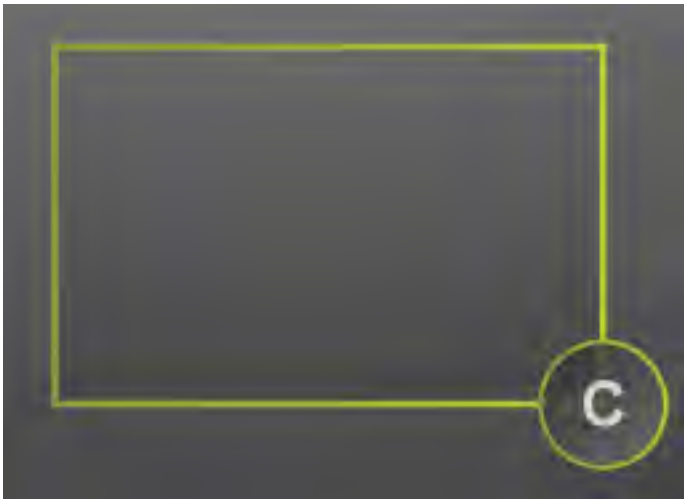

2 Position and resize the color box as needed:

- To position, do either of the following:
 - Drag the box.
 - With two or more fingers, drag anywhere on the touch panel.
- To resize, do either of the following:
 - Pinch or spread the box.



- Drag its handle

A green outline shows that the status of the change is active. On the clinical monitor, the current outline remains in gray until you stop moving or release your finger.

- To steer (linear transducers only), do either of the following:
 - Swipe left or right.
 - Tap **Steering** and select an angle.



3 Adjust controls as needed. See "Controls in Color" on page 110.

Dual

Dual is available in 2D and in Color.

- 1 Tap **Dual**. When **Dual** is on, the control is highlighted.



If this control is hidden, tapping **More Controls** displays it.

- 2 Do either of the following to make an image active:
 - Tap the right-hand or left-hand image.
 - Tap **Update** to switch between the right-hand and left-hand images.



If this control is hidden, tapping **More Controls** displays it.

When you switch between images in live imaging, the selected image inherits the settings of the previously selected image.

- 3 Adjust controls as needed. See "Controls in 2D" on page 101 and "Controls in Color" on page 110.
Tap **Dual** again to turn it off.

To view frames in the cine buffer in Dual

You can view independent cine buffers for side-by-side images in Dual.

- 1 Tap **Dual**. When **Dual** is on, the control is highlighted.



- 2 Do either of the following to make an image active:
 - Tap the right-hand or left-hand image.
 - Tap **Update** to switch between the right-hand and left-hand images.



If this control is hidden, tapping **More Controls** displays it.

- 3 Tap **FREEZE**.



4 Do any of the following:

- ▶ Drag the slider. Right moves forward, and left moves backward.




- ▶ Tap  or  to move forward or backward through frames one at a time. The current frame number appears on the touch panel. The frame number changes as you move forward or backward.
- ▶ Tap the right-hand or left-hand image, or tap **Update** to switch between the right-hand and left-hand images.

Note When you adjust an image in frozen Dual mode, the image may display incorrectly. To resolve this issue, pan the image or drag the cine buffer slider.

Doppler

- 1 Tap **Doppler**.

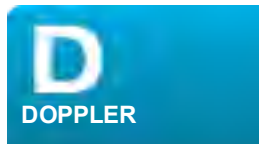

The D-line appears.



2 Do any of the following as needed:

- Adjust controls as needed. See "Controls in Doppler" on page 120.
- Position the D-line by dragging.
- Adjust the gate size by pinching or spreading.
- Correct the angle: Tap the double-headed arrows on the D-line, and then do either of the following:
 - Tap **+60°**, **0°**, or **-60°**.
 - Drag the round handle circularly.

Note You can also correct the angle by using the Angle Correct control. See "Controls in Doppler" on page 120.

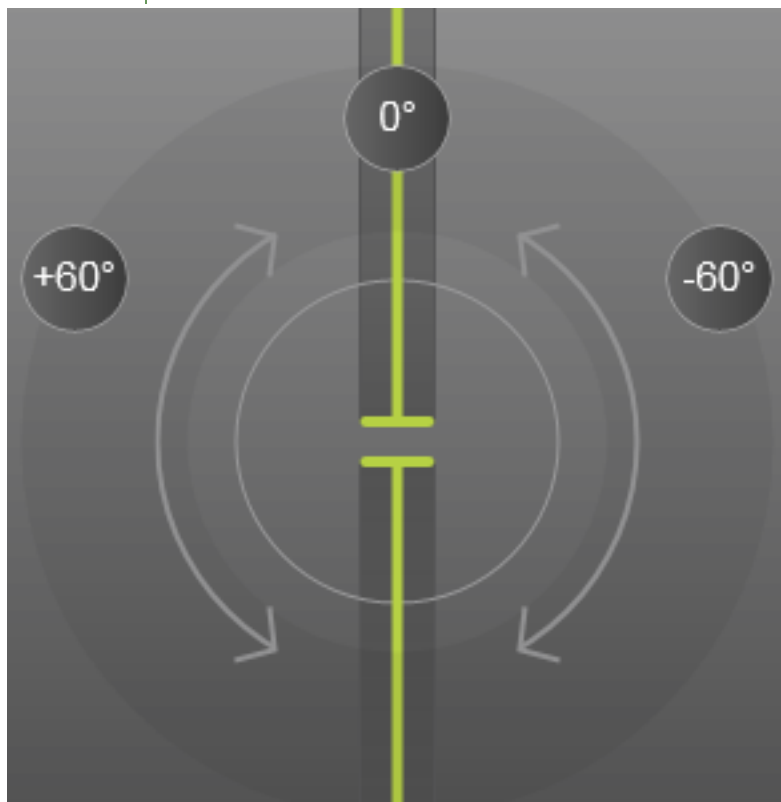


Figure 4-1 Correcting the D-line angle

- To steer the D-line, do one of the following:

- Swipe left or right.
- Tap **Steering**.




- 3 To display the spectral trace, do the following:

- a. Tap **Start** or **Update**.



- b. Do any of the following:

- Tap **Stop** and **Start** to stop and start the spectral trace.
- Tap **Sweep Speed** and then tap a sweep speed: **Fast**, **Medium**, or **Slow**.

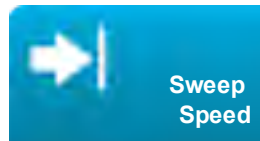

If this control is hidden, tapping **More Controls** displays it.

- Activate the trace or the 2D image by tapping them, or tap **Update** to toggle between the trace and the 2D pane.

If this control is hidden, tapping **More Controls** displays it.


- Adjust controls as needed. See "Controls in Doppler" on page 120.
-

M Mode

- 1 Tap **M Mode**.



The M-line appears.

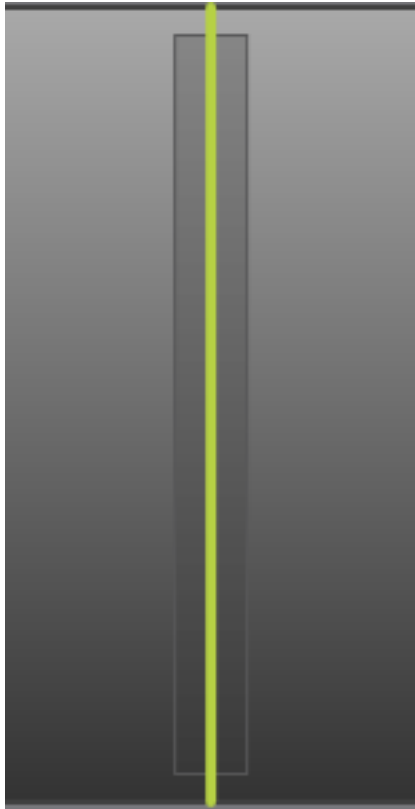

2 Drag the M-line to the desired position.

3 To display the M Mode trace, do the following:

a. Adjust the depth as needed. See "Adjusting depth and gain" on page 135.

b. Tap **Start**.

If Simultaneous M Mode is off, you can alternatively tap **Update** to display the M Mode trace.



If this control is hidden, tapping **More Controls** displays it.

c. Do any of the following:

▸ Tap **Stop** and **Start** to stop and start the M Mode trace.


▸ Tap **Simultaneous** to turn Simultaneous M Mode on or off.



If this control is hidden, tapping **More Controls** displays it.

For more information about Simultaneous M Mode, see "Controls in M Mode" on page 116.

▸ Tap **Sweep Speed** and then tap a sweep speed: **Fast**, **Medium**, or **Slow**.

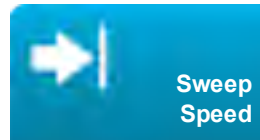

If this control is hidden, tapping **More Controls** displays it.

▸ Activate the trace or the 2D image by tapping them (when Simultaneous M Mode is off), or tap **Update** to toggle between the trace and the 2D pane (or to toggle display of 2D and M Mode controls and calculations).



If this control is hidden, tapping **More Controls** displays it.

4 Adjust controls as needed. See "Controls in M Mode" on page 116.

Imaging controls

The imaging controls let you adjust the image and related parameters, label, zoom, and more.

For each imaging mode, you can place your favorite and most-used controls in the Controls bar, located on the right side of the touch panel. You can place controls for the frozen image as well as for the live image. The remaining controls are still accessible in the More Controls window.

Available controls depend on the imaging mode, system configuration, and whether the image is live or frozen.

To access a control in the More Controls window

- 1 Tap **More Controls**.
- 2 If necessary, tap the imaging mode (for example, **2D**) at the top of the More Controls window. (The control for the active imaging mode is blue.)



Multiple imaging modes are available if one of them includes another. For example, because 2D is a subset of M Mode, you can choose from both 2D and M Mode controls when scanning in M Mode.

- 3 Tap the control.
- 4 If necessary, close the control: tap the X in the upper left corner of the control, or tap anywhere in the **More Controls** window.
To close the More Controls window, tap **More Controls**, or tap anywhere outside the More Controls window.

To place controls in the Controls bar

- 1 Start the imaging mode whose controls you want to place (see "Scanning" on page 88).
- 2 If placing controls for frozen imaging, freeze the image (see "To freeze the image" on page 137).
- 3 Tap **More Controls**.
The More Controls window appears.

- 4 Do the following for each imaging mode available:
- ▶ At the top of the window, tap the imaging mode (for example, **2D**).



Multiple imaging modes are available if one of them includes another. For example, because 2D is a subset of M Mode, you can choose from both 2D and M Mode controls when scanning in M Mode.

- ▶ Drag each control to the Controls bar, into the desired position.
You can drag the control up or down to another position, or you can drag it back into the More Controls window.
You can place up to seven controls in the Controls bar.

To close the More Controls window, tap **More Controls**, or tap anywhere outside the More Controls window.

Related topics

Controls in Color	110
Controls in Doppler	120
Controls in M Mode	116
Controls in 2D	101
Presets settings	66

Controls in 2D

To specify which controls appear in the Controls bar, see "Imaging controls" on page 99. Some controls appear only when the image is frozen. See "Controls in frozen 2D" on page 108.




Turns Tissue Harmonic Imaging (THI) on and off.

Tap this control to turn it on (highlighted icon) or off (dimmed icon).

Available in only certain exam types.

Available in	
Live	Frozen
✓	

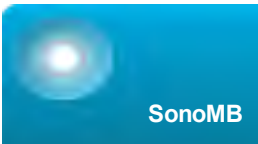


Turns SonoMB multi-beam imaging technology on and off.

SonoMB technology enhances the 2D image by viewing a target from multiple angles and then merging or averaging the data to improve overall image quality and to reduce noise and artifacts.

Available only in certain exam types. Not available with Simultaneous M Mode.

Tap this control to turn it on (highlighted icon) or off (dimmed icon).

Available in	
Live	Frozen
✓	

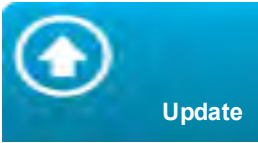


Updates the image. In Dual, tapping the control switches the active side from left to right or right to left. In Doppler or M Mode, tapping the control toggles between the trace and the 2D pane. If Simultaneous M Mode is on, tapping this control toggles display of 2D and M Mode controls and calculations.

Available in	
Live	Frozen
✓	✓



Enhances linear structures within a selected angle range and can facilitate needle guidance. See "Visualizing needles" on page 140.

Available in	
Live	Frozen
✓	

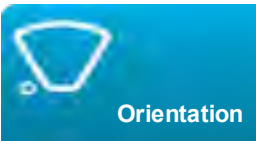

Sets the image orientation, allowing you to align the image with the transducer.
Tap to select the orientation: **Upper Right, Upper Left, Lower Left, Lower Right.**

Available in	
Live	Frozen
✓	



Optimizes the image by applying a specific group of settings such as focal zones, aperture size, frequency (center and bandwidth), line density, and waveform.

Note | The appearance and number of optimization settings for this button are determined by the selected transducer and exam type.

In general, use the Average optimization (middle or left setting). Use the right-hand (Difficult) optimization if the exam requires additional penetration. If three optimizations are available and resolution is more important than penetration, use the left-hand (Easy) optimization. Tap to select another optimization setting.

Available in	
Live	Frozen
✓	



Displays side-by-side 2D images, for which you can adjust some controls (for example, Depth and Gain) independently.

Tap this control to turn it on (highlighted icon) or off (dimmed icon).

With Dual on, tapping the right-hand or left-hand image (or tapping **Update**) makes the image active.

See "Dual" on page 88.

Available in	
Live	Frozen
✓	✓



Sets levels for the **AutoGain** control: adjusting to the positive side brightens images when you tap **AutoGain**, adjusting to the negative side makes images darker when you tap **AutoGain**. See "Gain" on page 136. The control is only available if **AutoGain** is on (see "Gain" on page 136). Tap this control and then drag the slider (the range is **-6** to **+6**) or tap **+3**, **0**, or **-3**.

The **AutoGain Adjust** control is disabled when you perform any action that turns off **AutoGain**, such as changing depth.


Available in	
Live	Frozen
✓	



Specifies the sector width. Tap this control and then tap **Narrow**, **Medium**, or **Full**.

Available only in the Cardiac exam type for certain transducers

Available in	
Live	Frozen
✓	

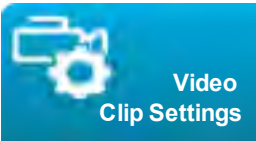

See "Saving video clips" on page 171.

Available in	
Live	Frozen
✓	✓




Turns needle guidelines on and off. For more information, including a list of transducers that are needle-guide capable and important safety information, see the transducer's user guide and the *FUJIFILM SonoSite Bracket and Needle Guide User Guide*.

Available in	
Live	Frozen
✓	



Displays ECG trace options. ECG is available only on the cardiac exam type. See "ECG" on page 174.

Available in	
Live	Frozen
✓	

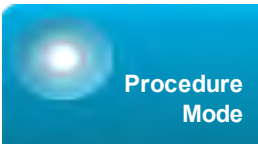


Allows you to view the image uninterrupted; for example, during a needle insertion procedure. When this control is enabled, the following are disabled:

- ▶ Dialog boxes or other system elements that block the image
- ▶ Sleep mode and automatic power-off

Tap this control to turn it on (highlighted icon) or off (dimmed icon).


Available in	
Live	Frozen
✓	

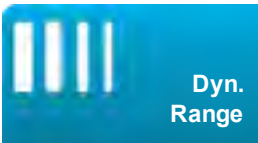

Adjusts the grayscale range. Tap the control and then tap **Up** or **Down** to select a setting: -3,-2,-1, 0, +1,+2,or +3. The selected setting is applied to both 2D and M Mode.

The positive range increases the number of grays displayed, and the negative range decreases the number of grays displayed.

Available in	
Live	Frozen
✓	✓

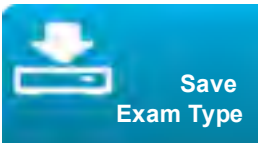

See "Customizing exam types" on page 70.

Available in	
Live	Frozen
✓	✓



Measures the distance from the skin line to a specified point on the image. For instructions, see "Measuring" on page 177.

Available in	
Live	Frozen
✓	✓



Provides tools that let you place text, pictographs, and arrows onto an image. See "Labeling images" on page 143.

Available in	
Live	Frozen
✓	✓



Magnifies the image. See "Zooming in on the image" on page 138.

Available in	
Live	Frozen
✓	✓



Turns the centerline graphic on or off. The centerline graphic aligns with the center mark of the transducer (if present) and serves as a reference mark for the center of the displayed image. This reference is helpful in maintaining alignment of the transducer with the anatomy of interest. This reference does not imply alignment with objects other than the transducer and image.

When performing out-of-plane procedures, the user should take care to understand the alignment of a needle with respect to the transducer. Needle and transducer motion affect intersection with the centerline graphic; if the needle is tilted outside the plane through the centerline perpendicular to the scan plane, the needle will not intersect the centerline graphic.

Tap this control to turn it on or off.

Available only with certain transducers. Not available during zoom mode or when guidelines are displayed (Needle Guide).


Available in	
Live	Frozen
✓	

Related topics

2D.....	82
Imaging controls.....	99
Controls in Color.....	110
Controls in Doppler.....	120
Controls in M Mode.....	116
Scanning.....	88

Controls in frozen 2D

The following controls are available only in frozen 2D.

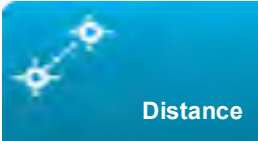

Measures the distance between two points.

See "Measuring" on page 177.



Uses an ellipse to measure a region's circumference or area.

See "Measuring" on page 177.




Lets you trace manually to measure an irregular shape.

See "Measuring" on page 177.



See "About calculations" on page 183.

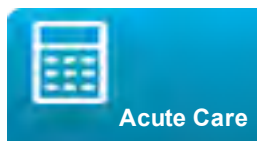

The Acute Care calculations are part of the licensed Acute Care feature. See "Acute Care calculations" on page 217.


Controls in Color

To specify which controls appear in the Controls bar, see "Imaging controls" on page 99. Some controls appear only when the image is frozen. See "Controls in frozen Color" on page 115.



Allows you to select **Color**, Color Power Doppler (**CPD**), or **Variance**. On the P21xp transducer in the Cardiac exam type, **Variance** is available instead of **CPD**. **CPD** shows amplitude and is useful for very low flow states. **Variance** displays a color map that shows regurgitant jet turbulence. The highest velocities appear as green.

Available in	
Live	Frozen
✓	

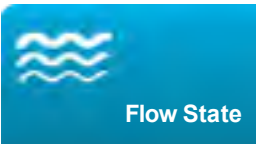

Lets you adjust the sensitivity of color parameters or settings to enhance the flow state. The system initially selects a setting based on the exam type.

- ▶ **High** optimizes for high flow states such as cardiac. The High setting minimizes flash artifacts.
- ▶ **Medium** optimizes for medium flow states such as abdominal or arterial flow.
- ▶ **Low** optimizes for low flow states such as breast or musculoskeletal.

For a more precise setting, adjust the Scale control.

Available in	
Live	Frozen
✓	



Sets the pulse repetition frequency (PRF). Tap this control, and then tap **Up** or **Down**.

Available in	
Live	Frozen
✓	



Helps to filter out echoes from low frequency signals.

Tap to select the strength of the filter: **Low**, **Med**, or **High**.

Available in	
Live	Frozen
✓	



(Linear transducers only)

Determines the angle of the color box (see "Color" on page 90), saving work in repositioning the transducer.

If adding Doppler, see "Controls in Doppler" on page 120.

Available in	
Live	Frozen
✓	





Switches the displayed direction of the flow, helping reduce the need of repositioning the transducer.

Tap this control to turn it on (highlighted icon) or off (dimmed icon).

Invert is unavailable when **Color** is set to **CPD**.

Available in	
Live	Frozen
✓	✓

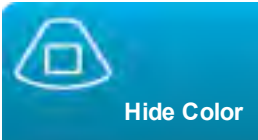

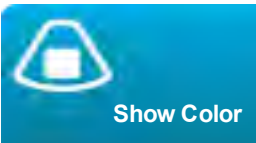


Shows and hides color. For example, you may want to hide color temporarily to view vascular plaque.

Available in	
Live	Frozen
✓	✓




Magnifies the image. See "Zooming in on the image" on page 138.

Available in	
Live	Frozen
✓	✓

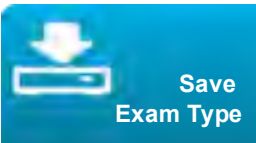

See "Customizing exam types" on page 70.

Available in	
Live	Frozen
✓	✓

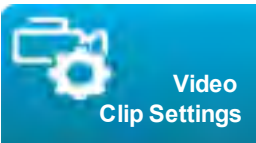

See "To set video clips controls" on page 171.

Available in	
Live	Frozen
✓	



Measures the distance from the skin line to a specified point on the image. For instructions, see "Measuring" on page 177.

Available in	
Live	Frozen
✓	✓



Provides tools that let you place text, pictographs, and arrows onto an image. See "Labeling images" on page 143.


Available in	
Live	Frozen
✓	✓

Related topics

Imaging controls	99
Controls in Doppler.....	120
Controls in M Mode.....	116
Controls in 2D.....	101
Color.....	85
Scanning.....	88

Controls in frozen Color

The following controls are available only in frozen Color.

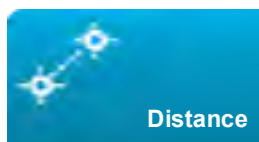

Measures the distance between two points.

See "Measuring" on page 177.



Uses an ellipse to measure a region's circumference or area.

See "Measuring" on page 177.




Lets you trace manually to measure an irregular shape.

See "Measuring" on page 177.



See "About calculations" on page 183.

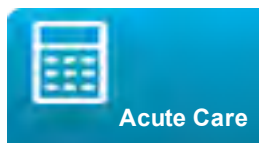


The Acute Care calculations are part of the licensed Acute Care feature. See "Acute Care calculations" on page 217.

Controls in M Mode

In M Mode imaging, you can adjust the following controls in addition to most of the 2D controls (the 2D controls are available in M-line--see "M-line" on page 83 and "Controls in 2D" on page 101). To specify which controls appear in the Controls bar, see "Imaging controls" on page 99. Some controls appear only when the image is frozen. See "Controls in frozen M Mode" on page 120.



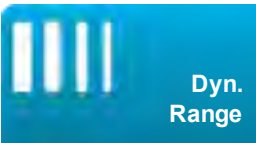

Adjusts the grayscale range. Tap the control and then tap **Up** or **Down** to select a setting: -3,-2,-1, 0, +1,+2, or +3. The selected setting is applied to both 2D and M Mode.

The positive range increases the number of grays displayed, and the negative range decreases the number of grays displayed.


Available in	
Live	Frozen
✓	✓



Displays the ECG trace. ECG is available only on the cardiac exam type. See "ECG" on page 174.


Tap this control to display ECG options.

Available in	
Live	Frozen
✓	

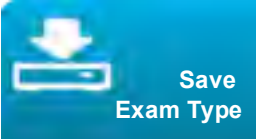

See "Customizing exam types" on page 70.

Available in	
Live	Frozen
✓	✓

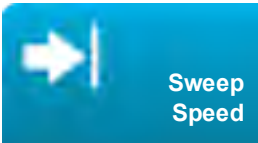

Available when displaying the M Mode trace. Sets the sweep speed. Tap this control and then select the speed: **Slow**, **Med**, or **Fast**.

Available in	
Live	Frozen
✓	



Lets you control the format of the 2D and sweeping images. Tap this control, and then tap the specified format:

- ▶ 1/3 2D, 2/3 Sweep
- ▶ 1/2 2D, 1/2 Sweep
- ▶ 2/3 2D, 1/3 Sweep
- ▶ Side by Side
- ▶ Full 2D, Full Sweep

You can adjust the images independently of each other. To make an image active, tap it, or tap **Update**.


Note | The format **Full 2D, Full Sweep** is not available during Simultaneous M Mode.

Available in	
Live	Frozen
✓	✓



Provides tools that let you place text, pictographs, and arrows onto an image. See "Labeling images" on page 143.

Available in	
Live	Frozen
✓	✓

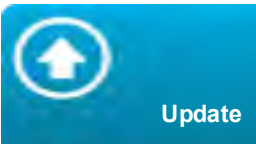

Tap the control to toggle between the trace and the 2D pane. If Simultaneous M Mode is on, tapping this control toggles display of 2D and M Mode controls and calculations.

Available in	
Live	Frozen
✓	✓



Turns Simultaneous M Mode on and off.

Tap this control to turn it on (highlighted icon) or off (dimmed icon).

Available in only certain exam types and transducers. Not available with SonoMB technology.


Available in	
Live	Frozen
✓	

Related topics


M Mode	83
Imaging controls	99
Controls in Color	110
Controls in Doppler	120
Controls in 2D	101
Scanning	88
M-line	83

Controls in frozen M Mode

The following controls are available only in frozen M Mode.

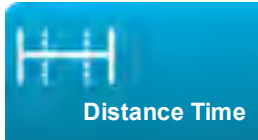

Available when displaying the M Mode trace. Provides calipers to measure distance in centimeters and measure time in seconds. See "Basic measurements in M Mode" on page 180.

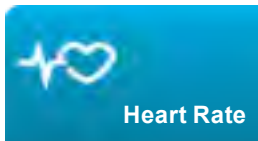

Available when displaying the M Mode trace. Provides calipers to measure heart rate. See "Basic measurements in M Mode" on page 180.

Controls in Doppler

In Doppler imaging, you can adjust controls in D-line (see "Controls in D-line" on the facing page) and in scrolling Doppler (see "Controls in scrolling Doppler" on page 127). To specify which controls appear in the Controls bar, see "Imaging controls" on page 99.

Controls in D-line

Some controls in D-line appear only when the image is frozen. See "Controls in frozen D-line" on page 125.





Select **PW** (pulsed wave Doppler), **CW** (continuous wave Doppler), or **PW TDI** (Tissue Doppler Imaging).

CW and **PW TDI** are available only in the Cardiac exam type.

Available in	
Live	Frozen
✓	



Linear transducers only. Determines the angle of the D-line.

Available in	
Live	Frozen
✓	



Available only in PW Doppler.

Corrects the angle. Tap this control, and then drag the slider or select a setting: **0°**, **+60°**, or **-60°**.

Note You can also adjust the angle by tapping the double-headed arrows on the D-line. See "Scanning" on page 88.

Available in	
Live	Frozen
✓	✓




Tap **Up** to increase the gate size. Tap **Down** to decrease the gate size.

Available in	
Live	Frozen
✓	




Magnifies the image. See "Zooming in on the image" on page 138.

Available in	
Live	Frozen
✓	✓

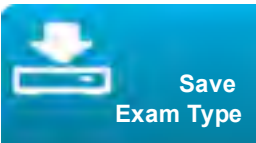

See "Customizing exam types" on page 70.

Available in	
Live	Frozen
✓	✓

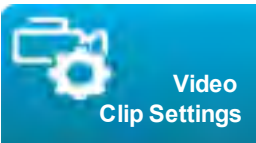

See "To set video clips controls" on page 171.

Available in	
Live	Frozen
✓	




Lets you control the format of the 2D and sweeping images. Tap this control, and then tap the specified format:

- ▶ 1/3 2D, 2/3 Sweep
- ▶ 1/2 2D, 1/2 Sweep
- ▶ 2/3 2D, 1/3 Sweep
- ▶ Side by Side
- ▶ Full 2D, Full Sweep

You can adjust the images independently of each other. To make an image active, tap it, or tap **Update**.

Note | The format **Full 2D, Full Sweep** is not available during Simultaneous M Mode.

Available in	
Live	Frozen
✓	✓

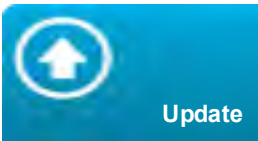

Updates the image. In Dual, tapping the control switches the active side from left to right or right to left. In Doppler or M Mode, tapping the control toggles between the trace and the 2D pane.

Available in	
Live	Frozen
✓	✓



Measures the distance from the skin line to a specified point on the image. For instructions, see "Measuring" on page 177.

Available in	
Live	Frozen
✓	✓




Provides tools that let you place text, pictographs, and arrows onto an image. See "Labeling images" on page 143.

Available in	
Live	Frozen
✓	✓

Controls in frozen D-line

The following controls are available only in frozen D-line.

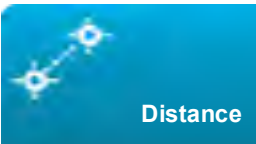

Measures the distance between two points.
See "Measuring" on page 177.



Uses an ellipse to measure a region's circumference or area.

See "Measuring" on page 177.




Lets you trace manually to measure an irregular shape.

See "Measuring" on page 177.



See "About calculations" on page 183.

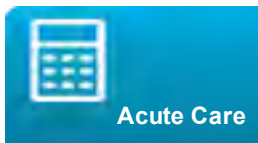

The Acute Care calculations are part of the licensed Acute Care feature. See "Acute Care calculations" on page 217.

Controls in scrolling Doppler

Some controls appear only when the image is frozen. See "Controls in frozen scrolling Doppler" on page 134.





Select **PW** (pulsed wave Doppler), **CW** (continuous wave Doppler), or **PW TDI** (Tissue Doppler Imaging).

CW and **PW TDI** are available only in the Cardiac exam type.

Available in	
Live	Frozen
✓	



Sets the pulse repetition frequency (PRF). Tap this control, and then tap **Up** or **Down**.

Available in	
Live	Frozen
✓	



Adjusts the Doppler speaker volume. Tap this control, and then tap **Up** to increase the volume or **Down** to decrease the volume.

Available in	
Live	Frozen
✓	




Tap **Up** to increase the gate size. Tap **Down** to decrease the gate size.

Available in	
Live	Frozen
✓	




Helps to filter out echoes from low frequency signals.
Tap to select the strength of the filter: **Low**, **Med**, or **High**.

Available in	
Live	Frozen
✓	

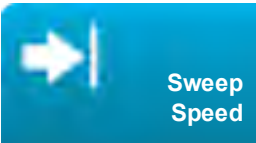


Sets the sweep speed. Tap this control and then select the speed: **Slow**, **Med**, or **Fast**.

Available in	
Live	Frozen
✓	

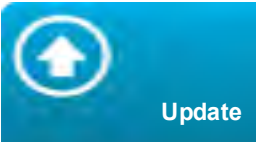

Tap the control to toggle between the D-line and the trace.

Available in	
Live	Frozen
✓	✓

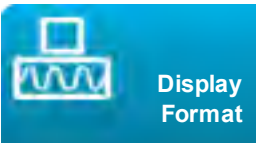

Lets you control the format of the 2D and sweeping images. Tap this control, and then tap the specified format:

- ▶ 1/3 2D, 2/3 Sweep
- ▶ 1/2 2D, 1/2 Sweep
- ▶ 2/3 2D, 1/3 Sweep
- ▶ Side by Side
- ▶ Full 2D, Full Sweep

You can adjust the images independently of each other. To make an image active, tap it, or tap **Update**.

Note | The format **Full 2D, Full Sweep** is not available during Simultaneous M Mode.

Available in	
Live	Frozen
✓	✓



Sets the baseline position. Tap this control, and then tap **Up** or **Down**.

Available in	
Live	Frozen
✓	✓



Switches the displayed direction of the flow, helping reduce the need of repositioning the transducer.

Tap this control to turn it on (highlighted icon) or off (dimmed icon).

Available in	
Live	Frozen
✓	✓



Corrects the angle. Tap this control, and then drag the slider or select a setting: **0°**, **+60°**, or **-60°**.

Note You can also adjust the angle by tapping the double-headed arrows on the D-line. See "Scanning" on page 88.

Available in	
Live	Frozen
✓	✓



Specifies the Doppler trace setting for measurements. Tap this control, and then tap **Off**, **Peak** (time average peak), or **Mean** (time average mean).

After you tap **Peak** or **Mean** in live imaging, you can choose between **Above**, **Below**, or **All** to control the display relative to the baseline. While **Peak** and **Mean** are available both during live imaging and when the image is frozen, the settings **Above**, **Below**, and **All** are available during live imaging only.


Available in	
Live	Frozen
✓	✓



Displays the ECG trace. ECG is available only on the cardiac exam type. See "ECG" on page 174.

Tap this control to display ECG options.

Available in	
Live	Frozen
✓	

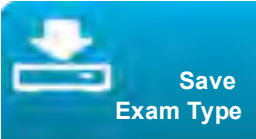

See "Customizing exam types" on page 70.

Available in	
Live	Frozen
✓	✓



Provides tools that let you place text, pictographs, and arrows onto an image. See "Labeling images" on page 143.

Available in	
Live	Frozen
✓	✓

Controls in frozen scrolling Doppler

The following controls are available only in frozen scrolling Doppler.



Displays velocity measurements. See "Basic measurements in Doppler " on page 180.




See "To measure time duration (Doppler)" on page 181.



Lets you trace manually to measure an irregular shape.


See "Measuring" on page 177.

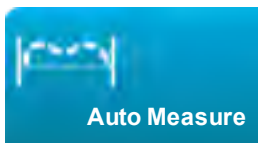

See "To measure automatically (Doppler)" on page 182.



See "About calculations " on page 183.

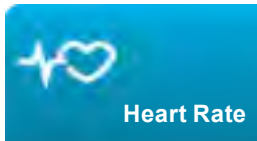

Provides calipers to measure heart rate.

Adjusting depth and gain

Depth

Depth refers to the depth of display. You can adjust the depth in all imaging modes except M Mode trace and Doppler trace. The vertical depth scale on the touch panel lists all available depth levels for the current transducer. The intervals depend on the active transducer.

To adjust the depth

As you adjust the depth, the depth value appears in a rectangle in the lower right corner of the image area or upper right corner if the image orientation is upside down.

Note | The depth value in the lower right corner of the clinical monitor is always the total acquired depth of the unzoomed image. When you zoom, this value stays the same.

- Do either of the following on the touch panel:
 - Tap a marker on the depth scale.
 - Drag the depth scale.



Gain


Gain refers to amplifying the intensity of the returning sound waves on the screen display. Increasing the gain brightens the image. Decreasing the gain darkens the image.

There are two ways to adjust gain: automatically and by using the gain sliders.

To adjust gain automatically

- ▶ Tap **AutoGain**.

Note The **AutoGain** control is present only if enabled as a custom control. See "User profile settings" on page 71.

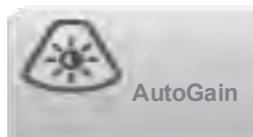

The system automatically balances the gain each time you tap this control. You can adjust the Auto Gain level by using the **AutoGain Adjust** control: see "Controls in 2D" on page 101.



To adjust gain by using the gain sliders

- ▶ Drag a slider left or right, to decrease or increase the gain respectively:
 - ▶ In Color or Doppler, drag the **C** (Color) or the **D** (Doppler) gain slider at the bottom of the touch panel.
 - ▶ In 2D or M Mode, tap the gain slider or the **M** (M Mode) gain slider at the bottom of the touch panel. Two additional sliders appear.
 - ▶ The upper slider adjusts the near gain of the 2D image, controlling the display intensity in the near field (nearer to the transducer).
 - ▶ The middle slider adjusts the far gain of the 2D image, controlling the display intensity in the far field (further from the transducer).
 - ▶ The lower slider adjusts the overall gain.

Note Near and Far correspond to the time gain compensation (TGC) controls on some other ultrasound systems.

Freezing, viewing frames, and zooming

Freezing the image

Before performing certain tasks, such as viewing the cine buffer or measuring, you stop live imaging or *freeze* the image.

When you freeze the image, you can perform calculations. See "About calculations" on page 183.

To freeze the image


- 1 Do either of the following:

- ▶ Tap **FREEZE**.

FREEZE

- ▶ With two or more fingers, double-tap the touch panel.
Additional features appear on the image.

Note You can also configure the footswitch to freeze (see "User profile settings" on page 71).

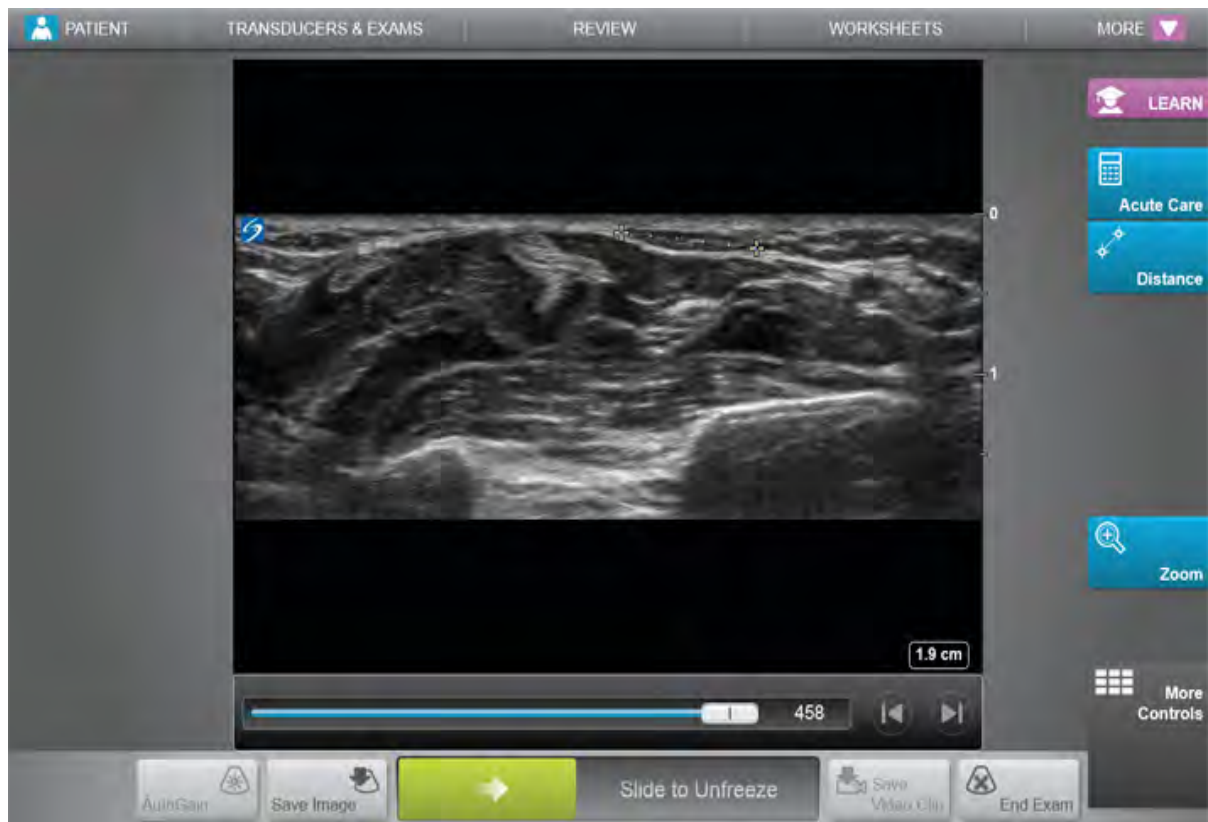

To unfreeze the image

- Do either of the following:
 - Drag or swipe the **Slide to Unfreeze** slider to the right.
 - With two or more fingers, double-tap the touch panel.

Related topics

About calculations 183

Viewing the cine buffer

While imaging, X-Porte always retains a certain number of frames in the *cine buffer*. You can move forward and backward in the cine buffer.

The system clears the cine buffer when you unfreeze the image.

You can view frames in the cine buffer in Dual. See "Dual" on page 91.

To view frames in the cine buffer

- 1 Freeze the image.
- 2 Do either of the following:
 - Drag the slider. Right moves forward, and left moves backward.
Simultaneous M Mode: While the slider bar represents the complete set of collected imaging data, the scrolling display shows only one cycle of imaging data at a time. The green CINE marker (▲) indicates a single column on the scrolling display, which corresponds in time to the displayed image.



- Tap or to move forward or backward through frames one at a time.
Simultaneous M Mode: Tapping these controls moves the green CINE marker (▲), showing a column on the scrolling display and its corresponding image.
Other modes: The current frame number appears on the touch panel. The frame number changes as you move forward or backward.

Zooming in on the image

You can zoom in on a 2D image in 2D, M Mode, Doppler, and Color imaging. You can freeze or unfreeze the image or change the imaging mode while zooming.



When you zoom in on an image, the Zoom icon appears on the image.

To zoom in live imaging

- 1 Tap **Zoom**.



A zoom box appears.

If this control is hidden, tapping **More Controls** displays it.

- 2 Size and position the zoom box over the area that you want to zoom:




- To size, pinch or spread the box, or drag.
- To position, drag the box.

- 3 Double-tap in the zoom box, or tap **Zoom**.



To exit zoom, double-tap in the zoom box, or tap **Unzoom**.

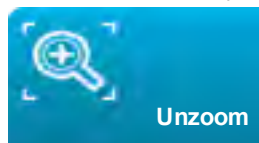

To zoom in Color


- 1 Position the Color box over the area you want to zoom.
- 2 Tap **Zoom**.



If this control is hidden, tapping **More Controls** displays it.
The area in the box is magnified.

Note | To change zoom magnification, you must first shrink your color box.

To exit zoom, double-tap in the zoom box, or tap **Unzoom**.

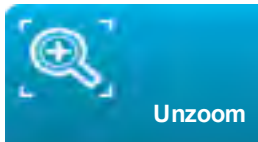

To zoom in on a frozen image

- 1 Tap **Zoom**.



If this control is hidden, tapping **More Controls** displays it.

- 2 Do any of the following:
 - Tap **4x** or **2x**. To unzoom, tap **Off**.
 - Drag the slider. To unzoom, drag the slider to **1.0**.
- 3 (Optional) Pan the image by dragging left, right, up, or down.

Note | You cannot pan the image if measurements, text, arrows, or labels exist on the image.

Visualizing needles

WARNING | To avoid incorrect needle placement when Needle Profiling is on:

- Using movement and fluid injection, verify the needle-tip location and trajectory. Needle Profiling enhances linear structures within a selected angle range on the ultrasound plane. Linear structures outside the selected angle range or the ultrasound plane—such as a bent needle—may be less apparent.

- Note that linear structures are enhanced only in an outlined portion of the image. The area outside the outline remains unchanged.

X-Porte features Steep Needle Profiling technology, which can facilitate needle guidance during catheter placement and nerve-block procedures. This technology enhances linear structures within an outlined area on the screen. Linear structures are best enhanced when perpendicular to the angle guide.

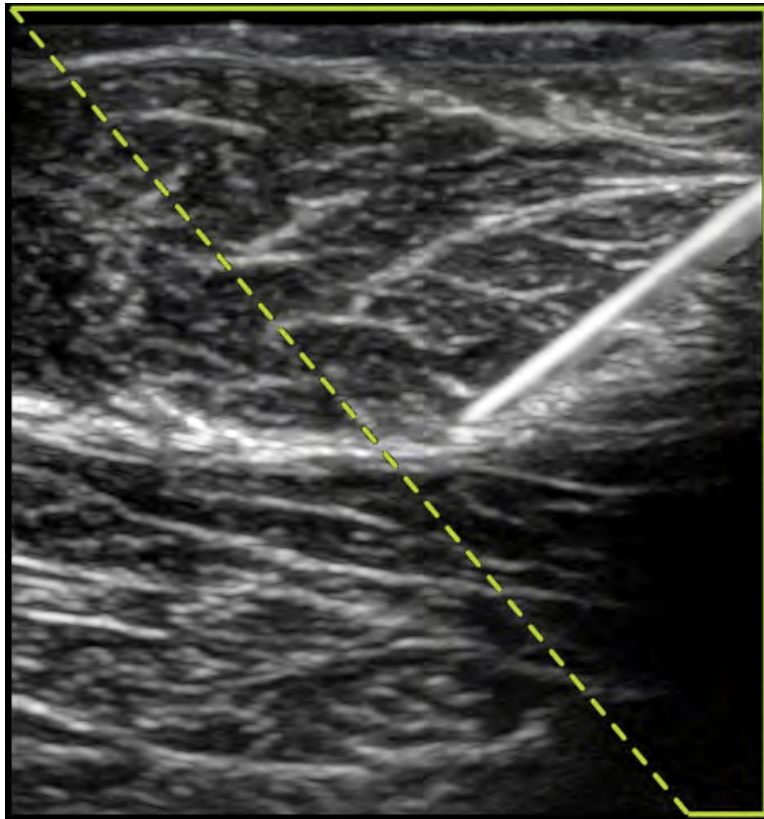


Figure 4-2 Image with Steep Needle Profiling on. The area within the green trapezoidal outline is the enhancement area.

Steep Needle Profiling is available in 2D full-screen imaging only and on the following exam types.

Transducer	Arterial	Breast	Musculoskeletal	Nerve	Small Parts	Venous
HFL38xp	✓	✓	✓	✓	✓	✓
HFL50xp		✓	✓	✓	✓	
HSL25xp	✓		✓	✓		✓
L25xp	✓		✓	✓		✓
L38xp	✓			✓	✓	✓

To use Steep Needle Profiling (basic steps)

- 1 In 2D imaging, tap **Needle Profiling**.



If this control is hidden, tapping **More Controls** displays it.

- 2 Ensure that the target is within the enhancement area (that is, within the outline) and that the needle, when inserted, will approach the angle guide (dashed line) perpendicularly.

- ▶ Tap Left or Right if you need to flip the image horizontally.



- ▶ Tap **Shallow**, **Medium**, or **Steep** to select a needle angle.
The needle angle sets the angle guide (dashed angled line).
Linear transducer: Use whichever setting best provides a perpendicular intersection with the angle guide. Within the enhancement area, the more perpendicular that a linear structure is to the angle guide, the more it is enhanced. Similarly, the less perpendicular (and more parallel) that a linear structure is to the angle guide, the less it is enhanced.

- 3 Insert the needle toward the angle guide.
- 4 (Optional) Tap **Off** to turn off Needle Profiling.
Temporarily turning off Needle Profiling can help you identify artifacts and other structures not of interest. Tap **On** to turn it on again.

Needle size and angle

Use a 17-gauge to 25-gauge needle (recommended). Enhancement results can depend on the type and brand of needle used. For more information, consult the medical literature on needle visibility in ultrasound-guided procedures.

You can angle the needle up to 50° from the transducer surface. Beyond 50°, the needle may be less enhanced. (Needle Profiling has little or no benefit to out-of-plane procedures. Needle Profiling is intended for in-plane procedures only.)

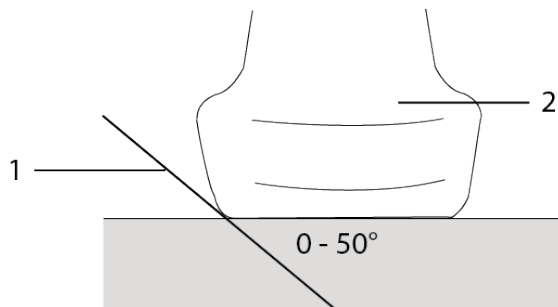


Figure 4-3 Needle size and angle: 1. Needle, 2. Transducer

For best results, angle the needle only up to 50° from the transducer surface.

Additional recommendations

Avoid setting the gain too high when using Needle Profiling, as unnecessarily high gain can cause artifacts in the image. Also, respiratory and cardiac movement in the image may cause bright pulsating artifacts.

Labeling images

You can label live, frozen, or saved images. You can label in full-screen 2D, full-screen trace, dual, or duplex. You can place text (including predefined labels), arrows, and pictographs.

To set preferences for labels, including predefined text labels, see "Presets settings" on page 66.

Tip | If you select **Clear Labels on Unfreeze** in the Presets setup page, the system clears all labels from an image when you unfreeze the image.

Text

You can add text manually or add a predefined label.

Note | If you zoom or pan an image, the system removes text labels and arrows from the image.

To place text on an image

- 1 Tap **Label**.



If this control is hidden, tapping **More Controls** displays it.

- 2 Tap **Text** and do any of the following:




- ▶ To specify that text and predefined labels connect when you enter them, instead of being placed separately, tap . Tap  to insert text and predefined labels separately.
- ▶ To place text manually, tap in the field at the top of the form, type text, and then tap **Done**.



- ▶ To add one or more predefined labels, tap them and then tap **Done**.
Tip To replace a label you've added with another label, tap the added label in the field at the top of the form, and then tap the label with which you want to replace the added label.
- ▶ To place text manually and add a predefined label, type text in the field at the top of the form, tap in the field again, and then tap one or more predefined labels. Tap **Done**.
- ▶ To clear text or predefined labels, tap **Clear**.



- ▶ To undo changes and return to the image, tap **Cancel**.

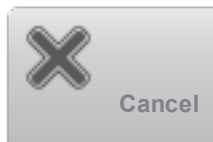


- ▶ To undo changes and return to 2D imaging, tap **2D**.


- 3 Drag the text label to the desired location.

4 Make changes as needed:

- To edit the text, select it in the **Label** list and tap **Edit**.

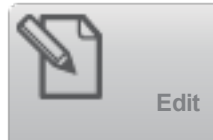

- To remove the text, select it in the **Label** list and tap **Delete**.

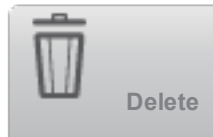

- To remove all text labels, tap **Remove All Text** in the **Label** list.

Arrows

You can add an arrow graphic to point out a specific part of the image. The system deletes arrows from the image when you zoom, pan, or unfreeze the image.


To place an arrow on an image

- 1 Tap **Label**.



If this control is hidden, tapping **More Controls** displays it.


- 2 Tap **Arrow**.

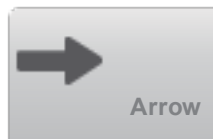

3 Position and orient the arrow:

- To position the arrow, drag it.
- To orient the arrow, drag the circle's edge.

To remove the arrow, select it and then tap **Delete** in the **Arrow** list.

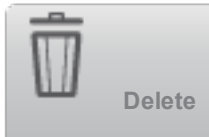

Pictographs

The pictograph set available depends on transducer and exam type. Each pictograph you place includes a transducer orientation icon.

To place a pictograph on an image

1 Tap **Label**.



If this control is hidden, tapping **More Controls** displays it.

2 Tap **Picto**.



3 Tap the desired pictograph, and then tap **Done**.




4 Drag the pictograph to the desired location.

5 Position and orient the transducer orientation icon:

- To position the icon, drag it.
- To orient the icon, drag the circle's edge.

To remove the pictograph, select it, and then tap **Delete** in the **Picto** list.

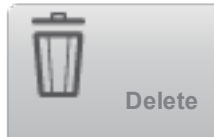


Setting the home position

The home position is the position in which the system places labels by default.

To reset the home position

- 1 Place text on an image.
- 2 Drag the label to the desired home position.
- 3 Tap **Set Home** in the **Label** list.

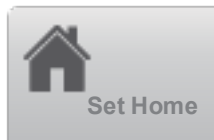

Labeling during review


You can label images during review. You can also label a frame from a video clip during review.

To review images and video clips

- Tap **REVIEW**, and then do either of the following:



- Review the current exam.
- Review a completed exam: Tap **Patient List**. In the patient list, select the exam, and then tap **Thumbnails**.

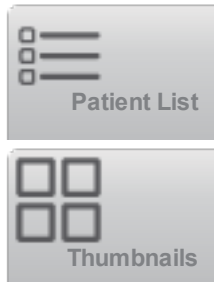


To label in review

- 1 In Review, tap the image or video clip you want to label:
To view the previous or next image or video clip, tap **Prev** or **Next**.



- 2 (Video clips only) Tap , and then drag the slider to the frame you want to label.
- 3 Tap **Labels**.

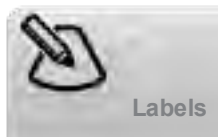


4 Do one of the following:

▸ Place the label:


- For text, see "To place text on an image" on page 144.
- For an arrow, see "To place an arrow on an image" on page 145.
- For a pictograph, see "To place a pictograph on an image" on page 146.

Note If you play a video clip, the system removes labels applied to the current frame.

- Cover an existing label: tap **Labels**, and then tap **Mark Out**. The system places a black rectangle on the image.

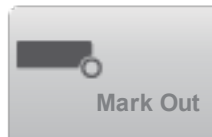


- To reposition the Mark Out box, drag it.
- To resize the Mark Out box, drag the round handle at the corner of the Mark Out box.
- To delete the Mark Out box, select the Mark Out box in the **Mark Out** list and then tap **Delete**.

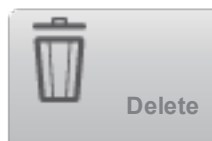

The system displays the image or video clip in full-screen mode.

5 Tap **Save**.

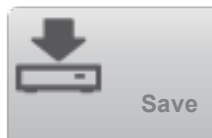

The system saves the labeled image to the exam.

6 Tap **Done** to exit labels and return to review.



Entering patient information

The patient form lets you enter identification, exam, and clinical details for the patient exam. This information appears in the patient report.

After you begin a new patient form, all images, video clips, and other information you save during the exam are linked to that patient.

You can enter patient information by:

- ▶ Entering the information manually. (See "Entering patient information manually" on the facing page.)
- ▶ Importing information from a DICOM worklist server. (See "Entering patient information from the worklist" on the facing page.)
- ▶ Scanning a patient bar code and obtaining patient information from a worklist server. (See "To enter patient information from the worklist with a bar code scanner" on the facing page.)
- ▶ Scanning a patient bar code to obtain the patient ID (See *Bar Code Scanner User Guide*).

Related topics

Connectivity settings	43
DICOM	46
Routing selections	74
Worksheets and reports	220

Editing patient information

You can edit patient information if the exam has not been archived or exported and if the information is not from a worklist.

To edit patient information

- 1 Tap **PATIENT**.



- 2 Make changes as desired.

- 3 Tap **Done**.



- 4 Do any of the following:
 - Tap **Modify** to save the changes to the current patient.
 - Tap **New** to start a new patient exam.
 - Tap **Cancel** to discard your changes.

Entering patient information manually

To enter patient information manually

- 1 End the previous exam (see "Ending the exam" on page 155).
- 2 Tap **PATIENT**.



- 3 Tap a text box, and fill in the fields. See "Patient form fields" on page 156.
- 4 Tap **Done**.



Entering patient information from the worklist

Using the worklist, you can import patient information from the Hospital Information System or Radiology Information System. (DICOM worklist is a licensed feature.)

The worklist is available after you do the following:

- Connect the system to the network
- Configure the system for DICOM transfer
- Select a worklist server for the active location

See "DICOM" on page 46.

To enter patient information from the worklist with a bar code scanner

You can specify that the system query the worklist for patient information when you scan a patient ID bar code.


- 1 On the User Profile setup page, select **Bar Code Auto Query** (see "User profile settings" on page 71).
- 2 Scan the patient's bar code.
The system queries the worklist and imports the information into the patient form.

To enter patient information from the worklist


- 1 Tap **PATIENT**.




- 2 Tap **Worklist**.

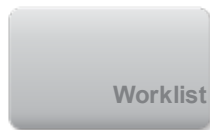

- 3 To sort the worklist, tap a column heading to sort by. By default, the worklist sorts in ascending order.
- 4 Do any of the following:
 - ▶ To select one or more procedures for the patient, tap the procedure or procedures. You can select multiple procedures only for the same patient.
 - ▶ To requery the server for an updated worklist, tap **Update**.

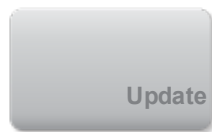


- 5 Tap **Select**.

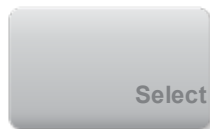

The system populates the patient form. (Patient information from the worklist is uneditable.)

To return to the patient form without selecting a procedure, tap **Back**.

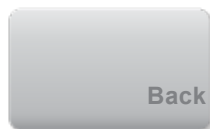

To view scheduled procedures

The patient information that you import from the worklist includes scheduled procedures.

- On the patient form, see the following lists under **Scheduled Procedures**:
 - Name** Scheduled procedures
 - Protocol** Protocol for the selected procedure
 - Meaning** Definition of the selected procedure

To change the procedure

You can select a procedure different from the scheduled procedure.

- Under **Performed Procedure** on the patient form, select the desired procedure from the **Code** list. The definition of the procedure appears in the **Meaning** field.
- If you want to modify the **Code** list, tap **Edit**, and do one of the following:

Note

Required fields are marked with asterisks.

 - Add an item: Tap **Add New**, and then fill in the **Code**, **Code Scheme Name**, and **Meaning** fields, and then tap **Save**.
 - Delete an item: Select the item, and then tap .

Updating and querying the worklist

The worklist automatically updates if set up for an automatic worklist query (see "DICOM" on page 46). You can also manually update the worklist, and you can query the worklist server for a matching patient procedure.

The following table shows the parameters used for worklist queries. You set parameters on the worklist configuration page (see "DICOM" on page 46).

Table 4-1 Query parameters


	Manual Patient Query	Manual Update from Worklist	Automatic Query Update
Patient Data	✓		
Date Range	✓	✓	✓
This Device Only		✓	✓
Automatic Query On/Off			✓

Table 4-1 Query parameters (continued)


	Manual Patient Query	Manual Update from Worklist	Automatic Query Update
Occurs Every			✓
Start Time			✓

To manually update the worklist

- 1 On the patient form, tap **Worklist**.

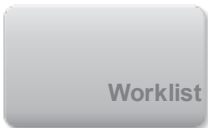


- 2 Tap **Update**.

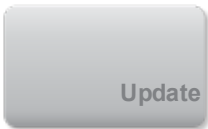

To perform a manual query


- 1 On a new patient form, do any of the following:
- ▶ Fill in any of the following fields for the query: **Last**, **First**, **Middle**, **Accession** , or **ID**. The query is on the characters you enter. For example, *Smith* returns *Smith*, *Smithson*, *Smithy*.
 - ▶ Under **Additional Worklist Query Parameters**, specify any of the following:
 - ▶ **Modality** Select a procedure type from the list. US (Ultrasound) is the default.
 - ▶ **Requested Procedure ID** Type a procedure ID.

- 2 Tap **Query**.

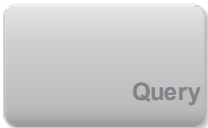

The worklist appears with the query results. The number of results is in the lower right screen. The time of the last query appears below the worklist.

- 3 Highlight the patient procedure you want, and then tap **Select**.

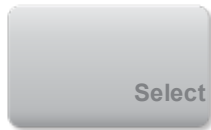


The system imports the information into the patient form.

Ending the exam


Note Although you cannot add images and video clips to a patient exam that is ended, you can automatically start a new patient exam that has the same patient information. See "To append images and video clips to a patient exam" on page 160.

When you end an exam, the system resets imaging settings to the default settings for the selected exam type.

- 1 Make sure that you have saved images and other information that you want to keep. (See "Saving images and video clips" on page 169.)
- 2 Do either of the following:
 - Tap **PATIENT**, and then tap **End Exam**.

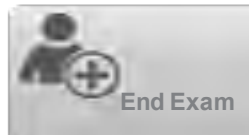

- Tap **End Exam** in imaging, in review, or in worksheets. The system returns to the startup location as specified in User Profile settings (see "User profile settings" on page 71).

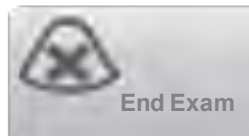

If **End Exam** isn't visible in imaging, specify that it appears as one of the four custom controls. See "User profile settings" on page 71.

Patient form fields

Field descriptions

The patient form fields available depend on exam type. In some fields, you can enter symbols and special characters. See "To enter text using the onscreen keyboard" on page 12.

Patient

- ▶ **Last, First, Middle** Patient name
- ▶ **ID** Patient identification number
 - Note** The Generate Patient ID option enables the system to auto-generate a unique patient ID to accommodate certain workflows. For more information, see "General preferences" on page 67.
- ▶ **Date of Birth**
- ▶ **Gender**
- ▶ **Accession** Enter number, if applicable
- ▶ **Indications** Enter desired text

Exam

- ▶ **Exam** The exam types available depend on transducer. See "Selecting a transducer and exam type" on page 16. An asterisk next to an exam type indicates that it is a system-defined exam type (see "Customizing exam types" on page 70).
- ▶ **BP** (Cardiac or Arterial exam) Blood Pressure
- ▶ **HR** (Cardiac or Arterial exam) Heart Rate. Enter the beats per minute. Saving the heart rate using a measurement overwrites this entry.
- ▶ **Height** (Cardiac exam) The patient height in feet and inches or meters and centimeters. (To change the units, see "Presets settings" on page 66.)
- ▶ **Weight** (Cardiac exam). The patient weight in pounds or kilos. (To change the units, see "Presets settings" on page 66.)
- ▶ **BSA** (Cardiac exam) Body Surface Area. Automatically calculated after you enter height and weight.
- ▶ **Twins** (Obstetrical exam) Select the Twins check box to display Twin A and Twin B measurements on the calculations menu and for access to Twin A and Twin B screens for previous exam data.
- ▶ **LMP or EDD** (Obstetrical or Gynecology exam) In an Obstetrical exam, select LMP or EDD and then enter either the date of the last menstrual period or the established due date. In a Gynecology exam, enter the date of the last menstrual period. The LMP date must precede the current system date.
- ▶ **Previous Exams** (Obstetrical exam) Displays fields for five previous exams. The date for a previous exam must precede the current system date. For twins, select Twin A/B to toggle between Twin A and Twin B screens. (If the Twin A/B control does not appear, tap **Back**, and make sure that the Twins check box is selected.)
- ▶ **NT Credential** (Obstetrical exam) A combination of letters and numbers representing an Nuchal Translucency (NT) credential ID number. For example, P12345.

Facility

- **Institution**
- **Department ID**
- **User**
- **Reading Doctor**
- **Referring Doctor**

Acute Care Worksheets

If the Acute Care Worksheets licensed option is installed on your system, the patient form displays these fields, which also appear in the patient's Acute Care worksheet:

- **MR #** Medical record number.
- **Clinical Category** The exam's clinical category.
- **Exam Visit** Initial or Repeat.
- **Primary Obtaining/Interpreting** The primary person obtaining or interpreting the images.
- **Secondary Obtaining/Interpreting** The secondary person obtaining or interpreting the images.
- **Additional Obtaining/Interpreting** Any additional person obtaining or interpreting the images.

Routing Selection

When you associate one or more routing selections with an exam, the system reflects the routing selections in the patient report and in the Acute Care worksheet (if the Acute Care Worksheets licensed option is installed). See "Worksheets and reports" on page 220.

If you have specified a DICOM archiver as an Educational archiver, setting routing selections in the patient form affects how exams are archived. For more information, see "Routing selections" on page 74.

- **Procedural**
- **Diagnostics**
- **Educational/Accreditation**

Scheduled Procedure

Procedure fields are available only if the DICOM worklist feature is licensed and configured. See "DICOM" on page 46.

- **Scheduled Procedure: Name**
- **Scheduled Procedure: Protocol** Protocol for the selected procedure
- **Scheduled Procedure: Meaning** Definition of the selected procedure
- **Performed Procedure: Code** Desired procedure
- **Performed Procedure: Edit** Enables you to modify the Code list

- **Performed Procedure: Meaning** Definition of the performed procedure
- **Additional Worklist Query Parameters: Modality** Procedure types
- **Additional Worklist Query Parameters: Requested Procedure ID** The procedure ID

Images and clips

Reviewing

You can review images and video clips from both current and completed exams.

Caution If the save available icon  does not appear in the system status area, internal storage may be defective. Contact FUJIFILM SonoSite Technical Support. (See "Getting help" on page 3.)

Patient list

The patient list organizes saved images and video clips in patient exams. You can sort the patient list, view and delete exams, edit patient information, and append images and video clips to an existing exam.

From the patient list, you can also export exams to a USB memory stick, select a routing selection to associate with an exam, or archive exams to a DICOM archive server. For more information, see "To connect a USB memory stick for importing or exporting" on page 9, "To associate a routing selection after an exam" on page 75, and "Archiving exams" on page 165.

Column headings in the patient list indicate:

- **Name** Patient name
- **ID** Patient ID
- **Date/Time** The date and time of the exam

-  The user who performed the exam

-  The number of images and video clips saved with the exam


- ▶  The archive status of the exam

- ▶  The export status of the exam

To display the patient list

- 1 Do one of the following:

- ▶ Tap **REVIEW** and then tap **Patient List**.

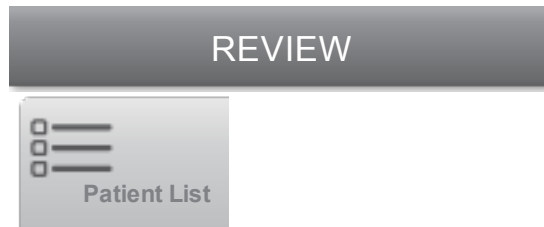

- ▶ Tap **Patient**, and then tap **Patient List**.




To sort the patient list

By default, the patient list is sorted by date and time, with the most recent patient listed first. You can re-sort the patient list.

- ▶ Tap the column heading that you want to sort by. Tap it again to sort in reverse order.

To select all exams in the patient list


- ▶ Tap **Select All**.

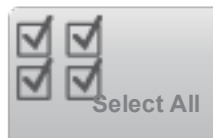

To deselect all exams, tap **Clear All**.


To delete an exam

- 1 In the patient list, select the check box next to one or more exams.
- 2 Tap **Delete**.

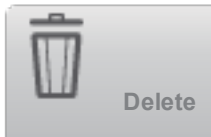

To view exam information


- 1 Tap the exam in the patient list.
- 2 Tap **Info**.

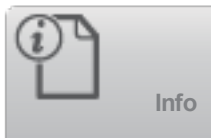

To edit patient information from the patient list

You can edit patient information from the patient list instead of from the patient form if the exam is ended but has not been exported or archived.

- 1 Tap the exam in the patient list.
- 2 Tap **Edit**.

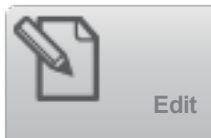


- 3 Fill in the text boxes, and tap **Done**.



To append images and video clips to a patient exam

Although you cannot add images and video clips to a patient exam that is ended, you can automatically start a new patient exam that has the same patient information. Depending on your archiver, the two exams appear as one study when exported or archived.

- 1 Tap the exam in the patient list.
- 2 Tap **Append**.

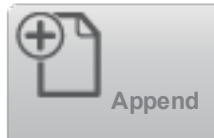

A new patient form appears. The form has the same information as the exam you selected.

Reviewing images and video clips

You can review images and video clips in only one patient exam at a time. The system displays images and video clips in the order in which they were saved.



Video clips play automatically after loading. The load time depends on video clip length.

To review images and video clips

- 1 Tap **REVIEW**, and then do either of the following:





- Review the current exam.
- Review a completed exam: Tap **Patient List**. In the patient list, tap the exam, and then tap **Thumbnails**.

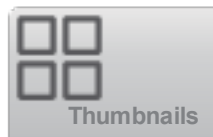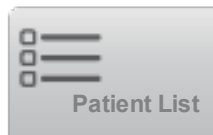


2 Do any of the following:


- ▶ To view the previous or next image or video clip, tap **Prev** or **Next**.

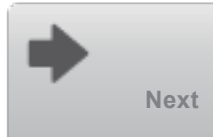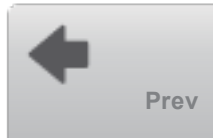

- ▶ To view images or video clips full-screen, tap **Full Screen**. Swipe left or right to view the previous or next item.

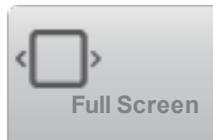

To return to multi-image display, tap **Thumbnails**.

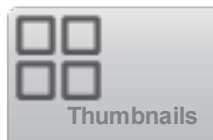


3 (Video clips only) Do any of the following:


- ▶ To pause the video clip, tap .
- ▶ To play the video clip, tap .
- ▶ To select a playback speed, tap .
- ▶ To move forward through frames one at a time, tap .
- ▶ To move backward through frames one at a time, tap .

4 (Optional) Do any of the following:

- ▶ Label an image or a frame from a video clip. (See "Labeling during review" on page 147.)
- ▶ View the report. (See "Worksheets and reports" on page 220.)
- ▶ Delete an image or video clip: select the image or video clip, and then tap **Delete**.

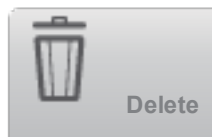

5 Return to the patient list by tapping **Patient List**, or return to imaging by tapping **Done**.

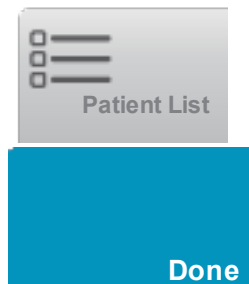


Printing images

For information about printing worksheets and reports or about DICOM printing, see "Printing reports and worksheets" on page 224 and "Archiving exams" on the facing page..

To print while imaging

- 1 Make sure that the power button on the printer is in the On position.
- 2 With the image displayed, tap **Print**.

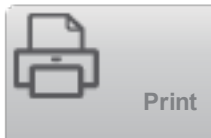


Note You can also configure the footswitch to print (see "User profile settings" on page 71).

To print a saved image from a current or ended exam


- 1 Make sure that the power button on the printer is in the On position.
- 2 Tap **REVIEW**.




- 3 Do one of the following:
- Select the check box for an image from a current exam.
 - Select an image from an ended exam: Tap **Patient List**, tap the exam, tap **Thumbnails**, and then select the check box for an image.

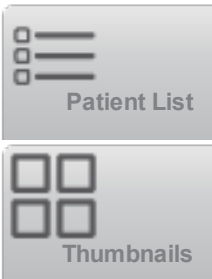

- 4 Tap **Full Screen**.

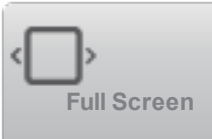

- 5 Tap **Print**.

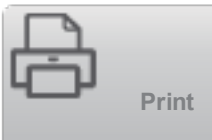

Related topics

Archiving and exporting	165
Reviewing	158
Worksheets and reports	220

Archiving and exporting

The patient list organizes saved images and video clips in patient exams.

You can *archive* exams from the patient list to a DICOM archive server or print them to a DICOM printer. You can *export* exams to a USB memory stick.

Archiving exams

If you have configured the system for DICOM transfer, the system automatically archives saved images and video clips, along with the patient report, to DICOM devices. If you have configured the system to connect to a DICOM printer, exams are also printed when they are archived.

The system archives images and video clips either during the exam or at the end of the exam, depending on the settings you have chosen in the **Transfer Images** setting on the **Location** configuration page. (Exams are printed only at the end of an exam.)

You can also manually archive exams.




The Archive in Progress icon is animated when patient exams are archiving. Pending exams are archived starting at the top of the patient list.

To verify that exams transferred

- 1 Tap **REVIEW**.



- 2 Tap **Patient List**.

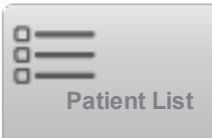

The column shows the status of exam transfer.

- ▶  Patient exam is archived.
- ▶  Patient exam is archive suspended. Network connection settings may be wrong (see "Connecting to the network" on page 47), or there may be a network or server problem. The exam needs to be manually archived (see "To manually archive exams" on the facing page).
- ▶  Storage commitment was successful.
- ▶  Storage commitment for this exam is suspended.

Exams with no status markers are pending archiving.


To display information about an exam

You can display information about an exam, including transfer details.

- 1 Tap **REVIEW**.




- 2 Tap **Patient List**.

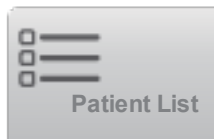

- 3 In the patient list, select the exam.

- 4 Tap **Info**.

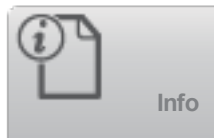

To manually archive exams

- 1 Verify the following:


- The correct location is selected. (See "To receive storage alerts" on page 43.)
- If the system is connected to a network via an Ethernet connection, the LAN link light (green LED) next to the Ethernet connector is on.

- For a wireless connection, the wireless connected icon  appears in the system status area on the clinical monitor.

- 2 Tap **REVIEW**.




- 3 Tap **Patient List**.

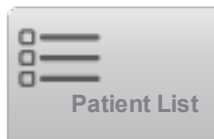

- 4 Select the check box next to one or more exams.


- 5 Tap **Archive**. (The system removes any status markers.)

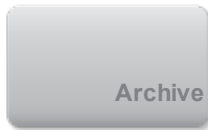

To discontinue MPPS for an exam

You can discontinue MPPS for an exam before completing the exam.

- ▶ On the patient form, tap **Discontinue**.

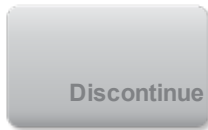

Exporting exams

Caution To avoid losing data from or damaging the USB memory stick, do not remove the USB memory stick or turn off the ultrasound system while exporting. In addition, do not bump or apply pressure to the USB memory stick while it is connected to the system. The connector could break.

You can export patient exams if they are ended and if the system administrator has not disabled USB export (See "Ending the exam" on page 155 and "Administration settings" on page 28.) Exams include images, video clips, and the patient report.


A USB memory stick is for temporary storage of images and clips. Archive patient exams regularly.

To export patient exams manually to a USB memory stick

- 1 Specify the file format for exported images. (See "USB settings" on page 77.)
- 2 Tap **REVIEW**.




- 3 Tap **Patient List**.

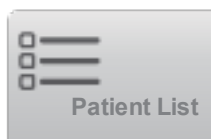

- 4 Connect a USB memory stick (see "To connect a USB memory stick for importing or exporting" on page 9).

5 In the patient list, select the check box next to one or more exams.

6 Tap **Export USB**. A list of USB devices appears.

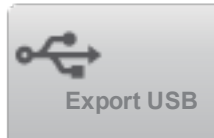


7 Select the USB memory stick. If you want to hide patient information, deselect **Include patient information on images and video clips**.

8 Tap **Export**.



The files are finished exporting approximately five seconds after the USB animation stops. Removing the USB memory stick or turning off the system while exporting may cause exported files to be corrupted or incomplete. To stop in-progress exporting, tap **Cancel**.

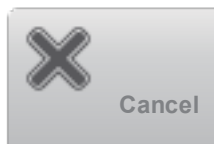

The column of the patient list indicates whether the system has exported the exam.

To export patient exams automatically to a USB memory stick

- 1 Make sure that automatic export is enabled. See "To specify export options" on page 78.
- 2 Insert a USB memory stick into a USB port on the side of the system.
- 3 End the exam.

Saving images and video clips

You can save images and video clips to internal storage. The system plays a sound if Beep Alert is on (see

"Audio settings" on page 34), and the Save in Progress icon  briefly appears on the clinical monitor.

The system limits the maximum number of images and video clips that you can save for an individual exam, and warns you when this limit is reached.

You can also save a labeled frame from a video clip. See "Labeling during review" on page 147.

To review saved images and video clips for the current patient, tap **REVIEW**. For patients whose exams have ended, then you must go to the Patient List by tapping **REVIEW** and then **Patient List**. See "Reviewing" on page 158. For information on viewing the cine buffer, see "Viewing the cine buffer" on page 138.

You can also save images from cine while the system is frozen.

Related topics

Audio settings.....	34
Reviewing.....	158
Archiving and exporting.....	165



Saving images

To avoid mixing images saved from multiple patients, make sure that the patient ID is displayed before you save an image. See "Entering patient information" on page 150 and "Display Information settings" on page 61.

To save an image

If the **Save Image** or **Save Image & Calcs** control is not available, configure the system to display it. See "To configure custom controls" on page 72.

- ▶ Do either of the following:
 - ▶ To save only the image, tap **Save Image**.
- ▶ To save the image and to save the current calculation to the patient report, tap **Save Image & Calcs**.

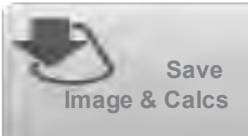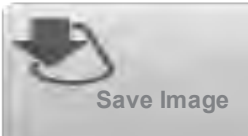

Note | You can also configure the footswitch to save (see "User profile settings" on page 71).

Saving video clips

To set video clips controls

- 1 Tap **Video Clip Settings**.



If this control is hidden, tapping **More Controls** displays it.

- 2 Set the following as desired:

- ▶ **Clip Method**

Prospective captures frames after you tap **Save Video Clip**. The system captures frames for the number of seconds you specify in the **Time** list.

Retrospective captures frames from presaved data available before you tap **Save Video Clip**. The system captures presaved frames for the number of seconds you specify in the **Time** list.



The system displays in the system status area when you save a video clip prospectively.

If Restrospective is selected in Video Clip Settings, the system displays in the system status area.



- ▶ **Clip Type**


Time captures based on number of seconds. Select the time duration from the **Time** list.

ECG captures based on the number of heart beats. Select the **R-Wave Count**.

- ▶ **Preview Video Clip**


Select this option to automatically play and optionally trim a captured video clip (the video clip does not save automatically to internal storage). If this option is deselected, the video clip saves to internal storage, and trim controls are not available.

- 3 Tap **Done**.

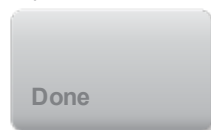

To capture and save a video clip

- 1 Make sure that video clips settings are set as desired. See "To set video clips controls" above.
- 2 Tap **Save Video Clip**.

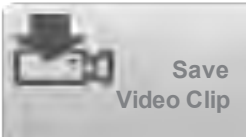

Note You can also configure the footswitch to save a video clip (see "User profile settings" on page 71).

If **Clip Method** is set to **Retrospective**, the system captures presaved frames for the number of seconds you specify in the **Time** list and, if Beep Alert is on (see "Audio settings" on page 34), beeps when acquisition is complete.


If **Clip Method** is set to **Prospective**, the system beeps if Beep Alert is on (see "Audio settings" on page 34), captures frames for the number of seconds you specify in the **Time** list, and then beeps again. Tap **Stop Video Clip** to end capture before the number of seconds specified in the **Time** list.

If **Preview Video Clip** is selected in **Video Clip Settings**, the video clip automatically begins playing on the clinical monitor.

- 3 (Optional, Preview Video Clip only) Trim the video clip:



- a. Tap .
- b. Drag the trim sliders to remove frames from the left or right sides of the video clip.
- c. Tap **Save**.

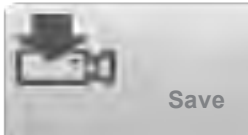

Note Tapping **Done** discards the video clip.

Managing internal storage space



The save available icon in the system status area shows the percentage of space available in internal storage.

To receive alerts when storage is near capacity, see "To receive storage alerts" on page 43.

If you try to save an image or video clip when space is low, the system alerts you that internal storage is low and prompts you to delete archived or storage committed patient exams.

If memory is full, the system displays the Memory Full icon .

DVR recording

You can record video displayed on the clinical monitor to a USB memory stick inserted into the DVR USB port. Clips save as MP4 files. You can copy the recordings to another device.

You cannot record video if the system administrator has selected **Disable USB Export** in Administration setups. (See "Administration settings" on page 28.)

To record to the DVR

- 1 Make sure that a USB memory stick is inserted into the DVR USB port.
The DVR USB port is located on the rear of the stand's dock. See "Ports" on page 21.

Note Wait at least five seconds between removing and reinserting a USB memory stick into the DVR USB port.

- 2 Tap **Rec** at the upper right of the touch panel.



Note You can also configure the footswitch to record (see "User profile settings" on page 71).

The system begins recording to the USB memory stick in the DVR port and displays a flashing recording indicator.



Note If **Rec** is not available, check to make sure that the DVR has a USB memory stick inserted and that the USB memory stick is not full.

- 3 To stop recording, tap **Rec**, or end the exam.



To copy DVR recordings to another device

- 1 After you stop recording, remove the USB memory stick from the DVR.
- 2 Insert the USB memory stick into another device (for example, a personal computer).

- 3 Display the USB memory stick's contents and copy the recordings (MP4 files) to the device.

Note If video recorded from the DVR is too bright when you view it on a personal computer, you can adjust the video color settings on your PC's monitor. For example, change the Dynamic Range to Full. See your computer's Help for information about available video color settings.

ECG

ECG monitoring requires an optional FUJIFILM SonoSite ECG module.

ECG is available only on the cardiac exam type.

- WARNING** To prevent misdiagnosis, do not use the ECG trace to diagnose cardiac rhythms. The FUJIFILM SonoSite ECG option is a non-diagnostic feature.
- Caution** Use only accessories recommended by FUJIFILM SonoSite with the system. Connecting an accessory not recommended by FUJIFILM SonoSite can damage the system.
- Caution** If you defibrillate a patient while the ECG module is connected to the system, the ECG signal may display incorrectly, and the ECG module may need to be replaced.

To monitor ECG

- 1 Select the cardiac exam type on the P21xp transducer. (See "Selecting a transducer and exam type" on page 16.)

- 2 Connect the ECG cable to the ECG port (**ECG**) on the ultrasound system.
ECG monitoring turns on automatically.

Note The ECG signal may take up to one minute to restabilize after defibrillator use on a patient.

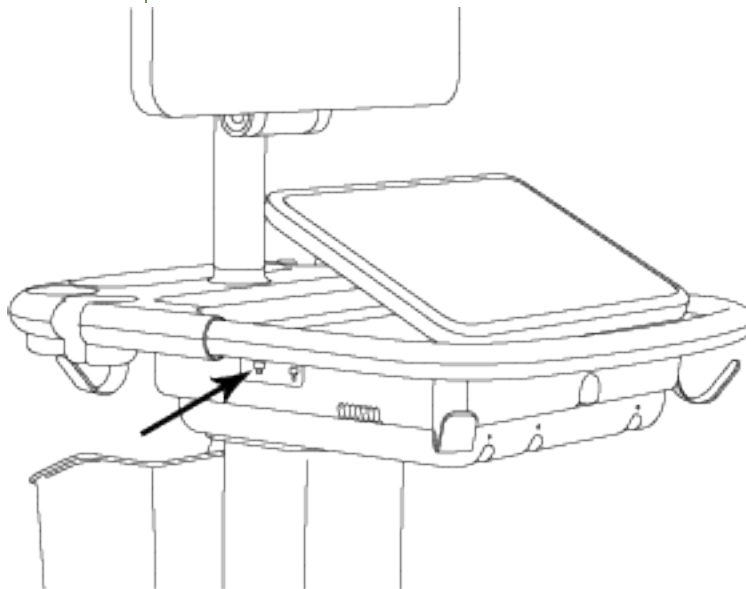


Figure 4-4 ECG port


- 3 Tap **ECG**.



If this control is hidden, tapping **More Controls** displays it.

- 4 Do any of the following in the **ECG Settings** window:
 - To display the ECG trace, select **Show**.
 - To adjust the value of the ECG gain, drag the **Gain** slider.
 - To set the ECG scroll speed, select **Slow**, **Medium**, or **Fast** under **Sweep Speed**.
If the Doppler or M Mode trace is active, the ECG scroll speed matches the sweep speed for the trace.
 - To move the ECG trace up or down vertically on the image, tap **Up** or **Down** under **Position**.

5 Tap **Done**.

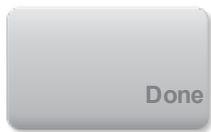

CHAPTER 5

Measurements and calculations

This section provides information about measurements, calculations, worksheets, and reports.

Measuring

X-Porte offers several types of measurements.

You can perform independent measurements, or you can perform a measurement and then assign it to a calculation. (See "Assigning measurements to calculations" on page 183.) If you want to save a measurement, you must assign it to a calculation (which saves results to the patient report), or you can save the image with the calipers displayed.

If you prefer to start within a calculation, see "About calculations" on page 183.

Calipers

You perform most measurements by using calipers that you drag into position.




The active caliper has a round highlighted handle

To activate calipers

- ▶ Tap the caliper.

To switch between calipers

- Do either of the following:
 - Tap either caliper.
 - Tap **Switch** beneath the measurement result on the left side of the touch panel.

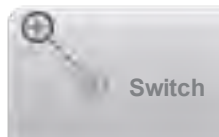

- Tap the caliper under **Measure**.

Viewing and deleting measurement results

The current measurement result appears on the left side of the touch panel under **Measure**. To assign it to a calculation, see "Assigning measurements to calculations" on page 183.

If you perform multiple measurements, tapping an entry under **Measure** highlights the corresponding calipers on the image.

To delete a measurement


- Tap the measurement under **Measure**, and then tap **Delete**.

Basic measurements in 2D

Note You can measure across images in Dual only with a linear transducer, and only when the images are at the same depth and magnification.

To measure the distance between two points (2D)

- 1 On a frozen image, tap **Distance**.

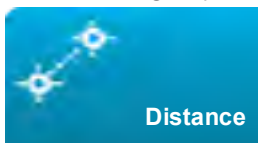

A caliper appears.

If this control is hidden, tapping **More Controls** displays it.

- 2 Drag the active caliper to the first point.
- 3 Drag the other caliper to the second point.
- 4 As needed, tap and drag each caliper until it is precisely positioned.

To measure circumference or area using an ellipse (2D)

- 1 On a frozen image, tap **Ellipse**.



If this control is hidden, tapping **More Controls** displays it.
An ellipse with three calipers appears.

- 2 Position and resize the ellipse:
 - To position, drag the calipers.
 - To resize, drag the calipers, the handle, or both.

To measure circumference or area by tracing (2D)

- 1 On a frozen image, tap **Trace**.



If this control is hidden, tapping **More Controls** displays it.
A caliper appears.

- 2 Drag the caliper to the start point, and then tap **Set**.
- 3 Drag the caliper around the area you want to trace.
To make a correction, trace backward on the dotted line.
- 4 Do one of the following:
 - To save the measurement and close the trace, tap **Save**.
 - To close the trace, tap **Close**.

To perform a Target Depth measurement (2D)

You can measure the distance from the skin line to a specified point on the image.

- 1 On a 2D image, tap **Target Depth**.



If this control is hidden, tapping **More Controls** displays it.
A dotted line appears from the skinline and with a single caliper at the end.


- 2 Drag the caliper to the desired location.

Basic measurements in M Mode

To measure distance and time (M Mode)

You can measure distance in centimeters and measure time in seconds.

- 1 On a frozen M Mode trace, tap **Distance Time**.

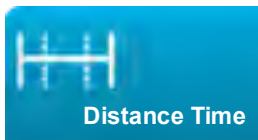


If this control is hidden, tapping **More Controls** displays it.
A caliper appears.

- 2 Drag the caliper to the desired location.
A second caliper appears.
- 3 Drag the second caliper to the desired location.

To measure heart rate (M Mode)

- 1 On a frozen M Mode trace, tap **Heart Rate**.

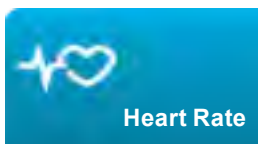

A vertical caliper appears.
If this control is hidden, tapping **More Controls** displays it.

- 2 Drag the caliper to the peak of the heartbeat.
A second vertical caliper appears.
- 3 Drag the second caliper to the peak of the next heartbeat.

Saving the heart rate measurement to the patient report overwrites any heart rate entered on the patient form.

Basic measurements in Doppler

The basic measurements that you can perform in Doppler imaging are Velocity (cm/s), Pressure Gradient, Elapsed Time, +/-x Ratio, Resistive Index (RI), and Acceleration. You can also trace a waveform manually or automatically.

For Doppler measurements, the Doppler scale must be set to cm/s. See "General preferences" on page 67.

To measure Velocity (cm/s) and Pressure Gradient (Doppler)

- 1 On a frozen Doppler spectral trace, tap **Velocity**.



A single caliper appears.

If this control is hidden, tapping **More Controls** displays it.

- 2 Drag the caliper to a peak velocity waveform.
This measurement involves a single caliper from the baseline.

To measure Velocities, Elapsed Time, +/x Ratio, Resistive Index (RI), and Acceleration (Doppler)

- 1 On a frozen Doppler spectral trace, tap **Velocity**.



A single caliper appears.

If this control is hidden, tapping **More Controls** displays it.

- 2 Drag the caliper to a peak systolic waveform.
- 3 Tap **Velocity**.
A second caliper appears.
- 4 Drag the second caliper to the end diastole on the waveform.

To measure time duration (Doppler)

- 1 On a frozen Doppler spectral trace, tap **Time**.



A vertical caliper appears.

If this control is hidden, tapping **More Controls** displays it.

- 2 Drag the caliper where desired.
- 3 Drag the second caliper where desired.

To trace manually (Doppler)

- 1 On a frozen Doppler spectral trace, tap **Manual Trace**.



If this control is hidden, tapping **More Controls** displays it.
A single caliper appears.

- 2 Drag the caliper to the beginning of the desired waveform, and tap **Set**.
If calipers are not positioned correctly, the result is inaccurate.
- 3 Drag the caliper to trace the waveform.
To make a correction, trace backward on the dotted line.
- 4 Tap **Close**.
The measurement results appear.

To measure automatically (Doppler)

After measuring automatically, confirm that the system-generated boundary is correct. If you are not satisfied with the trace, obtain a high-quality Doppler spectral trace image, or trace manually.

- 1 On a frozen Doppler spectral trace, tap **Auto Measure**.



If this control is hidden, tapping **More Controls** displays it.
A vertical caliper appears.

- 2 Drag the caliper to the beginning of the waveform.
If calipers are not positioned correctly, the calculation result is inaccurate.
- 3 Drag the second caliper to the end of the waveform, and tap **Done**.
The measurement results appear.

Automatic trace results

Automatic trace results

Depending on the exam type, the results from automatic tracing include the following:

- Velocity Time Integral (VTI)
- Peak Velocity (Vmax)
- Mean Pressure Gradient (PGmean)

- Mean Velocity on Peak Trace (Vmean)
- Pressure Gradient (PGmax)
- Cardiac Output (CO)
- Peak Systolic Velocity (PSV)
- Time Average Mean (TAM)*
- +/- or Systolic/Diastolic (S/D)
- Pulsatility Index (PI)
- End Diastolic Velocity (EDV)
- Acceleration Time (AT)
- Resistive Index (RI)
- Time Average Peak (TAP)
- Gate Depth

* The automatic trace tool must be used to calculate the TAM.

Assigning measurements to calculations

You can assign a measurement to a calculation that includes that measurement type.

When you assign a measurement to a calculation, a checkmark appears next to the measurement indicating that it has been successfully saved.

To assign a measurement to a calculation

- 1 After performing the measurement, select it, and tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

Note | If **Calcs** is unavailable, the measurement is unavailable for saving to a calculation.

A calculation list appears, along with a list of additional available lists.

- 2 Tap the desired calculation list.
- 3 Tap the desired measurement name in the calculation list.
- 4 Save the measurement: Tap **Save** under the measurement name.
A checkmark appears next to the measurement name.

About calculations

WARNING | To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.

- WARNING

To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.
- WARNING

To avoid misdiagnosis or harming the patient outcome, do not use single calculations as sole diagnostic criteria. Use calculations in conjunction with other clinical information.

In X-Porte you can perform calculations, whose results save to the patient report. (See "Worksheets and reports" on page 220.)


There are three kinds of calculations:

- Basic calculations that are available in multiple exam types:
 - Volume calculations
 - Volume Flow calculations
 - Percent reduction calculations
- Specialized calculations that are specific to one exam type:
 - Arterial calculations
 - Cardiac calculations
 - Gynecology calculations
 - Obstetrical calculations
 - Small Parts calculations
- Acute Care calculations, included with the Acute Care worksheets licensed feature.

Related topics	
Freezing the image.	136

Overview

Calculations are accessed from the *calculation lists*, which you display by tapping the **Calcs** control (for basic and specialized calculations) or **Acute Care** control (for Acute Care calculations), typically on a frozen image.

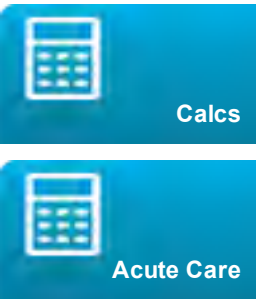

If these controls are hidden, tapping **More Controls** displays them.

After you tap the control, a calculation list appears on the left side of the touch panel. If the measurement you want to perform is in a different list, you can select that list (see "To select calculation lists" below).

Tapping  collapses the list contents. Tapping  expands them.

After you select a measurement name, it is highlighted and has **Save** and **Delete** controls, and calipers appear on the image.

You position the calipers by dragging. The measurement result appears beside the measurement name. After you save measurements, the calculation result appears on the clinical monitor.

If there are multiple saved measurements for a calculation, the system numbers them in the order they were completed in the calculation list on the left side of the touch panel.

To select calculation lists

If the measurement you want to perform is in a different list, you can select that list.

- 1 Tap  at the top of the current list.
- 2 Tap the desired list.


To save calculation measurements

When you save a measurement, a checkmark appears next to the measurement, indicating that the value has been saved.

- ▶ With one or more measurements on the screen, do any of the following:
 - ▶ Save the selected measurement: Tap **Save** under the measurement name.
 - ▶ Save all the measurements in a given set: Tap **Save Set**.
 - ▶ Save all performed measurements in the calculation: Tap **Save All**.



- ▶ Save both the selected measurement and the image: Tap **Save Image & Calcs**. (If the **Save Image & Calcs** control is not available, configure the system to display it (see "User profile settings" on page 71)).

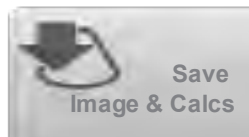

To redo or delete a saved measurement

- 1 On a frozen image, tap **Calcs** or, for the Acute Care calculations, tap **Acute Care**.



If these controls are hidden, tapping **More Controls** displays them.

- 2 In the calculation list that contains the measurement, do one of the following:
 - To redo, perform the measurement again.
To save the new measurement, tap **Save** under the measurement name. The new measurement saves to the patient report and appends the new measurement to the calculation list.
 - To delete, tap **Delete** under the measurement name.
The measurement last saved is deleted from the patient report. If it is the only measurement, the result is deleted from the report.

Some measurements can be deleted directly from the report preview. See "Worksheets and reports" on page 220.

Percent reduction calculations

- WARNING** To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.
- WARNING** To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

Percent reduction calculations are available in the following exam types: Abdomen, Arterial, Musculoskeletal, and Small Parts.

For a list of exam types on each transducer, see "Selecting a transducer and exam type" on page 16.

To calculate percent area reduction

- 1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 In the Area Reduction calculation list, do the following for **A¹** and then for **A²**:



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- a. Tap the measurement name.
- b. Drag the caliper to the trace starting point, and then tap **Set**.
- c. Trace the desired area.
To make a correction, trace backward on the dotted line.
- d. Complete the trace, and then do one of the following:
 - To save the measurement and close the trace, tap **Save**.
 - To close the trace, tap **Close**.

The percent area reduction result appears on the clinical monitor and saves to the patient report.

To calculate percent diameter reduction

- 1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the Diameter Reduction calculation list, do the following for **D¹** and then for **D²**:



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- a. Tap the measurement name.
- b. Position the calipers by dragging.
- c. Save the measurement.

The percent diameter reduction result appears on the clinical monitor and saves to the patient report.

Volume calculation

WARNING To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.

WARNING To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

The volume calculation involves three 2D distance measurements: Height, Width, and Length. You can calculate volume on one, two, or all three distance measurements. After any measurement is saved, the result appears on the clinical monitor and in the patient report.

The volume calculation is available in the following exam types: Abdomen, Arterial, Breast, Gynecological, Musculoskeletal, Nerve, Small Parts, Superficial.

For a list of exam types on each transducer, see "Selecting a transducer and exam type" on page 16.

To calculate volume

- Do the following for each image you need to measure:

- a. On the frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- b. On the **Volume 1**, **Volume 2**, or **Volume 3** calculation list, do the following for each measurement you need to take:
 - i. Tap the measurement name.
 - ii. Position the calipers by dragging.
 - iii. Save the measurement.

Volume flow calculation

WARNING To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.

WARNING To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

Volume flow calculations are available in the following exam types: Abdomen, Arterial.

For a list of exam types on each transducer, see "Intended uses" on page 25.

Required measurements

Calculation list	Measurement name (imaging mode)	Calculation result
Volume Flow	Diam (2D)	Vol Flow (Volume Flow ml/min)

Note | To calculate Volume Flow, you must measure the vessel diameter.

Considerations

Consider the following factors when performing volume flow measurements:

- You should follow current medical practice for volume flow calculation applications.
- The accuracy of the volume flow calculation largely depends on the user's measurement technique.
- The factors identified in the literature that affect the accuracy are as follows:
 - Using the diameter method for 2D area
 - Precision in placing the caliper
 - Difficulty ensuring uniform insonation of the vessel


For more information about the considerations and degree of accuracy for volume flow measurements and calculations, see the following reference:

Allan, Paul L. et al. *Clinical Doppler Ultrasound*, 4th Ed., Harcourt Publishers Limited, (2000) 36-38.

Calculating volume flow

To calculate volume flow

- 1 If measuring the diameter instead of using the sample volume size, perform the 2D measurement:
 - a. On a frozen full-screen 2D image or duplex image, tap **Calcs**.

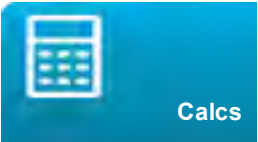

If this control is hidden, tapping **More Controls** displays it.

- b. On the **Volume Flow** calculation list, tap **Diam**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- c. Position the calipers by dragging.
- d. Tap **Save** under the measurement name.

2 Perform the Doppler measurement:

- a. If full-screen imaging, start and then freeze the Doppler spectral trace.
- b. On the **Volume Flow** calculation list, tap **TAM** or **TAP**.
A vertical caliper appears.
- c. Drag the vertical caliper to the beginning of the waveform.
A second vertical caliper appears.
If calipers are not positioned precisely, the calculation result is inaccurate.
- d. Drag the second vertical caliper to the end of the waveform, and tap **Done**.
- e. If you need to reposition either caliper, tap it (or tap its name in the calculation list) to make it active, drag it where desired, and then tap **Done**.
- f. Tap **Save** under the measurement name.
Results appear on the clinical monitor and save to the patient report.

Exam-based calculations

Arterial calculations

- WARNING**

To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.
- WARNING**

To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

The Arterial calculations that you can perform are listed in the table below.

In the Arterial exam, you can also calculate ICA/CCA ratio, volume, volume flow, and percent reduction. See "Volume calculation" on page 188, "Volume flow calculation" on page 188, and "Percent reduction calculations" on page 186.

Table 5-1 Arterial calculations

Calculation list	Measurement name
Right	CCA ▶ Proximal ▶ Middle ▶ Distal ▶ Bulb
	ICA ▶ Proximal ▶ Middle ▶ Distal
	ECA ▶ Proximal ▶ VA
Left	CCA ▶ Proximal ▶ Middle ▶ Distal ▶ Bulb
	ICA ▶ Proximal ▶ Middle ▶ Distal
	ECA ▶ Proximal ▶ VA

Calculation results for each measurement include s (systolic) and d (diastolic).

To perform an Arterial calculation

- 1 On a frozen Doppler spectral trace, tap **Calcs**.



- 2 Do the following for each measurement you want to take:
- a. Select the calculation list that contains the measurement name.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- b. Drag the caliper to the peak systolic waveform, and tap **Next**.
A second caliper appears.
- c. Drag the caliper to the end diastole point on the waveform.
- d. Tap **Save** or **Save Set** under the measurement name.

Cardiac calculations

- WARNING**

WARNING

WARNING
- To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.

To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

Moving the baseline, scrolling, or inverting the trace while frozen clears the displayed results.

In the Cardiac exam, you can perform the following calculations:

Measurements required to complete Cardiac calculations

Note This table is organized to show calculation results. The order of items in the calculation lists on the system may differ somewhat.

Table 5-2 Cardiac calculations and results

Calculation list	Measurement name (imaging mode)	Results
LV	Diastole (2D or M Mode)	CO
	▶ RVWd (2D or M Mode)	CI
	▶ RVDd (2D or M Mode)	EF
	▶ IVSd (2D or M Mode)	SV
	▶ LVDd (2D or M Mode)	SI
	▶ LVPWd (2D or M Mode)	LVESV
		LVEDV
	Systole	IVS
	▶ RVWs (2D or M Mode)	LVPWFT
	▶ RVDs (2D or M Mode)	LVDFS
	▶ IVSs (2D or M Mode)	LV Mass*
	▶ LVDs (2D or M Mode)	
	▶ LVPWs (2D or M Mode)	
<i>HR² needed for CO & CI</i>		
<i>*If measurements taken in M Mode.</i>		
Ao/LA	Ao (2D or M Mode)	Ao
		LA/Ao
	LA (2D or M Mode)	LA
		LA/Ao
	AAo (2D)	AAo
	LVOT D (2D)	LVOT D
		LVOT area
MV	ACS (M Mode)	ACS
	LVET (M Mode)	LVET
	E-F Slope (M Mode)	E-F SLOPE
	EPSS (M Mode)	EPSS

Table 5-2 Cardiac calculations and results (continued)

Calculation list	Measurement name (imaging mode)	Results
MV/MR	MV ▸ E (Doppler) ▸ A (Doppler)	E EPG A APG E:A
	▸ PHT (Doppler)	PHT MVA Decel
	▸ MV VTI (Doppler)	VTI Vmax PGmax Vmean PGmean
	▸ IVRT (Doppler)	time
	▸ Adur (Doppler)	time
	MR ▸ dP:dT^b (CW Doppler)	dP:dT
	MVA (2D)	MV Area
	AVA (2D)	AV Area
Area		
LV Vol	A4Cd (2D) A4Cs (2D) A2Cd (2D) A2Cs (2D)	LV Vol LV Area EF CO SV CI SI Biplane d Biplane s

Table 5-2 Cardiac calculations and results (continued)

Calculation list	Measurement name (imaging mode)	Results
LV Mass	Epi (2D) Endo (2D) Apical (2D)	LV Mass Epi Area Endo Area D Apical
AV/LVOT/AI	AV ▸ VMAX (Doppler)	Vmax PGmax
	▸ AV VTI (Doppler)	VTI Vmax PGmax Vmean PGmean AVA
	LVOT ▸ VMAX (Doppler) ▸ VTI (Doppler) ▸ LVOT D (2D)	VTI Vmax PGmax Vmean PGmean LVOT D
	AI ▸ PHT (Doppler)	AI PHT AI slope

Table 5-2 Cardiac calculations and results (continued)

Calculation list	Measurement name (imaging mode)	Results
TV/TR	TR VMax (Doppler)	Vmax PGmax
	RA pressure ^c	RVSP
	E (Doppler) A (Doppler)	E E PG A A PG E:A
	PHT (Doppler)	PHT TVA Decel
	TV VTI (Doppler)	VTI Vmax PGmax Vmean PGmean
PV	VMAX (Doppler)	Vmax PGmax
	PV VTI (Doppler) AT (Doppler)	VTI Vmax PGmax Vmean PGmean AT
CO	LVOT Diam (2D) HR (Doppler) AV VTI (Doppler)	CO CI HR VTI

Table 5-2 Cardiac calculations and results (continued)

Calculation list	Measurement name (imaging mode)	Results
P Vein	A (Doppler) Adur (Doppler) S (Doppler) D (Doppler)	VMax time VMax S/D ratio VMax
PISA	Radius (Color) MR VTI (Doppler) Ann Diam (2D) MV VTI (Doppler)	PISA Area VAlas ERO MV Regurgitant Volume Regurgitant Fraction VMax PGmax
Qp/Qs	LVOT Diam (2D) RVOT D (2D) LVOT VTI (Doppler) RVOT VTI (Doppler)	LVOT D RVOT D VTI VMax PGmax Vmean PGmean SV Qp/Qs
TDI	Sep e' (Doppler) Sep a' (Doppler) Lat e' (Doppler) Lat a' (Doppler) Inf e' (Doppler) Inf a' (Doppler) Ant e' (Doppler) Ant a' (Doppler)	E(MV)/e' ratio

^aYou can enter the HR measurement on the patient form, by Doppler measurement (See "To measure Heart Rate (HR)" on page 206), or by M Mode measurement (See "To measure heart rate (M Mode)" on page 180).

^bPerformed at 300 cm/s on the negative side of the baseline.

^cSpecified on the cardiac patient report.

To calculate Proximal Isovelocity Surface Area (PISA)

The PISA calculation requires a measurement in 2D, a measurement in Color, and two measurements in Doppler spectral trace. After all measurements are saved, the result appears in the patient report.

1 Measure from Ann D:

- a. On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- b. On the **PISA** calculation list, tap **Ann D**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- c. Position the calipers by dragging.
- d. Save the calculation: Tap **Save** under the measurement name.

2 Measure from Radius:

- a. On a frozen Color image, tap **Calcs**.
- b. On the **PISA** calculation list, tap **Radius**.
- c. Position the calipers by dragging.
- d. Save the calculation: Tap **Save** under the measurement name.

3 On a frozen Doppler spectral trace, tap **Calcs**.

4 Do the following to measure from MR VTI and again to measure from MV VTI:

- a. On the **PISA** calculation list, tap **MR VTI** or **MV VTI**.
- b. Drag the caliper to the start of the waveform, and tap **Set** to start the trace.
- c. Using your finger, trace the waveform.
To make a correction, trace backward on the dotted line.
- d. Tap **Close** to complete the trace.
- e. Save the calculation: Tap **Save** under the measurement name.

To measure LVd and LVs

1 On a frozen 2D image or M Mode trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **LV** calculation list, tap the name of the first measurement you want to take.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- 3 Position the calipers by dragging.
- 4 If you want to take additional measurements, tap either of the following and then position the calipers:
 - **Next** to advance to the next item in the calculation list.
 - The measurement name in the calculation list.

If you need to redo a measurement, select the measurement name and tap **Delete**

- 5 Save the calculation: Tap **Save Set** under the measurement name. The system saves all on-screen measurements for the calculation.

To measure Ao, LA, AAO, or LVOT Diam

- 1 On a frozen 2D image or M Mode trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **Ao/LA** calculation list, tap the measurement name.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- 3 Position the calipers by dragging.
- 4 Save the calculation: Tap **Save** under the measurement name.

To calculate LV Volume (Simpson's Rule)

- 1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

2 Do the following for each measurement:

- a. On the **LV Vol** calculation list, tap the desired view and phase.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- b. Position the caliper at the mitral annulus, and tap **Set** to start the trace.
- c. Using your finger, trace the left ventricular (LV) cavity.
To make a correction, trace backward on the dotted line.
- d. Complete the trace, and tap **Close**.
- e. Adjust the axis as needed.
- f. Save the calculation: Tap **Save** under the measurement name.

To calculate MV or AV area

1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

2 On the **Area** calculation list, tap **MVA** or **AVA**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- 3 Drag the caliper to where you want to begin the trace, and tap **Set**.
- 4 Using your finger, trace the desired area.
To make a correction, trace backward on the dotted line.
- 5 Do one of the following:
 - To save the measurement and close the trace, tap **Save**.
 - To close the trace, tap **Close**.

To calculate LV Mass

1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

2 Do the following for Epi and then for Endo:

- ▶ On the **LV Mass** calculation list, tap the measurement name.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- ▶ Drag the caliper to where you want to begin the trace, and tap **Set**.
- ▶ Using your finger, trace the desired area.
To make a correction, trace backward on the dotted line.

3 Tap **Save** under the measurement name to save the calculation and close the trace.

4 On the **LV Mass** calculation list, tap **Apical**.

5 Positioning the calipers, measure the ventricular length.

6 Save the calculation: Tap **Save** under the measurement name.

To measure peak velocity

For each cardiac measurement, the system saves up to five individual measurements and calculates their average. If you take more than five measurements, the most recent measurement replaces the oldest measurement. If you delete a saved measurement from the patient report, the next measurement taken replaces the deleted one in the patient report.

1 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

2 On the **MV/MR**, **TV/TR**, **TDI**, or **P Vein** calculation list, do the following for each measurement you want to take:



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- Tap the measurement name.
- Position the calipers.
- Save the calculation: tap **Save** or **Save Set**.

3 If you want to take additional measurements, tap either of the following and then position the calipers:

- ▶ **Next** to advance to the next item in the calculation list.
- ▶ The measurement name in the calculation list.

To calculate Velocity Time Integral (VTI)

This calculation computes other results in addition to VTI. See "Measurements required to complete Cardiac calculations" on page 192.

- 1 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **MV/MR**, **AV/LVOT/AI**, **TV/TR**, or **PV** calculation list, tap **VTI**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- 3 Drag the caliper to the start of the waveform, and tap **Set**.
- 4 Using your finger, trace the waveform.
To make a correction, trace backward on the dotted line.
- 5 Complete the trace, and tap **Close**.
- 6 Save the calculation: Tap **Save** under the measurement name.

To calculate Right Ventricular Systolic Pressure (RVSP)

- 1 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **TV/TR** calculation list, tap **TR VMax**.
- 3 Position the caliper by dragging.
- 4 Save the calculation: Tap **Save** under the measurement name.
To adjust the RA pressure, see "(Cardiac) To adjust the RA pressure" on page 222.
Changing the RA pressure from the default 5 affects the RVSP calculation in the patient report.

To calculate Pressure Half Time (PHT) in MV, AI, or TV

- 1 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **MV/MR**, **AV/LVOT/AI**, or **TV/TR** calculation list, tap **PHT**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- 3 Drag the first caliper to the peak.
A second caliper appears.
- 4 Position the second caliper:
 - a. In MV, drag the caliper along the E-F slope.
 - b. In AV, drag the caliper to the end diastole.
- 5 Save the calculation: Tap **Save** under the measurement name.

To calculate Isovolumic Relaxation Time (IVRT)

- 1 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **MV/MR** calculation list, tap **IVRT**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

A vertical caliper appears.

- 3 Drag the caliper to the aortic valve closure.
A second vertical caliper appears.
- 4 Drag the second caliper to onset of mitral inflow.
- 5 Save the calculation: Tap **Save** under the measurement name.

To calculate Delta Pressure: Delta Time (dP:dT)

To perform the dP:dT measurements, the CW Doppler scale must include velocities of 300 cm/s or greater on the negative side of the baseline. (See "Controls in Doppler" on page 120.)

- 1 On a frozen CW Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **MV/MR** calculation list, tap **dP:dT**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.


A horizontal dotted line with an active caliper appears at 100 cm/s.

- 3 Drag the first caliper along the waveform at 100 cm/s.
A second horizontal dotted line with an active caliper appears at 300 cm/s.
- 4 Drag the second caliper along the waveform at 300 cm/s.
- 5 Save the calculation: Tap **Save** under the measurement name.

To calculate Aortic Valve Area (AVA)

The AVA calculation requires a measurement in 2D and two measurements in Doppler. After the measurements are saved, the result appears in the patient report.

- 1 In 2D, measure from LVOT:
 - a. On a frozen 2D image, tap **Calcs**.

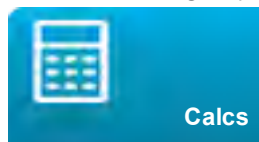

If this control is hidden, tapping **More Controls** displays it.

- b. On the **AV/LVOT/AI** calculation list, tap **LVOT D**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- c. Position the calipers by dragging.
 - d. Save the calculation: Tap **Save** under the measurement name.
- 2 In Doppler, measure LVOT Vmax or VTI, and then measure AV Vmax or VTI.
 - For VMAX, see "To measure peak velocity" on page 201. From the **AV/LVOT/AI** calculation list, select **VMAX** under **AV**, and place calipers at the sample site.
 - For VTI, see "To calculate Velocity Time Integral (VTI)" on page 202. From the **AV/LVOT/AI** calculation list, select **VTI** under **AV**, and place calipers at the sample site.

To calculate Qp/Qs

The Qp/Qs calculation requires two measurements in 2D and two measurements in Doppler. After the measurements are saved, the result appears in the patient report.

- 1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 Do the following to measure from LVOT D and again to measure from RVOT D:
 - a. From the **Qp/Qs** calculations list, select **LVOT D** or **RVOT D**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- b. Position the calipers by dragging.
 - c. Save the calculation: Tap **Save** under the measurement name.
- 3 On a frozen Doppler spectral trace, tap **Calcs**.
 - 4 Do the following to measure from LVOT VTI and again to measure from RVOT VTI:
 - a. From the **Qp/Qs** calculations list, select **LVOT VTI** or **RVOT VTI**.
 - b. Tap **Set** to start the trace.
 - c. Using your finger, trace the waveform.
To make a correction, trace backward on the dotted line.
 - d. Tap **Close** to complete the trace.
 - e. Save the calculation: Tap **Save** under the measurement name.

For information on the automatic trace tool, see "To measure automatically (Doppler)" on page 182.

To calculate Stroke Volume (SV) or Stroke Index (SI)

The SV and SI calculations require a measurement in 2D and a measurement in Doppler. SI also requires Body Surface Area (BSA). After the measurements are saved, the result appears in the patient report.

- 1 (SI Only) Fill in the **Height** and **Weight** fields on the patient form. The BSA is calculated automatically. (See "Entering patient information" on page 150.)

2 Measure LVOT:

- a. On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- b. From the **AV/LVOT/AI** calculation list, tap **LVOT D**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- c. Position the calipers at LVOT area by dragging.
- d. Save the calculation: Tap **Save** under the measurement name.

3 Measure from aorta (Doppler). See "To calculate Velocity Time Integral (VTI)" on page 202. From the **AV/LVOT/AI** calculation list, select **VTI** under **AV**.

For information on the automatic trace tool, see "To measure automatically (Doppler)" on page 182.

To measure Heart Rate (HR)

Heart Rate is available in all cardiac packages. The Heart Rate is not calculated using the ECG trace.

- 1 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the CO calculations list, tap **HR**.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.
A vertical caliper appears.

- 3 Drag the first vertical caliper to the peak of the heartbeat.
A second vertical caliper appears. The active caliper is highlighted green.
- 4 Drag the second vertical caliper to the peak of the next heartbeat.
- 5 Save the calculation: Tap **Save** under the measurement name.

To calculate Cardiac Output (CO) or Cardiac Index (CI)

The CO and CI calculations require Stroke Volume and Heart Rate calculations. CI also requires Body Surface Area (BSA). After the measurements are saved, the result appears in the patient report.

- 1 (CI Only) Fill in the **Height** and **Weight** fields on the patient form. The BSA is calculated automatically. (See "Entering patient information" on page 150.)
- 2 Calculate SV. See "To calculate Stroke Volume (SV) or Stroke Index (SI)" on page 205.
- 3 Calculate HR. See "To measure Heart Rate (HR)" on the previous page.

To measure a Tissue Doppler Imaging (TDI) waveform

- 1 Ensure that TDI is on. (See "Controls in Doppler" on page 120.)
- 2 On a frozen Doppler spectral trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 3 Do the following for each measurement you want to take:
 - a. On the TDI calculation list, tap the measurement name.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- b. Position the calipers by dragging.
- c. Save the calculation: Tap **Save** under the measurement name.

Gynecological calculations

WARNING To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.

WARNING To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

The Gynecological calculations include measurements for uterus, ovaries, and follicles.

In the Gynecological exam, you can also calculate volume (see "Volume calculation" on page 188).

Measuring uterus

You can measure uterus length (L), width (W), height (H), and endometrial thickness. If you measure length, width, and height, the system also calculates the volume.

To measure uterus

- 1 On the frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the Uterus list, do the following for each measurement you want to take:



To select a calculation list, tap at the top of the current list, and then tap the desired list.

- a. In the Uterus list, tap the measurement name.
- b. Position the calipers by dragging.
- c. Save the measurement.

Measuring ovaries

You can take up to three distance (D) measurements on each ovary. The system also calculates the volume based on the three measurements.

To measure ovaries

- 1 On the frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **Ovary** calculation list, do the following for each measurement you want to take:



To select a calculation list, tap at the top of the current list, and then tap the desired list.

- a. Tap the measurement name under **Right Ovary** or **Left Ovary**.
- b. Position the calipers by dragging.
- c. Save the measurement.

Measuring follicles

On each side, you can save up to three distance (D) measurements per follicle, for up to ten follicles. The system can display up to eight measurements at once.

If you measure a follicle twice, the average appears in the report. If you measure a follicle three times, the average and a volume calculation appear in the report. Volumes are derived based on one distance, two distance, and three distance measurements.

To measure follicles

- 1 On the frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 On the **Follicle Right** (if measuring the right side) or **Follicle Left** (if measuring the left side) calculation list, do the following for each measurement you want to take:



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- a. Tap the measurement name.
The measurement names appear under **1**, **2**, **3**, and so on, to pertain to the 10 follicle calculations available.
- b. Position the calipers by dragging.
- c. Under the measurement name, tap **Save Set** or to measure another follicle, tap **Next**.
A number appears before the measurement name and on the image. For the first measurement, 1 is used. For the second measurement, 2 is used, and so on.

Obstetrics calculations

- WARNING** Make sure that you have selected the Obstetrics exam type and the OB author for the Obstetrical calculation table you intend to use. See "Obstetrics calculations settings" on page 36.
- WARNING** To avoid incorrect obstetrics calculations, verify with a local clock and calendar that the system's date and time settings are correct before each use of the system. The system does not automatically adjust for daylight savings time changes.
- WARNING** Prior to use, verify that Obstetrical custom table data entries are correct. The system does not confirm the accuracy of the custom table data entered by the user.
- WARNING** To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.

WARNING | To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

In X-Porte, you can calculate gestational age, fetal heart rate, middle cerebral artery, and umbilical artery blood flow velocities.

You can select authors for Obstetrical calculations. See "Obstetrics calculations settings" on page 36 and "Measurement publications and terminology" on page 229.

EFW is calculated only after appropriate measurements are completed. If any one of these parameters results in an EDD greater than what the Obstetrical calculation tables provide, the EFW is not displayed.

Note | If you change the calculation author during the exam, the common measurements are retained.

Table 5-3 Results from system-defined obstetrical measurements and table authors

Calculation result	Gestational OB measurements	Available authors
Gestational Age ^a	YS	—
	GS	Hansmann, Nyberg, Tokyo U.
	CRL	Hadlock, Hansmann, Osaka, Tokyo U.
	BPD	Chitty, Hadlock, Hansmann, Osaka, Tokyo U.
	OFD	Hansmann
	HC	Chitty, Hadlock, Hansmann
	TTD	Hansmann, Tokyo U. ^b
	APTD	Tokyo U. ^b
	AC	Hadlock, Hansmann, Tokyo U.
	FTA	Osaka
	FL	Chitty, Hadlock, Hansmann, Osaka, Tokyo U.
	HL	Jeanty
	TL	Jeanty
	Cereb D	—
	CM	—
	Lat Vent	—
	Cx Len	—

Table 5-3 Results from system-defined obstetrical measurements and table authors (continued)

Calculation result	Gestational OB measurements	Available authors
Estimated Fetal Weight (EFW) ^c	HC, AC, FL	Hadlock 1
	BPD, AC, FL	Hadlock 2
	AC, FL	Hadlock 3
	BPD, TTD	Hansmann
	BPD, FTA, FL	Osaka U.
	BPD, AC	Shepard
	BPD, TTD, APTD, FL	Tokyo U.
Ratios	HC/AC	Campbell
	FL/AC	Hadlock
	FL/BPD	Hohler
	FL/HC	Hadlock
Amniotic Fluid Index	Q ¹ , Q ² , Q ³ , Q ⁴	Jeng
Growth Analysis Tables ^d	BPD	Chitty, Hadlock, Jeanty
	HC	Chitty, Hadlock, Jeanty
	AC	Chitty, Hadlock, Jeanty
	FL	Chitty, Hadlock, Jeanty
	EFW	Brenner, Hadlock, Jeanty
	HC/AC	Campbell

^aThe Gestational Age is automatically calculated and displayed next to the Obstetrical measurement you selected. The average of the results is the AUA. Only measurements of the same type are averaged.

^bFor Tokyo U., APTD and TTD are used only to calculate EFW. No age or growth tables are associated with these measurements.

^cThe Estimated Fetal Weight (EFW) calculation uses an equation that consists of one or more fetal biometry measurements. The author for the Obstetrical tables, which you choose on a system setup page, determines the measurements you must perform to obtain an EFW calculation.

Individual selections for Hadlock's EFW equations 1, 2, and 3 are not determined by the user. The selected equation is determined by the measurements that have been saved to the patient report with priority given to the order listed above.

^dThe Growth Analysis tables are used by the Report Graphs feature. Three growth curves are drawn using the table data for the selected growth parameter and published author. Growth tables are available only with a user-entered LMP or EDD.

To measure gestational growth (2D)

For each 2D Obstetrical measurement (except AFI), the system saves up to three individual measurements and their average. If you take more than three measurements, the earliest measurement is deleted.

- 1 Make sure that the Obstetrics exam type is selected.
- 2 On the patient form, select **LMP** or **EDD**. Select **Twins** if twins.
- 3 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 4 Do the following for each measurement you want to take:
 - a. Select the calculation list that contains the measurement.



To select a calculation list, tap  at the top of the current list, and then tap the desired list. For twins, there are two sets of calculation lists: Each list in one set is labeled **Twin A**, and each list in the other set is labeled **Twin B**.

- b. Tap the measurement name.
The caliper tool may change depending on the measurement selected, but the position remains constant.
 - c. Position the calipers by dragging.
 - d. Tap **Save** under the measurement name.

To measure heart rate (M Mode or Doppler)

- 1 Make sure that the Obstetrics exam type is selected.
- 2 On a frozen M Mode or Doppler trace, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

3 Tap **HR** on the following calculation list:

- For M Mode, the **HR** calculation list (or, if twins, the **Twin A HR** list or **Twin B HR** list).
- For Doppler, the **Doppler Measurements** list (or, if twins, the **Twin A Doppler Measurements** list or **Twin B Doppler Measurements** list).

A vertical caliper appears.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

4 Drag the vertical caliper to the peak of the heartbeat.


A second vertical caliper appears.

5 Drag the second vertical caliper to the peak of the next heartbeat.

6 Tap **Save** under the measurement name.

To calculate the S/D ratio, RI or PI of the Middle Cerebral Artery (MCA) or Umbilical Artery (UmbA) (Doppler)

- 1 Make sure that the Obstetrics exam type is selected.
- 2 On the patient form, select **LMP** or **EDD**. Select **Twins** if twins.
- 3 On a frozen Doppler spectral trace, tap **Calcs**.

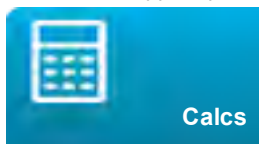

If this control is hidden, tapping **More Controls** displays it.

4 Do the following for each measurement you need to take:

- a. On the **Doppler Measurements** calculations list, tap the measurement name under **MCA** or **UmbA**.
- b. Position the calipers:
 - For **S/D**, **RI**, drag the first caliper to the peak systolic waveform. Tap **Next**, and then drag the second caliper to the end diastole on the waveform.
 - For **S/D**, **RI**, **PI**, drag the caliper to the beginning of the desired waveform, and tap **Set**. Trace the desired area, and then tap **Close**.

If calipers are positioned imprecisely, the calculation result is inaccurate.

5 Save the calculation.

Only one calculation (**S/D**, **RI** or **S/D**, **RI**, **PI**) can be saved.

Required measurements for MCA or UmbA

Table 5-4 MCA or UmbA calculations and results

Calculation list	Section heading	Measurement name	Results
Doppler Measurements	MCA	Systolic Diastolic S/D, RI	SD RI
		Systolic Diastolic S/D, RI, PI*	SD RI PI
	Umb A	Systolic Diastolic S/D, RI	RI SD
		Systolic Diastolic S/D, RI, PI*	SD RI PI

*Calculation requires a trace measurement.

Related topics

Cardiac calculations	192
Calculations settings	35
Worksheets and reports	220
Acute Care calculations	217

Measuring gestational sacs

You can make more than one Gestational Sac measurement on the same frozen image: tap **Next**, make an additional measurement, and then tap **Save**.

To make a single Gestational Sac measurement, tap **Save** after the measurement.

Small Parts calculations

- WARNING

To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.
- WARNING

To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

The Small Parts calculations include hip angle and hip ratio.

In the Small Parts exam, you can also calculate volume and percent reduction. See "Volume calculation" on page 188 and "Percent reduction calculations" on page 186.

To calculate hip angle

- 1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

- 2 Do the following under **Right Hip** and again under **Left Hip**:



To select a calculation list, tap at the top of the current list, and then tap the desired list.

- a. Under **Hip Angle**, tap **Baseline**.
A baseline with calipers appears.
- b. Position the baseline by dragging the calipers.
- c. Tap **Line A** (alpha line).
Calipers for Line A appear.
- d. Position Line A by dragging calipers, and tap **Save**.
- e. Tap **Line B** (beta line).
Calipers for Line B appear.
- f. Position Line B by dragging calipers, and tap **Save**.

To calculate hip ratio

- 1 On a frozen 2D image, tap **Calcs**.



If this control is hidden, tapping **More Controls** displays it.

2 Do the following under **Right Hip** and again under **Left Hip**:



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- a. Under **d:D Ratio**, tap **Fem Hd** (femoral head).
An ellipse with calipers appears.
- b. Position and resize the ellipse by dragging its calipers.
- c. Tap **Baseline**.
The baseline appears automatically.
- d. Position the baseline by dragging its caliper.

Acute Care calculations

- WARNING

To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.
- WARNING

To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared. See "Entering patient information" on page 150.

The Acute Care calculations are part of the licensed Acute Care feature. Results from Acute Care calculations appear in the Acute Care worksheets. (See "Worksheets and reports" on page 220.)

You can perform any of the following calculations, in any exam type. Distance measurements are in mm.

Table 5-5 *Acute Care calculations*

Calculation list	Measurement name (imaging mode)
Abdominal Aorta	Maximal aortic diameter (2D)

Table 5-5 Acute Care calculations (continued)


Calculation list	Measurement name (imaging mode)
Obstetrical Pelvic	Yolk Sac (2D) Gestational Sac (2D) Myometrial Thickness (2D) CRL (2D)* BPD (2D)* Fetal HR (M Mode) Adnexa ▶ Right Ovarian Cyst (2D) ▶ Left Ovarian Cyst (2D) <i>*Calculates a gestational age. See "Obstetrics calculations " on page 209for related information. Uses the selected gestational age table. See "Calculations settings" on page 35.</i>
Non-Obstet- rical Pelvic	Endometrial Stripe (2D) Uterine Fibroid (2D) Adnexa Right Ovarian Size (2D)* ▶ Length ▶ Width ▶ Height ▶ Right Ovarian Cyst ▶ Resistive index (Doppler) Adnexa Left Ovarian Size (2D)* ▶ Length ▶ Width ▶ Height ▶ Left Ovarian Cyst ▶ Resistive index (Doppler) <i>* Volume is calculated in ML after Length, Width, and Height measurements are taken.</i>

Table 5-5 Acute Care calculations (continued)

Calculation list	Measurement name (imaging mode)
Biliary	Gallstone (2D) Gallbladder Wall (2D) CBD (2D) Transverse Gallbladder (2D) Longitudinal Gallbladder (2D)
Renal/ Urinary Tract	Right Kidney (2D) ▸ Kidney Stone ▸ Renal Cyst Left Kidney (2D) ▸ Kidney Stone ▸ Renal Cyst Bladder (2D)* ▸ Width ▸ Height ▸ Length <i>* Bladder volume is calculated in ML after Width, Height, and Length measurements are taken.</i>
Cardiac	TR VMAx (Doppler) Aortic Root (2D) Thoracic Aorta Diameter (2D) IVC ▸ Maximum Diameter (2D) ▸ Minimum Diameter (2D)
Soft Tissue/ MSK	Tissue Thickness (2D) Subcutaneous Collection Diameter (2D)
Ocular	Eye ▸ Right ONSD (2D) ▸ Left ONSD (2D)

To perform an Acute Care calculation

- 1 On a frozen image, tap **Acute Care**.

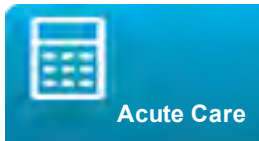

If this control is hidden, tapping **More Controls** displays it.

- 2 Do the following for each measurement you want to take:
- a. Select the calculation list that contains the measurement.



To select a calculation list, tap  at the top of the current list, and then tap the desired list.

- b. Tap the measurement name.
- c. Position the calipers by dragging.
If a Doppler measurement, drag the caliper to a peak systolic waveform, and tap **Next**. A second caliper appears. Drag the second caliper to the end diastole on the waveform.
- d. Save the measurement: Tap **Save** or **Save Set** under the measurement name.

Related topics

Worksheets and reports	220
Cardiac calculations	192
Obstetrics calculations	209

Worksheets and reports

The system collects patient information, calculation results, and worksheet findings into a *patient report*. You can view and edit the report preview and worksheets at any time until the exam has ended. After the exam has ended, you can view the patient report (see "Displaying reports and worksheets after the exam has ended" on page 225).

The value for a calculation appears in the report only if the calculation is saved. The pound symbol (#) indicates a value that is out of range (for example, too large or small). Calculation values that are out of range are not included in derived calculations (for example, mean). Calculations appear in the order that they were saved.

When you archive or export the exam, the report is included in the exam.

If they are licensed on your system, Acute Care worksheets and MSK worksheets are available. See "Acute Care and MSK worksheets" on page 223.

Related topics

Reviewing	158
DICOM	46
Entering patient information	150

Printing images	164
Acute Care calculations	217
Cardiac calculations	192
Obstetrics calculations	209
Archiving and exporting	165

Report preview

To display the report preview

- 1 Tap **WORKSHEETS**.




- 2 (Arterial or Cardiac exam) Tap **Summary** or **Calculations** in the **Report Preview** list. The mean of the detail entries is used in the summary.
You can switch between these views as needed.
To exit the report preview and return to imaging, tap **Done**.



To delete calculation values from the report preview

- 1 Display the report preview.
2 (Arterial or Cardiac exam) Tap **Calculations**.
3 Select the value, and then tap **Delete**.

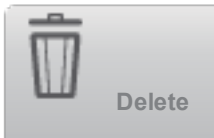

Deleting some measurements also deletes related measurements. Deleted measurements are not included in the summary information.

Arterial and cardiac reports

(Arterial) To modify the ICA/CCA ratio

- 1 Display the arterial report preview.
2 Tap **Summary**.

- 3 Select measurements for the ICA/CCA ratio for both the right and left sides from the **Ratio** list.

(Cardiac) To adjust the RA pressure

- 1 Display the cardiac report preview.
- 2 Tap **Summary**.
- 3 Select the desired value from the RA list.
Changing the RA pressure from the default value of 5 MmHG affects the RVSP calculation result.

Obstetrics reports

The Obstetrics calcs reports have a space for signing printed reports.

To display the Obstetrics Twins report preview

- In the Obstetrics report preview, do one of the following:
 - To display an individual twin report preview, tap **Twin A** or **Twin B**.
 - To display both twins in one report preview, tap **Compare**.

To fill out the Obstetrics anatomy checklist

You can document reviewed anatomy.

- 1 In the Obstetrics report preview, tap **Anatomy Checklist**.
- 2 Do any of the following:
 - Select the check boxes.
 - Enter the **Fetal Lie**.
 - Enter the **Plac Location** and select the grading method (**0**, **I**, **II**, or **III**) from the corresponding list.
 - Select a **Gender**.

To complete the Obstetrics biophysical profile

- In the Obstetrics report preview, select values under **Biophysical Profile**.
The total is calculated when values are selected. NST (non-stress test) is optional.

Obstetrics graphs


You can display Obstetrics graphs if the **LMP** or **EDD** text boxes are filled in the patient form, and if a Growth Analysis author has been selected for the measurement (see "Obstetrics calculations settings" on page 36).

To display Obstetrics graphs


- 1 Tap **WORKSHEETS**.



- 2 Tap **Graphs**.
- 3 In the **Graphs** list, select the desired measurement and author.
The graph for the selected measurement appears.
For twins, both measurement sets are plotted on the same graph.
- 4 (Optional) Tap **Save** to save the current graph page as an image that you can review as part of the exam. (See "Reviewing" on page 158.)

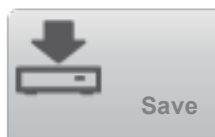

- 5 (Optional) Tap **Print Page** to print the current graph page.

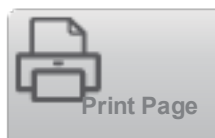

- 6 Tap **Done** to return to live imaging.



Acute Care and MSK worksheets

Acute Care and MSK (Musculoskeletal) worksheets are licensed features that include patient information, text boxes, and procedure information from the ultrasound system. Acute Care worksheets include results from measurements and calculations you perform from the Acute Care calculation lists.

To display an Acute Care or MSK worksheet

- 1 Tap **WORKSHEETS**.



- 2 Tap **Acute Care Worksheets** or **MSK Worksheets**.


- 3 Tap a worksheet in the list.
To exit the worksheet and return to imaging, tap **Done**.



To edit an Acute Care or MSK worksheet

You can edit worksheets on the system until the exam has ended.

- 1 Enter information in the worksheet:
 - **Indications**
 - (Acute Care worksheets) Select multiple indications, or select **Other Indications** to enter additional information.
 - (MSK worksheets) Enter indications in the Indications text box.
 - (Acute Care worksheets only) **Views** Select options to reflect the views obtained. Select **Other Views** to enter additional information.
 - (Acute Care worksheets only) **Findings** You can make multiple selections in the Findings section. Select **Other Findings** to enter additional information.
 - (Acute Care worksheets only) **Interpretation** You can make multiple selections in the Interpretation section. Select **Other Interpretations** to enter additional information.
 - (MSK worksheets only) **Procedure Details and Conclusions**. You can make multiple selections in the Procedure Details and Conclusions section. Enter comments in the Comments text box.
- 2 (Acute Care worksheets only) To delete a calculation value, select the value and then tap **Delete**.

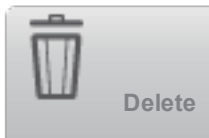

- 3 Tap **Done**.



Printing reports and worksheets


You can print the current page of a report preview or worksheet or the entire report preview or worksheet.

To print a report preview or worksheet

- 1 Display the report preview or worksheet.
- 2 Do one of the following:
 - To print the entire report preview or worksheet, tap **Print All**.



- To print the current page, tap **Print Page**.

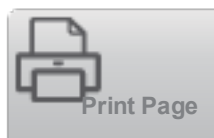

Displaying reports and worksheets after the exam has ended


When you end an exam, the system saves the *patient report* with all measurements and calculations performed during the exam, including Acute Care or MSK worksheet data, if available.

To display the report after the exam has ended

- 1 Tap **REVIEW**.




- 2 Tap **Patient List**.

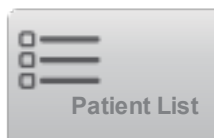

- 3 Select an exam.
- 4 Tap **Thumbnails**.

- 5 Tap **Report**.

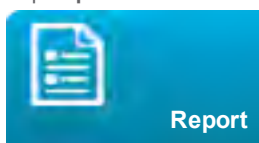

The system displays the read-only report and any associated worksheets

- 6 Tap **Done** to return to the patient list.



CHAPTER 6

Measurement references

This section provides information about measurement accuracy, publications, and terminology.

Measurement accuracy

The measurements from the system are of a physical property such as distance for evaluation by the clinician. The accuracy values require that you can place the calipers over one pixel. The values do not include acoustic anomalies of the body.

The 2D linear distance measurement results are displayed in centimeters with one place past the decimal point, if the measurement is ten or greater; two places past the decimal point, if the measurement is less than ten.

The linear distance measurement components have the accuracy and range shown in the following tables.

Table 6-1 2D Measurement Accuracy and Range

2D Measurement Accuracy and Range	System Tolerance ^a	Accuracy By	Test Method ^b	Range (cm)
Axial Distance	< ±2% plus 1% of full scale	Acquisition	Phantom	0-26 cm
Lateral Distance	< ±2% plus 1% of full scale	Acquisition	Phantom	0-35 cm
Diagonal Distance	< ±2% plus 1% of full scale	Acquisition	Phantom	0-44 cm
Area ^c The area accuracy is defined using the following equation: % tolerance = ((1 + lateral error) * (1 + axial error) – 1) * 100 + 0.5%.	< ±4% plus (2% of full scale/smallest dimension) * 100 plus 0.5%	Acquisition	Phantom	0.01-720 cm ²
Circumference ^d The circumference accuracy is defined as the greater of the lateral or axial accuracy and by the following equation: % tolerance = (2 (maximum of 2 errors) * 100) + 0.5%.	< ±3% plus (1.4% of full scale/ smallest dimension) * 100 plus 0.5%	Acquisition	Phantom	0.01-96 cm

Table 6-2 M Mode Measurement and Calculation Accuracy and Range

M Mode Measurement Accuracy and Range	System Tolerance	Accuracy By	Test Method	Range
Distance	< +/- 2% plus 1% of full scale ^a	Acquisition	Phantom ^b	0-26 cm
Time	< +/- 2% plus 1% of full scale ^c	Acquisition	Phantom ^d	0.01-10 sec
Heart Rate	< +/- 2% plus (Full Scale ^e * Heart Rate/100) %	Acquisition	Phantom ^f	5-923 bpm

Table 6-3 PW Doppler Mode Measurement and Calculation Accuracy and Range

Doppler Mode Measurement Accuracy and Range	System Tolerance	Accuracy By	Test Method ^f	Range
Velocity cursor	< +/- 2% plus 1% of full scale ^g	Acquisition	Phantom	0.01 cm/sec-550 cm/sec
Frequency cursor	< +/- 2% plus 1% of full scale ^g	Acquisition	Phantom	0.01kHz-20.8 kHz
Time	< +/- 2% plus 1% of full scale ^e	Acquisition	Phantom	0.01-10 sec

^aFull scale for distance implies the maximum depth of the image.

^bAn RMI 413a model phantom with 0.7 dB/cm MHz attenuation was used.

^cThe area accuracy is defined using the following equation:
% tolerance = ((1 + lateral error) * (1 + axial error) – 1) * 100 + 0.5%.

^dThe circumference accuracy is defined as the greater of the lateral or axial accuracy and by the following equation:
% tolerance = (2 (maximum of 2 errors) * 100) + 0.5%.

^eFull scale for time implies the total time displayed on the scrolling graphic image.

^fFUJIFILM SonoSite special test equipment was used.

^gFull scale for frequency or velocity implies the total frequency or velocity magnitude, displayed on the scrolling graphic image.

Sources of measurement errors

In general, two types of errors can be introduced into the measurement:

Acquisition Error Includes errors introduced by the ultrasound system electronics relating to signal acquisition, signal conversion, and signal processing for display. Additionally, computational and display errors are introduced by the generation of the pixel scale factor, application of that factor to the caliper positions on the screen, and the measurement display.

Algorithmic Error The error introduced by measurements, which are input to higher order calculations. This error is associated with floating-point versus integer-type math, which is subject to errors introduced by round- versus truncating results for display of a given level of significant digit in the calculation.

Measurement publications and terminology

The following are the publications and terminology used for each calculation result.

Terminology and measurements comply with American Institute of Ultrasound in Medicine (AIUM) published standards.

Cardiac references

Acceleration (ACC) in cm/s²

Zwibel, W.J. *Introduction to Vascular Ultrasonography*. 4th ed., W.B. Saunders Company, (2000), 52.

ACC = abs (delta velocity/delta time)

Acceleration Time (AT) in msec

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Lippincott, Williams, and Wilkins, (1999), 219.

Aortic Valve Area (AVA) by Continuity Equation in cm²

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 393, 442.

$$A_2 = A_1 * V_1 / V_2$$

where:

A_2 = A_o valve area

A_1 = LVOT area; V_1 = LVOT

velocity; V_2 = A_o valve velocity

LVOT = Left Ventricular Outflow Tract

AVA (PV_{LVOT} / PV_{AO}) * CSA_{LVOT}

AVA (VTI_{LVOT} / VTI_{AO}) * CSA_{LVOT}

Body Surface Area (BSA) in m²

Grossman, W. *Cardiac Catheterization and Angiography*. Philadelphia: Lea and Febiger, (1980), 90.

$$BSA = 0.007184 * \text{Weight}^{0.425} * \text{Height}^{0.725}$$

Weight = kilograms

Height = centimeters

Cardiac Index (CI) in l/min/m²

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd Edition, Boston: Little, Brown and Company, (1999), 59.

$$CI = CO / BSA$$

where:

CO = Cardiac Output

BSA = Body Surface Area

Cardiac Output (CO) in l/min

Oh, J.K., J.B. Seward, A.J. Tajik *The Echo Manual*. 2nd ed., Lippincott, Williams, and Wilkins, (1999), 59.

$$CO = (SV * HR) / 1000$$

where:

CO = Cardiac Output

SV = Stroke Volume

HR = Heart Rate

Cross Sectional Area (CSA) in cm²

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 383.

$$CSA = 0.785 * D^2$$

where: D = diameter of the anatomy of interest

Deceleration Time in msec

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 453.

$$|time\ a - time\ b|$$

Delta Pressure: Delta Time (dP:dT) in mmHg/s

Otto, C.M. *Textbook of Clinical Echocardiography*. 2nd ed., W.B. Saunders Company, (2000), 117, 118.

$$32\text{ mmHg/time interval in seconds}$$

E:A Ratio in cm/sec

$$E:A = \text{velocity E} / \text{velocity A}$$

E/Ea Ratio

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 225.

$$E\text{ Velocity} / Ea\text{ velocity}$$

where:

E velocity = Mitral Valve E velocity

Ea = annular E velocity, also known as: E prime

Effective Regurgitant Orifice (ERO) in mm²

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 455.

$$\text{ERO} = 6.28 (r^2) * \text{Va} / \text{MR Vel}$$

where:

r = radius

Va = aliasing velocity

Ejection Fraction (EF), percent

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Lippincott, Williams, and Wilkins, (1999), 40.

$$\text{EF} = ((\text{LVEDV} - \text{LVESV}) / \text{LVEDV}) * 100\%$$

where:

EF = Ejection Fraction

LVEDV = Left Ventricular End Diastolic Volume

LVESV = Left Ventricular End Systolic Volume

Elapsed Time (ET) in msec

ET = time between velocity cursors in milliseconds

Heart Rate (HR) in bpm

HR = 3 digit value input by user or measured on M Mode and Doppler image in one heart cycle

Interventricular Septum (IVS) Fractional Thickening, percent

Laurenceau, J. L., M.C. Malergue. *The Essentials of Echocardiography*. Le Hague: Martinus Nijhoff, (1981), 71.

$$\text{IVSFT} = ((\text{IVSS} - \text{IVSD}) / \text{IVSD}) * 100\%$$

where:

IVSS = Interventricular Septal Thickness at Systole

IVSD = Interventricular Septal Thickness at Diastole

Isovolumic Relaxation Time (IVRT) in msec

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. School of Cardiac Ultrasound, Arizona Heart Institute, (1993), 146.

|time a - time b|

Left Atrium/Aorta (LA/Ao)

Feigenbaum, H. *Echocardiography*. Philadelphia: Lea and Febiger, (1994), 206, Figure 4-49.

Left Ventricular End Volumes (Teichholz) in ml

Teichholz, L.E., T. Kreulen, M.V. Herman, et. al. "Problems in echocardiographic volume determinations: echocardiographic-angiographic correlations in the presence or absence of asynergy." *American Journal of Cardiology*, (1976), 37:7.

$$\text{LVESV} = (7.0 * \text{LVDS}^3) / (2.4 + \text{LVDS})$$

where:

LVESV = Left Ventricular End Systolic Volume

LVDS = Left Ventricular Dimension at Systole

$$\text{LVEDV} = (7.0 * \text{LVDD}^3) / (2.4 + \text{LVDD})$$

where:

LVEDV = Left Ventricular End Diastolic Volume

LVDD = Left Ventricular Dimension at Diastole

Left Ventricular Mass in gm for 2D

Schiller, N.B., P.M. Shah, M. Crawford, et.al. "Recommendations for Quantification of the Left Ventricle by Two-Dimensional Echocardiography." *Journal of American Society of Echocardiography*. September-October 1998, 2:364.

$$\text{LV Mass} = 1.05 * \{[(5/6) * A1 * (a + d + t)] - [(5/6) * A2 * (a + d)]\}$$

where:

A1 = short axis area, diastole (Epi)

A2 = short axis area, diastole (Endo)

a = long or semi major axis

d = truncated semi major axis from the widest short axis diameter to mitral annulus plane.

t = myocardial thickness

Left Ventricular Mass in gm for M Mode

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd Edition, Boston: Little, Brown and Company, (1999), 39.

$$\text{LV Mass} = 1.04 [(\text{LVID} + \text{PWT} + \text{IVST})^3 - \text{LVID}^3] * 0.8 + 0.6$$

where:

LVID = Internal Dimension
PWT = Posterior Wall Thickness
IVST = Interventricular Septal Thickness
1.04 = Specific gravity of the myocardium
0.8 = Correction factor

Left Ventricular Volume: Biplane Method in ml

Schiller, N.B., P.M. Shah, M. Crawford, et.al. "Recommendations for Quantitation of the Left Ventricle by Two-Dimensional Echocardiography." *Journal of American Society of Echocardiography*. September-October 1989, 2:362.

$$V = \left(\frac{\pi}{4}\right) \sum_{i=1}^n a_i b_i \left(\frac{L}{n}\right)$$

where:

V = Volume in ml
a = Diameter
b = Diameter
n = Number of segments (n=20)
L = Length
i = Segment

Left Ventricular Volume: Single Plane Method in ml

Schiller, N.B., P.M. Shah, M. Crawford, et.al. "Recommendations for Quantitation of the Left Ventricle by Two-Dimensional Echocardiography." *Journal of American Society of Echocardiography*. September-October 1989, 2:362.

$$V = \left(\frac{\pi}{4}\right) \sum_{i=1}^n a_i^2 \left(\frac{L}{n}\right)$$

where:

V = Volume
a = Diameter
n = Number of segments (n=20)
L = Length
i = Segment

Left Ventricular Dimension (LVD) Fractional Shortening, percent

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. Boston: Little, Brown and Company, (1994), 43-44.

$$\text{LVDFS} = ((\text{LVDD} - \text{LVDS}) / \text{LVDD}) * 100\%$$

where:

LVDD = Left Ventricle Dimension at Diastole
LVDS = Left Ventricle Dimension at Systole

LV Ejection Fraction

Schiller, N.B., Shah, P.M., Crawford, M., et. al. "Recommendations for Quantification of the Left Ventricle by Two-Dimensional Echocardiography." *Journal of American Society of Echocardiography*. September-October 1989, 2:364.

$$\text{EF} = ((\text{End Diastolic Volume} - \text{End Systolic Volume}) / \text{End Diastolic Volume}) * 100 (\%).$$

Left Ventricular Posterior Wall Fractional Thickening (LVPWFT), percent

Laurenceau, J. L., M.C. Malergue. *The Essentials of Echocardiography*. Le Hague: Martinus Nijhoff, (1981), 71.

$$\text{LVPWFT} = ((\text{LVPWS} - \text{LVPWD}) / \text{LVPWD}) * 100\%$$

where:

LVPWS = Left Ventricular Posterior Wall Thickness at Systole
LVPWD = Left Ventricular Posterior Wall Thickness at Diastole

Mean Velocity (Vmean) in cm/s

Vmean = mean velocity

Mitral Valve Area (MVA) in cm²

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 391, 452.

$$\text{MVA} = 220 / \text{PHT}$$

where: PHT = pressure half time

220 is an empirical derived constant and may not accurately predict mitral valve area in mitral prosthetic heart valves. The mitral valve area continuity equation may be utilized in mitral prosthetic heart valves to predict effective orifice area.

MV Flow Rate in cc/sec

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 396.

$$\text{Flow} = 6.28 (r^2) * V_a$$

where:

r = radius

V_a = aliasing Velocity

Pressure Gradient (PGr) in mmHG

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Lippincott, Williams, and Wilkins, (1999), 64.

$$\text{PGr} = 4 * (\text{Velocity})^2$$

Peak E Pressure Gradient (E PG)

$$\text{E PG} = 4 * \text{PE}^2$$

Peak A Pressure Gradient (A PG)

$$\text{A PG} = 4 * \text{PA}^2$$

Peak Pressure Gradient (PGmax)

$$\text{PGmax} = 4 * \text{PV}^2$$

Mean Pressure Gradient (PGmean)

PGmean = Average of pressure gradients/Duration of flow

Pressure Half Time (PHT) in msec

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 391.

$$\text{PHT} = \text{DT} * 0.29$$

where: DT = deceleration time

Proximal Isovelocity Surface Area (PISA) in cm²

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Boston: Little, Brown and Company, (1999), 125.

$$\text{PISA} = 2 \pi r^2$$

where:

$$2\pi = 6.28$$

r = aliasing radius

Qp/Qs

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 400.

$Qp/Qs = SV_{Qp\ site} / SV_{Qs\ site}$

SV sites will vary depending upon the location of the shunt.

Regurgitant Fraction (RF) in percent

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. Boston: Little, Brown and Company, (1999), 125.

$RF = RV / MV_{SV}$

where:

RV = Regurgitant Volume

MV SV = Mitral Stroke Volume

Regurgitant Volume (RV) in cc

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 396, 455.

$RV = ERO * MR_{VTI}$

Right Ventricular Systolic Pressure (RVSP) in mmHg

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. School of Cardiac Ultrasound, Arizona Heart Institute, (1993), 152.

$RVSP = 4 * (V_{max\ TR})^2 + RAP$

where: RAP = Right Atrial Pressure

S/D

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 217.

S velocity/D velocity

where:

S velocity = Pulmonary vein S wave

D velocity= Pulmonary vein D wave

Stroke Index (SI) in cc/m²

Mosby's Medical, Nursing, & Allied Health Dictionary, 4th ed., (1994), 1492.

$SI = SV / BSA$

where:

SV = Stroke Volume

BSA = Body Surface Area

Stroke Volume (SV) Doppler in ml

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Lippincott, Williams, and Wilkins, (1999), 40, 59, 62.

$$SV = (CSA \times VTI)$$

where:

CSA = Cross Sectional Area of the orifice (LVOT area)

VTI = Velocity Time Integral of the aortic valve

Stroke Volume (SV) 2D and M Mode in ml

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Boston: Little, Brown and Company, (1994), 44.

$$SV = (LVEDV - LVESV)$$

where:

SV = Stroke Volume

LVEDV = End Diastolic Volume

LVESV = End Systolic Volume

Tricuspid Valve Area (TVA)

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 55, 391, 452.

$$TVA = 220 / PHT$$

Velocity Time Integral (VTI) in cm

Reynolds, Terry. *The Echocardiographer's Pocket Reference*. 2nd ed., School of Cardiac Ultrasound, Arizona Heart Institute, (2000), 383.

VTI = absolute peak velocities integrated between two points in time, expressed in units of cm.

When using Auto Measure, VTI is always based on the peak trace and represents the distance (cm) that blood travels in the time interval represented by the delimiters (presumably an ejection period).

Obstetrical references

Amniotic Fluid Index (AFI)

Jeng, C. J., et al. "Amniotic Fluid Index Measurement with the Four Quadrant Technique During Pregnancy." *The Journal of Reproductive Medicine*, 35:7 (July 1990), 674-677.

Average Ultrasound Age (AUA)

The system provides an AUA derived from the component measurements from the measurement tables.

Estimated Date of Delivery (EDD) by Average Ultrasound Age (AUA)

Results are displayed as month/day/year.

EDD = system date + (280 days – AUA in days)

Estimated Date of Delivery (EDD) by Last Menstrual Period (LMP)

The date entered into the patient information for LMP must precede the current date.

Results are displayed as month/day/year.

EDD = LMP date + 280 days

Estimated Fetal Weight (EFW)

Hadlock, F., et al. "Estimation of Fetal Weight with the Use of Head, Body, and Femur Measurements, A Prospective Study." *American Journal of Obstetrics and Gynecology*, 151:3 (February 1, 1985), 333-337.

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 154.

Osaka University. *Ultrasound in Obstetrics and Gynecology*. (July 20, 1990), 103-105.

Shepard M.J., V. A. Richards, R. L. Berkowitz, et al. "An Evaluation of Two Equations for Predicting Fetal Weight by Ultrasound." *American Journal of Obstetrics and Gynecology*, 142:1 (January 1, 1982), 47-54.

University of Tokyo, Shinozuka, N. FJSUM, et al. "Standard Values of Ultrasonographic Fetal Biometry." *Japanese Journal of Medical Ultrasonics*, 23:12 (1996), 880, Equation 1.

Gestational Age (GA) by Last Menstrual Period (LMP)

The gestational age derived from the LMP date entered on the patient form.

Results are displayed in weeks and days, and is calculated as follows:

GA(LMP) = System date – LMP date

Gestational Age (GA) by Last Menstrual Period (LMPd) Derived from Established Due Date (EDD)

Same as GA by EDD.

The gestational age derived from the system derived LMP using the Established Due Date entered on the patient form.

Results are displayed in weeks and days, and is calculated as follows:

GA(LMPd) = System Date – LMPd

Last Menstrual Period Derived (LMPd) by Established Due Date (EDD)

Results are displayed as month/day/year.

$\text{LMPd(Estab. DD)} = \text{Estab. DD} - 280 \text{ days}$

Gestational age tables

Abdominal Circumference (AC)

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 431.

University of Tokyo, Shinozuka, N. FJSUM, et al. "Standard Values of Ultrasonographic Fetal Biometry." *Japanese Journal of Medical Ultrasonics*, 23:12 (1996), 885.

WARNING | The gestational age calculated by your FUJIFILM SonoSite system does not match the age in the aforementioned reference at the 20.0 cm and 30.0 cm abdominal circumference (AC) measurements. The implemented algorithm extrapolates the gestational age from the slope of the curve of all table measurements, rather than decreasing the gestational age for a larger AC measurement indicated in the referenced table. This results in the gestational age always increasing with an increase in AC.

Biparietal Diameter (BPD)

Chitty, L. S. and D.G. Altman. "New charts for ultrasound dating of pregnancy." *Ultrasound in Obstetrics and Gynecology* 10: (1997), 174-179, Table 3.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 440.

Osaka University. *Ultrasound in Obstetrics and Gynecology*. (July 20, 1990), 98.

University of Tokyo, Shinozuka, N. FJSUM, et al. "Standard Values of Ultrasonographic Fetal Biometry." *Japanese Journal of Medical Ultrasonics*, 23:12 (1996), 885.

Cisterna Magna (CM)

Mahony, B.; P. Callen, R. Filly, and W. Hoddick. "The fetal cisterna magna." *Radiology*, 153: (December 1984), 773-776.

Crown Rump Length (CRL)

Hadlock, F., et al. "Fetal Crown-Rump Length: Re-evaluation of Relation to Menstrual Age (5-18 weeks) with High-Resolution, Real-Time Ultrasound." *Radiology*, 182: (February 1992), 501-505.

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 439.

Osaka University. *Ultrasound in Obstetrics and Gynecology*. (July 20, 1990), 20 and 96.

Tokyo University. "Gestational Weeks and Computation Methods." *Ultrasound Imaging Diagnostics*, 12:1 (1982-1), 24-25, Table 3.

Femur Length (FL)

Chitty, L. S. and D.G. Altman. "New charts for ultrasound dating of pregnancy." *Ultrasound in Obstetrics and Gynecology* 10: (1997), 174-179, Table 8, 186.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 431.

Osaka University. *Ultrasound in Obstetrics and Gynecology*. (July 20, 1990), 101-102.

University of Tokyo, Shinozuka, N. FJSUM, et al. "Standard Values of Ultrasonographic Fetal Biometry." *Japanese Journal of Medical Ultrasonics*, 23:12 (1996), 886.

Fetal Trunk Cross-Sectional Area (FTA)

Osaka University. *Ultrasound in Obstetrics and Gynecology*. (July 20, 1990), 99-100.

Gestational Sac (GS)

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986).

Nyberg, D.A., et al. "Transvaginal Ultrasound." *Mosby Yearbook*, (1992), 76.

Gestational sac measurements provide a fetal age based on the mean of one, two, or three distance measurements; however, Nyberg's gestational age equation requires all three distance measurements for an accurate estimate.

Tokyo University. "Gestational Weeks and Computation Methods." *Ultrasound Imaging Diagnostics*, 12:1 (1982-1).

Head Circumference (HC)

Chitty, L. S. and D.G. Altman. "New charts for ultrasound dating of pregnancy." *Ultrasound in Obstetrics and Gynecology* 10: (1997), 174-191, Table 5, 182.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 431.

Humerus (HL)

Jeanty, P.; F. Rodesch; D. Delbeke; J. E. Dumont. "Estimate of Gestational Age from Measurements of Fetal Long Bones." *Journal of Ultrasound in Medicine*. 3: (February 1984), 75-79

Occipito-Frontal Diameter (OFD)

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 431.

Tibia

Jeanly, P.; F. Rodesch; D. Delbeke; J. E. Dumont. "Estimate of Gestational Age from Measurements of Fetal Long Bones." *Journal of Ultrasound in Medicine*. 3: (February 1984), 75-79.

Transverse Trunk Diameter (TTD)

Hansmann, M., et al. *Ultrasound Diagnosis in Obstetrics and Gynecology*. New York: Springer-Verlag, (1986), 431.

Growth analysis tables

Abdominal Circumference (AC)

Chitty, Lyn S. et al. "Charts of Fetal Size: 3. Abdominal Measurements." *British Journal of Obstetrics and Gynaecology* 101: (February 1994), 131, Appendix: AC-Derived.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Jeanly P., E. Cousaert, and F. Cantraine. "Normal Growth of the Abdominal Perimeter." *American Journal of Perinatology*, 1: (January 1984), 129-135.

(Also published in Hansmann, Hackeloer, Staudach, Wittman. *Ultrasound Diagnosis in Obstetrics and Gynecology*. Springer-Verlag, New York, (1986), 179, Table 7.13.)

Biparietal Diameter (BPD)

Chitty, Lyn S. et al. "Charts of Fetal Size: 2. Head Measurements." *British Journal of Obstetrics and Gynaecology* 101: (January 1994), 43, Appendix: BPD-Outer-Inner.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Jeanly P., E. Cousaert, and F. Cantraine. "A Longitudinal Study of Fetal Limb Growth." *American Journal of Perinatology*, 1: (January 1984), 136-144, Table 5.

(Also published in Hansmann, Hackeloer, Staudach, Wittman. *Ultrasound Diagnosis in Obstetrics and Gynecology*. Springer-Verlag, New York, (1986), 176, Table 7.8.)

Estimated Fetal Weight (EFW)

Brenner, William E.; D. A. Edelman; C. H. Hendricks. "A standard of fetal growth for the United States of America," *American Journal of Obstetrics and Gynecology*, 126: 5 (November 1, 1976), 555-564; Table II.

Hadlock F., et al. "In Utero Analysis of Fetal Growth: A Sonographic Weight Standard." *Radiology*, 181: (1991), 129-133.

Jeanty, Philippe, F. Cantraine, R. Romero, E. Cousaert, and J. Hobbins. "A Longitudinal Study of Fetal Weight Growth." *Journal of Ultrasound in Medicine*, 3: (July 1984), 321-328, Table 1.

(Also published in Hansmann, Hackeloer, Staudach, and Wittman. *Ultrasound Diagnosis in Obstetrics and Gynecology*. Springer-Verlag, New York, (1986), 186, Table 7.20.)

Femur Length (FL)

Chitty, Lyn S. et al. "Charts of Fetal Size: 4. Femur Length." *British Journal of Obstetrics and Gynaecology* 101: (February 1994), 135.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Jeanty P, E. Cousaert, and F. Cantraine. "A Longitudinal Study of Fetal Limb Growth." *American Journal of Perinatology*, 1: (January 1984), 136-144, Table 5.

(Also published in Hansmann, Hackeloer, Staudach, Wittman. *Ultrasound Diagnosis in Obstetrics and Gynecology*. Springer-Verlag, New York, (1986), 182, Table 7.17.)

Head Circumference (HC)

Chitty, Lyn S., et al. "Charts of Fetal Size: 2. Head Measurements." *British Journal of Obstetrics and Gynaecology* 101: (January 1994), 43, Appendix: HC-Derived.

Hadlock, F., et al. "Estimating Fetal Age: Computer-Assisted Analysis of Multiple Fetal Growth Parameters." *Radiology*, 152: (1984), 497-501.

Jeanty P, E. Cousaert, and F. Cantraine. "A longitudinal study of Fetal Head Biometry." *American J of Perinatology*, 1: (January 1984), 118-128, Table 3.

(Also published in Hansmann, Hackeloer, Staudach, Wittman. *Ultrasound Diagnosis in Obstetrics and Gynecology*. Springer-Verlag, New York, (1986), 176, Table 7.8.)

Head Circumference (HC)/Abdominal Circumference (AC)

Campbell S., Thoms Alison. "Ultrasound Measurements of the Fetal Head to Abdomen Circumference Ratio in the Assessment of Growth Retardation," *British Journal of Obstetrics and Gynaecology*, 84: (March 1977), 165-174.

Ratio calculations

FL/AC Ratio

Hadlock F.P., R. L. Deter, R. B. Harrist, E. Roecker, and S.K. Park. "A Date Independent Predictor of Intrauterine Growth Retardation: Femur Length/Abdominal Circumference Ratio," *American Journal of Roentgenology*, 141: (November 1983), 979-984.

FL/BPD Ratio

Hohler, C.W., and T.A. Quetel. "Comparison of Ultrasound Femur Length and Biparietal Diameter in Late Pregnancy," *American Journal of Obstetrics and Gynecology*, 141:7 (Dec. 1 1981), 759-762.

FL/HC Ratio

Hadlock F.P., R. B. Harrist, Y. Shah, and S. K. Park. "The Femur Length/Head Circumference Relation in Obstetric Sonography." *Journal of Ultrasound in Medicine*, 3: (October 1984), 439-442.

HC/AC Ratio

Campbell S., Thoms Alison. "Ultrasound Measurements of the Fetal Head to Abdomen Circumference Ratio in the Assessment of Growth Retardation," *British Journal of Obstetrics and Gynaecology*, 84: (March 1977), 165-174.

General references

+/x or S/D Ratio

$+/x = (\text{Velocity A} / \text{Velocity B})$

where:

A = velocity cursor +

B = velocity cursor x

Acceleration Index (ACC)

Zwiebel, W.J. *Introduction to Vascular Ultrasonography*, 4th ed., W.B. Saunders Company, (2000), 52.

$ACC = \text{abs}(\Delta \text{velocity} / \Delta \text{time})$

Elapsed Time (ET)

ET = time between velocity cursors in milliseconds

Hip Angle/d:D Ratio

Graf, R. "Fundamentals of Sonographic Diagnosis of Infant Hip Dysplasia." *Journal of Pediatric Orthopedics*, Vol. 4, No. 6: 735-740, 1984.

Morin, C., Harcke, H., MacEwen, G. "The Infant Hip: Real-Time US Assessment of Acetabular Development." *Radiology* 177: 673-677, December 1985.

Percent Area Reduction

Taylor K.J.W., P.N. Burns, P. Breslau. *Clinical Applications of Doppler Ultrasound*, Raven Press, N.Y., (1988), 130-136.

Zwiebel W.J., J.A. Zagzebski, A.B. Crummy, et al. "Correlation of peak Doppler frequency with lumen narrowing in carotid stenosis." *Stroke*, 3: (1982), 386-391.

$\% \text{ Area Reduction} = (1 - A2(\text{cm}^2) / A1(\text{cm}^2)) * 100$

where:

A1 = original area of the vessel in square cm

A2 = reduced area of the vessel in square cm

Percent Diameter Reduction

Handa, Nobuo et al., "Echo-Doppler Velocimeter in the Diagnosis of Hypertensive Patients: The Renal Artery Doppler Technique," *Ultrasound in Medicine and Biology*, 12:12 (1986), 945-952.

$$\% \text{ Diameter Reduction} = (1 - D2(\text{cm})/D1(\text{cm})) * 100$$

where:

D1 = original diameter of the vessel in cm

D2 = reduced diameter of the vessel in cm

Pressure Gradient (PGr) in mmHG

Oh, J.K., J.B. Seward, A.J. Tajik. *The Echo Manual*. 2nd ed., Lippincott, Williams, and Wilkins, (1999), 64.

$$4 * (\text{Velocity})^2$$

Peak E Pressure Gradient (E PG)

$$E \text{ PG} = 4 * PE^2$$

Peak A Pressure Gradient (A PG)

$$A \text{ PG} = 4 * PA^2$$

Peak Pressure Gradient (PGmax)

$$PG_{\text{max}} = 4 * PV^2$$

Mean Pressure Gradient (PGmean)

$$PG_{\text{mean}} = 4 * V_{\text{max}}^2$$

Pulsatility Index (PI)

Kurtz, A.B., W.D. Middleton. *Ultrasound-the Requisites*. Mosby Year Book, Inc., (1996), 469.

$$PI = (PSV - EDV)/V$$

where:

PSV = peak systolic velocity

EDV = end diastolic velocity

V = mean flow velocity throughout the entire cardiac cycle

Note | When using **Auto Measure**, the mean flow velocity is TAP (Time Average Peak).

Zwiebel, W. J. *Introduction to Vascular Ultrasonography*, 6th Edition, W.B. Saunders Company, 2012.

$$PI = (PSV - MDV) / V \text{ (no units)}$$

where:

PSV = peak systolic velocity

MDV = Minimum diastolic velocity

V = TAP (Time Averaged Peak) flow velocity throughout the cardiac cycle

Resistive Index (RI)

Kurtz, A.B., W.D. Middleton. *Ultrasound-the Requisites*. Mosby Year Book, Inc., (1996), 467.

RI = ((Velocity A – Velocity B)/Velocity A) in measurements

where:

A = velocity cursor +

B = velocity cursor x

Time Averaged Mean (TAM) in cm/s

TAM = mean (mean Trace)

Time Averaged Peak (TAP) in cm/s

TAP = peak (peak Trace)

Volume (Vol)

Beyer, W.H. *Standard Mathematical Tables*, 28th ed., CRC Press, Boca Raton, FL, (1987), 131.

Volume Bladder

Dicuio, M., et.al. "Measurements of urinary bladder volume: comparison of five ultrasound calculation methods in volunteers." *Arch. Ital. Urol Androl*, (2005) Mar:77(1):60-2.

Volume Flow (VF) in ml/m

Allan, Paul L. et al. *Clinical Doppler Ultrasound*, 4th ed., Harcourt Publishers Limited. (2000), 36-38.

One of the following, depending on the Live Trace setting:

VF = CSA * TAM * 60

VF = CSA * TAP * 60

Volume Follicle

Raine-Fenning, N. et al. *Ultrasound in Obstetrics & Gynecology* (2008), Volume 31, Issue 6, John Wiley & Sons, Ltd., 691–696.

Volume Ovarian

Balen, A.H., Laven, J.S., Tan, S.L., Dewailly, D. "Ultrasound assessment of the polycystic ovary: international consensus definitions." *Human Reproduction Update*, (2003), Vol.9, No.6 pp. 505-514.

Volume Uterine

Wiener, J. J. and Newcombe, R. G. "Measurements of uterine volume: a comparison between measurements by ultrasonography and by water displacement." *J. Clin. Ultrasound*, (1992) 20 (7), 457–460.

CHAPTER 7

Troubleshooting and maintenance

This section contains information to help correct problems with system operation, to enter a software license, and to take proper care of the system, transducer, and accessories.

Troubleshooting

If you encounter difficulty with the system, use the following list to help troubleshoot the problem. If the problem persists, contact FUJIFILM SonoSite Technical Support. (See "FUJIFILM SonoSite Technical Support" on page 3.)

System does not turn on. Check all power connections.

Unplug and plug back in the power supply AC and DC power cables (desktop system).

Check that all three battery switches are in the ON position, which is indicated by the **ⓘ** symbol, and ensure that the batteries are charged (stand system).

Plug the system into AC power to reactivate battery operation if the battery switches have been in the OFF position, which is indicated by the **·ⓘ** symbol (stand system).

System image quality is poor. Adjust the clinical monitor to improve viewing angle. Adjust the brightness.

Adjust the gain.

No CPD image. Adjust the gain. CPD has a hide control. Ensure that this control is not activated.

No Color image. Adjust the gain or the PRF scale. Color has a hide control. Ensure that this control is not activated.

No measurement selections. Ensure the desired exam type has been selected and that the image is frozen. Tap **Calcs** in the Controls bar, or from **More Controls**.

No sound. If you do not hear system sounds when you use the system on the desktop, make sure that you have configured audio settings correctly (see "Audio settings" on page 34). If you still do not hear system sounds, cycle power: Turn the isolation transformer power switch to the OFF position and then turn it to the ON position.

Printing does not work. If you are printing to a DICOM printer, check with your system administrator to make sure that DICOM settings are accurate and that your system is connected to the network.

If you are printing to a local printer, do any of the following:

- Check the printer connections. The system will automatically detect the printer.
- Ensure that the local printer is turned on and set up properly. See the printer manufacturer's instructions, if necessary.

Only saved images and video clips and the current patient's worksheets can be printed.

DVR does not record. Make sure that the USB memory stick has enough available space.

System does not recognize the transducer. Disconnect and reconnect the transducer.

Maintenance icon () **appears on screen.** Restart the system. If the issue recurs, system maintenance may be required. Note the number that appears in parentheses on the C: line and contact FUJIFILM SonoSite or your FUJIFILM SonoSite representative.

System prompts you to ensure the USB device is valid. Use the USB memory stick included with the system.

Make sure that the USB memory stick does not have password protection enabled and is not defective.

System prompts you to ensure the USB device contains valid data. Make sure that the data are present on the USB memory stick.

Re-export the original data onto the USB memory stick. Contact your system administrator.

USB device does not appear in the list. Check that the USB device has been properly inserted into an available USB slot. Use the USB memory stick that came with the system.

System displays the alert "...internal storage device is full." Free internal storage space by ending the current exam, archiving or exporting patient exams, and then deleting them from the system.

Cannot access Patient Form. Cannot access Patient List. Ensure that you are logged in as a user, not as a guest.

System does not export or transfer video clips (DICOM). In the **Archiver** settings of DICOM Config, make sure that **Exclude Video Clips** is not checked.

When you position the caliper on the touch panel, it appears higher than the point you actually touched. Reposition the caliper by aiming for the bottom half of the blue circle.

Software licensing

FUJIFILM SonoSite software is controlled by a license key. After you install new software, the system prompts you for a license key. You must obtain one key for each system and transducer package that uses the software.

The software will operate for a short time (the grace period) without a license key. During the grace period, all system functions are available. After the grace period, the system is not usable until you enter a valid license key. Grace period time is not used while the system is off or asleep. Grace period time remaining appears on the license update screen.

Caution After the grace period expires, all system functions except licensing are unavailable until you enter a valid license key.

To obtain a license key

- 1 Turn on the system.
- 2 Navigate to System Information to gather version information:
 - a. Tap **MORE** and then tap **System Settings**.
 - b. Tap **System Information** in the list on the left and scroll to display System Licensing and Scan-head Licensing.
- 3 Contact FUJIFILM SonoSite Technical Support. (See "FUJIFILM SonoSite Technical Support" on page 3.) You will be asked for the following information from System Information:
 - a. Your name
 - b. System serial number
 - c. PCBA serial number
 - d. Transducer package version
- 4 After you obtain a license key, you must enter it into the system. You can enter it either at startup or in System Setup.

To enter the license key at startup

- 1 Turn on the system.
The license update screen appears.
- 2 Enter the license key in the **Enter license number** box.
- 3 Tap **Done**.
- 4 If the license update screen reappears, verify that you entered the license key correctly. If the license update screen still appears, contact FUJIFILM SonoSite Technical Support. (See "FUJIFILM SonoSite Technical Support" on page 3.)

To enter the license key in System Settings

- 1 Tap **MORE**, and then tap **System Settings**.
- 2 Tap **System Information** in the list on the left.

- 3 Enter the license key in the **Enter license key** boxes in the **System Licensing** or **Scanhead Licensing** section.
- 4 Tap **ENTER**.

Maintenance

WARNING No modification of this equipment, except as described in this manual or the *X-Porte Service Manual*, is allowed.

WARNING Do not service or perform maintenance procedures on the system while it is in use with a patient.

No periodic or preventive maintenance is required for the system, transducer, or accessories other than cleaning and disinfecting the transducer after every use. For information on cleaning and disinfecting your ultrasound system, see *Cleaning and Disinfecting X-Porte Products*. No internal components require periodic testing or calibration.

In addition to cleaning and disinfecting the transducer, FUJIFILM SonoSite recommends that you plug the system in when not in use to fully charge the batteries. When charging, all three battery switches must be in the ON position.

Performing maintenance procedures not described in this document or the service manual may void the product warranty. Contact FUJIFILM SonoSite Technical Support for any maintenance questions. (See "FUJIFILM SonoSite Technical Support" on page 3.)

Disinfecting

Information for cleaning the ultrasound system, transducers, and accessories can be found in *Cleaning and Disinfecting X-Porte Products*.

System backups

To safeguard against loss of data, FUJIFILM SonoSite recommends that you routinely back up:

- Patient data
- System configuration settings

Patient data

Digital Imaging and Communications in Medicine (DICOM) provides a way to archive patient data by connecting your ultrasound system over a local area network (LAN) with various archivers for storage after every patient exam. FUJIFILM SonoSite recommends that you configure and use DICOM transfer to prevent patient data loss in the event of a system fault. For more information, see "DICOM" on page 46 in the X-Porte Help.

If you do not use DICOM networking, then FUJIFILM SonoSite recommends that you configure your system for automatic export of patient data to a USB memory stick after every exam. For more information, see "DICOM" on page 46 in the X-Porte Help.

System configuration settings

In addition to patient data, FUJIFILM SonoSite recommends that you back up ultrasound system configuration settings, called presets, after you have fully configured the system and any time you modify these settings. These backups preserve

your customized settings in case of a fault in the system. For more information, see "Presets settings" on page 66 in the X-Porte Help.

Servicing

Your ultrasound engine may be repaired or replaced at the manufacturer's discretion. If servicing is necessary, then you must remove the ultrasound engine. Before the engine is removed and shipped to a repair facility, you must take precautions to protect patient data and to preserve your customized settings.

Caution To protect patient privacy, all patient procedure information must be exported to a USB memory stick or archived to a secure repository via DICOM transfer and then deleted from the Patient List.

Caution To preserve your configuration settings, export Presets and DICOM settings to a USB memory stick and store the stick in a secure location.

To prepare your system for service

- 1 End any in-progress procedures.
- 2 Export all patient procedure information to a USB stick or archive it to a DICOM device. For complete instructions, see "Archiving and exporting" on page 165 in the X-PorteHelp.
- 3 To delete all patient data, tap **Patient** and then **Patient List**.
- 4 Tap **Select All** and then tap **Delete**.
- 5 Export the following items to a USB memory stick:
 - System preferences (that is, presets)
 - System log file
 - Assert log file
 - Note: Exporting the Assert log file requires administrator access.
 - User log file
 - DICOM log file (DICOM users only)
 - DICOM settings (DICOM users only)

For information on importing and exporting, see "Importing and exporting" on page 73 and "Importing and exporting connectivity settings" on page 43.

To remove the engine from the stand

Caution Turn off the system before removing the ultrasound engine from the stand. Removing the ultrasound system from the stand while the system is powered on may damage the dock.

Caution Disconnect any devices attached to the ultrasound engine (such as the Triple Transducer Connect, USB memory sticks, transducers, power cables, or ECG cables) before removing it from the stand.

- 1 Locate the two gray latch levers under the left and right sides of the stand head.


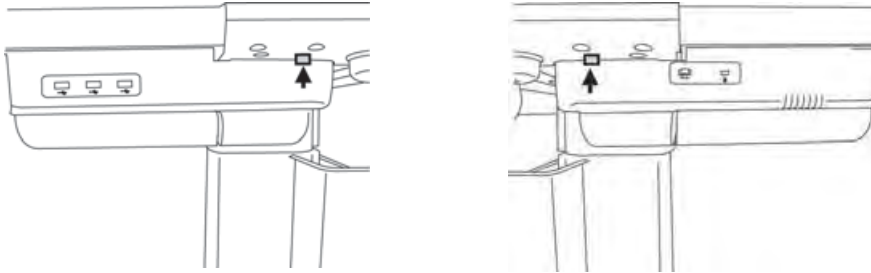


Figure 7-1 Two gray latch levers under the left and right sides of the stand

- 2 Simultaneously pull each latch outward, away from the engine and towards the stand's outer edges. The engine ejects from the stand.

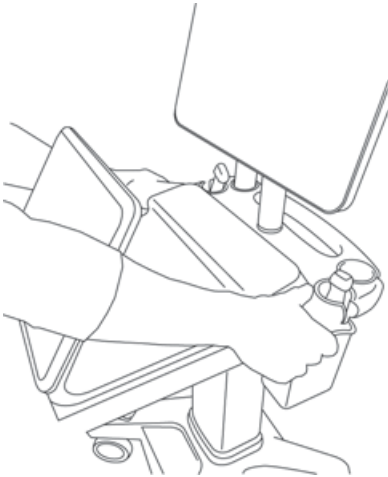

- 3 Lift the engine from the back edge and lift forward and up away from the top of the stand.

To remove a transducer

- 1 Pull the transducer latch down away from the connector housing, and rotate it clockwise.
- 2 Pull the transducer connector away from the system.

Cleaning and disinfecting

Introduction

FUJIFILM SonoSite tests disinfectants and disinfectant devices for use with its systems, transducers, and accessories. This document includes instructions for cleaning and disinfecting the FC ultrasound system, the X-Porte ultrasound system, transducers, and accessories.

This document does not have the following regulatory information for disinfectants:

- ▶ USA FDA registration and listing, 510(k) clearance, and/or EPA registration
- ▶ CE approval

Before using a disinfectant, confirm that its regulatory status is appropriate for your jurisdiction and use. Verify expiration dates on chemicals.

When disposing of chemicals, follow manufacturer recommendations and local regulations.

WARNING

Disinfectants and cleaning methods listed are recommended by FUJIFILM SonoSite for compatibility with product materials, not for biological effectiveness. Refer to the disinfectant label instructions for guidance on disinfection efficacy and appropriate clinical uses.

The level of disinfection required for a device is dictated by the type of tissue it contacts during use. To avoid infection, ensure that the disinfectant type and the solution strength and duration are appropriate for the equipment. For information, see the disinfectant label instructions and the recommendations of the Association for Professionals in Infection Control and Epidemiology (APIC) and the FDA.

WARNING

To avoid infection:

- ▶ Always use protective eyewear and gloves when performing cleaning and disinfecting procedures.
- ▶ Ensure that the solution expiration date has not passed.

Ultrasound system

The ultrasound system includes the stand, clinical monitor (X-Porte), touch panel, system (engine), cups, and cable hooks.

- WARNING**
- To avoid electrical shock, before cleaning, disconnect the system from the power supply.
- Caution**
- Do not spray cleaners or disinfectant directly on the system surfaces. Doing so may cause solution to leak into the system, damaging the system and voiding the warranty.
- Caution**
- Do not use strong solvents such as thinner or benzene, or abrasive cleansers, since these will damage the exterior surfaces.
- Caution**
- Use only recommended cleaners or disinfectants on system surfaces. Immersion-type disinfectants are not approved for use on system surfaces.
- Caution**
- When you clean the system, ensure that the solution does not get inside the system controls or the battery compartment.

Caution | Do not scratch the LCD screen.

Cleaning and disinfecting the ultrasound system

Clean the ultrasound system prior to disinfecting to remove any particulate matter or bodily fluids.

To clean and disinfect the ultrasound system

- 1 Turn off the system.
 - 2 Disconnect the system from power:
 - Press the Power button, listen for the audio tone, and then disconnect the AC input power cord from the stand base.
- Caution** | Unplugging the AC power cord before you hear the tone may result in loss of data. If you don't hear a tone, then your system may be set up to mute all sounds. To reinstate sounds, see "Audio settings" on page 34 in the X-PorteHelp.
- 3 Clean the exterior surfaces using a soft cloth lightly dampened in a mild soap or detergent cleaning solution to remove any particulate matter or body fluids.
Apply the solution to the cloth rather than the surface.
 - 4 Mix the disinfectant solution compatible with the system, following disinfectant label instructions for solution strengths and disinfectant contact duration. See "Compatible disinfectants for the ultrasound system" below.
 - 5 Wipe surfaces with the disinfectant solution.
 - 6 Air dry or towel dry with a clean cloth.

Compatible disinfectants for the ultrasound system

You can disinfect the exterior surface of the ultrasound system using any of the following chemically compatible disinfectants approved in your locale. The exterior surface includes the stand, clinical monitor, touch panel, system (engine), cups, and cable hooks.

Table 7-1 *Compatible disinfectants for the ultrasound system*

Disinfectant	Active Ingredient
Airkem A-33	Quaternary Ammonia
Anios Wipes	Quaternary Ammonium, Isopropyl Alcohol
Aniosyme DD1 (5%)	Quaternary Ammonium
Anioxide 1000	Peracetic Acid, Peroxide
Asepti-HB	Quaternary Ammonium
Asepti Steryl	Orthophenylphenol, Para-tertiary Amylphenol, Ethanol

Table 7-1 Compatible disinfectants for the ultrasound system (continued)

Disinfectant	Active Ingredient
Asepti-Wipe II	Quaternary Ammonium
Cidalkan Lingettes	Ethanol
Clinell Universal Sanitizing Wipes	Quaternary Ammonium, Benzalkonium Chloride, Propane-1,2-diol
Dynacide PA	Ispopropanol
Endosporine	Glutaraldehyde
Hydrogen Peroxide (3%)	Hydrogen Peroxide
Incidin Plus (1%)	Glucoprotamin
Medzyme	
Mikrobac Forte 2%	Benzylalkyldimethyammoniumchlorides,N-N-Dodecylpropane-1,3-diamine
Sanicloth HB	Quaternary Ammonium
Steranios 2%	Glutaraldehyde
Steranios 20%	Glutaraldehyde
Trigene Advance Wipes	Quaternary Ammonium, Plymeric Biguanide Hydrochloride
Wex-Cide	Ortho-Phenylphenol, Ortho-Benzyl-para-Chlorphenol

Transducers

- WARNING

To avoid electrical shock, before cleaning, disconnect the transducer from the system.
- Caution

Transducers must be cleaned after every use. Cleaning transducers is necessary prior to effective disinfection. Ensure that you follow the manufacturer's instructions when using disinfectants.
- Caution

Do not use a surgeon's brush when cleaning transducers. Even the use of soft brushes can damage a transducer. Use a soft cloth.
- Caution

Using a non-recommended cleaning or disinfection solution, incorrect solution strength, or immersing a transducer deeper or for a longer period of time than recommended can damage or discolor the transducer and void the transducer warranty.
- Caution

Do not allow cleaning solution or disinfectant into the transducer connector.
- Caution

Do not allow disinfectant to contact metal surfaces. Use a soft cloth lightly dampened in a mild soap or compatible cleaning solution to remove any disinfectant that remains on metal sur-

faces.

Caution Attempting to disinfect a transducer or transducer cable using a method other than the one included here can damage the transducer and void the warranty.

Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. To disinfect the transducer and its cable, use the immersion method or the wipe method.

You can disinfect the transducer, scanning surface, and cable using a compatible disinfectant or disinfectant device:

- ▶ For disinfectants compatible with standard transducers, see "Compatible disinfectants for standard transducers" on the facing page.
- ▶ For disinfectants compatible with specialty transducers, such as the ICTxp, see "Compatible disinfectants for specialty transducers" on page 263. FUJIFILM SonoSite recommends the use of high-level disinfectants with specialty transducers, such as the ICTxp.

Cleaning and disinfecting transducers

To clean a transducer

- 1 Disconnect the transducer from the system.
- 2 Remove any transducer sheath.
- 3 Clean the surface using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.
- 4 Apply the solution to the cloth rather than the surface.
- 5 Rinse with water or wipe with water-dampened cloth; then wipe with a dry cloth.

To disinfect a transducer (wipe method)

- 1 Clean the transducer.
- 2 Mix the disinfectant solution compatible with the transducer, following disinfectant label instructions for solution strengths and disinfectant contact duration.
- 3 Wipe surfaces with the disinfectant solution.
- 4 If the disinfectant's instructions include rinsing, rinse with water or wipe with a water-dampened cloth. Then air dry, or dry with a clean cloth.
- 5 Examine the transducer and cable for damage such as cracks, splitting, or fluid leaks.
If damage is evident, do not use of the transducer, and contact FUJIFILM SonoSite or your local representative.

To disinfect a transducer (immersion method)

- 1 Clean the transducer.
- 2 Mix the disinfectant solution compatible with the transducer, following disinfectant label instructions for solution strengths and disinfectant contact duration.
- 3 Immerse the transducer into the disinfection solution not more than 12-18 inches (31-46 cm) from the point where the cable enters the connector.
Follow the instructions on the disinfectant label for the duration of the transducer immersion.

- 4 Using the instructions on the disinfectant label, rinse to the point of the previous immersion, and then air dry or towel dry with a clean cloth.
- 5 Examine the transducer and cable for damage such as cracks, splitting, or fluid leaks. If damage is evident, discontinue use of the transducer, and contact FUJIFILM SonoSite or your local representative.

Compatible disinfectants for standard transducers

Table 7-2 Disinfectant-device compatibility with standard transducers

Device	C60xp	L38xp	HFL38xp	HFL50xp	L25xp	P21xp
Nanosonics Trophon EPR**	✓	✓	✓	✓	✓	✓

✓ - Acceptable for use

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer’s user manual.

*Not approved for use in Canada.

Table 7-3 Disinfectant compatibility with standard transducers

Disinfectant	C60xp	L38xp	HFL38xp	HFL50xp	HSL25xp	L25xp	P21xp
Accel Intervention Wipes			✓		✓	✓	✓
Accel TB Wipes				✓			
Airkem A-33	✓	✓	✓	✓	✓	✓	✓
Alcohol-Reagent-Ethanol		✓			✓	✓	
Anios Wipes	✓		✓	✓	✓	✓	✓
Aniosyme DD1 (5%)	✓	✓	✓	✓	✓	✓	✓
Asepti-HB		✓			✓	✓	
Asepti-Wipe II	✓		✓	✓	✓		✓

Table 7-3 Disinfectant compatibility with standard transducers (continued)

Disinfectant	C60xp	L38xp	HFL38xp	HFL50xp	HSL25xp	L25xp	P21xp
Bacoban			✓		✓	✓	✓
Baxedin Pre-Op	✓	✓	✓	✓	✓	✓	✓
Carpe Diem TB Wipes	✓	✓	✓	✓	✓		✓
CaviWipes	✓	✓	✓	✓	✓	✓	✓
Cidalkan		✓	✓		✓		✓
Cidalkan Lingettes		✓	✓		✓	✓	✓
Cidex	✓	✓	✓	✓	✓	✓	✓
Cidex OPA	✓	✓	✓	✓	✓	✓	✓
Cidex Plus	✓	✓	✓	✓	✓	✓	✓
Cleanisept Wipes		✓	✓		✓		✓
Clinell Universal San- itising Wipes	✓	✓	✓	✓	✓	✓	✓
Clorox Germicidal Wipes	✓	✓	✓	✓	✓	✓	✓
Control III		✓	✓		✓		✓
Coverage Germicidal Surface Wipes	✓	✓	✓	✓	✓	✓	✓
Coverage Plus Germ- icidal Surface Wipes	✓	✓	✓	✓	✓	✓	✓
Coverage Spray					✓	✓	

Table 7-3 Disinfectant compatibility with standard transducers (continued)

Disinfectant	C60xp	L38xp	HFL38xp	HFL50xp	HSL25xp	L25xp	P21xp
DisCide Ultra Disinfecting Towlettes		✓	✓		✓		✓
Dispatch Wipes		✓	✓		✓		✓
Dynacide PA		✓	✓		✓	✓	✓
Echo Clean Lingettes	✓		✓	✓	✓		
End-Bac II	✓		✓	✓	✓		
Endosporine	✓	✓	✓	✓	✓	✓	✓
Envirocide		✓			✓	✓	✓
Gluco Chlor Wipes	✓	✓	✓	✓	✓	✓	✓
Hydrogen Peroxide (3%)	✓	✓	✓	✓	✓	✓	✓
Incidin Plus (1%)	✓	✓	✓	✓	✓	✓	✓
Incidin Plus (3%)	✓	✓	✓	✓	✓	✓	✓
Korsolex Extra		✓	✓		✓		✓
Madacide 1	✓	✓	✓	✓	✓	✓	✓
Medzyme	✓	✓	✓	✓	✓	✓	✓
Metricide OPA Plus	✓	✓	✓	✓	✓	✓	✓
MetriSponge	✓	✓	✓	✓	✓	✓	✓

Table 7-3 Disinfectant compatibility with standard transducers (continued)

Disinfectant	C60xp	L38xp	HFL38xp	HFL50xp	HSL25xp	L25xp	P21xp
Mikrobac Forte 2%	✓	✓	✓	✓	✓		✓
Mikrobac Tissues	✓	✓	✓	✓	✓	✓	✓
Oxivir TB Wipes	✓	✓	✓	✓	✓		✓
PerCept RTU Wipes	✓	✓	✓	✓	✓	✓	✓
Protex Spray	✓	✓	✓	✓	✓	✓	✓
Protex Wipes	✓	✓	✓	✓	✓	✓	✓
Sani-Cloth AF	✓	✓	✓	✓	✓		✓
Sani-Cloth AF3	✓	✓	✓	✓	✓	✓	✓
Sani-Cloth Bleach Wipes	✓	✓	✓	✓	✓	✓	✓
Sani-Cloth HB	✓	✓	✓	✓	✓	✓	✓
Sani-Cloth Plus	✓	✓	✓	✓	✓		✓
Sekusept		✓	✓		✓		✓
Staphene	✓	✓	✓	✓	✓		✓
Steranios (2%)	✓	✓	✓	✓	✓	✓	✓
Trigene Advance Wipes	✓	✓	✓	✓	✓	✓	✓
Tristel Solo	✓	✓	✓	✓	✓	✓	✓

Table 7-3 Disinfectant compatibility with standard transducers (continued)

Disinfectant	C60xp	L38xp	HFL38xp	HFL50xp	HSL25xp	L25xp	P21xp
T-Spray	✓	✓	✓	✓	✓	✓	✓
T-Spray II	✓	✓	✓	✓	✓	✓	✓
Virex II 256	✓	✓		✓	✓	✓	✓
Virex TB	✓	✓		✓	✓	✓	✓
Virox 5 RTU Wipes		✓	✓		✓	✓	✓
Wet Task Wipes	✓	✓	✓	✓	✓	✓	✓
Wet Wipe Disinfection		✓	✓		✓	✓	✓
Wipes Plus Disinfecting Wipes	✓	✓	✓	✓	✓	✓	✓

✓ - Acceptable for use

Compatible disinfectants for specialty transducers

FUJIFILM SonoSite recommends the use of high-level disinfectants with specialty transducers. In the U.S., 510 (k)-approved high-level disinfectants compatible with FUJIFILM SonoSite specialty transducers include Cidex, Cidex OPA, Cidex Plus, Metricide OPA Plus, and the Nanosonics Trophon EPR disinfectant device.

Disinfectant-device compatibility with specialty transducers (ICTxp)

The Nanosonics Trophon EPR disinfectant device is acceptable for use with specialty transducers. For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer's user manual.

Disinfectant compatibility with specialty transducers (ICTxp)

The following disinfectants are acceptable for use with specialty transducers:

- Cidex
- Cidex OPA

- Cidex Plus
- Metricide OPA Plus

Accessories

To clean and disinfect the stand or Triple Transducer Connect, (TTC), see "Cleaning and disinfecting the stand or Triple Transducer Connect (TTC)" below.

To clean the footswitch or ECG cable, see "Cleaning the footswitch or ECG cable" on the facing page.

Use the cleaning and disinfecting recommendations in the peripheral manufacturer's instructions when cleaning or disinfecting peripherals.

WARNING | To avoid electrical shock, always disconnect the power supply before cleaning the PowerPark.

Cleaning and disinfecting accessories

Clean accessories prior to disinfecting. You can disinfect the exterior surface of accessories using a recommended disinfectant. See "Compatible disinfectants for accessories" on the facing page.

To clean and disinfect accessories

- 1 If necessary, unplug the power supply, and detach any cables.
- 2 Clean the exterior surfaces of the accessory using a soft cloth lightly dampened in a mild soap or detergent cleaning solution to remove any particulate matter or body fluids.
Apply the solution to the cloth rather than the surface.
- 3 Mix the disinfectant solution compatible with the accessory, following disinfectant label instructions for solution strengths and disinfectant contact duration. See "Compatible disinfectants for accessories" on the facing page.
- 4 Wipe surfaces with the disinfectant solution. See "Cleaning and disinfecting the stand or Triple Transducer Connect (TTC)" below.
- 5 Air dry or towel dry with a clean cloth.

Cleaning and disinfecting the stand or Triple Transducer Connect (TTC)

To clean and disinfect the stand or TTC

You can clean and disinfect the exterior surface of the stand or TTC using a recommended cleaner or disinfectant.

- 1 Do one of the following:
 - If cleaning the stand, unplug the power supply, detach any cables from the system, and remove the system.
 - If cleaning the TTC, remove it from the system platform.
- 2 Clean the surfaces using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.
Apply the solution to the cloth rather than the surface.
- 3 Wipe the surfaces with the disinfecting solution. Follow the manufacturer's instructions for the disinfecting solu-

tion, including exposure time and temperature for effective disinfection.

- 4 Air dry or towel dry with a clean cloth.

Cleaning the footswitch or ECG cable

To clean the footswitch

Use the disinfecting recommendations in the peripheral manufacturer's instructions when disinfecting the footswitch.

Caution | To avoid damaging the footswitch, do not sterilize. It is not intended for use in a sterile environment.

- 1 Dampen a non-abrasive cloth with one of the following products:
- Isopropyl alcohol
 - Soap and water
 - Sodium Hypochlorite 5.25% (Bleach) diluted 10:1
- 2 Wring out cloth until slightly wet and then gently rub soiled area until clean.

To clean and disinfect the ECG cable (wipe method)

Caution | To avoid damaging the ECG cable, do not sterilize.

- 1 Remove the cable from the system.
- 2 Clean the surface using a soft cloth lightly dampened in a mild soap or detergent cleaning solution. Apply the solution to the cloth rather than the surface.
- 3 Wipe the surfaces with any of the following products:
- Bleach (sodium hypochlorite)
 - Green soap
- 4 Air dry or towel dry with a clean cloth.

Compatible disinfectants for accessories

Table 7-4 Disinfectant compatibility with FUJIFILM SonoSite accessories

Disinfectant	PowerPark Docking Station	ECG cable	AC/DC power supply	Digital video cable	Triple Transducer Connect
Airkem A-33	✓		✓	✓	✓
Alcohol, Ethanol					✓

Table 7-4 Disinfectant compatibility with FUJIFILM SonoSite accessories (continued)

Disinfectant	PowerPark Docking Station	ECG cable	AC/DC power supply	Digital video cable	Triple Transducer Connect
Alcohol, Isopropyl (100%)					✓
Alcohol, Isopropyl (70%)		✓			
Anios Wipes	✓		✓	✓	✓
Aniosyme DD1 (5%)	✓		✓	✓	✓
Anioxyde 1000	✓		✓	✓	✓
Asepti-HB	✓		✓	✓	✓
Asepti-Steryl			✓		✓
Asepti-Wipe II	✓		✓	✓	✓
Baxedin Pre-Op					✓
CaviWipes	✓	✓	✓		✓
Cidalkan			✓		✓
Cidalkan Lingettes				✓	✓
Cidalkan Wipes					✓
Clinell Universal Sanitising Wipes			✓	✓	✓
Clorox Germicidal Wipe			✓		

Table 7-4 Disinfectant compatibility with FUJIFILM SonoSite accessories (continued)

Disinfectant	PowerPark Docking Station	ECG cable	AC/DC power supply	Digital video cable	Triple Transducer Connect
Control III	✓		✓	✓	✓
Coverage Germicidal Surface Wipes			✓		✓
Coverage Plus Germicidal Surface Wipes			✓		✓
Coverage Plus Wipes					✓
Coverage Spray			✓		✓
DisCide Ultra Disinfecting Towelettes			✓		✓
Dynacide PA	✓		✓	✓	✓
Echo Clean Lingettes	✓		✓		✓
End-Bac II			✓		
Endosporine	✓		✓	✓	✓
Envirocide	✓		✓		✓
Gluko Chlor Wipes			✓		
Hydrogen Peroxide (3%)	✓		✓	✓	✓
Incidin Plus (1%)	✓		✓	✓	✓
Incidin Plus (3%)	✓		✓	✓	✓

Table 7-4 Disinfectant compatibility with FUJIFILM SonoSite accessories (continued)

Disinfectant	PowerPark Docking Station	ECG cable	AC/DC power supply	Digital video cable	Triple Transducer Connect
Madacide 1			✓		✓
Medzyme	✓		✓	✓	✓
Mikrobac Forte 2%				✓	
Nu-Cidex	✓				
PerCept RTU Wipes	✓		✓		
Rely+On PeraSafe	✓		✓		✓
Sani-Cloth Bleach Wipes			✓		
Sani-Cloth HB			✓	✓	✓
Sani-Cloth Plus	✓	✓	✓		✓
Simple Chlor				✓	✓
Sporicidin	✓		✓		✓
Staphene			✓		✓
Steranios 2%	✓		✓	✓	✓
Steranios 20%	✓		✓	✓	✓
Super Sani-Cloth		✓	✓		✓
T-Spray			✓		

Table 7-4 Disinfectant compatibility with FUJIFILM SonoSite accessories (continued)

Disinfectant	PowerPark Docking Station	ECG cable	AC/DC power supply	Digital video cable	Triple Transducer Connect
T-Spray II	✓		✓		✓
Trigene Advance Wipes			✓	✓	✓
Tristel Solo	✓		✓		✓
Tristel Wipes			✓		✓
Virex 256	✓		✓		
Virex II 256					✓
Virex TB			✓		✓
Virox 5 RTU Wipes	✓		✓	✓	✓
Wet Task Wipes			✓		✓
Wex-Cide	✓		✓	✓	✓
Wipes Plus Disinfecting Wipes			✓		✓
Wood Wyant Ultra Wipes					✓

✓ - Acceptable for use

CHAPTER 8

Safety

This section contains general safety information that applies to the ultrasound system, transducers, accessories, and peripherals.

Ergonomic safety

These scanning guidelines are intended to assist you in the comfortable and effective use of your ultrasound system.

WARNING To prevent musculoskeletal disorders, follow the guidelines in this section.

Use of an ultrasound system may be linked to musculoskeletal disorders (MSDs)^{abc}.

Use of an ultrasound system is defined as the physical interaction between the operator, the ultrasound system, and the transducer.

When using an ultrasound system, as with many similar physical activities, you may experience occasional discomfort in your hands, fingers, arms, shoulders, eyes, back, or other parts of your body. However, if you experience symptoms such as constant or recurring discomfort, pain, throbbing, aching, tingling, numbness, burning sensation, or stiffness, do not ignore these warning signs. Promptly see a qualified health professional. Symptoms such as these can be linked with MSDs. MSDs can be painful and may result in potentially disabling injuries to the nerves, muscles, tendons, or other parts of the body. Examples of MSDs include carpal tunnel

^aMagnavita, N., L. Bevilacqua, P. Mirk, A. Fileni, and N. Castellino. "Work-related Musculoskeletal Complaints in Sonologists." *Occupational Environmental Medicine*. 41:11 (1999), 981-988.

^bCraig, M. "Sonography: An Occupational Hazard?" *Journal of Diagnostic Medical Sonography*. 3 (1985), 121-125.

^cSmith, C.S., G.W. Wolf, G. Y. Xie, and M. D. Smith. "Musculoskeletal Pain in Cardiac Ultrasonographers: Results of a Random Survey." *Journal of American Society of Echocardiography*. (May 1997), 357-362.

syndrome and tendonitis.

While researchers are not able to definitively answer many questions about MSDs, there is a general agreement that certain factors are associated with their occurrence including preexisting medical and physical conditions, overall health, equipment and body position while doing work, frequency of work, duration of work, and other physical activities that may facilitate the onset of MSDs^a. This section provides guidelines that may help you work more comfortably and may reduce your risk of MSDs^{bc}.

^aWihlidal, L.M. and S. Kumar. "An Injury Profile of Practicing Diagnostic Medical Sonographers in Alberta." *International Journal of Industrial Ergonomics*. 19 (1997), 205-216.

^bHabes, D.J. and S. Baron. "Health Hazard Report 99-0093-2749." *University of Medicine and Dentistry of New Jersey*. (1999).

^cVanderpool, H.E., E.A. Friis, B.S. Smith, and K.L. Harms. "Prevalence of Carpal Tunnel Syndrome and Other Work-related Musculoskeletal Problems in Cardiac Sonographers." *Journal of Medicine*. 35:6 (1993), 605-610.

Position the system

Minimize eye and neck strain

- If possible, position the system within reach.
- Adjust the angle of the clinical monitor or touch panel to minimize glare.
- Adjust the height so that the clinical monitor is at or slightly below eye level.

Position yourself

Support your back during an exam

- Use a chair that supports your lower back, that adjusts to your work surface height, that promotes a natural body posture, and that allows quick height adjustments.
- Always sit or stand upright. Avoid bending or stooping.

Minimize reaching and twisting

- Use a bed that is height adjustable.
- Position the patient as close to you as possible.
- Face forward. Avoid twisting your head or body.
- Move your entire body front to back, and position your scanning arm next to or slightly in front of you.
- Stand for difficult exams to minimize reaching.
- Position the monitor directly in front of you.

Promote comfortable shoulder and arm postures

- Keep your elbow close to your side.
- Relax your shoulders in a level position.
- Support your arm using a support cushion or pillow, or rest it on the bed.

Promote comfortable hand, wrist, and finger postures

- Hold the transducer lightly in your fingers.
- Minimize the pressure applied on the patient.
- Keep your wrist in a straight position.

Take breaks, exercise, and vary activities

- Minimizing scanning time and taking breaks can effectively allow your body to recover from physical activity and help you avoid MSDs. Some ultrasound tasks may require longer or more frequent breaks. However, simply changing tasks can help some muscle groups relax while others remain or become active.
- Work efficiently by using the software and hardware features correctly.
- Keep moving. Avoid sustaining the same posture by varying your head, neck, body, arm, and leg positions.
- Do targeted exercises. Targeted exercises can strengthen muscle groups, which may help you avoid MSDs. Contact a qualified health professional to determine stretches and exercises that are right for you.

Electrical safety

This system meets EN60601-1, Class I/internally-powered equipment requirements and Type BF (transducers) and Type CF (ECG leads) isolated patient-applied parts safety requirements.

This system complies with the applicable medical equipment requirements published in the Canadian Standards Association (CSA), European Norm Harmonized Standards, and Underwriters Laboratories (UL) safety standards. See "Standards" on page 295.

For maximum safety observe the following warnings and cautions.

WARNING To avoid the risk of injury, do not operate the system in the presence of flammable gasses or anesthetics. Explosion can result.

WARNING To avoid the risk of electrical shock or injury, do not open the system enclosures. All internal adjustments and replacements, must be made by a qualified technician.

WARNING To avoid the risk of electrical shock:

- Use only properly grounded equipment. Shock hazards exist if the power supply is not properly grounded. Grounding reliability can only be achieved when equipment is connected to a receptacle marked “Hospital Only” or “Hospital Grade” or the equivalent. The grounding wire must not be removed or defeated.
- This equipment must be connected only to a supply mains with protective earth.
- Do not let any part of the system (including the bar code scanner, power supply, or power supply connector), except for the transducer or ECG leads, touch the patient.
- Do not touch the power supply and the patient at the same time.
- Do not touch any of the following:
 - The signal input/output connectors on the back of the ultrasound system.
 - The system transducer connector when the transducer or Triple Transducer Connect (TTC) is disconnected.
 - Any unused TTC transducer connector when the TTC is connected.
- Do not connect the system's power supply to an MPSO or extension cord.
- Before using the transducer, inspect the transducer face, housing, and cable. Do not use the transducer if the transducer or cable is damaged.
- Turn the engine off when cleaning.
- Do not use any transducer that has been immersed beyond the specified cleaning or disinfection level. See "Troubleshooting and maintenance" on page 249.
- Use only accessories and peripherals recommended by FUJIFILM SonoSite, including the power supply. Connection of accessories and peripherals not recommended by FUJIFILM SonoSite could result in electrical shock. Contact FUJIFILM SonoSite or your local representative for a list of accessories and peripherals available from or recommend by FUJIFILM SonoSite.

WARNING To avoid the risk of electrical shock and fire hazard:

- Inspect the AC power cords, cables, and plugs on a regular basis. Ensure that they are not damaged.
- The power cord set that connects the power supply of the ultrasound system or to mains power must only be used with the power supply, and cannot be used to connect other devices to mains power.

WARNING To prevent injury to the operator/bystander, the transducer must be removed from patient contact before the application of a high-voltage defibrillation pulse.

WARNING To avoid possible electrical shock or electromagnetic interference, verify proper operation and compliance with relevant safety standards for all equipment before clinical use. Connecting additional equipment to the ultrasound system constitutes configuring a medical system. FUJIFILM SonoSite recommends verifying that the system, all combinations of equipment, and accessories connected to the ultrasound system comply with JACHO installation requirements and/or safety standards such as AAMI-ES1, NFPA 99 or IEC Standard 60601-1-1 and electromagnetic compatibility standard IEC 60601-1-2 (Electromagnetic compatibility), and are certified according to IEC Standard 60950 Information Technology Equipment (ITE) or IEC

- 60601-1.
- WARNING

Because the only method of completely removing AC power from the stand is to disconnect the AC input power cord from the stand base, ensure that you place the stand in a location in which you can easily remove the AC input power cord if necessary.
- WARNING

Failures in the electrical safety design of connected devices may result in a voltage on the ultrasound system. To minimize the risk of electrical shock to the patient and/or operator:

 - Use medical-grade devices.
 - After connections are made, test the electrical safety utilizing biomedical department electrical safety procedures.
 - For added protection, use an isolation transformer between the connecting device and AC power.
- Caution

Do not use the system if an error message appears on the image display: note the error code; call FUJIFILM SonoSite or your local representative; turn off the system by pressing and holding the power key until the system powers down.
- Caution

To avoid increasing the system and transducer connector temperature, do not block the airflow to the ventilation holes on the front and back of the system.

Electrical safety classification

Class I equip- The ultrasound system is classified as Class I equipment when powered from the external power ment supply or mounted on the stand because the external power supply is a Class 1 protectively earthed power supply.

Note AC-powered peripherals that may be used with the system are Class I and are individually protectively earthed. Ground bond testing may be conducted on each AC-powered peripheral.

- Internally powered equipment
- Ultrasound system not connected to the AC supply
- Type BF applied parts
- Ultrasound transducers
- Type CF applied parts
- ECG module/ECG leads
- IPX-7
- Ultrasound transducers
- IPX-8
- Footswitch
- Non AP/APG
- Ultrasound system power supply, docking system, and peripherals. Equipment is not suitable for use in the presence of flammable anaesthetics.

Isolating the X-Porte ultrasound system from power


The X-Porte ultrasound system does not become completely isolated from power by pressing the Power button. The only way to completely remove power is to press the Power button, disconnect the AC input power cord from the stand base, and set all three battery switches to the off position.

To isolate the system from power

- 1 Press the Power button.
- 2 Listen for the audio tone.

Caution | Unplugging the AC power cord before you hear the tone may result in loss of data. If you don't hear a tone, then your system may be set up to mute all sounds. To reinstate sounds, see "Audio settings" on page 34 in the X-PorteHelp.

- 3 Disconnect the AC input power cord from the stand base.
- 4 Depress all three battery switches to the **•O** symbol, which is the OFF position.

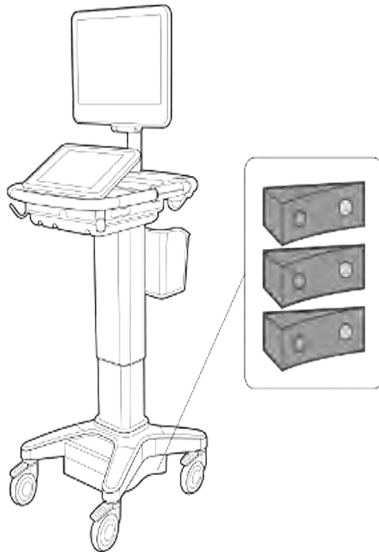

Equipment safety

To protect your ultrasound system, transducer, and accessories, follow these precautions.

WARNING | When transporting your system, to avoid possible injury from the system tipping, always collapse the clinical monitor and push forward on the bar on the platform instead of pushing downward on the bar or pushing the clinical monitor.

Caution | Excessive bending or twisting of cables can cause a failure or intermittent operation.

Caution | Improper cleaning or disinfecting of any part of the system can cause permanent damage. For cleaning and disinfecting instructions, see Cleaning and Disinfecting X-Porte Products.

Caution | Do not submerge the transducer connector in solution. The cable is not liquid-tight beyond the transducer connector/cable interface.

- Caution** | Do not use solvents such as thinner or benzene, or abrasive cleaners on any part of the system.
- Caution** | Do not spill liquid on the system.
- Caution** | Position the system to allow access to the mains power-cord connector.
- Caution** | At high elevations (greater than 1948 meters / 6391 feet above sea level), always operate the ultrasound engine in the stand.

Clinical safety

- WARNING** | To avoid injury, inspect all fasteners and connections.
- WARNING** | To avoid the risk of a burn hazard, do not use the transducer with high frequency surgical equipment. Such a hazard may occur in the event of a defect in the high frequency surgical neutral electrode connection.
- WARNING** | Do not use the system if it exhibits erratic or inconsistent behavior. Discontinuities in the scanning sequence are indicative of a hardware failure that must be corrected before use.
- WARNING** | Some transducer sheaths contain natural rubber latex and talc, which can cause allergic reactions in some individuals. FUJIFILM SonoSite recommends identifying your latex- and talc-sensitive patients and being prepared to treat allergic reactions promptly.
- WARNING** | Perform ultrasound procedures prudently. Use the ALARA (as low as reasonably achievable) principle and follow the prudent use information concerning MI and TI.
- WARNING** | FUJIFILM SonoSite does not currently recommend a specific brand of acoustic standoff. If an acoustic standoff is used, it must have a minimum attenuation of .3dB/cm/MHz.
- WARNING** | Use market-cleared, sterile transducer sheaths and sterile coupling gel for transrectal, transvaginal, or guided-needle procedures. Do not apply the transducer sheath and coupling gel until you are ready to perform the procedure. After use, remove and discard the single-use sheath, and clean and disinfect the transducer using a FUJIFILM SonoSite recommended high-level disinfectant.
- WARNING** | To avoid injury or reduce the risk of infection to the patient, observe the following:
 - Follow Universal Precautions when inserting and maintaining a medical device for interventional procedures.
 - Appropriate training in interventional procedures as dictated by current relevant medical practices as well as in proper operation of the ultrasound system and transducer is required. During vascular access, the potential exists for serious complications including without limitation the following: pneumothorax, arterial puncture, and guidewire misplacement.
- WARNING** | To avoid device damage or patient injury, do not use the P21xp needle guide bracket on patients with pacemakers or medical electronic implants. The needle guide bracket for the P21xp transducers contains a magnet that is used to ensure the bracket is correctly oriented on the transducer. The magnetic field in direct proximity to the pacemaker or medical electronic implant may have an adverse effect.
- WARNING** | To avoid applying unsafe voltage levels to the patient while a device is connected to the external VGA port, do not touch the ultrasound system and the patient simultaneously.

Hazardous materials

WARNING Products and accessories may contain hazardous materials. When disposing of products and accessories, be environmentally responsible and meet federal and local regulations for disposing hazardous materials.

Electromagnetic compatibility

The ultrasound system has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices to IEC 60601-1-2:2007. These limits are designed to provide reasonable protection against harmful interference in a typical medical installation.

WARNING To avoid the risk of increased electromagnetic emissions or decreased immunity, use only accessories and peripherals recommended by FUJIFILM SonoSite. Connection of accessories and peripherals not recommended by FUJIFILM SonoSite could result in malfunctioning of your ultrasound system or other medical electrical devices in the area. Contact FUJIFILM SonoSite or your local representative for a list of accessories and peripherals available from or recommended by FUJIFILM SonoSite. See "Compatible accessories and peripherals" on page 281.

Caution Medical electrical equipment requires special precautions regarding EMC and must be installed and operated according to these instructions. It is possible that high levels of radiated or conducted radio-frequency (RF) electromagnetic interference (EMI) from portable and mobile RF communications equipment or other strong or nearby radio-frequency sources, could result in performance disruption of the ultrasound system. Evidence of disruption may include image degradation or distortion, erratic readings, equipment ceasing to operate, or other incorrect functioning. If this occurs, survey the site to determine the source of disruption, and take the following actions to eliminate the source(s).

- Turn equipment in the vicinity off and on to isolate disruptive equipment.
- Relocate or re-orient interfering equipment.
- Increase distance between interfering equipment and your ultrasound system.
- Manage use of frequencies close to ultrasound system frequencies.
- Remove devices that are highly susceptible to EMI.
- Lower power from internal sources within facility control (such as paging systems).
- Label devices susceptible to EMI.
- Educate clinical staff to recognize potential EMI-related problems.
- Eliminate or reduce EMI with technical solutions (such as shielding).
- Restrict use of personal communicators (cell phones, computers) in areas with devices susceptible to EMI.
- Share relevant EMI information with others, particularly when evaluating new equipment purchases which may generate EMI.
- Purchase medical devices that comply with IEC 60601-1-2 EMC Standards.

Caution Do not stack other equipment on the ultrasound system or use other equipment in close proximity and

adjacent to the ultrasound system. If stacking or using other equipment in close proximity is unavoidable, then you must observe the system to verify normal operation.

Wireless transmission


The X-Porte ultrasound system contains an IEEE 802.11 transmitter that utilizes the ISM frequency bands from 2.412 to 2.484GHz or 4.915 to 5.824 GHz. It implements four different methods of transmission:

- ▶ IEEE 802.11a (4.915 to 5.824 GHz) with Orthogonal Frequency Division Multiplexing (OFDM) at 11 dBm $\pm$ 1.5 dBm @ 54 Mbps
- ▶ IEEE 802.11b (2.412 to 2.484GHz) with Direct Sequence Spread Spectrum (DSSS) at 16 dBm $\pm$ 1.5 dBm @ 11 Mbps
- ▶ IEEE 802.11g (2.412 to 2.484GHz) with Orthogonal Frequency Division Multiplexing (OFDM) at 13 dBm $\pm$ 1.5 dBm @ 54 Mbps
- ▶ IEEE 802.11n (2.412 to 2.484GHz or 4.915 to 5.824 GHz) Orthogonal Frequency Division Multiplexing (OFDM) at 10 dBm $\pm$ 1.5 dBm (802.11an) or at 12 dBm $\pm$ 1.5 dBm (802.11gn)

Electrostatic discharge

Caution Electrostatic discharge (ESD), or static shock, is a naturally occurring phenomenon. ESD is common in conditions of low humidity, which can be caused by heating or air conditioning. ESD is a discharge of the electrical energy from a charged body to a lesser or non-charged body. The degree of discharge can be significant enough to cause damage to a transducer or an ultrasound system. The following precautions can help reduce ESD: anti-static spray on carpets, anti-static spray on linoleum, and anti-static mats.

WARNING Unless following ESD precautionary procedures, do not connect to or touch (with body or hand-held tools) pins (contacts) of connectors that have the ESD Sensitive Devices label:

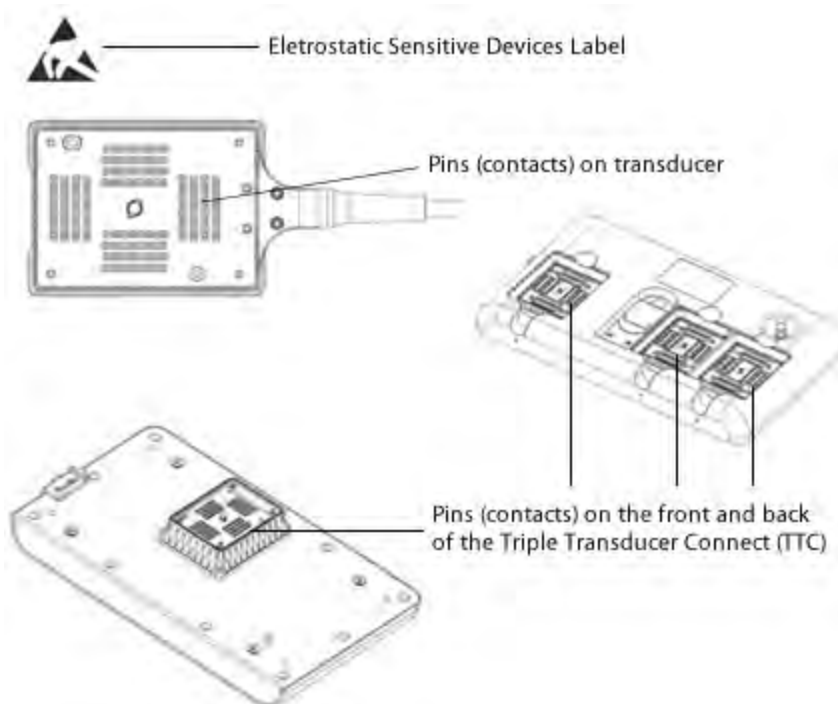

If the symbol is on a border surrounding multiple connectors, the symbol pertains to all connectors within the border.

ESD precautionary procedures include the following:

- All staff involved must receive training about ESD, including the following at a minimum: an explanation of the ESD warning symbol, ESD precautionary procedures, an introduction to the physics of electrostatic charge, the voltage levels that can occur in normal practice, and the damage that can occur to electronic components if equipment is touched by an individual who is electrostatically charged (IEC 60601-1-2, section 5.2.1.2 d).
- Prevent the buildup of electrostatic charge. For example, use humidification, conductive floor coverings, nonsynthetic clothing, ionizers, and minimizing insulating materials.
- Discharge your body to earth.
- Use a wrist strap to bond yourself to the ultrasound system or to earth.

Separation distance

Recommended separation distances between portable and mobile RF communications equipment and the X-Porte ultrasound system

The X-Porte ultrasound system is intended for use in an electromagnetic environment in which radiated radio frequency (RF) disturbances are controlled. The customer or the user of the X-Porte ultrasound system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the X-Porte ultrasound system as recommended below, according to the maximum output power of the communications equipment.

Table 8-1 Separation distance

Rated maximum out-put power of transmitter Watts	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d=1.2 \sqrt{P}$	80 MHz to 800 MHz $d=1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d=2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

Compatible accessories and peripherals

FUJIFILM SonoSite has tested the X-Porte ultrasound system with the following accessories and peripherals and has demonstrated compliance to the requirements of IEC 60601-1-2:2007.

You may use these FUJIFILM SonoSite accessories and third-party peripherals with the X-Porte ultrasound system.

WARNING Use of the accessories with medical systems other than the X-Porte ultrasound system may result in increased emissions or decreased immunity of the medical system.

WARNING Use of accessories other than those specified may result in increased emissions or decreased immunity of the ultrasound system.

Table 8-2 *Accessories and peripherals compatible with X-Porte ultrasound system*

Description	Maximum Cable Length
HSL25xp transducer ^a	7.5 ft / 2.3 m
L25xp transducer ^a	7.5 ft / 2.3 m
L38xp transducer ^a	5.5 ft / 1.7 m
P21xp transducer ^a	6.0 ft / 1.8 m
ICTxp transducer ^a	5.7 ft / 1.7 m
HFL38xp transducer ^a	5.7 ft / 1.7 m
HFL50xp transducer ^a	5.7 ft / 1.7 m
C60xp transducer ^a	5.7 ft / 1.7 m
Bar code scanner	4.8 ft / 1.5 m
Batteries (set of 3)	-
Clinical monitor	-
Dock	-
DVR USB flash memory (64 GB)	-
ECG lead wires	24 in / 0.6 m (USA/Japan) 24 in / 0.6 m (EU)
ECG module	5.8 ft / 1.8 m (USA) 5.8 ft / 1.8 m (Japan) 5.8 ft / 1.8 m (EU)
Ethernet cable	10 ft / 3 m
Footswitch	10 ft / 3 m
Kensington security cable	6 ft / 1.8 m
PowerPark Docking Station	-

Table 8-2 Accessories and peripherals compatible with X-Porte ultrasound system (continued)

Description	Maximum Cable Length
Printer (black and white)	-
SPMU (Stand Power Management Unit)	-
Stand	-
Stand power cord	10 ft / 3.1 m
Touch panel	
Triple Transducer Connect	
Ultrasound engine	
USB flash memory (16 GB)	
USB flash memory (32 GB)	
USB flash memory (64 GB)	
Isolation transformer (desktop configuration only) ^b	
Power cord, isolation transformer (desktop configuration only)	6 ft / 1.8 m Monitor pedestal mount (desktop configuration only)
Power supply, monitor (desktop configuration only)	6 ft / 1.8
Power supply (engine, desktop configuration only)	4 ft / 1.2 m
Power cords (engine and monitor, desktop configuration only)	3.3 ft / 1 m
Mouse (desktop configuration only)	6 ft / 1.8 m
Digital video cable (desktop configuration only)	22.5 in / 0.57 m

^aFor transducers, maximum cable length is measured between the strain reliefs. The stated lengths do not include the lengths of cable in the following locations: underneath the strain reliefs, inside the transducer enclosure, or inside the transducer connector.

^bBefore you assemble X-Porte Desktop, you must configure the input and output voltages on the isolation transformer. Incorrect voltage settings can cause damage to the equipment. For instructions, see the *Toroid Superior Transformers Voltage Selections W series IsoBox*, included with the isolation transformer.

Manufacturer's declaration

The tables in this section document the intended use environment and EMC compliance levels of the system. For maximum performance, ensure that the system is used in the environments described in this table.

The system is intended for use in the electromagnetic environment specified below.

Table 8-3 Manufacturer's Declaration - Electromagnetic Emissions

Emissions Test	Compliance	Electromagnetic Environment
RF emissions CISPR 11	Group 1	The X-Porte ultrasound system uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The X-Porte ultrasound system is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network which supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

The system is intended for use in the electromagnetic environment specified below.

Table 8-4 Manufacturer's Declaration - Electromagnetic Immunity

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
Electrostatic Discharge (ESD) IEC 61000-4-2	± 2.0KV, ± 4.0KV, ± 6.0KV contact ± 2.0KV, ± 4.0KV, ± 8.0KV air	± 2.0KV, ± 4.0KV, ± 6.0KV contact ± 2.0KV, ± 4.0KV, ± 8.0KV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast Transient burst IEC 61000-4-4	± 2KV on the mains ± 1KV on signal lines	± 2KV on the mains ± 1KV on signal lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1KV line(s) to line(s) ± 2KV line(s) to earth	± 1KV line(s) to line(s) ± 2KV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.

Table 8-4 Manufacturer's Declaration - Electromagnetic Immunity (continued)

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$<5\% U_T$ ($>95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $<5\% U_T$ ($>95\%$ dip in U_T) for 5s	$<5\% U_T$ ($>95\%$ dip in U_T) for 0.5 cycle $40\% U_T$ (60% dip in U_T) for 5 cycles $70\% U_T$ (30% dip in U_T) for 25 cycles $<5\% U_T$ ($>95\%$ dip in U_T) for 5s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the FUJIFILM SonoSite ultrasound system requires continued operation during power mains interruptions, it is recommended that the FUJIFILM SonoSite ultrasound system be powered from an uninterruptible power supply or a battery.
Power Frequency Magnetic Field IEC 61000-4-8	3 A/m	3 A/m	If image distortion occurs, it may be necessary to position the FUJIFILM SonoSite ultrasound system further from sources of power frequency magnetic fields or to install magnetic shielding. The power frequency magnetic field should be measured in the Intended installation location to assure that it is sufficiently low.
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the FUJIFILM SonoSite ultrasound system including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended Separation Distance $d = 1.2 \sqrt{P}$

Table 8-4 Manufacturer's Declaration - Electromagnetic Immunity (continued)

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment
Radiated RF IEC 61000-4-3	3 Vim 80 MHz to 2.5 GHz	3 V/m	$d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic Site survey ^a , should be less than the compliance level in each frequency range ^b . Interference may occur in the vicinity of equipment marked with the following symbol:  (IEC 60417 No. 417-IEC-5140: "Source of non-ionizing radiation")

Note | Note: U_T is the AC mains voltage prior to application of the test level. At 80 MHz and 800 MHz, the higher frequency range applies.
These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

FCC Caution Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- ▶ This device may not cause harmful interference.
- ▶ This device must accept any interference received, including interference that may cause undesired operation.

^aField strengths from fixed transmitters such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the FUJIFILM SonoSite ultrasound system is used exceeds the applicable RF compliance level above, the FUJIFILM SonoSite ultrasound system should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the FUJIFILM SonoSite ultrasound system.

^bOver the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Immunity testing requirements

TheX-Porte ultrasound system complies with the essential performance requirements specified in IEC 60601-1-2 and IEC 60601-2-37. Results of immunity testing show that the X-Porte ultrasound system meets these requirements and is free from the following:

- ▶ Noise on a waveform or artifacts or distortion in an image or error of a displayed numerical value that cannot be attributed to a physiological effect and that may alter the diagnosis
- ▶ Display of incorrect numerical values associated with the diagnosis to be performed
- ▶ Display of incorrect safety related indications
- ▶ Production of unintended or excessive ultrasound output
- ▶ Production of unintended or excessive transducer assembly surface temperature
- ▶ Production of unintended or uncontrolled motion of transducer assemblies intended for intracorporeal use

Labeling Symbols

The following symbols are used on the products, packaging, and containers.

Table 8-5 Labeling Symbols

Symbol	Definition
	Alternating Current (AC)
	Class 1 device indicating manufacturer's declaration of conformance with Annex VII of 93/42/EEC
	Class 1 device requiring verification by the Notified Body of sterilization or measurement features, or to a Class IIa, IIb, or III device requiring verification or auditing by the Notified Body to applicable Annex(es) of 93/42/EEC
	Attention, see the user guide

Table 8-5 Labeling Symbols (continued)

Symbol	Definition
	Device complies with relevant Australian regulations for electronic devices.
	Batch code, date code, or lot code type of control number Biological risk
	Biological risk
	Device complies with relevant Brazilian regulations for electro-medical devices.
	Canadian Standards Association certification mark. The "C" and "US" indicators next to this mark signify that the product has been evaluated to the applicable CSA and ANSI/UL Standards, for use in Canada and the U.S., respectively.
	Canadian Standards Association component certification mark.
	Catalog number
	Collect separately from other household waste (see European Commission Directive 93/86/EEC). Refer to local regulations for disposal.

Table 8-5 Labeling Symbols (continued)

Symbol	Definition
	Corrugated recycle
	Date of manufacture
	Manufacturer and, if co-located, date of manufacture
	Direct Current (DC)
	Do not get wet.
	Do not stack over # high, where # represents the number on the label.
	Electrostatic sensitive devices
	Device complies with relevant FCC regulations for electronic devices.
	Fragile
GEL	Gel
	Sterilized using irradiation
	Sterilized using ethylene oxide

Table 8-5 Labeling Symbols (continued)

Symbol	Definition
	Hot
	Device emits a static (DC) magnetic field.
	Non-ionizing radiation
	Paper recycle
	Serial number type of control number
	Temperature limitation
	Atmospheric pressure limitation
	Humidity limitation
IPX7	Submersible. Protected against the effects of temporary immersion.
IPX8	Water-Tight Equipment. Protected against the effects of extended immersion.
	Handle transducer with care.

Table 8-5 Labeling Symbols (continued)

Symbol	Definition
	Follow manufacturer's instructions for disinfecting time.
	Disinfect transducer.
	Type BF patient applied part (B = body, F = floating applied part)
	Defibrillation-proof type CF applied part
	Underwriter's Laboratories labeling
	Pollution Control Logo. (Applies to all parts/products listed in the China RoHS disclosure table. May not appear on the exterior of some parts/products because of space limitations.)
	China Compulsory Certificate mark ("CCC Mark"). A compulsory safety mark for compliance to Chinese national standards for many products sold in the People's Republic of China.
	Follow instructions for use.
WARNING: Connect Only Accessories and Peri- pherals Recommended by FUJIFILM SonoSite	WARNING: Connect Only Accessories and Peripherals Recommended by FUJIFILM SonoSite

Table 8-5 Labeling Symbols (continued)

Symbol	Definition
	Authorized representative in the European Community
	Equipotentiality
	Pushing Prohibited Do not push on monitor. System may tip due to pushing or leaning on the monitor.

Specifications

For information on accessories and peripherals, see "Compatible accessories and peripherals" on page 281.

Dimensions

Stand

Length	26.4 in. (67.1 cm)
Width	21.2 in. (53.8 cm)
Height (max)	64 in. (162.6 cm)
Height (min)	42.2 in. (107.2 cm)

Clinical monitor

Length (exterior dimension)	18.8 in. (47.8 cm)
Height (exterior dimension)	14.2 in. (36 cm)
Diagonal (viewable area)	17 in. (43 cm) minimum

Touch panel

Length (exterior dimension)	16.1 in. (40.8 cm)
Height (exterior dimension)	10.2 in. (25.8 cm)
Diagonal (viewable area)	12.1 in. (30.7 cm)

Environmental limits

Note | The temperature, pressure, and humidity limits apply only to the ultrasound system and trans-

ducers.

Operating

System and transducer

10–40°C (50–104°F), 15–95% R.H.

800 to 1060hPa (0.79 to 1.05 ATM)

Shipping and storage

System and transducer

-35–60°C (-31–140°F), 15–95% R.H.

500 to 1060hPa (0.5 to 1.05 ATM)

Electrical

Power Supply Input: 100-240V ~ 50/60 Hz 6.0A maximum

Imaging modes

- 2D (256 gray shades)
- Color Power Doppler (CPD) (256 colors)
- Color Doppler (Color) (256 colors)
- M Mode
- Continuous wave (CW) Doppler

Additionally, the system includes advanced imaging technologies:

- Tissue Doppler Imaging (TDI)
- Tissue Harmonic Imaging (THI)

Image and video clip storage capacity

The number of images and video clips you can save depends on imaging mode and file format.

Standards

Electromechanical safety standards

Table 8-6 *Electromechanical safety standards*

Standard	Description
IEC 60601-1:1988	Medical Electrical Equipment–Part 1. General Requirements for Safety (2nd Edition including Amendment 1 and 2).
IEC 60601-2-37:2001 + Amendment A1:2004	Particular requirements for the safety of ultrasonic medical diagnostic and monitoring equipment; as applied to IEC 60601-1:1988, Medical Electrical Equipment–Part 1. General Requirements for Safety (2nd Edition including Amendment 1 and 2); including all national differences necessary to meet IECEE CB Scheme certification.
IEC 60601-1-1:2000	Medical Electrical Equipment–Part 1-1. General Requirements for Safety–Section 1-1. Collateral Standard. Safety Requirements for Medical Electrical Systems.
IEC 60601-1-4:2000	Medical Electrical Equipment–Part 1-4. General Requirements for Safety–Section 1-4. Collateral Standard. Programmable Electrical Medical Systems.
IEC 60601-1:2005/A1:2012	Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.
IEC 60601-2-37:2007	Medical electrical equipment - Part 2-37: Particular requirements for the basic safety and essential performance of ultrasonic medical diagnostic and monitoring equipment; as applied to IEC 60601-1:2005/A1:2012, Medical electrical equipment - Part 1: General requirements for basic safety and essential performance (3rd Edition including Amendment 1); including all national differences necessary to meet IECEE CB Scheme certification.
IEC 61157:2007	Standard Means for the Reporting of the Acoustic Output of Medical Diagnostic Ultrasonic Equipment.

EMC standards classification

Table 8-7 *EMC standards classification*

Standard	Description
IEC 60601-1-2:2007	Medical Electrical Equipment. General Requirements for Basic Safety and Essential Performance–Collateral Standard. Electromagnetic Compatibility. Requirements and Tests.

Table 8-7 EMC standards classification (continued)

Standard	Description
CISPR 11:2004	Industrial, Scientific, and Medical (ISM) Radio-Frequency Equipment Electromagnetic Disturbance Characteristics-Limits and Methods of Measurement.

The Classification for the ultrasound system, docking system, accessories, and peripherals when configured together is: Group 1, Class A.

DICOM standard

NEMA PS 3.15: 2000, Digital Imaging and Communications in Medicine (DICOM)-Part 15: Security Profiles.

The system conforms to the DICOM standard as specified in the X-Porte DICOM Conformance Statement, available at www.sonosite.com. This statement provides information about the purpose, characteristics, configuration, and specifications of the network connections supported by the system.

HIPAA standard

The system includes security settings that help you to meet the applicable security requirements listed in the HIPAA standard. Users are ultimately responsible for ensuring the security and protection of all electronic protected health information collected, stored, reviewed, and transmitted on the system.

The Health Insurance and Portability and Accountability Act, Pub.L. No. 104-191 (1996). 45 CFR 160, General Administrative Requirements.

45 CFR 164, Security and Privacy.

CHAPTER 9

Acoustic output

This section contains information about the ALARA (as low as reasonably achievable) principle, the output display standard, and acoustic power and intensity tables. The information applies to the ultrasound system, transducer, accessories, and peripherals.

ALARA principle

ALARA is the guiding principle for the use of diagnostic ultrasound. Sonographers and other qualified ultrasound users, using good judgment and insight, determine the exposure that is “as low as reasonably achievable.” There are no set rules to determine the correct exposure for every situation. The qualified ultrasound user determines the most appropriate way to keep exposure low and bioeffects to a minimum, while obtaining a diagnostic examination.

A thorough knowledge of the imaging modes, transducer capability, system setup and scanning technique is necessary. The imaging mode determines the nature of the ultrasound beam. A stationary beam results in a more concentrated exposure than a scanned beam, which spreads that exposure over that area. The transducer capability depends upon the frequency, penetration, resolution, and field of view. The default system presets are reset at the start of each new patient. It is the scanning technique of the qualified ultrasound user along with patient variability that determines the system settings throughout the exam.

The variables which affect the way the qualified ultrasound user implements the ALARA principle include patient body size, location of the bone relative to the focal point, attenuation in the body, and ultrasound exposure time. Exposure time is an especially useful variable, because the qualified ultrasound user can control it. The ability to limit the exposure over time supports the ALARA principle.

Applying the ALARA principle

The system imaging mode selected by the qualified ultrasound user is determined by the diagnostic information required. 2D imaging provides anatomical information; CPD imaging provides information about the energy or amplitude strength of the Doppler signal over time at a given anatomical location and is used for detecting the presence of blood flow; Color imaging provides information about the energy or amplitude strength of the Doppler signal over time at a given anatomical location and is used for detecting the presence, velocity, and direction of blood flow; Tissue Harmonic Imaging uses higher received frequencies to reduce clutter, artifact, and improve resolution on the 2D image. Understanding the nature of the imaging mode used allows the qualified ultrasound user to apply the ALARA principle.

Prudent use of ultrasound means limiting ultrasound to situations in which it is medically useful and limiting patient exposure to the lowest ultrasound output for the shortest time necessary to achieve acceptable diagnostic results. Although there are no direct user controls for acoustic output, users can indirectly control output by varying depth. Decisions that support prudent use are based on the type of patient, exam type, patient history, ease or difficulty of obtaining diagnostically useful information, and potential localized heating of the patient due to transducer surface temperature. See "Transducer surface temperature rise" on page 302. In the event of a device malfunction, there are redundant controls that limit transducer power. This is accomplished by an electrical design that limits both power supply current and voltage to the transducer.

The sonographer uses the system controls to adjust image quality and limit ultrasound output. The system controls are divided into three categories relative to output: controls that directly affect output, controls that indirectly affect output, and receiver controls.

Direct, indirect, and receiver controls

Direct controls The system does not have a direct user control for output. Rather, the system has been designed to automatically adjust output to ensure that acoustic and thermal limits are not exceeded for all imaging modes. Since there is no direct user control for output, the sonographer should rely on controlling exposure time and scanning technique to implement the ALARA principle.

The system does not exceed a spatial peak temporal average intensity (ISPTA) of 720 mW/cm² for all imaging modes. The mechanical index (MI) and thermal index (TI) may exceed values greater than 1.0 on some transducers in some imaging modes. For either the Ophthalmic or Orbital exam, the acoustic output is limited to the following values: ISPTA does not exceed 50 mW/cm²; TI does not exceed 1.0, and MI does not exceed 0.23. Ultrasound users can monitor the MI and TI values on the right side of the clinical monitor and implement the ALARA principle accordingly. For more information on MI and TI, see BS EN 60601-2-37:2001: Annex HH.

Indirect controls The controls that indirectly affect output are controls affecting imaging mode, freeze, and depth. The imaging mode determines the nature of the ultrasound beam. Freeze stops all ultrasound output but keeps the last image displayed on screen. Freeze can be used by the ultrasound user to limit exposure time while studying an image and maintaining probe position during a scan. Some controls, such as depth, show a rough correspondence with output, and may be used as a general means for indirectly reducing MI or TI. See "Guidelines for reducing MI and TI" on the facing page.

Receiver controls The receiver controls are the gain controls. Receiver controls do not affect output. They should be used, if possible, to improve image quality before using controls that directly or indirectly affect output.

Acoustic artifacts

An acoustic artifact is information, present or absent in an image, that does not properly indicate the structure or flow being imaged. There are helpful artifacts that aid in diagnosis and those that hinder proper interpretation. Examples of artifacts include shadowing, through transmission, aliasing, reverberations, and comet tails.

For more information on detecting and interpreting acoustic artifacts, see the following reference:

Kremkau, Frederick W. *Diagnostic Ultrasound: Principles and Instruments*. 7th ed., W.B. Saunders Company, (Oct. 17, 2005).

Guidelines for reducing MI and TI

The following are general guidelines for reducing MI or TI. If multiple parameters are given, the best results may be achieved by minimizing these parameters simultaneously. In some modes changing these parameters does not affect MI or TI. Changes to other parameters may also result in MI and TI reductions. Please note the MI and TI values on the right side of the screen.

Table 9-1 Guidelines for Reducing MI

Transducer	Depth
C60xp	↑
HFL38xp	↑
HFL50xp	↑
HSL25xp	↑
ICTxp	↑
L25xp	↑
L38xp	↑
P21xp	↑

↓ Decrease or lower setting of parameter to reduce MI.
↑ Increase or raise setting of parameter to reduce MI.

Table 9-2 Guidelines for Reducing TI (TIS, TIC, TIB)

Transducer	CPD Settings						Depth
	Box Width	Box Height	Box Depth	PRF	Depth	Optimize	
C60xp	↑		↑				↑(Depth

Table 9-2 Guidelines for Reducing TI (TIS, TIC, TIB) (continued)

Transducer	CPD Settings						Depth
	Box Width	Box Height	Box Depth	PRF	Depth	Optimize	
HFL38xp	↑		↑				↓(Depth)
HFL50xp	↑		↑				↓(Depth)
HSL25xp			↑				↓(Depth)
ICTxp			↑				↑(Depth)
L25xp			↑				↓(Depth)
L38xp			↑				↓(Depth)
P21xp			↑				↓(PRF)

↓ Decrease or lower setting of parameter to reduce MI.

↑ Increase or raise setting of parameter to reduce MI.

Output display

The system meets the AIUM (American Institute of Ultrasound in Medicine) output display standard for MI and TI (see "Related guidance documents" on page 302). The following table indicates for each transducer and operating mode when either the TI or MI is greater than or equal to a value of 1.0, thus requiring display.

Table 9-3 TI or MI ≥ 1.0

Transducer Model	Index	2D/ M Mode	CPD/Color	PW Doppler	CW Doppler
C60xp	MI	Yes	Yes	Yes	-
	TIC, TIB, or TIS	Yes	Yes	Yes	-
HFL38xp	MI	Yes	Yes	Yes	-
	TIC, TIB, or TIS	No	No	Yes	-
HFL50xp	MI	Yes	Yes	No	-
	TIC, TIB, or TIS	No	No	Yes	-
HSL25xp	MI	Yes	Yes	No	-

Table 9-3 TI or MI ≥ 1.0 (continued)

Transducer Model	Index	2D/ M Mode	CPD/Color	PW Doppler	CW Doppler
	TIC, TIB, or TIS	No	No	Yes	-
ICTxp	MI	Yes	No	Yes	-
	TIC, TIB, or TIS	No	No	Yes	-
L25xp	MI	Yes	Yes	No	-
	TIC, TIB, or TIS	No	No	Yes	-
L38xp	MI	Yes	Yes	Yes	-
	TIC, TIB, or TIS	Yes	No	Yes	-
P21xp	MI	Yes	Yes	Yes	No
	TIC, TIB, or TIS	Yes	Yes	Yes	Yes

Even if MI is less than 1.0, the system provides a continuous real-time display of MI in all imaging modes, in increments of 0.1.

The system meets the output display standard for TI and provides a continuous real-time display of TI in all imaging modes, in increments of 0.1.

The TI consists of three user-selectable indices, and only one of these is displayed at any one time. In order to display TI properly and meet the ALARA principle, the user selects an appropriate TI based on the specific exam being performed. FUJIFILM SonoSite provides a copy of *AIUM Medical Ultrasound Safety*, which contains guidance on determining which TI is appropriate (see "Related guidance documents" on page 1).

MI and TI output display accuracy

The accuracy result for the MI is stated statistically. With 95% confidence, 95% of the measured MI values will be within +18% to -25% of the displayed MI value, or +0.2 of the displayed value, whichever value is larger.

The accuracy result for the TI is stated statistically. With 95% confidence, 95% of the measured TI values will be within +21% to -40% of the displayed TI value, or +0.2 of the displayed value, whichever value is larger. The values equate to +1dB to -3dB.

A displayed value of 0.0 for MI or TI means that the calculated estimate for the index is less than 0.05.

Factors that contribute to display uncertainty

The net uncertainty of the displayed indices is derived by combining the quantified uncertainty from three sources: measurement uncertainty, system and transducer variability, and engineering assumptions and approximations made when calculating the display values.

Measurement errors of the acoustic parameters when taking the reference data are the major source of error that contributes to the display uncertainty. The measurement error is described in "Acoustic measurement precision and uncertainty" on page 345.

The displayed MI and TI values are based on calculations that use a set of acoustic output measurements that were made using a single reference ultrasound system with a single reference transducer that is representative of the population of transducers of that type. The reference system and transducer are chosen from a sample population of systems and transducers taken from early production units, and they are selected based on having an acoustic output that is representative of the nominal expected acoustic output for all transducer-system combinations that might occur. Of course every transducer-system combination has its own unique characteristic acoustic output, and will not match the nominal output on which the display estimates are based. This variability between systems and transducers introduces an error into displayed value. By doing acoustic output sampling testing during production, the amount of error introduced by the variability is bounded. The sampling testing ensures that the acoustic output of transducers and systems being manufactured stays within a specified range of the nominal acoustic output.

Another source of error arises from the assumptions and approximations that are made when deriving the estimates for the display indices. Chief among these assumptions is that the acoustic output, and thus the derived display indices, are linearly correlated with the transmit drive voltage of the transducer. Generally, this assumption is very good, but it is not exact, and thus some error in the display can be attributed to the assumption of voltage linearity.

Related guidance documents

Information for Manufacturers Seeking Marketing Clearance of Diagnostic Ultrasound Systems and Transducers, FDA, 1997.

Medical Ultrasound Safety, American Institute of Ultrasound in Medicine (AIUM), 1994. (A copy is included with each system.)

Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment, NEMA UD2-2004.

Acoustic Output Measurement and Labeling Standard for Diagnostic Ultrasound Equipment, American Institute of Ultrasound in Medicine, 1993.

Standard for Real-Time Display of Thermal and Mechanical Acoustic Output Indices on Diagnostic Ultrasound Equipment, NEMA UD3-2004.

Guidance on the interpretation of TI and MI to be used to inform the operator, Annex HH, BS EN 60601-2-37 reprinted at P05699.

Transducer surface temperature rise

The tables in this section list the measured surface temperature rise from ambient ($23^{\circ}\text{C} \pm 3^{\circ}\text{C}$) of transducers used on the ultrasound system. The temperatures were measured in accordance with EN 60601-2-37 with controls and settings positioned to give maximum temperatures.

Table 9-4 Transducer Surface Temperature Rise, External Use (°C)

Test	C60xp	HFL38xp	HFL50xp	HSL25xp	L25xp	L38xp	P21xp
Still air	14.8	10.4	11.1	13.8	12.8	12.9	14.7
Simulated use	9.5 (≤10°C)	7.0 (≤10°C)	9.0 (≤10°C)	7.9 (≤10°C)	8.6 (≤10°C)	9.0 (≤10°C)	9.5 (≤10°C)

Table 9-5 Transducer Surface Temperature Rise, Non-External Use (°C)

Test	ICTxp
Still air	8.9
Simulated use	4.7 (<6°C)

Acoustic output measurement

Since the initial use of diagnostic ultrasound, the possible human biological effects (bioeffects) from ultrasound exposure have been studied by various scientific and medical institutions. In October 1987, AIUM ratified a report from its Bioeffects Committee (Bioeffects Considerations for the Safety of Diagnostic Ultrasound, J Ultra-sound Med., Sept. 1988: Vol. 7, No. 9 Supplement). The report, sometimes referred to as the Stowe Report, reviewed available data on possible effects of ultrasound exposure. Another report, “Bioeffects and Safety of Diagnostic Ultrasound,” dated January 28, 1993, provides more current information.

The acoustic output for this ultrasound system has been measured and calculated in accordance with “Acoustic Output Measurement Standard for Diagnostic Ultrasound Equipment” (NEMA UD2-2004), and “Standard for Real-Time Display of Thermal and Mechanical Acoustic Output Indices on Diagnostic Ultrasound Equipment” (NEMA UDe3-2004).

In Situ, derated, and water value intensities

All intensity parameters are measured in water. Since water does not absorb acoustic energy, these water measurements represent a worst case value. Biological tissue does absorb acoustic energy. The true value of the intensity at any point depends on the amount, type of tissue, and the frequency of the ultrasound passing through the tissue. The intensity value in the tissue, *In Situ*, has been estimated by using the following formula:

In Situ= Water [e^{-(0.23alf)}] where:

where:

In Situ = *In Situ* intensity value

Water = Water intensity value

e = 2.7183

a = attenuation factor (dB/cm MHz)

Attenuation factor (a) for various tissue types are given below:

brain = 0.53

heart = 0.66

kidney = 0.79

liver = 0.43

muscle = 0.55

l = skinline to measurement depth in cm

f = center frequency of the transducer/system/mode combination in MHz

Since the ultrasonic path during the exam is likely to pass through varying lengths and types of tissue, it is difficult to estimate the true *In Situ* intensity. An attenuation factor of 0.3 is used for general reporting purposes; therefore, the *In Situ* value commonly reported uses the formula:

$$In\ Situ\ (derated) = Water\ e^{-(0.069lf)}$$

Since this value is not the true *In Situ* intensity, the term “derated” is used to qualify it.

The maximum derated and the maximum water values do not always occur at the same operating conditions; therefore, the reported maximum water and derated values may not be related by the *In Situ* (derated) formula. For example: a multi-zone array transducer that has maximum water value intensities in its deepest zone, but also has the smallest derating factor in that zone. The same transducer may have its largest derated intensity in one of its shallowest focal zones.

Tissue models and equipment survey

Tissue models are necessary to estimate attenuation and acoustic exposure levels *In Situ* from measurements of acoustic output made in water. Currently, available models may be limited in their accuracy because of varying tissue paths during diagnostic ultrasound exposures and uncertainties in the acoustic properties of soft tissues. No single tissue model is adequate for predicting exposures in all situations from measurements made in water, and continued improvement and verification of these models is necessary for making exposure assessments for specific exam types.

A homogeneous tissue model with attenuation coefficient of 0.3 dB/cm MHz throughout the beam path is commonly used when estimating exposure levels. The model is conservative in that it overestimates the *In Situ* acoustic exposure when the path between the transducer and site of interest is composed entirely of soft tissue. When the path contains significant amounts of fluid, as in many first and second-trimester pregnancies scanned transabdominally, this model may underestimate the *In Situ* acoustic exposure. The amount of underestimation depends upon each specific situation.

Fixed-path tissue models, in which soft tissue thickness is held constant, sometimes are used to estimate *In Situ* acoustic exposures when the beam path is longer than 3 cm and consists largely of fluid. When this model is used to estimate maximum exposure to the fetus during transabdominal scans, a value of 1 dB/cm MHz may be used during all trimesters.

Existing tissue models that are based on linear propagation may underestimate acoustic exposures when significant saturation due to non-linear distortion of beams in water is present during the output measurement.

The maximum acoustic output levels of diagnostic ultrasound devices extend over a broad range of values:

- A survey of 1990-equipment models yielded MI values between 0.1 and 1.0 at their highest output settings. Maximum MI values of approximately 2.0 are known to occur for currently available equipment. Maximum MI values are similar for real-time 2D and M Mode imaging.
- Computed estimates of upper limits to temperature elevations during transabdominal scans were obtained in a survey of 1988 and 1990 pulsed Doppler equipment. The vast majority of models yielded upper limits less than 1° and 4°C (1.8° and 7.2°F) for exposures of first-trimester fetal tissue and second-trimester fetal bone, respectively. The largest values obtained were approximately 1.5°C (2.7°F) for first-trimester fetal tissue and 7°C (12.6°F) for second-trimester fetal bone. Estimated maximum temperature elevations given here are for a "fixed path" tissue model and are for devices having ISPTA values greater than 500 mW/cm². The temperature elevations for fetal bone and tissue were computed based on calculation procedures given in Sections 4.3.2.1-4.3.2.6 in "Bioeffects and Safety of Diagnostic Ultrasound" (AIUM, 1993).

Acoustic output tables

The tables in this section indicate the acoustic output for the system and transducer combinations with a TI or MI equal to or greater than one. These tables are organized by transducer model and imaging mode. For a definition of terms used in the tables, see "Terminology in acoustic output tables" on page 350.

Table 9-6 Transducer Model: C60xp, Operating Mode: 2D	307
Table 9-7 Transducer Model: C60xp, Operating Mode: M Mode	308
Table 9-8 Transducer Model: C60xp, Operating Mode: Color/CPD Mode	309
Table 9-9 Transducer Model: C60xp, Operating Mode: PW Doppler	310
Table 9-10 Transducer Model: HFL38xp, Operating Mode: 2D	311
Table 9-11 Transducer Model: HFL38xp, Operating Mode: M Mode	312
Table 9-12 Transducer Model: HFL38xp, Operating Mode: Color/CPD	313
Table 9-13 Transducer Model: HFL38xp, Operating Mode: PW Doppler	314
Table 9-14 Transducer Model: HFL50xp, Operating Mode: 2D	315
Table 9-15 Transducer Model: HFL50xp, Operating Mode: M Mode	316
Table 9-16 Transducer Model: HFL50xp, Operating Mode: Color/CPD	317
Table 9-17 Transducer Model: HFL50xp, Operating Mode: PW Doppler	318
Table 9-18 Transducer Model: HSL25xp, Operating Mode: 2D	319
Table 9-19 Transducer Model: HSL25xp, Operating Mode: M Mode	320
Table 9-20 Transducer Model: HSL25xp, Operating Mode: Color/CPD Mode	321
Table 9-21 Transducer Model: HSL25xp, Operating Mode: PW Doppler	322
Table 9-22 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: 2D	323

Table 9-23 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: M Mode	324
Table 9-24 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: Color/CPD Mode	325
Table 9-25 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: PW Doppler	326
Table 9-26 Transducer Model: ICTxp, Operating Mode: M Mode	327
Table 9-27 Transducer Model: ICTxp, Operating Mode: PW Doppler	328
Table 9-28 Transducer Model: L25xp, Operating Mode: 2D	329
Table 9-29 Transducer Model: L25xp, Operating Mode: M Mode	330
Table 9-30 Transducer Model: L25xp, Operating Mode: Color/CPD	331
Table 9-31 Transducer Model: L25xp, Operating Mode: PW Doppler	332
Table 9-32 Transducer Model: L25xp (Ophthalmic use), Operating Mode: 2D	333
Table 9-33 Transducer Model: L25xp (Ophthalmic use), Operating Mode: M Mode	334
Table 9-34 Transducer Model: L25xp (Ophthalmic use), Operating Mode: Color/CPD Mode	335
Table 9-35 Transducer Model: L25xp (Ophthalmic use), Operating Mode: PW Doppler	336
Table 9-36 Transducer Model: L38xp, Operating Mode: 2D	337
Table 9-37 Transducer Model: L38xp, Operating Mode: M Mode	338
Table 9-38 Transducer Model: L38xp, Operating Mode: Color/CPD	339
Table 9-39 Transducer Model: L38xp, Operating Mode: PW Doppler	340
Table 9-40 Transducer Model: P21xp, Operating Mode: 2D	341
Table 9-41 Transducer Model: P21xp, Operating Mode: M Mode	342
Table 9-42 Transducer Model: P21xp, Operating Mode: Color/CPD	343
Table 9-43 Transducer Model: P21xp, Operating Mode: PW Doppler	344
Table 9-44 Transducer Model: P21xp, Operating Mode: CW Doppler	345
Table 9-45 Acoustic Measurement Precision and Uncertainty	346

Table 9-6 Transducer Model: C60xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.3	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.15					
	W_0 (mW)			#	-		-	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						-	
	$z@PII_{.3max}$ (cm)	4.4						
	$d_{ed}(z_{sd})$ (cm)						-	
	f_c (MHz)	2.73	#	-	-	-		#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
	Y (cm)		#	-	-	-	#	
Other Information	PD (μsec)	0.517						
	PRF (Hz)	6027						
	$p_r@PII_{max}$ (MPa)	3.26						
	$d_{ed}@PII_{max}$ (cm)						-	
	Focal Length FL_x (cm)		#	-	-			#
	FL_v (cm)		#	-	-			#
	$I_{PA,3}@MI_{max}$ (W/cm ²)	322						
Operating Control Conditions	Control 1: Exam Type	Nrv						
	Control 2: Optimization	Diff						
	Control 3: Depth	9.9cm						
	Control 4: MB/THI	On/Off						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-7 Transducer Model: C60xp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$		
Global Maximum Index Value			1.5	-	(a)	-	1.7	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	2.48					
	W_0	(mW)		-	#		55.0	#
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					4.90	
	$z@PII_{,3max}$	(cm)	4.2					
	$d_{ed}(z_{sd})$	(cm)					0.34	
	f_c	(MHz)	2.73	-	#	-	2.16	#
	Dim of A_{aprt}	X (cm)		-	#	-	1.87	#
Y (cm)			-	#	-	1.30	#	
Other Information	PD	(μsec)	0.534					
	PRF	(Hz)	800					
	$p_r@PII_{\text{max}}$	(MPa)	3.70					
	$d_{ed}@PII_{\text{max}}$	(cm)					0.33	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
		$I_{PA,3}@MI_{\text{max}}$ (W/cm ²)	392					
Operating Control Conditions	Control 1: Exam Type	Abd					Abd	
	Control 2: Optimization	Diff					Avg	
	Control 3: Depth	7.7 cm					11 cm	
	Control 4: THI	Off					On	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-8 Transducer Model: C60xp, Operating Mode: Color/CPD Mode

Index Label			M.I.	TIS		TIB		TIC
				Scan	Non-scan		Non-scan	
					A _{aprt} ≤1	A _{aprt} >1		
Global Maximum Index Value			1.2	1.0	-	-	-	(b)
Associated Acoustic Parameter	p _{r,3}	(MPa)	1.99					
	W ₀	(mW)		81.3	-		-	#
	min of W ₃ (z ₁), I _{TA,3} (z ₁)]	(mW)				-		
	z ₁	(cm)				-		
	z _{bd}	(cm)				-		
	z _{sd}	(cm)					-	
	z@P _{II,3max}	(cm)	4.4					
	d _{ed} (z _{sd})	(cm)					-	
	f _c	(MHz)	2.63	2.63	-	-	-	#
	Dim of A _{aprt}	X (cm)		0.81	-	-	-	#
		Y (cm)		1.3	-	-	-	#
Other Information	PD	(μsec)	1.415					
	PRF	(Hz)	2645					
	p _r @P _{II} _{max}	(MPa)	2.96					
	d _{ed} @P _{II} _{max}	(cm)					-	
	Focal Length	FL _x (cm)		2.71	-	-		#
		FL _y (cm)		6.5	-	-		#
	I _{PA,3} @MI _{max}	(W/cm ²)	346					
Operating Control Conditions	Control 1: Exam Type		ABD	OB				
	Control 2: Optimization		Color	Color				
	Control 3: 2D Optimization/THI/Depth		Avg/On/11 cm	Avg/Off/6.0 cm				
	Control 4: Color Optimization/PRF		High/3049Hz	Low/414Hz				
	Control 5: Color Box Position/Size		Def/Wide & Tall	Def/Narrow				

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-9 Transducer Model: C60xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	-	-	1.1	3.2	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.44					
	W_0 (mW)			-	-		66.9	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					84.1		
	z_1 (cm)					3.9		
	z_{bd} (cm)					3.8		
	z_{sd} (cm)						0.40	
	$z@Pll_{,3max}$ (cm)		1.9					
	$d_{ea}(z_{sd})$ (cm)						0.45	
	f_c (MHz)		2.64	-	-	2.67	2.67	#
	Dim of A_{aprt}	X (cm)		-	-	3.98	0.31	#
	Y (cm)		-	-	1.30	1.30	#	
Other Information	PD (μsec)		1.233					
	PRF (Hz)		1302					
	$p_r@Pll_{max}$ (MPa)		2.89					
	$d_{ea}@Pll_{max}$ (cm)						0.45	
	Focal Length	FL_x (cm)		-	-	15.90		#
		FL_y (cm)		-	-	6.50		#
	$I_{PA,3}@MI_{max}$ (W/cm ²)		239					
Operating Control Conditions	Control 1: Exam Type		OB			OB	OB	
	Control 2: Sample Volume Size		1 mm			12 mm	12 mm	
	Control 3: Sample Volume Position		Zone 2			Zone 10	Zone 0	
	Control 4: PRF		1302 Hz			1302 Hz	2604 Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-10 Transducer Model: HFL38xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC
				Scan	Non-scan	Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$	
Global Maximum Index Value			1.1	(a)	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	1.17				
	W_0	(mW)	#		-		#
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-	
	z_1	(cm)				-	
	z_{bd}	(cm)				-	
	z_{sd}	(cm)					-
	$z@PII_{.3max}$	(cm)	0.4				
	$d_{ed}(z_{sd})$	(cm)					-
	f_c	(MHz)	7.22	#	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	#
Other Information		Y (cm)		#	-	-	#
	PD	(μ sec)	0.194				
	PRF	(Hz)	7977				
	$p_r@PII_{max}$	(MPa)	2.16				
	$d_{ed}@PII_{max}$	(cm)					-
	Focal Length	FL_x (cm)		#	-	-	#
		FL_v (cm)		#	-	-	#
	$I_{PA,3}@MI_{max}$	(W/cm ²)	62				
Operating Control Conditions	Control 1: Exam Type	Ven					
	Control 2: Optimization	Avg					
	Control 3: Depth	3.5cm					
	Control 4: Needle Profiling	On					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-11 Transducer Model: HFL38xp, Operating Mode: M Mode

Index Label		M.I.	TIS		TIB	TIC
			Scan	Non-scan	Non-scan	
				$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$	
Global Maximum Index Value		1.2	-	(a)	-	(a)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)	1.58				
	W_0 (mW)		-	#		#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)				-	
	z_1 (cm)				-	
	z_{bd} (cm)				-	
	z_{sd} (cm)					#
	$z@P_{II,3max}$ (cm)	0.4				
	$d_{ed}(z_{sd})$ (cm)					#
	f_c (MHz)	7.22	-	#	-	#
	Dim of A_{aprt}					
Other Information						
	PD (μsec)	0.200				
	PRF (Hz)	800				
	$p_r@P_{II,max}$ (MPa)	2.82				
	$d_{ed}@P_{II,max}$ (cm)					#
	Focal Length FL_x (cm)		-	#	-	#
	FL_v (cm)		-	#	-	#
Operating Control Conditions	$I_{PA,3}@MI_{max}$ (W/cm^2)	121				
	Control 1: Exam Type	Breast				
	Control 2: Optimization	Diff				
	Control 3: Depth	1.9cm				

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-12 Transducer Model: HFL38xp, Operating Mode: Color/CPD

Index Label			M.I.	Scan	TIS		TIB	TIC
					Non-scan		Non-scan	
					$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$		
Global Maximum Index Value			1.0	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$	(MPa)	1.08					
	W_0	(mW)		#	-		-	#
	min of $W_3(z_1), I_{TA.3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					-	
	$z@P_{II.3\text{max}}$	(cm)	0.4					
	$d_{eq}(z_{sd})$	(cm)					-	
	f_c	(MHz)	7.22	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
Other Information		Y (cm)		#	-	-	-	#
	PD	(μsec)	0.193					
	PRF	(Hz)	6640					
	$p_r@P_{II\text{max}}$	(MPa)	1.97					
	$d_{eq}@P_{II\text{max}}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_v (cm)		#	-	-		#
	$I_{PA.3}@M_{I\text{max}}$	(W/cm ²)	51					
Operating Control Conditions	Control 1: Exam Type	Nerve						
	Control 2: Mode	Color						
	Control 3: 2D Optimization/Depth	Avg/4.0cm						
	Control 4: Color Optimization/PRF	High/7812Hz						
	Control 5: Color Box Position/Size	Def/Wide						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-13 Transducer Model: HFL38xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			(a)	-	1.1	-	1.6	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	#					
	W_0	(mW)		-	40.1		24.3	#
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)			-	-		
	z_{bd}	(cm)			-	-		
	z_{sd}	(cm)					0.40	
	$z@P_{II,3max}$	(cm)	#					
	$d_{ed}(z_{sd})$	(cm)					0.25	
	f_c	(MHz)	#	-	7.22	-	7.22	#
	Dim of A_{aprt}	X (cm)		-	0.12	-	0.12	#
	Y (cm)		-	0.40	-	0.40	#	
Other Information	PD	(μsec)	#					
	PRF	(Hz)	#					
	$p_r@P_{II,max}$	(MPa)	#					
	$d_{ed}@P_{II,max}$	(cm)					0.25	
	Focal Length	FL_x (cm)		-	4.68	-		#
		FL_v (cm)		-	2.5	-		#
	$I_{PA,3}@M_{I,max}$	(W/cm ²)	#					
Operating Control Conditions	Control 1: Exam Type				SmP		SmP	
	Control 2: Sample Volume Size				6mm		1mm	
	Control 3: Sample Volume Position				Zone 11 (47mm)		Zone 7 (28mm)	
	Control 4: PRF				1563Hz		1563Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-14 Transducer Model: HFL50xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC
				Scan	Non-scan	Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$	
Global Maximum Index Value			1.1	(a)	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$	(MPa)	2.61				
	W_0	(mW)	#	-		-	#
	min of $W_3(z_1), I_{TA.3}(z_1)$	(mW)				-	
	z_1	(cm)				-	
	z_{bd}	(cm)				-	
	z_{sd}	(cm)					-
	$z@PII_{.3max}$	(cm)	1.3				
	$d_{ed}(z_{sd})$	(cm)					-
	f_c	(MHz)	5.30	#	-	-	-
	Dim of A_{aprt}	X (cm)		#	-	-	-
Other Information		Y (cm)		#	-	-	-
	PD	(μ sec)	0.493				
	PRF	(Hz)	1084				
	$p_r@PII_{max}$	(MPa)	3.31				
	$d_{ed}@PII_{max}$	(cm)					-
	Focal Length	FL_x (cm)		#	-	-	#
		FL_v (cm)		#	-	-	#
	$I_{PA.3}@MI_{max}$	(W/cm ²)	427				
	Control 1: Exam Type	SmP					
	Control 2: Optimization	Avg					
Operating Control Conditions	Control 3: Depth	4.8 cm					
	Control 4: MB	On/Off					
	Control 5: Needle Profiling	On					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-15 Transducer Model: HFL50xp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.2	-	(a)	-	(a)	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		3.40					
	W_0 (mW)			-	#		#	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						#	
	$z@P_{II,3max}$ (cm)		1.1					
	$d_{ed}(z_{sd})$ (cm)						#	
	f_c (MHz)		7.82	-	#	-	#	#
	Dim of A_{aprt}	X (cm)		-	#	-	#	#
	Y (cm)		-	#	-	#	#	
Other Information	PD (μsec)		0.230					
	PRF (Hz)		800					
	$p_r@P_{II,max}$ (MPa)		4.51					
	$d_{ed}@P_{II,max}$ (cm)						#	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
		$I_{PA,3}@MI_{max}$ (W/cm ²)		401				
Operating Control Conditions	Control 1: Exam Type		Bre					
	Control 2: Optimization		Diff					
	Control 3: Depth		1.9 cm					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-16 Transducer Model: HFL50xp, Operating Mode: Color/CPD

Index Label			M.I.	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$		
Global Maximum Index Value			1.1	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)	2.62						
	W_0 (mW)		#	-		-	#	
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)				-			
	z_1 (cm)				-			
	z_{bd} (cm)				-			
	z_{sd} (cm)					-		
	$z@P_{II,3\text{max}}$ (cm)	1.2						
	$d_{ea}(z_{sd})$ (cm)					-		
	f_c (MHz)	5.93	#	-	-	-	#	
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
	Y (cm)		#	-	-	-	#	
Other Information	PD (μsec)	0.629						
	PRF (Hz)	6649						
	$p_r@P_{II\text{max}}$ (MPa)	3.35						
	$d_{ea}@P_{II\text{max}}$ (cm)					-		
	Focal Length	FL_x (cm)		#	-	-	#	
		FL_v (cm)		#	-	-	#	
	$I_{PA,3}@M_{I\text{max}}$ (W/cm ²)	398						
Operating Control Conditions	Control 1: Exam Type	SmP						
	Control 2: Mode	CVD						
	Control 3: 2D Optimization/Depth	Avg/4.0 cm						
	Control 4: Color Optimization/PRF	High/7812 Hz						
	Control 5: Color Box Position/Size	Def/Wide and Tall						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-17 Transducer Model: HFL50xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			(a)	-	1.0	(a)	1.5	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)	#						
	W_0 (mW)			-	33.7		23.0	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					#		
	z_1 (cm)					#		
	z_{bd} (cm)					#		
	z_{sd} (cm)						1.68	
	$z@P_{II,3max}$ (cm)	#						
	$d_{ed}(z_{sd})$ (cm)						0.19	
	f_c (MHz)	#	-	6.00	#	5.99	#	
	Dim of A_{aprt}	X (cm)		-	1.20	#	0.72	#
	Y (cm)		-	0.40	#	0.40	#	
Other Information	PD (μ sec)	#						
	PRF (Hz)	#						
	$p_r@P_{II,max}$ (MPa)	#						
	$d_{ed}@P_{II,max}$ (cm)						0.18	
	Focal Length	FL_x (cm)		-	4.68	#		#
		FL_y (cm)		-	2.50	#		#
	$I_{PA,3}@M_{I,max}$ (W/cm ²)	#						
Operating Control Conditions	Control 1: Exam Type				SmP		SmP	
	Control 2: Sample Volume Size				6 mm		1 mm	
	Control 3: Sample Volume Position				Zone 11		Zone 7	
	Control 4: PRF				1562 Hz		1953 Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-18 Transducer Model: HSL25xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.0	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$	(MPa)	2.42					
	W_0	(mW)		#	-		-	#
	min of $W_{.3}(z_1), I_{TA.3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					-	
	$z@PII_{.3max}$	(cm)	0.9					
	$d_{ed}(z_{sd})$	(cm)					-	
	f_c	(MHz)	6.08	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
	Y (cm)		#	-	-	-	#	
Other Information	PD	(μsec)	0.606					
	PRF	(Hz)	783					
	$p_r@PII_{max}$	(MPa)	2.92					
	$d_{ed}@PII_{max}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_v (cm)		#	-	-		#
	$I_{PA.3}@MI_{max}$	(W/cm ²)	328					
Operating Control Conditions	Control 1: Exam Type	Ven						
	Control 2: Optimization	Avg						
	Control 3: Depth	2.4cm						
	Control 4: MB	On						
	Control 5: Needle Profiling	On						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-19 Transducer Model: HSL25xp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.2	-	(a)	-	(a)	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	3.17					
	W_0	(mW)		-	#		#	#
	min of W_3 (z_1), $I_{TA,3}(z_1)$]	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					#	
	$z@PII_{,3max}$	(cm)	1.0					
	$d_{ed}(z_{sd})$	(cm)					#	
	f_c	(MHz)	6.81	-	#	-	#	#
	Dim of A_{aprt}	X (cm)		-	#	-	#	#
	Y (cm)		-	#	-	#	#	
Other Information	PD	(μsec)	0.192					
	PRF	(Hz)	800					
	$p_r@PII_{max}$	(MPa)	3.98					
	$d_{ed}@PII_{max}$	(cm)					#	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
	$I_{PA,3}@MI_{max}$	(W/cm ²)	408					
Operating Control Conditions	Control 1: Exam Type		Ven					
	Control 2: Optimization		Diff					
	Control 3: Depth		3.1cm					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-20 Transducer Model: HSL25xp, Operating Mode: Color/CPD Mode

Index Label			M.I.	Scan	TIS		TIB	TIC
					Non-scan		Non-scan	
					$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$		
Global Maximum Index Value			1.1	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	2.81					
	W_0	(mW)		#	-		-	#
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)					-	
	z_1	(cm)					-	
	z_{bd}	(cm)					-	
	z_{so}	(cm)					-	
	$z@P_{II,3max}$	(cm)	1.0					
	$d_{ed}(z_{so})$	(cm)					-	
	f_c	(MHz)	7.08	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
		Y (cm)		#	-	-	-	#
Other Information	PD	(μsec)	0.178					
	PRF	(Hz)	1717					
	$p_r@P_{II,max}$	(MPa)	3.60					
	$d_{ed}@P_{II,max}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_v (cm)		#	-	-		#
	$I_{PA,3}@M_{I,max}$	(W/cm ²)	321					
Operating Control Conditions	Control 1: Exam Type	Msk						
	Control 2: Mode	CPD						
	Control 3: Optimization/Depth	Avg/2.0cm						
	Control 4: Color Optimization/PRF	Low/312Hz						
	Control 5: Color Box Position/Size	Def/Narrow						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-21 Transducer Model: HSL25xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			(a)	-	(a)	-	1.2	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	#					
	W_0	(mW)		-	#		15.6	#
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{so}	(cm)					1.40	
	$z@Pll_{,3max}$	(cm)	#					
	$d_{ed}(z_{so})$	(cm)					0.17	
	f_c	(MHz)	#	-	#	-	6.00	#
	Dim of A_{aprt}	X (cm)		-	#	-	0.60	#
	Y (cm)		-	#	-	0.30	#	
Other Information	PD	(μ sec)	#					
	PRF	(Hz)	#					
	$p_r@Pll_{max}$	(MPa)	#					
	$d_{ed}@Pll_{max}$	(cm)					0.16	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
	$I_{PA,3}@MI_{max}$	(W/cm ²)	#					
Operating Control Conditions	Control 1: Exam Type						Ven	
	Control 2: Sample Volume Size						7mm	
	Control 3: Sample Volume Position						Zone 6	
	Control 4: PRF						2604 Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-22 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	0.03	-	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$ (MPa)		0.45					
	W_0 (mW)			0.8	-		-	#
	min of $W_3(z_1), I_{TA.3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						-	
	$z@PII_{.3max}$ (cm)	2.3						
	$d_{ed}(z_{sd})$ (cm)						-	
	f_c (MHz)	6.85	7.15	-	-	-		#
	Dim of A_{aprt} X (cm)		1.12	-	-	-		#
	Y (cm)		0.30	-	-	-		#
Other Information	PD (μsec)	0.183						
	PRF (Hz)	10710						
	$p_r@PII_{max}$ (MPa)	0.78						
	$d_{ed}@PII_{max}$ (cm)						-	
	Focal Length FL_x (cm)		3.80	-	-			#
	FL_v (cm)		2.70	-	-			#
	$I_{PA.3}@MI_{max}$ (W/cm ²)	11						
Operating Control Conditions	Control 1: Exam Type	Oph	Oph					
	Control 2: Optimization	Diff	Diff					
	Control 3: Depth	4.9cm	6.0cm					
	Control 4: MB	On	On					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-23 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: M Mode

Index Label		M.I.	TIS		TIB	TIC
			Scan	Non-scan	Non-scan	
				$A_{aprt} \leq 1$	$A_{aprt} > 1$	
Global Maximum Index Value		0.17	-	0.02	-	0.02 (b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)	0.47				
	W_0 (mW)		-	0.5		0.3 #
	min of $W_3(z_1, I_{TA,3}(z_1))$ (mW)				-	
	z_1 (cm)				-	
	z_{bd} (cm)				-	
	z_{sd} (cm)					0.95
	$z@PII_{,3max}$ (cm)	1.8				
	$d_{ed}(z_{sd})$ (cm)					0.13
	f_c (MHz)	7.57	-	7.64	-	7.44 #
	Dim of A_{aprt} X (cm)		-	0.60	-	0.40 #
	Y (cm)		-	0.30	-	0.30 #
Other Information	PD (μ sec)	0.181				
	PRF (Hz)	800				
	$p_r@PII_{max}$ (MPa)	0.75				
	$d_{ed}@PII_{max}$ (cm)					0.12
	Focal Length FL_x (cm)		-	1.95	-	#
	FL_v (cm)		-	2.70	-	#
	$I_{PA,3}@MI_{max}$ (W/cm ²)	12				
Operating Control Conditions	Control 1: Exam Type	Oph		Oph	Oph	
	Control 2: Optimization	Diff		Diff	Diff	
	Control 3: Depth	2.7cm		4.5cm	3.5cm	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-24 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: Color/CPD Mode

Index Label			M.I.	Scan	TIS		TIB	TIC
					Non-scan	Non-scan	Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	0.08	-	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$	(MPa)	0.47					
	W_0	(mW)		2.6	-		-	#
	min of $W_3(z_1), I_{TA.3}(z_1)$	(mW)					-	
	z_1	(cm)					-	
	z_{bd}	(cm)					-	
	z_{sd}	(cm)					-	
	$z@P_{II.3max}$	(cm)	1.9					
	$d_{eq}(z_{sd})$	(cm)					-	
	f_c	(MHz)	7.77	6.10	-	-	-	#
	Dim of A_{aprt}	X (cm)		1.00	-	-	-	#
Other Information		Y (cm)		0.30	-	-	-	#
	PD	(μ sec)	0.164					
	PRF	(Hz)	2424					
	$p_r@P_{II_{max}}$	(MPa)	0.78					
	$d_{eq}@P_{II_{max}}$	(cm)					-	
	Focal Length	FL_x (cm)		4.34	-	-		#
		FL_v (cm)		2.70	-	-		#
	$I_{PA.3}@M_{I_{max}}$	(W/cm ²)	18					
	Control 1: Exam Type	Oph	Oph					
	Control 2: Mode	CPD	CPD					
Operating Control Conditions	Control 3: 2D Optimization/Depth	Avg/4.2cm	Avg/4.9cm					
	Control 4: Color Optimization/PRF	Low/1250Hz	Low/781Hz					
	Control 5: Color Box Position/Size	Top/Wide & Short	Bottom /Def					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-25 Transducer Model: HSL25xp (Ophthalmic use), Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	-	0.19	-	0.26	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		0.42					
	W_0 (mW)			-	6.8		6.8	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{so} (cm)						0.90	
	$z@PII_{,3max}$ (cm)	0.8						
	$d_{ed}(z_{so})$ (cm)						0.40	
	f_c (MHz)	6.00	-	6.01	-	6.01	#	
	Dim of A_{aprt}	X (cm)		-	1.20	-	1.20	#
Y (cm)			-	0.30	-	0.30	#	
Other Information	PD (μsec)	2.586						
	PRF (Hz)	1562						
	$p_r@PII_{max}$ (MPa)	0.50						
	$d_{ed}@PII_{max}$ (cm)					0.34		
	Focal Length	FL_x (cm)		-	4.68	-		#
		FL_v (cm)		-	2.70	-		#
	$I_{PA,3}@MI_{max}$ (W/cm ²)	6						
Operating Control Conditions	Control 1: Exam Type	Oph		Oph		Oph		
	Control 2: Sample Volume Size	2mm		3mm		3mm		
	Control 3: Sample Volume Position	Zone 2		Zone 11		Zone 11		
	Control 4: PRF	1562Hz		5208Hz		5208Hz		

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-26 Transducer Model: ICTxp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.1	-	(a)	-	(a)	(b)
Associated Acoustic Parameter	$p_{r.3}$ (MPa)		2.46					
	W_0 (mW)			-	#		#	#
	min of $W_3(z_1), I_{TA.3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						#	
	$z@PII_{.3max}$ (cm)		1.4					
	$d_{ed}(z_{sd})$ (cm)						#	
	f_c (MHz)		4.64	-	#	-	#	#
	Dim of A_{aprt}	X (cm)		-	#	-	#	#
	Y (cm)		-	#	-	#	#	
Other Information	PD	(μ sec)	0.322					
	PRF	(Hz)	1600					
	$p_r@PII_{max}$	(MPa)	3.07					
	$d_{ed}@PII_{max}$	(cm)					#	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
		$I_{PA.3}@MI_{max}$ (W/cm ²)	347					
Operating Control Conditions	Control 1: Exam Type		Gyn					
	Control 2: Optimization		Diff					
	Control 3: Depth		4.1 cm					
	Control 4: THI		Off					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-27 Transducer Model: ICTxp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$		
Global Maximum Index Value			1.0	-	(a)	-	1.4	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.01					
	W_0 (mW)			-	#		19.8	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{so} (cm)						1.66	
	$z@PII_{,3max}$ (cm)		1.4					
	$d_{ed}(z_{so})$ (cm)						0.20	
	f_c (MHz)		4.38	-	#	-	4.36	#
	Dim of A_{aprt}	X (cm)		-	#	-	0.72	#
	Y (cm)		-	#	-	0.50	#	
Other Information	PD (μsec)		1.144					
	PRF (Hz)		1302					
	$p_r@PII_{\text{max}}$ (MPa)		2.49					
	$d_{ed}@PII_{\text{max}}$ (cm)						0.19	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
	$I_{PA,3}@MI_{\text{max}}$ (W/cm ²)		263					
Operating Control Conditions	Control 1: Exam Type		OB				OB	
	Control 2: Sample Volume Size		1 mm				12 mm	
	Control 3: Sample Volume Position		Zone 2				Zone 3	
	Control 4: PRF		1302 Hz				1562 Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-28 Transducer Model: L25xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.0	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$	(MPa)	2.42					
	W_0	(mW)		#	-		-	#
	min of $W_3(z_1), I_{TA.3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					-	
	$z@PII_{.3max}$	(cm)	0.9					
	$d_{ed}(z_{sd})$	(cm)					-	
	f_c	(MHz)	6.08	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
	Y (cm)		#	-	-	-	#	
Other Information	PD	(μsec)	0.606					
	PRF	(Hz)	783					
	$p_r@PII_{max}$	(MPa)	2.92					
	$d_{ed}@PII_{max}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_v (cm)		#	-	-		#
	$I_{PA.3}@MI_{max}$	(W/cm ²)	328					
Operating Control Conditions	Control 1: Exam Type	Ven						
	Control 2: Optimization	Avg						
	Control 3: Depth	2.4cm						
	Control 4: MB	On						
	Control 5: Needle Profiling	On						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-29 Transducer Model: L25xp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.2	-	(a)	-	(a)	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		3.17					
	W_0 (mW)			-	#		#	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						#	
	$z@PII_{,3max}$ (cm)		1.0					
	$d_{ed}(z_{sd})$ (cm)						#	
	f_c (MHz)		6.81	-	#	-	#	#
	Dim of A_{aprt}	X (cm)		-	#	-	#	#
	Y (cm)		-	#	-	#	#	
Other Information	PD (μsec)		0.192					
	PRF (Hz)		800					
	$p_r@PII_{max}$ (MPa)		3.98					
	$d_{ed}@PII_{max}$ (cm)						#	
	Focal Length	FL_x (cm)		-	#	-		#
		FL_v (cm)		-	#	-		#
		$I_{PA,3}@MI_{max}$ (W/cm ²)		408				
Operating Control Conditions	Control 1: Exam Type		Ven					
	Control 2: Optimization		Diff					
	Control 3: Depth		3.1cm					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-30 Transducer Model: L25xp, Operating Mode: Color/CPD

Index Label			M.I.	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					$A_{\text{aprt}} \leq 1$	$A_{\text{aprt}} > 1$		
Global Maximum Index Value			1.1	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.81					
	W_0 (mW)			#	-		-	#
	$\min(W_{.3}(z_1), I_{TA,3}(z_1))$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						-	
	$z@P_{II,3\text{max}}$ (cm)		1.0					
	$d_{ed}(z_{sd})$ (cm)						-	
	f_c (MHz)		7.08	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
	Y (cm)		#	-	-	-	#	
Other Information	PD (μsec)		0.178					
	PRF (Hz)		1717					
	$p_r@P_{II\text{max}}$ (MPa)		3.60					
	$d_{ed}@P_{II\text{max}}$ (cm)						-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_y (cm)		#	-	-		#
	$I_{PA,3}@M_{I\text{max}}$ (W/cm ²)		321					
Operating Control Conditions	Control 1: Exam Type		Msk					
	Control 2: Mode		CPD					
	Control 3: 2D Optimization/Depth		Avg/2.0cm					
	Control 4: Color Optimization/PRF		Low/312Hz					
	Control 5: Color Box Position/Size		Def/Narrow					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-31 Transducer Model: L25xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC
				Scan	Non-scan	Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$	
Global Maximum Index Value			(a)	-	(a)	-	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		#				
	W_0 (mW)			-	#	15.6	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-	
	z_1 (cm)					-	
	z_{bd} (cm)					-	
	z_{so} (cm)					1.40	
	$z@Pll_{,3max}$ (cm)	#					
	$d_{ed}(z_{so})$ (cm)					0.17	
	f_c (MHz)	#	-	#	-	6.00	#
	Dim of A_{aprt}	X (cm)		-	#	0.60	#
Other Information		Y (cm)		-	#	0.30	#
	PD (μ sec)		#				
	PRF (Hz)		#				
	$p_r@Pll_{max}$ (MPa)		#				
	$d_{ed}@Pll_{max}$ (cm)					0.16	
	Focal Length	FL_x (cm)		-	#	-	#
		FL_v (cm)		-	#	-	#
Operating Control Conditions	$I_{PA,3}@MI_{max}$ (W/cm ²)		#				
	Control 1: Exam Type					Ven	
	Control 2: Sample Volume Size					7mm	
	Control 3: Sample Volume Position					Zone 6	
	Control 4: PRF					2604 Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-32 Transducer Model: L25xp (Ophthalmic use), Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	0.03	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		0.45					
	W_0 (mW)			0.8	-		-	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						-	
	$z@PII_{.3max}$ (cm)	2.3						
	$d_{ed}(z_{sd})$ (cm)						-	
	f_c (MHz)	6.85	7.15	-	-	-	-	#
	Dim of A_{aprt}	X (cm)		1.12	-	-	-	#
	Y (cm)		0.30	-	-	-	#	
Other Information	PD (μsec)	0.183						
	PRF (Hz)	10710						
	$p_r@PII_{max}$ (MPa)	0.78						
	$d_{ed}@PII_{max}$ (cm)						-	
	Focal Length FL_x (cm)		3.80	-	-			#
	FL_v (cm)		2.70	-	-			#
	$I_{PA,3}@MI_{max}$ (W/cm ²)	11						
Operating Control Conditions	Control 1: Exam Type	Oph	Oph					
	Control 2: Optimization	Diff	Diff					
	Control 3: Depth	4.9cm	6.0cm					
	Control 4: MB	On	On					

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-33 Transducer Model: L25xp (Ophthalmic use), Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	-	0.02	-	0.02	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		0.47					
	W_0 (mW)			-	0.5		0.3	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						0.95	
	$z@PII_{,3max}$ (cm)		1.8					
	$d_{ed}(z_{sd})$ (cm)						0.13	
	f_c (MHz)		7.57	-	7.64	-	7.44	#
	Dim of A_{aprt} X (cm)			-	0.60	-	0.40	#
		Y (cm)		-	0.30	-	0.30	#
Other Information	PD (μsec)		0.181					
	PRF (Hz)		800					
	$p_r@PII_{max}$ (MPa)		0.75					
	$d_{ed}@PII_{max}$ (cm)						0.12	
	Focal Length FL_x (cm)			-	1.95	-		#
		FL_v (cm)		-	2.70	-		#
	$I_{PA,3}@MI_{max}$ (W/cm ²)		12					
Operating Control Conditions	Control 1: Exam Type		Oph		Oph	Oph		
	Control 2: Optimization		Diff		Diff	Diff		
	Control 3: Depth		2.7cm		4.5cm	3.5cm		

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-34 Transducer Model: L25xp (Ophthalmic use), Operating Mode: Color/CPD Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	0.08	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		0.47					
	W_0 (mW)			2.6	-		-	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						-	
	$z@P_{II,3max}$ (cm)		1.9					
	$d_{eq}(z_{sd})$ (cm)						-	
	f_c (MHz)		7.77	6.10	-	-	-	#
	Dim of A_{aprt}	X (cm)		1.00	-	-	-	#
	Y (cm)		0.30	-	-	-	#	
Other Information	PD (μ sec)		0.164					
	PRF (Hz)		2424					
	$p_r@P_{II,max}$ (MPa)		0.78					
	$d_{eq}@P_{II,max}$ (cm)						-	
	Focal Length	FL_x (cm)		4.34	-	-		#
		FL_v (cm)		2.70	-	-		#
	$I_{PA,3}@M_{I,max}$ (W/cm ²)		18					
Operating Control Conditions	Control 1: Exam Type		Oph	Oph				
	Control 2: Mode		CPD	CPD				
	Control 3: 2D Optimization/Depth		Avg/4.2cm	Avg/4.9cm				
	Control 4: Color Optimization/PRF		Low/1250Hz	Low/781Hz				
	Control 5: Color Box Position/Size		Top/Wide & Short	Bottom /Def				

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-35 Transducer Model: L25xp (Ophthalmic use), Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			0.17	-	0.19	-	0.26	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		0.42					
	W_0 (mW)			-	6.8		6.8	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{so} (cm)						0.90	
	$z@PII_{,3max}$ (cm)	0.8						
	$d_{ed}(z_{so})$ (cm)						0.40	
	f_c (MHz)	6.00	-	6.01	-	6.01	#	
	Dim of A_{aprt}	X (cm)		-	1.20	-	1.20	#
Y (cm)			-	0.30	-	0.30	#	
Other Information	PD (μsec)	2.586						
	PRF (Hz)	1562						
	$p_r@PII_{max}$ (MPa)	0.50						
	$d_{ed}@PII_{max}$ (cm)					0.34		
	Focal Length	FL_x (cm)		-	4.68	-		#
		FL_v (cm)		-	2.70	-		#
	$I_{PA,3}@MI_{max}$ (W/cm ²)	6						
Operating Control Conditions	Control 1: Exam Type	Oph		Oph		Oph		
	Control 2: Sample Volume Size	2mm		3mm		3mm		
	Control 3: Sample Volume Position	Zone 2		Zone 11		Zone 11		
	Control 4: PRF	1562Hz		5208Hz		5208Hz		

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-36 Transducer Model: L38xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r.3}$	(MPa)	3.32					
	W_0	(mW)		#	-		-	#
	min of $W_3(z_1), I_{TA.3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					-	
	$z@PII_{.3max}$	(cm)	1.1					
	$d_{ed}(z_{sd})$	(cm)					-	
	f_c	(MHz)	4.83	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
	Y (cm)		#	-	-	-	#	
Other Information	PD	(μsec)	0.426					
	PRF	(Hz)	790					
	$p_r@PII_{max}$	(MPa)	4.02					
	$d_{ed}@PII_{max}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_v (cm)		#	-	-		#
	$I_{PA.3}@MI_{max}$	(W/cm ²)	622					
Operating Control Conditions	Control 1: Exam Type	Ven						
	Control 2: Optimization	Avg						
	Control 3: Depth	3.5 cm						
	Control 4: MB	On/Off						
	Control 5: Needle Profiling	On						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-37 Transducer Model: L38xp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	-	1.5	-	2.0	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		3.37					
	W_0 (mW)			-	66.8		66.8	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						1.0	
	$z@PII_{,3max}$ (cm)		1.2					
	$d_{ed}(z_{sd})$ (cm)						0.62	
	f_c (MHz)		5.22	-	4.40	-	4.40	#
	Dim of A_{aprt}	X (cm)		-	2.40	-	2.40	#
	Y (cm)		-	0.40	-	0.40	#	
Other Information	PD (μsec)		0.277					
	PRF (Hz)		800					
	$p_r@PII_{max}$ (MPa)		4.18					
	$d_{ed}@PII_{max}$ (cm)						0.62	
	Focal Length	FL_x (cm)		-	7.02	-		#
		FL_v (cm)		-	1.50	-		#
		$I_{PA,3}@MI_{max}$ (W/cm ²)		538				
Operating Control Conditions	Control 1: Exam Type		Ven		Ven		Ven	
	Control 2: Optimization		Diff		Diff		Diff	
	Control 3: Depth		4.0 cm		9.0 cm		9.0 cm	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-38 Transducer Model: L38xp, Operating Mode: Color/CPD

Index Label			M.I.	Scan	TIS		TIB	TIC
					Non-scan	Non-scan	Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	(a)	-	-	-	(b)
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	3.20					
	W_0	(mW)		#	-		-	#
	min of W_3 (z_1), $I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					-	
	$z@PIL_{3max}$	(cm)	1.1					
	$d_{ed}(z_{sd})$	(cm)					-	
	f_c	(MHz)	4.63	#	-	-	-	#
	Dim of A_{aprt}	X (cm)		#	-	-	-	#
		Y (cm)		#	-	-	-	#
Other Information	PD	(μ sec)	0.578					
	PRF	(Hz)	5064					
	$p_r@PIL_{max}$	(MPa)	3.78					
	$d_{ed}@PIL_{max}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		#
		FL_v (cm)		#	-	-		#
	$I_{PA,3}@MI_{max}$	(W/cm ²)	678					
Operating Control Conditions	Control 1: Exam Type	Ven						
	Control 2: Mode	Color						
	Control 3: 2D Optimization/Depth	Avg/3.5 cm						
	Control 4: Color Optimization/PRF	High/6250 Hz						
	Control 5: Color Box Position/Size	Def/Wide-Tall						

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-39 Transducer Model: L38xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.0	-	2.0	-	2.9	(b)
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.21					
	W_0 (mW)			-	80.5		80.5	#
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{so} (cm)						1.00	
	$z@PII_{,3max}$ (cm)	0.3						
	$d_{ed}(z_{so})$ (cm)						0.43	
	f_c (MHz)	5.28	-	5.33	-	5.33	#	
	Dim of A_{aprt}	X (cm)		-	1.80	-	1.80	#
	Y (cm)		-	0.40	-	0.40	#	
Other Information	PD (μ sec)		1.201					
	PRF (Hz)		1562					
	$p_r@PII_{max}$ (MPa)		2.33					
	$d_{ed}@PII_{max}$ (cm)						0.40	
	Focal Length	FL_x (cm)		-	7.02	-		#
		FL_v (cm)		-	1.5	-		#
	$I_{PA,3}@MI_{max}$ (W/cm ²)		210					
Operating Control Conditions	Control 1: Exam Type		Nrv		Nrv		Nrv	
	Control 2: Sample Volume Size		1 mm		3 mm		3 mm	
	Control 3: Sample Volume Position		Zone 0		Zone 11		Zone 11	
	Control 4: PRF		1562 Hz		1562 Hz		1562 Hz	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-40 Transducer Model: P21xp, Operating Mode: 2D

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	(a)	-	-	-	2.0
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	2.02					
	W_0	(mW)		#	-		-	146.2
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					-	
	$z@PII_{.3max}$	(cm)	4.1					
	$d_{ed}(z_{sd})$	(cm)					-	
	f_c	(MHz)	1.85	#	-	-	-	2.09
	Dim of A_{aprt}	X (cm)		#	-	-	-	2.10
		Y (cm)		#	-	-	-	1.30
Other Information	PD	(μsec)	1.134					
	PRF	(Hz)	2933					
	$p_r@PII_{max}$	(MPa)	2.63					
	$d_{ed}@PII_{max}$	(cm)					-	
	Focal Length	FL_x (cm)		#	-	-		18.94
		FL_v (cm)		#	-	-		9.00
	$I_{PA,3}@MI_{max}$	(W/cm ²)	240					
Operating Control Conditions	Control 1: Exam Type	Crd						Crd
	Control 2: Optimization	Avg						Diff
	Control 3: Depth	9.0 cm						35 cm
	Control 4: MB/THI	Off/On						Off/On
	Control 5: Sector Size	Full						Narrow

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-41 Transducer Model: P21xp, Operating Mode: M Mode

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	-	(a)	-	1.9	1.9
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	2.02					
	W_0	(mW)		-	#		61.1	133.3
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					5.00	
	$z@PII_{,3max}$	(cm)	4.1					
	$d_{ed}(z_{sd})$	(cm)					0.36	
	f_c	(MHz)	1.85	-	#	-	1.99	2.26
	Dim of A_{aprt}	X (cm)		-	#	-	2.10	0.79
Y (cm)			#	-	-	1.30	1.30	
Other Information	PD	(μsec)	1.134					
	PRF	(Hz)	800					
	$p_r@PII_{max}$	(MPa)	2.63					
	$d_{ed}@PII_{max}$	(cm)					0.35	
	Focal Length	FL_x (cm)		-	#	-		2.25
		FL_v (cm)		-	#	-		9.00
	$I_{PA,3}@MI_{max}$	(W/cm ²)	240					
Operating Control Conditions	Control 1: Exam Type		Crd				Abd	OB
	Control 2: Optimization		Avg				Avg	Diff
	Control 3: Depth		9.0 cm				11 cm	5.0 cm
	Control 4: THI		On				On	Off
	Control 5: Simultaneous		Off				Off	On

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-42 Transducer Model: P21xp, Operating Mode: Color/CPD

Index Label			M.I.	TIS		TIB		TIC
				Scan	Non-scan		Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	2.4	-	-		2.8
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.02					
	W_0 (mW)			141.3	-		-	150.9
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1 (cm)					-		
	z_{bd} (cm)					-		
	z_{sd} (cm)						-	
	$z@P_{II,3max}$ (cm)		4.1					
	$d_{ed}(z_{sd})$ (cm)						-	
	f_c (MHz)		1.85	2.03	-	-	-	2.03
	Dim of A_{aprt}	X (cm)		0.33	-	-	-	0.33
	Y (cm)		1.30	-	-	-	1.30	
Other Information	PD (μ sec)		1.134					
	PRF (Hz)		800					
	$p_r@P_{II_{max}}$ (MPa)		2.63					
	$d_{ed}@P_{II_{max}}$ (cm)						-	
	Focal Length	FL_x (cm)		1.30	-	-		1.30
		FL_v (cm)		9.0	-	-		9.0
		$I_{PA,3}@M_{I_{max}}$ (W/cm ²)	240					
Operating Control Conditions	Control 1: Exam Type		Crđ	Abđ				Abđ
	Control 2: Mode		Color	Color				Color
	Control 3: 2D Optimization/THI/Depth		Avg/On/9.0 cm	Avg/Off/5.0 cm				Avg/Off/5.0 cm
	Control 4: Color Optimization/PRF		Low/880 Hz	High/2500 Hz				High/2500 Hz
	Control 5: Color Box Position/Size		Top/Short-Wide	Top/Def				Top/Def

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-43 Transducer Model: P21xp, Operating Mode: PW Doppler

Index Label			M.I.	TIS		TIB		TIC
				Scan	Non-scan		Non-scan	
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			1.5	-	-	1.3	3.7	2.8
Associated Acoustic Parameter	$p_{r,3}$ (MPa)		2.15					
	W_0 (mW)			-	-		204.3	82.3
	min of $W_3(z_1), I_{TA,3}(z_1)$ (mW)					126.9		
	z_1 (cm)					3.1		
	z_{bd} (cm)					2.8		
	z_{sd} (cm)						3.90	
	$z@Pll_{,3max}$ (cm)	1.4						
	$d_{ea}(z_{sd})$ (cm)						0.71	
	f_c (MHz)	2.07	-	-	2.09	2.09		2.08
	Dim of A_{aprt}	X (cm)		-	-	2.10	2.10	0.33
	Y (cm)		-	-	1.30	1.30	1.30	
Other Information	PD (μ sec)	1.195						
	PRF (Hz)	1562						
	$p_r@Pll_{max}$ (MPa)	2.37						
	$d_{ea}@Pll_{max}$ (cm)						0.49	
	Focal Length	FL_x (cm)		-	-	14.01		1.30
		FL_y (cm)		-	-	9.0		9.0
	$I_{PA,3}@MI_{max}$ (W/cm ²)	184						
Operating Control Conditions	Control 1: Exam Type	Abd			Crđ	Abd	Abd	
	Control 2: Sample Volume Size	0 mm			14 mm	14 mm	7 mm	
	Control 3: Sample Volume Position	Zone 1			Zone 8	Zone 8	Zone 0	
	Control 4: PRF	1562 Hz			1562 Hz	1562 Hz	1953 Hz	
	Control 5: TDI	-			Off	-	-	

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Table 9-44 Transducer Model: P21xp, Operating Mode: CW Doppler

Index Label			M.I.	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					$A_{aprt} \leq 1$	$A_{aprt} > 1$		
Global Maximum Index Value			(a)	-	1.1	-	4.0	2.6
Associated Acoustic Parameter	$p_{r,3}$	(MPa)	#					
	W_0	(mW)		-	113.2		110.9	113.2
	min of $W_3(z_1), I_{TA,3}(z_1)$	(mW)				-		
	z_1	(cm)				-		
	z_{bd}	(cm)				-		
	z_{sd}	(cm)					1.25	
	$z@P_{II,3max}$	(cm)	#					
	$d_{ed}(z_{sd})$	(cm)					0.52	
	f_c	(MHz)	#	-	2.0	-	2.00	2.00
	Dim of A_{aprt}	X (cm)		-	0.72	-	0.85	0.72
	Y (cm)		-	1.30	-	1.30	1.30	
Other Information	PD	(μsec)	#					
	PRF	(Hz)	#					
	$p_r@P_{II,max}$	(MPa)	#					
	$d_{ed}@P_{II,max}$	(cm)					0.47	
	Focal Length	FL_x (cm)		-	1.30	-		1.30
		FL_v (cm)		-	9.0	-		9.0
	$I_{PA,3}@MI_{max}$	(W/cm ²)	#					
Operating Control Conditions	Control 1: Exam Type				Crd		Crd	Crd
	Control 2: Sample Volume				Zone 0		Zone 1	Zone 0

(a) This index is not required for this operating mode; value is <1.

(b) This transducer is not intended for transcranial or neonatal cephalic uses.

No data are reported for this operating condition since the global maximum index value is not reported for the reason listed. (Reference Global Maximum Index Value line.)

- Data are not applicable for this transducer/mode.

Acoustic measurement precision and uncertainty

All table entries have been obtained at the same operating conditions that give rise to the maximum index value in the first column of the table. Measurement precision and uncertainty for power, pressure, intensity, and other quantities that are used to derive the values in the acoustic output table are shown in the table below. In accord-

ance with Section 6.4 of the Output Display Standard, the following measurement precision and uncertainty values are determined by making repeat measurements and stating the standard deviation as a percentage.

Table 9-45 Acoustic Measurement Precision and Uncertainty

Quantity	Precision (% of standard deviation)	Uncertainty (95% confidence)
Pr	1.9%	±11.2%
Pr _{.3}	1.9%	±12.2%
Wo	3.4%	±10%
fc	0.1%	±4.7%
PII	3.2%	+12.5 to -16.8%
PII _{.3}	3.2%	+13.47 to -17.5%

Terminology in acoustic output tables

- A_{aprt}

Area of the active aperture measured in cm².
- d_{eq}(z)

Equivalent beam diameter as a function of axial distance z, and is equal to $\sqrt{(4/(\pi))((Wo)/(I_{TA}(z)))}$, where I_{TA}(z) is the temporal-average intensity as a function of z in centimeters.
- d_{eq}@PII

Equivalent beam diameter at the point where the free-field, spatial-peak pulse intensity integral is a maximum in centimeters.
- Dim_{max}

Active aperture dimensions for the azimuthal (x) and elevational (y) planes in centimeters.
- A_{aprt}

Center frequency in MHz.
- FL

Focal length, or azimuthal (x) and elevational (y) lengths, if different measured in centimeters.
- I_{pa.3}@MI_{max}

Derated pulse average intensity at the maximum MI in units of W/cm².
- I_{SPTA.3}

Derated spatial peak, temporal average intensity in units of milliwatts/cm².
- I_{SPTA.3}

Derated spatial-peak temporal-average intensity at axial distance z₁ (milliwatts per square centimeter).
- z₁

Mechanical index.
- PD

Pulse duration (microseconds) associated with the transmit pattern giving rise to the reported value of MI.

$P_{r,3}$	Derated peak rarefactional pressure associated with the transmit pattern giving rise to the value reported under MI (Megapascals).
$p_r@PII$	Peak rarefactional pressure at the point where the free-field, spatial-peak pulse intensity integral is a maximum in Megapascals.
PRF^{max}	Pulse repetition frequency associated with the transmit pattern giving rise to the reported value of MI in Hertz.
TI	Applicable thermal index for the transducer, imaging mode, and exam type.
TI type	Thermal index value for the transducer, imaging mode, and exam type.
TI value	(Bone thermal index) is a thermal index for applications in which the ultrasound beam passes through soft tissue and a focal region is in the immediate vicinity of bone. TIB non-scan is the bone thermal index in the non-auto-scanning mode.
TIC	(Cranial bone thermal index) is the thermal index for applications in which the ultrasound beam passes through bone near the beam entrance into the body.
TIS	(Soft tissue thermal index) is a thermal index related to soft tissues. TIS scan is the soft tissue thermal index in an auto-scanning mode. TIS non-scan is the soft tissue thermal index in the non-autoscanning mode.
$W_{.3}(z_1)$	Derated ultrasonic power at axial distance z_1 in units of milliwatts.
W_o	Ultrasonic power, except for TIS_{scan} , in which case it is the ultrasonic power passing through a one centimeter window in units of milliwatts.
z_1	Axial distance corresponding to the location of maximum $\min\{W_{.3}(z_1), I_{TA,3}(z) \times 1 \text{ cm}^2\}$, where $z \geq z_{bp}$ in centimeters.
z_{bp}	1.69 (A_{aprt}) in centimeters.
z_{sp}	For MI, the axial distance at which $p_{r,3}$ is measured. For TIB, the axial distance at which TIB is a global maximum (for example, $z_{sp} = z_{b,3}$) in centimeters.

General terms

For ultrasound terms not included in this glossary, refer to Recommended Ultrasound Terminology, Second Edition, published in 1997 by AIUM.

AIUM	American Institute of Ultrasound in Medicine
as low as reasonably achievable (ALARA)	The guiding principle of ultrasound use, which states that you should keep patient exposure to ultrasound energy as low as reasonably achievable for diagnostic results.
curved array transducer	Identified by the letter C a number (60). The number corresponds to the radius of curvature of the array expressed in millimeters. The transducer elements are electrically configured to control the characteristics and direction of the acoustic beam. For example, C60xp.
depth	Refers to the depth of the display. A constant speed of sound of 1538.5 meters/second is assumed in the calculation of echo position in the image.
<i>in situ</i>	In the natural or original position.

LCD	liquid crystal display
linear array transducer	Identified by the letter L (linear) and a number (38). The number corresponds to the radius of width of the array expressed in millimeters. The transducer elements are electrically configured to control the characteristics and direction of the acoustic beam. For example, L38xp.
mechanical index (MI)	An indication of the likelihood of mechanical bioeffects occurring: the higher the MI, the greater the likelihood of mechanical bioeffects. See "Acoustic output" on page 297 for a more complete description of MI.
MI/TI	See mechanical index (MI) and thermal index (TI).
phased array	A transducer designed primarily for cardiac scanning. Forms a sector image by electronically steering the beam direction and focus.

SonoMB technology A subset of the 2D imaging mode in which the 2D image is enhanced by looking at a target from multiple angles and then merging or averaging the scanned data together to improve overall image quality and, in parallel, reducing noise and artifacts.

target depth A depth on the display that corresponds to the skin/transducer interface.

thermal index (TI) The ratio of total acoustic power to the acoustic power required to raise tissue temperature by 1°C under defined assumptions. See "Acoustic output" on page 297 for a more complete description of TI.

TIB (bone thermal index) A thermal index for applications in which the ultrasound beam passes through soft tissue and a focal region is in the immediate vicinity of bone.

TIC (cranial bone thermal index) A thermal index for applications in which the ultrasound beam passes through bone near the beam entrance into the body.

TIS (soft tissue thermal index) A thermal index related to soft tissues.

Tissue Doppler Imaging (TDI) A pulsed wave Doppler technique used to detect myocardial motion.

Tissue Harmonic Imaging Transmits at one fre-

quency and receives at a higher harmonic frequency to reduce noise and clutter and improve resolution.

transducer A device that transforms one form of energy into another form of energy. Ultrasound transducers contain piezoelectric elements, which when excited electrically, emit acoustic energy. When the acoustic energy is transmitted into the body, it travels until it encounters an interface, or change in tissue properties. At the interface, an echo is formed that returns to the transducer, where this acoustic energy is transformed into electrical energy, processed, and displayed as anatomical information.

variance Displays a variation in Color Doppler flow imaging within a given sample. Variance is mapped to the color green and is used to detect turbulence.

Terminology in acoustic output tables

A_{aprt} Area of the active aperture measured in cm^2 .

$d_{eq}(z)$	Equivalent beam diameter as a function of axial distance z , and is equal to $\sqrt{(4/(\pi))(W_o)/(I_{TA}(z))}$, where $I_{TA}(z)$ is the temporal-average intensity as a function of z in centimeters.	TI value	Thermal index value for the transducer, imaging mode, and exam type.
$d_{eq@PII}$	Equivalent beam diameter at the point where the free-field, spatial-peak pulse intensity integral is a maximum in centimeters.	TIB	(Bone thermal index) is a thermal index for applications in which the ultrasound beam passes through soft tissue and a focal region is in the immediate vicinity of bone. TIB non-scan is the bone thermal index in the non-autoscanning mode.
Dim. of A_{aprt}	Active aperture dimensions for the azimuthal (x) and elevational (y) planes in centimeters.	TIC	(Cranial bone thermal index) is the thermal index for applications in which the ultrasound beam passes through bone near the beam entrance into the body.
fc	Center frequency in MHz.	TIS	(Soft tissue thermal index) is a thermal index related to soft tissues. TIS scan is the soft tissue thermal index in an auto-scanning mode. TIS non-scan is the soft tissue thermal index in the non-autoscanning mode.
FL	Focal length, or azimuthal (x) and elevational (y) lengths, if different measured in centimeters.		
$I_{pa.3@Mimax}$	Derated pulse average intensity at the maximum MI in units of W/cm^2 .	W_{z_1}	Derated ultrasonic power at axial distance z_1 in units of milliwatts.
$I_{SPTA.3}$	Derated spatial peak, temporal average intensity in units of milliwatts/cm ² .	W_o	Ultrasonic power, except for TIS_{scan} , in which case it is the ultrasonic power passing through a one centimeter window in units of milliwatts.
$I_{SPTA.3}$	Derated spatial-peak temporal-average intensity at axial distance z_1 (milliwatts per square centimeter).	z_1	Axial distance corresponding to the location of maximum $\min(W_{z_1}, I_{TA.3}(z) \times 1 \text{ cm}^2)$, where $z \geq z_{bp}$ in centimeters.
MI	Mechanical index.	z_{bp}	1.69 (A_{aprt}) in centimeters.
PD	Pulse duration (microseconds) associated with the transmit pattern giving rise to the reported value of MI.	z_{sp}	For MI, the axial distance at which $p_{r.3}$ is measured. For TIB, the axial distance at which TIB is a global maximum (for example, $z_{sp} = z_{b.3}$) in centimeters.
$P_{r.3}$	Derated peak rarefactional pressure associated with the transmit pattern giving rise to the value reported under MI (Megapascals).		
$p_{r@PII}$	Peak rarefactional pressure at the point where the free-field, spatial-peak pulse intensity integral is a maximum in Megapascals.		
PRF	Pulse repetition frequency associated with the transmit pattern giving rise to the reported value of MI in Hertz.		
TI type	Applicable thermal index for the transducer, imaging mode, and exam type.		



SonoSite
FUJIFILM

P14645-02

